

## **SSCP Part B**

Kurzbericht über Sicherheit und klinische Leistung  
für den Patienten gemäß Verordnung (EU)  
2017/ 745 Artikel 32 und MDCG 2019-9 Rev.1

**Produkt/e:**  
**Dentale Implantate und Dentale Abutments**

Ritter Implants GmbH & Co. KG	SSCP Part B	I55-0526-2301
		Datum: 2023-02-28

# 1 Inhaltsverzeichnis

1	Inhaltsverzeichnis.....	2
2	Tabellenverzeichnis .....	3
3	Abkürzungen .....	4
4	Kurzbericht über Sicherheit und klinische Leistung für den Patienten.....	5
4.1	<b>Identifizierung des Produkts und des Herstellers</b> .....	5
4.1.1	Handelsname(n) des Produkts .....	5
4.1.2	Name und Adresse des Herstellers.....	5
4.1.3	Basis UDI-DI .....	5
4.1.4	Jahr, indem das Produkt erstmalig mit CE-Kennzeichnung auf den Markt gekommen ist .....	6
4.2	<b>Zweckbestimmung des Medizinprodukts</b> .....	6
4.2.1	Vorgesehene Anwendung .....	6
4.2.2	Indikation(en) und vorgesehene Patientengruppe(n) .....	6
4.2.3	Kontraindikationen .....	7
4.3	<b>Produktbeschreibung</b> .....	11
4.3.1	Produktbeschreibung und Materialien/Substanzen, die mit dem Patienten in Kontakt kommen. ....	11
4.3.2	Informationen zu medizinischen Substanzen im Medizinprodukt, falls zutreffend .....	12
4.3.3	Beschreibung, wie das Medizinprodukt seine beabsichtigte Wirkung erzielt .....	12
4.3.4	Beschreibung von Zubehör, falls zutreffend .....	12
4.4	<b>Risiken und Warnhinweise</b> .....	12
4.4.1	Wie werden potenzielle Risiken kontrolliert und verarbeitet.....	12
4.4.2	Verbleibende Risiken und unerwünschte Nebenwirkungen .....	12
4.4.3	Warnhinweise und Vorbeugungsmaßnahmen .....	13
4.4.4	Zusammenfassung von Korrekturmaßnahmen im Feld (FSCA inklusive FSN), falls zutreffend .....	15
4.5	<b>Zusammenfassung der klinischen Bewertung und des Berichts der klinischen Nachbeobachtung nach dem Inverkehrbringen</b> .....	15
4.5.1	Klinischer Hintergrund des Produkts .....	18
4.5.2	Der klinische Nachweis für die CE-Kennzeichnung .....	18
4.5.3	Sicherheit.....	18
4.6	<b>Mögliche diagnostische oder therapeutische Alternativen</b> .....	19
4.7	<b>Empfohlene Anwender-Schulungen</b> .....	21
5	Anhang .....	21
5.1	Implantatsysteme .....	21
5.2	Dentale Implantataufbauten .....	34
6	Verweis.....	199

Ritter Implants GmbH & Co. KG	SSCP Part B	I55-0526-2301
		Datum: 2023-02-28

## 2 Tabellenverzeichnis

Tabelle 1 Basis-UDI-DI.....	5
Tabelle 2 Nutzen-Risiko .....	18
Tabelle 3 therapeutische Alternativen .....	20
Tabelle 4 QSI Ri-Quadro Spiral Implantate .....	21
Tabelle 5 NL-QSI Ri-Quadro Spiral Implantate .....	22
Tabelle 6 SB/LA Spiral Implantate.....	23
Tabelle 7 SB/LA Spiral Implantate (NL).....	24
Tabelle 8 iOS3/ Ora3/ iT (EM).....	24
Tabelle 9 Lamina/ iH (EM).....	24
Tabelle 10 QSI (EM).....	26
Tabelle 11 Augmentat iH/ Lamina Augmentat (EM) .....	26
Tabelle 12 Artic/ Ora/ iV (EM) .....	27
Tabelle 13 Augmentat iV/ Artic Augmentat (EM).....	29
Tabelle 14 Fortis/ iT (EM).....	30
Tabelle 15 Augmentation iT/ Fortis Augmentat (EM).....	31
Tabelle 16 i8 (EM).....	32
Tabelle 17 SB/LA Spiral (EM) .....	33
Tabelle 18 Titan Abutments .....	34
Tabelle 19 Titan Abutments (EM).....	36
Tabelle 20 Zirkon Abutments .....	51
Tabelle 21 Zirkon Abutments (EM).....	52
Tabelle 22 Pop Up Transfer Abutments (PUT).....	59
Tabelle 23 Clicq-Pression Transfer Abutments (EM) .....	62
Tabelle 24 Clicq-Pression Kits (EM).....	70
Tabelle 25 Ball Attachments.....	72
Tabelle 26 Ball Attachments (EM).....	73
Tabelle 27 Ball Attachments Kits (EM).....	75
Tabelle 28 Clicq Overdenture.....	77
Tabelle 29 Pop-Clicq Overdenture (EM) .....	81
Tabelle 30 Q-Clicq (EM).....	94
Tabelle 31 Q-Clicq Kits (EM).....	97
Tabelle 32 One Piece Multi-Unit.....	99
Tabelle 33 Multi-Clicq On-X (EM).....	103
Tabelle 34 Multi-Clicq Kits (EM) .....	130
Tabelle 35 2 Piece Multi-Unit .....	147
Tabelle 36 Clicq-Base (EM) .....	148
Tabelle 37 Abutments zum Angießen .....	162
Tabelle 38 Castable & UCLA Abutments (EM).....	164
Tabelle 39 Milling Blanks.....	165
Tabelle 40 Pre-Milled Blank (EM).....	166
Tabelle 41 Ti-Base CAD/CAM.....	173
Tabelle 42 Uni-Base One Piece Ti-Base (EM) .....	175
Tabelle 43 Scan Bodies .....	196
Tabelle 44 Scan Abutments (EM).....	197
Tabelle 45 Clicq-Scan Abutment (EM) .....	197
Tabelle 46 Multi-Clicq Scan Abutments (EM).....	197

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

### 3 Abkürzungen

*Tabelle 1 Abkürzungen*

	<b>Erklärung englisch</b>	<b>Erklärung deutsch</b>	<b>MDR</b>
Eudamed	European database on medical devices	Europäische Datenbank für Medizinprodukte	Art. 33
FSCA	Field safety corrective action	Sicherheitskorrekturmaßnahmen im Feld	Art.2 (68)
FSN	Field safety notice	Sicherheitsanweisung im Feld	Art.2 (69)
IFU	Instructions for use	Gebrauchsanweisung	Annex I, Kapitel 3
NB	Notified body	Benannte Stelle	Art.2 (42)
PMCF	Post-market clinical follow-up	Klinische Nachbeobachtung nach dem Inverkehrbringen	Annex XIV Teil B
PMS	Post-market surveillance	Überwachung nach dem Inverkehrbringen	Art.2 (60)
PSUR	Periodic safety update report	Regelmäßig aktualisierter Bericht über die Sicherheit („Sicherheitsbericht“)	Art.86
SRN	Single registration number for an economic operator	Einmalige Registrierungsnummer	Art. 31
SSCP	Summary of safety and clinical performance	Kurzbericht über Sicherheit und klinische Leistung („Kurzbericht“)	Art.32
UDI-DI	Unique Device Identification - Device identifier	Produktkennung (DI) vom System zur eindeutigen Produktidentifikation (UDI)	Art.2 (15) Art. 27

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

## 4 Kurzbericht über Sicherheit und klinische Leistung für den Patienten

Dieser Kurzbericht über Sicherheit und klinische Leistung (SSCP) soll den öffentlichen Zugang zu einer aktuellen Zusammenfassung der Hauptaspekte bezüglich Sicherheit und klinische Leistung von Medizinprodukten gewährleisten. Die nachfolgende Information ist für Patienten oder Laien ausgelegt. Ein ausführlicherer Kurzbericht über Sicherheit und klinische Leistung für Angehörige von Gesundheitsberufen ist im ersten Teil dieses Dokuments aufgeführt.

Der Kurzbericht dient nicht dazu allgemeine Ratschläge über die Behandlung einer Krankheit zu geben. Bitte nehmen Sie Kontakt zum Gesundheitspersonal auf den Fall, dass Sie Fragen zu Ihrer Krankheit haben oder bezüglich des Umgangs mit dem Medizinprodukt in Ihrem medizinischen Fall. Der Kurzbericht (SSCP) ersetzt weder einen Implantatausweis noch die Gebrauchsanweisung, um die Informationen des sicheren Gebrauchs des Medizinprodukts zur Verfügung zu stellen.

### 4.1 Identifizierung des Produkts und des Herstellers

#### 4.1.1 Handelsname(n) des Produkts

Die Handelsnamen und Produktbeschreibungen der Produkte befinden sich im Anhang des Berichts.

#### 4.1.2 Name und Adresse des Herstellers

Ritter Implants GmbH & Co. KG  
Freiburger Str. 45  
88400 Biberach

#### 4.1.3 Basis UDI-DI

*Tabelle 1 Basis-UDI-DI*

Produktgruppe		Basis-UDI-DI
1	QSI Ri-Quadro Spiral Implantate	++ERITP0102010101MJ
2	NL-QSI Ri-Quadro Spiral Implantate-Narrow Line	++ERITP0102010102ML
3	SB/LA Spiral Implantate	++ERITP0102010103MN
4	SB/LA Spiral Implantate Narrow Line	++ERITP0102010104MQ
5	iOs3/ Ora3/ iT3	++ERITP0102010105MS
6	Lamina/ iH/ QSI	++ERITP0102010106MU
7	Augmentat iH/ Lamina Augmentat/ Augmentat iV/ Artic Augmentat/ Augmentat iT/ Fortis Augmentat	++ERITP0102010107MW
8	Artic/ Ora/ iV	++ERITP0102010108MY
9	Fortis/ iT	++ERITP0102010109N2
10	i8	++ERITP0102010110MK

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

Produktgruppe		Basis-UDI-DI
11	Titanium Abutments	++ERITP0102010112MP
12	Zirconium Abutments	++ERITP0102010113MR
13	Transfer Abutment System	++ERITP0102010114MT
14	Ball Systems	++ERITP0102010115MV
15	Clicq Systems	++ERITP0102010116MX
16	Multi-Unit Systems	++ERITP0102010117MZ

#### 4.1.4 Jahr, indem das Produkt erstmalig mit CE-Kennzeichnung auf den Markt gekommen ist

Die Produkte der Ritter Implants GmbH & Co. KG sind seit 2012 CE-Zertifiziert und die Produkte der Produktlinie Edison Medical (EM) sind seit 2021 CE-Zertifiziert.

## 4.2 Zweckbestimmung des Medizinprodukts

Die Zweckbestimmung der Dentalen Implantate und Dentalen Implantataufbauten besteht in der Wiederherstellung der Kaufunktion.

Die Kaufunktion wird durch die Implantate als künstliche Zahnwurzel und den Implantataufbauten, die den Verbund zwischen der prothetischen Versorgung (Prothese bzw. Zahnersatz) darstellen, wiederhergestellt.

### 4.2.1 Vorgesehene Anwendung

Die **Implantate und Abutments** sind zur langzeitigen oralen Anwendung bestimmt und stehen mit der Mund- / Rachenschleimhaut in Kontakt.

### 4.2.2 Indikation(en) und vorgesehene Patientengruppe(n)

Die Indikation des Dental Implants und Dental Implants Accessories stellt den Ersatz der verlorengegangenen Anatomie dar.

#### **Die Implantate werden für zweistufige und einteilige Implantationsprozesse eingesetzt.**

Die Implantate sind für den einfachen oder mehrfachen Ersatz von verlorenen Zähnen vorgesehen und bieten eine Möglichkeit, die prothetischen Teile bei ganz oder teilweise zahnlosen Patienten einzusetzen.

#### **Spezifische Verwendungszwecke für Implantate mit einem Durchmesser von 3.0 mm**

Aufgrund ihrer reduzierten mechanischen Stabilität werden Implantate mit kleinem Durchmesser 3.0 mm nur in Fällen mit geringer mechanischer Belastung verwendet, wie z.B. seitliche Schneidezähne im Oberkiefer oder Schneidezähne im Unterkiefer. Eine Platzierung im Seitenzahnbereich wird nicht empfohlen.

#### **Spezifische Verwendungszwecke für Implantate der Länge 6 mm**

Aufgrund der reduzierten Verankerungsfläche im Knochen sind diese Implantate nur für die folgenden Indikationen zu verwenden:

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

a) Als zusätzliches Implantat, zusammen mit längeren Implantaten zur Unterstützung der prothetischen Versorgung.

b) Als Hilfsimplantat für Implantatstegkonstruktionen, die Vollprothesen in einem stark atrophierten Unterkiefer tragen.

**Die Abutments sind für den ein- oder mehrfachen Ersatz verlorener Zähne vorgesehen und bieten eine Möglichkeit, die prothetischen Teile bei ganz oder teilweise zahnlosen Patienten zu befestigen.**

### Position im Mund

- Gerade Abutments: Befinden sich in allen Sektoren (Bereichen) des Mundes.
- Abgewinkelte Abutments: Befinden sich im vorderen Sektor (Bereiche) des Oberkiefers für 15° abgewinkelte Abutments  
  
Sektoren, in denen die vorhandenen Defekte ein Implantat senkrecht zur okklusalen Ebene 25° abgewinkelt unmöglich machen
- Kugelaufsatz/ Clicq Deckprothese/ Mehrfachsystem: Werden in allen Bereichen des Mundes eingesetzt, wird aber üblicherweise im Frontzahnbereich für Deckprothesen verwendet
- Gingivaformer / Kappen: Befinden sich in allen Bereichen des Mundes

Als vorgesehene Patientengruppe der Dental Implants und Dental Implants Accessories werden Erwachsene in unterschiedlichen Zahnverluststadien behandelt. Zur Wiederherstellung der Kaufunktion werden Patienten mit natürlichen Restzahnbestand und/ oder in Kombination mit Implantaten und/ oder mit Implantat-getragenen Restaurationen im Ober-/ Unterkiefer versorgt.

Die vorgesehenen Anwender sind zahnmedizinisches Fachpersonal (Zahnärzte/ zahnmedizinisches Fachangestellte/ Zahntechniker) und geschultes medizinisches Fachpersonal (Pflegepersonal). Die zahnmedizinischen Personengruppen sind aufgrund ihres Ausbildungsstandes als Anwender qualifiziert.

Die Produkte der Dental Implants und Dental Implants Accessories sind nur für zertifizierte Zahnärzte und autorisierte Personen mit spezifischer Implantat Ausbildung bestimmt

### 4.2.3 Kontraindikationen

Das Setzen von Zahnimplantaten ist bei den folgenden Patienten kontraindiziert:

1. Patienten, die medizinisch nicht für einen oralchirurgischen Eingriff geeignet sind (Patienten, die Kortikosteroide, Antikoagulanzen, Antikonvulsiva oder Bisphosphonate einnehmen, sowie Patienten, die eine Bestrahlung oder eine andere immunsuppressive Therapie erhalten).
2. Stillende oder schwangere Frauen sind keine Kandidaten, ebenso wenig Patienten mit abnormalen Laborwerten für Blutharnstoff-Stickstoff, Kreatinin oder Serumkalzium.
3. Patienten mit unkontrolliertem Diabetes, kardiovaskulären Erkrankungen, Endokarditis und Bluthochdruck über 170/110 mm Hg.
4. Osteoporotische Stauchungsfrakturen, Atemwegserkrankungen, Schilddrüsen- oder Nebenschilddrüsenenerkrankungen sowie Patienten mit diagnostizierter bösartiger Erkrankung oder unerklärlichen Knoten oder Massen im Kopf oder Hals.

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

**5.** Patienten mit unkontrollierten Krankheiten wie: Hämophilie, Granulozytopenie oder anderen Blutungsproblemen, Steroidgebrauch, prophylaktischer Antibiotikagebrauch, brüchiger Diabetes, Ehler-Danlos-Syndrom, Osteoradionekrose, Nierenversagen, Organtransplantation, Antikoagulationstherapie, ungeklärte Überempfindlichkeit, fibröse Dysplasie, regionale Enteritis.

**6.** Krankheiten oder Behandlungen, die die Heilung stark beeinträchtigen, z. B. einschließlich Strahlentherapie.

**7.** Mangel an angemessener Ausbildung des Behandlers.

**8.** Unzureichende Motivation des Patienten, wie z. B. psychiatrische Störungen, die das Verständnis des Patienten und die Einhaltung der notwendigen Verfahren beeinträchtigen, unrealistische Erwartungen des Patienten, unerreichbare prothetische Rekonstruktion, Unfähigkeit des Patienten, die Mundhygiene zu handhaben, Überempfindlichkeit des Patienten gegenüber bestimmten Komponenten des Verfahrens, z. B. Titanüberempfindlichkeit.

**9.** Elektrochirurgie: Zahnimplantate bestehen aus einer metallischen Legierung; sie zeichnen sich daher durch eine hohe Leitfähigkeit aus. Aus diesem Grund ist die Elektrochirurgie in der Nähe von Zahnimplantaten streng kontraindiziert.

**10.** Das System ist nicht für die Verwendung bei einem begrenzten interokklusalen Abstand von weniger als 7,0 mm geeignet. Es ist nicht zur Verwendung geeignet, wenn die Implantate mit dem 25° abgewinkelten Abutment um mehr als 45° divergieren oder wenn die Implantate mit dem 15° abgewinkelten Abutment um mehr als 30° divergieren. Das abgewinkelte Abutment darf nicht übermäßig präpariert werden. Die Narrow Line (NL) der Implantate hat eine ähnliche Art von Abutments. Die NL-QSI-Implantate (Narrow line) sind in den Ausführungen gerade und 15° erhältlich. Die NL-Abutments sind nur für die NL-QSI und NL-SNAP\* 3,0mmD\* und 3,3mmD vorgesehen. Ritter Zahnimplantate sollten nicht gesetzt werden, wenn das Volumen des Alveolarknochens nicht ausreicht, um das Implantat zu tragen (mindestens 2 mm zirkumferenziell und 2 mm apikal). Im Oberkiefer eingesetzte Implantate sollten die Sinusbodenmembran nicht perforieren.

**11.** Schlechte Knochenqualität, schlechte Mundhygiene des Patienten, starkes Rauchen, Verwendung von Kautabak, unkontrollierte systematische Krankheiten (Diabetes usw.), reduzierte Immunität, AIDS, Alkoholismus, Drogenabhängigkeit, psychische Instabilität, Aggression, Kinder mit unterentwickeltem Knochen und Knochenstoffwechselstörungen können zu mangelnder Integration und/oder späterem Implantatversagen beitragen. Starker Bruxismus, Zähneknirschen und Überbelastung können zu Knochenverlust, Schraubenlockerung, Komponentenfraktur und/oder Implantatversagen führen. Strahlenbelastung und Chemotherapie können die Gesundheit und den Erfolg des Implantats beeinträchtigen.

**Temporäre Kontraindikationen für Implantate sind:**

1. Systemische Infektionen, lokale orale und respiratorische Infektionen, Entzündungen um das Implantat

2. Anatomische oder pathologische Implikationen wie;

a) Unzureichende Alveolarknochenbreite und -höhe, um das Implantat mit mindestens 1,5 Millimeter Knochen zu umgeben

b) Unzureichende Knochenhöhe, bei der eine korrekte Implantatinsertion innerhalb von 2 mm in den Mandibularkanal, den Sinusboden usw. eingreifen würde

3. Bösartige Erkrankungen

4. Schwangerschaft

5. Unzureichendes Knochenvolumen, es sei denn, ein Augmentationsverfahren kann in Betracht gezogen werden. Zahnimplantat-Patienten sollten angewiesen werden, ihren Arzt zu konsultieren, bevor sie sich solchen Behandlungsoptionen unterziehen

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

Die Anwendung von Abutments ist in den folgenden Fällen Kontraindiziert:

Unsachgemäße Technik kann zu Knochenschwund und Implantatversagen führen. Ritter Dental Implantatsysteme sind nur für den Einsatz mit Ritter Dental speziell entwickelten Knochenbohrern und Prothesen vorgesehen. Implantate, die in einem starken Winkel zum bestehenden Gebiss gesetzt werden, oder mehrere Implantate, die konvergent oder divergent gesetzt werden, können zu komplexen Restaurationen führen, die Implantate überlasten können.

Eine gründliche diagnostische Abklärung und die Verwendung einer chirurgischen Schablone wird empfohlen, um die korrekte Positionierung des oder der Implantate zu gewährleisten.

Relative Kontraindikationen sind die Verwendung von Steroiden, Chemotherapeutika, Bisphosphonaten und Antikoagulantien. Diese und andere Medikamente, die den Operationsbereich, das umgebende Gewebe oder die Heilungsfunktion des Patienten beeinflussen können, können den Erfolg des Implantats beeinflussen. Eine sorgfältige Patientenauswahl einschließlich der Rücksprache mit dem behandelnden Arzt wird vor der Implantatbehandlung für Patienten mit solchen Medikamenten dringend empfohlen.

Das Einsetzen eines Implantats angrenzend an einen infizierten Zahn oder einen versagenden wurzelkanalbehandelten Zahn kann zum Versagen des Implantats führen. Übermäßige Beweglichkeit, Knochenschwund oder Infektion können darauf hinweisen, dass das Implantat versagt. Jedes Implantat, das anscheinend versagt, sollte so schnell wie möglich behandelt oder entfernt werden. Wenn eine Entfernung erforderlich ist, kürettieren Sie eventuelles Weichgewebe aus dem Implantatbett. Man kann entweder die Heilung der Stelle wie bei einer traumatischen Extraktion zulassen oder wie angegeben geführte geweberegenerative Eingriffe durchführen.

Aufgrund der Metallleitfähigkeit können Elektrochirurgie um die Implantate und intraorale Aufbaupräparationen ohne Spülung zu Gewebeschäden und Implantatversagen führen.

Die richtige Fallplanung ist entscheidend für den langfristigen Erfolg von Prothese und Implantat. Überlastung ist einer der Hauptgründe für das Versagen von Implantaten. Man sollte sicherstellen, dass die Implantatgröße und die Abutment-Winkelungen für die okklusale Belastung geeignet sind. Stark abgewinkelte Abutments (>25°) sollten vermieden werden und werden nicht empfohlen. Zur besseren Unterstützung kann eine Schienung von außeraxial belasteten Implantaten erforderlich sein.

**FRAKTUREN** Implantat- und Abutmentfrakturen können auftreten, wenn die aufgebrachten Lasten die normalen funktionalen Designtoleranzen der Implantatkomponenten überschreiten. Mögliche Überlastbedingungen können sich aus Fehlern bei der Anzahl, Länge und/oder Länge der Implantate ergeben. Fehllursachen können ferner sein: unangepasste Durchmesser, um eine Restauration angemessen zu stützen, übermäßige Kragarmlänge, unvollständige Abutmentsitzungen, Abutmentwinkel größer als 25 Grad, okklusale Störungen, die übermäßige Querkräfte verursachen, Patientenparafunktion (z.B. Bruxing-Clenching), unsachgemäße Gussverfahren, unzureichender Prothesensitz und körperliches Trauma.

**VERÄNDERUNGEN DER LEISTUNG** Es liegt in der Verantwortung des Arztes, den Patienten über alle geeigneten Kontraindikationen, Nebenwirkungen und Vorsichtsmaßnahmen aufzuklären, sowie über die Notwendigkeit, bei Veränderungen in der Leistung des Implantats einen ausgebildeten Zahnarzt aufzusuchen (z.B. Lockerung der Prothese, Infektion oder Exsudat um das Implantat, Schmerzen oder andere ungewöhnliche Symptome, die der Patient nicht erwartet hat).

**Multi-Unit** Die Wahl des richtigen Aufsatzes liegt in der Verantwortung des Zahnarztes oder Zahntechnikers. Sicherheit, Verantwortung und Garantie: Ritter-Geräte und -Komponenten werden in Übereinstimmung mit den europäischen und US-amerikanischen Normen für Medizinprodukte hergestellt.

**MRT-Sicherheit** Ritter Titanimplantate werden aus Titan (Ti) der Güteklasse 5 hergestellt. Diese Materialien gelten als paramagnetisch und wechselwirken daher nur schwach mit Magnetfeldern. Ritter-Sekundärteile wie Abutments, Kappen, Verschlusschrauben und Einheilkappen werden aus Titan 5, PEEK oder Keramik (ZrO2) hergestellt. Diese Materialien sind entweder paramagnetisch oder nichtmagnetisch. Aus der Literatur kann geschlossen werden, dass es unwahrscheinlich ist, dass die Komponenten des Ritter Dental Implantat Systems mit der Patientensicherheit interferieren. In wissenschaftlichen Artikeln wurde gezeigt, dass die

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

magnetische Verschiebung von Komponenten des Zahnimplantatsystems geringer ist als die durch die Schwerkraft auf das Medizinprodukt ausgeübte Kraft, und die HF-Erwärmung führt zu einem maximalen Temperaturanstieg unter die Hitzeschmerz-Schwelle von 8 °C - 10 °C, ohne Berücksichtigung des Kühleffekts des umgebenden Gewebes und des Blutflusses [1]. Bildartefakte sind jedoch zu erwarten und müssen bei der Bildanalyse berücksichtigt werden [2], [3] und [4].

Beachten Sie, dass diese Literatur nicht ausreicht, um bei den Komponenten des Ritter Dental Implant Systems eine Bewertung von MR Sicherheit zuzuweisen. Das Scannen eines Patienten, der mit diesen Komponenten ausgestattet ist, kann zu Verletzungen des Patienten führen.

Aufgrund der großen Vielfalt der auf dem Markt erhältlichen MRT-Scanner kann Ritter keine Vorhersagen bezüglich der Sicherheit machen oder zu dem Verhalten unserer Implantate und Komponenten in einem bestimmten MRT-System. Patienten sollten sich mit ihrem Arzt und Bildgebungstechniker vor der Durchführung eines MRT-Verfahrens beraten. Abnehmbare Restaurationen sollten vor dem Scannen herausgenommen werden, wie dies u.a. auch für Uhren, Schmuck usw. üblich ist. Schließlich kann Ritter keine Verantwortung für die Zusammensetzung und das Verhalten von Produkten Dritter übernehmen (einschließlich der Krone, Brücke, Steg, Prothese usw.), die nicht von Ritter vertrieben werden und Materialien enthalten könnten, die nicht kompatibel mit MRT-Bildgebung sind.

Während des Transports dürfen keine Schäden an der Verpackung entstehen. Produkte haben kein Verfallsdatum. Das Produkt wird in einer NON STERILE Verpackung verkauft, es wird empfohlen, den Sterilisationsprozess der Metallteile unter Einhaltung der üblichen medizinischen Verfahren (Dampfautoklav-Sterilisation) fortzusetzen. Alle Kunststoffteile (nicht gießbar) sollten durch Kaltsterilisation mit Benzalcynhydrinlösung sterilisiert werden. Technischer Support: Unsere technischen Mitarbeiter stehen Ihnen bei allen Fragen rund um den Einsatz der Ritter-Komponenten zur Verfügung. Weitere Informationen zur Verwendung des Ball Attachment & Clicq Overdenture SYSTEM finden Sie in unserem Hauptkatalog und unseren technischen Merkblättern.

#### Referenzen

[1] Sherin Jose Chockattu, Deepak Byathnal Suryakant, Sophia Thakur, "Unwanted effects due to interactions between dental materials and magnetic resonance imaging: a review of the literature", Restorative Dentistry & Endodontics 2018 Nov;43(4): e39

[2] Margit-Ann Geibel, Benjamin Gelißen, Anna-Katinka Bracher, Volker Rasche „ Artifact Properties of Dental Ceramic and Titanium Implants in MRI", Georg Thieme Verlag KG, Published online:2018-11-12.

[3] Chalakupzhiyl Abraham Mathew, Sudhakara Maller, Maheshwaran, "Interactions between magnetic resonance imaging and dental material" Journal of Pharmacy & BioAllied Science.2013 Jun;5(Suppl1): S113-S116

[4] Ralf Smeets, Maximilian Schöllchen, Tobias Gauer, Ghazal Arabi, Alexandre T. Assaf, Carsten Rendenbach, Benedicta Beck-Broichstter, Jan Semmusch, Jan Sedlacik, Mx Heiland, Jens Fiehler, Susanne Siemonsen "Artifacts in multimodal imaging of titanium, zirconium and binary titanium-zirconium alloy dental implants: an in vitro study" Dentomaxillofacial Radiology (2017) 46,20160267

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

### 4.3 Produktbeschreibung

#### 4.3.1 Produktbeschreibung und Materialien/Substanzen, die mit dem Patienten in Kontakt kommen

Das Implantatsystem stellt den Ersatz verlorengegangener anatomischer Strukturen durch künstliche Wurzeln und künstliche Zahnstümpfe dar. Die Implantate und Implantataufbauten (Abutments) bilden die Grundlage für die Verankerung von prothetischen Versorgungen, als herausnehmbarer oder festsitzenden Zahnersatz z.B. Kronen/ Brücken oder Prothesen.

Das **Implantat-System** beinhaltet 10 unterschiedliche Implantate:

QSI Ri-Quadro Spiral Implantate

NL-QSI Ri-Quadro Spiral Implantate – Narrow Line

SB/ LA Spiral Implantate

SB/ LA Spiral Implantate – Narrow Line

iOs3/ Ora3/ iT3 (Edison Medical - EM)

Lamina/ iH/ QSI (EM)

Augmentat iH/ Lamina Augmentat/ Augmentat iV/ Artic Augmentat/ Augmentat iT/ Fortis Augmentat (EM)

Artic/ Ora/ iV (EM)

Fortis/ iT (EM)

i8 (EM)

Alle **Implantate** bestehen aus einer Titanlegierung TiAl6V4 ELI, mit einer gestrahlten und säuregeätzten Makrooberfläche von 20-40 Mikrometern und einer Mikrooberfläche von etwa 2 Mikrometern.

Das **Implantataufbau-System** umfasst 6 unterschiedliche Systeme:

Anatomische Systeme – Temporäre-/ Titan-/ Zirkonabutments

Transfer Abutment Systeme – Pop Up Transfer Abutment/ Clicq-Pression Transfer Abutment

Ball Systeme – Ball Attachments & Caps

Clicq Systeme – Clicq Overdenture/ Pop-Clicq Overdenture/ Q-Clicq Attachment

Multi-Unit Systeme – One Piece Multi-Unit Abutments/ Multi-Clicq On-X/ 2-Piece Multi-Unit Abutments/

Clicq-Base

Labor Systeme – Analogs for Implants/ Lab Analogs/ Abutments for Casting/ Castable & UCLA

Abutments/ Milling Blanks/ Pre-Milled Blank/ Ti-Base CAD&CAM/ Uni-Base One

Piece Ti-Base/ Scan Bodies/ Scan Abutments

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

Die **Implantataufbauten** bestehen aus den Werkstoffen Titan, Kobalt-Chrom, PEEK, Zirkon und POM. Die Unterschiede zwischen den Systemen bestehen in der zahntechnischen Handhabung und der anatomischen Situation des Patienten.

#### 4.3.2 Informationen zu medizinischen Substanzen im Medizinprodukt, falls zutreffend

Nicht anwendbar, da die Produkte (dentale Implantate und dentale Implantataufbauten) keine medizinischen Substanzen enthalten.

#### 4.3.3 Beschreibung, wie das Medizinprodukt seine beabsichtigte Wirkung erzielt

Nicht anwendbar, da die Produkte (dentale Implantate und dentale Implantataufbauten) keine medizinischen Substanzen enthalten.

#### 4.3.4 Beschreibung von Zubehör, falls zutreffend

Nicht anwendbar, da für die Handhabung der Produkte (dentale Implantate und dentale Implantataufbauten) durch den Patienten kein notwendiges Zubehör erforderlich ist.

### 4.4 Risiken und Warnhinweise

Kontaktieren Sie Ihr Gesundheitspersonal, falls Sie glauben, dass Sie aufgrund des Medizinprodukts oder der Anwendung des Produkts Nebenwirkungen haben oder falls Sie von Risiken betroffen sind. Dieses Dokument ersetzt nicht die Beratung mit Ihrem Angehörigen eines Gesundheitsberufs, falls notwendig.

#### 4.4.1 Wie werden potenzielle Risiken kontrolliert und verarbeitet

Potenzielle Risiken im Zusammenhang mit den Produkten werden im jährlichen Risikomanagement, im Bericht zur Überwachung nach dem Inverkehrbringen und in dem Klinischen Bewertungsbericht analysiert, bewertet und dokumentiert. Daraus können Maßnahmen entstehen, die zu Produktänderungen und zu Änderungen in Produktbegleitdokumenten z.B. Gebrauchsanweisung/ Implantatpass führen.

#### 4.4.2 Verbleibende Risiken und unerwünschte Nebenwirkungen

**Zu den Risiken gehören:** unmittelbare anästhetische und chirurgische Risiken psychiatrische Risiken, medizinische Bedrohungen der Langzeitretention, langfristige Auswirkungen auf die Gesundheit und Komplikationen, das Folgendes umfassen können: verzögerte Heilung, Ödeme, Blutungen, Dehiszenz, Klammern, Hämatome, allergische Reaktionen, Entzündungen der Nasennebenhöhlen, Nervenschäden, Sprachprobleme und Gingivitis langfristige Probleme können Folgendes beinhalten: Nervenschäden, Knochenverlust, Hyperplasie, lokale oder systemische bakterielle Infektion, endokarditische, langfristige Schmerzen und Knochenbrüche.

**Folgende Organsysteme können betroffen** sein: Herz-Kreislauf-Koronarerkrankungen; Atemwegserkrankungen - chronische Lungenerkrankungen; Nieren - chronisches Nierenversagen; Endokrin - Diabetes, Schilddrüsenerkrankungen, Hypophysen- und Nebennierenerkrankungen; Hämatologische - Anämie, Leukämie, Blutgerinnungsstörungen; Osteoporose der Musculoskelett-Arthritis; Neurologischer Schlaganfall, Lähmung, mentale Retardierung.

Mögliche **unerwünschte Ereignisse im Zusammenhang mit der Verwendung** von Zahnimplantaten können sein: Nicht integrierbar; Integrationsverlust; Dehiszenz, die eine Knochentransplantation erfordert; Perforation der Kieferhöhle, unterer Rand, Lingualplatte, Labialplatte, unterer Alveolarkanal, Gingiva; Infektion wie

Ritter Implants GmbH & Co. KG	SSCP Part B	I55-0526-2301
		Datum: 2023-02-28

berichtet durch: Abszess, Fistel, Eiterung, Entzündung, Röntgendurchblutung; anhaltende Schmerzen, Taubheitsgefühl, Parästhesie; Hyperplasie, Übermäßiger Knochenschwund, der ein Eingreifen erfordert, Implantatbruch oder -fraktur, Systemische Infektion und Nervenverletzung

**Allgemeine Krankheiten und Medikamente:** Herz-Kreislauf-Erkrankungen im Zusammenhang mit hohem endokarditischem Risiko (SBE); Koronarinsuffizienz; Blutdyskrasien; Immundefizienz, AIDS; Krebs und Bestrahlung der Gesichtsregion in den letzten fünf Jahren; Atemwegserkrankungen; Schilddrüsen- oder Nebenschilddrüsenenerkrankungen: Patienten mit knötchenförmigen Vergrößerungen oder unerklärlichen Knoten im Kopf- oder Halsbereich; Knochenstoffwechselstörungen; Diabetes; Bluthochdruck über 170/110 mmHg; Drogenmissbrauch, Alkoholismus; Titanüberempfindlichkeit: Patienten mit Kortikosteroiden, Antikoagulantien, Antikonvulsiva, einer Immunsuppressionstherapie; Patienten mit abnormalen Werten für Kreatin, BUN oder Serumkalzium; Hämophilie; Granulozytopenie; Steroidgebrauch; Prophylaktische Antibiotika; Ehler-Danlos-Syndrom; Nierenversagen; Organtransplantation; Fasrige Dysplasie.

Im Falle von unerwarteten Schmerzen muss der Chirurg sofort kontaktiert werden, da eine körperliche Anstrengung nach der Operation vermieden werden sollte.

Die gewonnenen klinischen Daten stammen aus unterschiedlichen Quellen: Recherchen auf wissenschaftlichen und medizinischen Datenbanken, aus dem Berichtswesen der Überwachung nach dem Inverkehrbringen, aus Trendreporting und Kundenfeedback, aus dem Risikomanagement und dem klinischen Bewertungsbericht.

#### 4.4.3 Warnhinweise und Vorbeugungsmaßnahmen

##### **Implantate**

Bei kurzen Implantaten sollten Ärzte die Patienten auf eine der folgenden Bedingungen hin genau überwachen: periimplantärer Knochenverlust, Veränderungen der Reaktion des Implantats auf Schläge oder röntgenologische Veränderungen des Knochens auf den Implantatkontakt entlang der Implantatlänge. Wenn das Implantat Mobilität oder mehr als 50% Knochenschwund aufweist, sollte das Implantat auf eine mögliche Entfernung hin untersucht werden. Wenn die Ärzte ein kurzes Implantat wählen, dann sollten sie einen zweistufigen chirurgischen Ansatz in Betracht ziehen, indem sie ein kurzes Implantat mit einem zusätzlichen Implantat verbinden und eine möglichst breite Befestigung platzieren. Längere Zeiträume für die Osseointegration sind einzuplanen und Sofortbelastung ist zu vermeiden

##### **OP-Protokoll – Obligatorische Erstuntersuchungen**

Patientenuntersuchung; Anamnese des Patienten; Klinische Untersuchung der Patientenhygiene, der Zähne, der Okklusion, des Parodontiums; Biologische Beobachtungen; Röntgenuntersuchung: CT-Scan, Intraoral, Röntgen, pan-oral. etc. Der Mangel an adäquater Praxistraining ist einer der Hauptfaktoren, die den Erfolg der Implantatchirurgie und damit die langfristige Gesundheit der Patienten beeinflussen.

##### **Chirurgische und Restauratorische Verfahren**

Das Hart- und Weichgewebe muss sorgfältig behandelt werden, um die Osseointegration zu gewährleisten. Die OP-Eingriffsstelle muss mit äußerster Präzision vorbereitet werden. Die verwendeten Hilfsinstrumente müssen ordnungsgemäß sterilisiert werden. Das chirurgische Verfahren erfordert Bohrgeschwindigkeiten von 1000 U/min für den ersten Bohrer bis 500 U/min oder den letzten. Physiologische Kochsalzlösung muss das Gebiet bewässern, während die Keulungssequenz strikt eingehalten werden muss. Die Implantatgröße (Höhe und Breite) wird nach vorheriger Röntgenaufnahme gewählt. Es muss ein Abstand von 2 mm zu anatomischen Hindernissen und eine maximale Knochenhöhe vorhanden sein. – Die Implantate werden in sterilem Zustand geliefert.

- Implantate dürfen nicht wieder sterilisiert werden.
- Implantate sind nur für den einmaligen Gebrauch bestimmt.
- Alle Produkte sollten während der Operation in einem sterilen Operationsfeld platziert werden.

1. Dieses System darf nur von geschultem Fachpersonal bedient werden. Die chirurgischen und restaurativen Techniken, die erforderlich sind, um das System richtig einzusetzen, sind hochspezialisierte und komplexe

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

Verfahren. Unsachgemäße Technik kann zu Implantatversagen, Verlust von Stützknochen, Restauraionsfrakturen, Schraubenlockerungen und Aspiration führen. Daher sollte keine Implantation ohne vorherige angemessene Schulung durch einen zertifizierten Zahnarzt durchgeführt werden.

2. Bei pädiatrischen Patienten wird eine routinemäßige Behandlung erst empfohlen, wenn das Ende des Kieferknochenwachstums ordnungsgemäß dokumentiert ist.

3. Dieses Produkt ist ein Einwegprodukt. Jeder Versuch, das Produkt wiederzuverwenden, kann ein schwerwiegendes Gesundheitsrisiko darstellen, wie z.B. Produktkontamination, akute Infektion, Verschlechterung der mechanischen Eigenschaften, was zu einer schmerzhaften Revisionsoperation zum Austausch des Implantats führt.

4. Verwenden Sie keine Implantate, wenn die Verpackung geöffnet oder beschädigt wird, da dies zu einer Beeinträchtigung der Sterilität oder einer Fehlfunktion des Produkts führen kann.

5. **BREAKAGE** Implantat- und Abutmentfrakturen können auftreten, wenn die aufgetragenen Lasten die normalen funktionalen Designtoleranzen der Implantatkomponenten überschreiten. Mögliche Überlastbedingungen können sich aus Mängeln bei der Anzahl, Länge und/oder dem Durchmesser der Implantate ergeben, um eine Restauration angemessen zu stützen, übermäßiger Kragarmlänge, unvollständiger Abutmentsitz, Abutmentwinkel größer als 30 Grad, okklusalen Interferenzen, die übermäßige Querkräfte verursachen, Patientenparafunktion (z.B. Zusammenpressen), unsachgemäßen Gussverfahren, unzureichendem Prothesensitz und physischem Trauma.

6. Verwenden Sie keine übermäßige Kraft, die das Einsteckmoment von 60Ncm übersteigt: Ein zu starkes Anziehen eines Implantats kann zu einer Beschädigung des Implantats und/oder der inneren Verbindung zu Frakturen oder Nekrosen des Knochenareals führen und die Osseointegration beeinträchtigen.

7. Ausgewogene Okklusion.

8. Richtige Zementierung.

9. Nachsorge der Patienten (Recall)

Die Implantatchirurgie ist ein sehr komplexes Verfahren, und es wird empfohlen, die notwendigen Schulungen zu absolvieren, die die Implantatchirurgie vermitteln. Unsachgemäße Implantattechniken können zu Implantatversagen und Knochenverlust führen. Ritter-Implantate sind nur für die Verwendung gemäß dem oben beschriebenen Protokoll mit Ritter-Implantatbohrern vorgesehen. Implantate, die in scharfen Winkeln gesetzt werden, können zu einem Implantatversagen führen. Knochenschwund, Infektion und Bewegung des Implantats können darauf hinweisen, dass das Implantat versagt. Wird eines davon beobachtet, sollte das Problem so schnell wie möglich behandelt oder das Implantat entfernt werden. Es muss eine adäquate Palpation und visuelle Inspektion des zukünftigen Implantatlagers durchgeführt werden, um festzustellen, ob für ein Implantat genügend Knochenqualität und -volumen vorhanden ist. Nach dem Implantatversagen müssen die Qualität und das Volumen des Restknochens bewertet werden.

Das Implantat wird in einer sterilen Verpackung geliefert. Nicht wieder sterilisieren. Eine geöffnete, beschädigte oder defekte Verpackung ist an den Lieferanten zum kostenlosen Austausch zurückzugeben. Die Verwendung eines Implantats erfordert keine ungewöhnliche präoperative Antibiotikaphylaxe. Implantate sollten nicht gereinigt und unter keinen Umständen wieder sterilisiert werden. Autoklavieren Sie keine harten Kunststoffteile, die bei ca. 170°C (338°F) schmelzen können. Abutments werden unsteril bereitgestellt und sollten vor dem Einsetzen in die Mundhöhle sterilisiert werden.

#### **Implantataufbauten (Abutments)**

Chirurgische und restaurative Techniken, die zur Platzierung von Zahnimplantaten erforderlich sind, sind hochspezialisierte und komplexe Verfahren. Chirurgen und alle Ärzte sollten in solchen Verfahren umfassend geschult sein und in solchen Implantatpraxen kompetent sein. Alle Praktiker sollten Studiengänge besuchen, um sich mit implantologischen Techniken vertraut zu machen. Unsachgemäße Technik kann zu Knochenschwund und Implantatversagen führen. Ritter Dental Implantatsysteme sind nur für den Einsatz mit

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

Ritter Dental speziell entwickelten Knochenbohrern vorgesehen. Implantate, die in einem starken Winkel zum bestehenden Gebiss gesetzt werden, oder mehrere Implantate, die konvergent oder divergent gesetzt werden, können zu komplexen Restaurationen führen, die Implantate überlasten können.

Eine gründliche diagnostische Abklärung und die Verwendung einer chirurgischen Schablone wird empfohlen, um die korrekte Positionierung des oder der Implantate zu gewährleisten. Die richtige Fallplanung ist entscheidend für den langfristigen Erfolg von Prothese und Implantat. Überlastung ist einer der Hauptgründe für das Versagen von Implantaten. Man sollte sicherstellen, dass die Implantatgröße und die Abutment-Winkelungen für die okklusale Belastung geeignet sind. Stark abgewinkelte Abutments (>25°) sollten vermieden werden und werden nicht empfohlen. Zur besseren Unterstützung kann eine Schienung von außeral axial belasteten Implantaten erforderlich sein.

Relative Kontraindikationen sind die Verwendung von Steroiden, Chemotherapeutika, Bisphosphonaten und Antikoagulantien. Diese und andere Medikamente, die den Operationsbereich, das umgebende Gewebe oder die Heilungsfunktion des Patienten beeinflussen können, können den Erfolg des Implantats beeinflussen. Eine sorgfältige Patientenauswahl einschließlich der Rücksprache mit dem behandelnden Arzt wird vor der Implantatbehandlung für Patienten mit solchen Medikamenten dringend empfohlen.

Das Einsetzen eines Implantats angrenzend an einen infizierten Zahn oder einen versagenden wurzelkanalbehandelten Zahn kann zum Versagen des Implantats führen. Übermäßige Beweglichkeit, Knochenschwund oder Infektion können darauf hinweisen, dass das Implantat versagt. Jedes Implantat, das anscheinend versagt, sollte so schnell wie möglich behandelt oder entfernt werden. Wenn eine Entfernung erforderlich ist, kürettieren Sie eventuelles Weichgewebe aus dem Implantatbett. Man kann entweder die Heilung der Stelle wie bei einer traumatischen Extraktion zulassen oder wie angegeben geführte gewebe-regenerative Eingriffe durchführen. Aufgrund der Metalleitfähigkeit können Elektrochirurgie um die Implantate und intraorale Aufbaupräparationen ohne Spülung zu Gewebeschäden und Implantatversagen führen.

Multi-Unit Die Wahl des richtigen Aufsatzes liegt in der Verantwortung des Zahnarztes oder Zahntechnikers. Sicherheit, Verantwortung und Garantie: Ritter-Geräte und -Komponenten werden in Übereinstimmung mit den europäischen und US-amerikanischen Normen für Medizinprodukte hergestellt.

Während des Transports dürfen keine Schäden an der Verpackung entstehen. Produkte haben kein Verfallsdatum. Das Produkt wird in einer NON STERILE Verpackung verkauft, es wird empfohlen, den Sterilisationsprozess der Metallteile unter Einhaltung der üblichen medizinischen Verfahren (Dampfautoklav-Sterilisation) fortzusetzen. Alle Kunststoffteile (nicht gießbar) sollten durch Kaltsterilisation mit Benzalcynhydrinlösung sterilisiert werden.

Alle aktuellen Gebrauchsanweisungen befinden sich unter den folgenden Link:  
<https://ritterimplants.de/ifu/>

#### 4.4.4 Zusammenfassung von Korrekturmaßnahmen im Feld (FSCA inklusive FSN), falls zutreffend

Nicht anwendbar, da für die Produkte (dentale Implantate und dentale Implantataufbauten) keine Korrekturmaßnahmen im Feld ergriffen werden mussten.

### 4.5 Zusammenfassung der klinischen Bewertung und des Berichts der klinischen Nachbeobachtung nach dem Inverkehrbringen

#### Angemessenheit des Nutzens – Risiko - Profil nach derzeitigem Kenntnisstand

Die gewonnenen Daten aus dem Risikomanagement, die ermittelten Daten aus der wissenschaftlichen Literatur und die Daten auf Basis der Grundlegenden Sicherheits- und Leistungsanforderungen belegen die Angemessenheit des Nutzens – Risiko – Profil der dentalen Implantate und Implantataufbauten nach derzeitigem Kenntnisstand.

Ritter Implants GmbH & Co. KG	SSCP Part B	I55-0526-2301
		Datum: 2023-02-28

Die Betrachtung der möglichen Restrisiken im Vergleich zu dem klinischen Nutzen der Patienten wie z.B. Wiedererlangen der ästhetischen und phonetischen Funktion, eine uneingeschränkte Kaufunktion durch vollständige Bezahnung, der Erhalt von Gewebestrukturen durch Vorbeugen einer möglichen Kieferatrophie, mehr Lebensqualität durch bedingt festsitzende Versorgungen, gute Reinigungsmöglichkeit der Restauration und eine optimale Mundhygiene sowie das Vorbeugen von Folgeerkrankungen durch das bedingte herausnehmen der Prothetik überwiegt der Nutzen des Patienten gegenüber der Restrisiken.

Das Nutzen – Risiko –Profil dentalen Implantate und Implantataufbauten belegt, dass bei bestimmungsgemäßer Anwendung der Produkte die angegebene Zweckbestimmung erfüllen und die positiven Auswirkungen der Produkte auf die Gesundheit der Patienten anhand patientenrelevanter messbarer klinischer Ergebnisse gegeben sind.

### **Angemessenheit der von Ritter Implants GmbH & Co.KG gelieferten Informationsmaterialien**

Der derzeitige Stand der Technik bezieht sich auf die medizinischen Felder: der Stomatologie, Kariologie, Parodontologie, Gnathologie, Implantologie und aus den folgenden Therapiefeldern; der systemischen, lokalen, mechanischen Behandlung, Füllungs-, Wurzel- und Schienentherapie sowie der Alternativen Behandlungsmethoden der Laser-, Icon® -, Physiotherapie und zahntechnischen Restaurationen. Die gewonnenen Daten aus dem Risikomanagement und der ermittelten Daten aus der wissenschaftlichen Literatur belegen den derzeitigen Stand der Technik.

Die von Ritter Implants GmbH & Co.KG gelieferten Informationsmaterialien bestehen aus Werbematerialien (z.B. Produktkataloge) und den Produktbegleitdokumenten (z.B. Gebrauchsanweisung, Implantationsausweis).

Die Werbematerialien stellen die bildliche und schriftliche Darstellung der wichtigsten Arbeitsschritte (Usability), die Produktidentifikation (Bezeichnung und REF), Produktdetails sowie Produkte zur gemeinsamen Anwendung dar. Die Gebrauchsanweisungen beschreiben die Zweckbestimmung, Patientenzielgruppe, die vorgesehenen Anwender, Indikationen, Kontraindikationen, Hinweise zur Sicherheit und Gefahrenvermeidung, Lagerung und Haltbarkeit, Handhabung sowie Technische Daten aller benannten Produkte.

Die Daten aus den Produktbegleitdokumenten und den Werbematerialien, sowie die regelmäßige Aktualisierung durch die Erkenntnisse aus der Überwachung nach dem Inverkehrbringen und der daraus folgenden Risikobetrachtung belegen die angemessenen Maßnahmen zur Risikominimierung der betreffenden Produkte.

### **Übereinstimmung zwischen den klinischen Daten**

Die Risikomanagementdaten belegen, dass die bereits identifizierten Risiken der dentalen Implantate und Implantataufbauten dem Stand der Technik entsprechen. Die klinischen Betrachtungen zu Nebenwirkungen befinden sich in der Gebrauchsanweisung.

Die ermittelten Daten aus der Literatur ergänzen das klinische Risikomanagement durch neu identifizierte Risiken. Die Betrachtung bezieht sich auf medizinische Felder, medizinische Krankheitsbilder, und Therapieverfahren. Die neu gewonnenen Erkenntnisse fließen in die Risikomanagement - Dokumente und in die Gebrauchsanweisungen.

Daraus resultiert die Übereinstimmung der klinischen Daten, der Stand der Technik mit den erforderlichen Dokumenten und Informationsmaterialien für die zu bewertenden Produkte (dentale Implantate und Implantataufbauten).

Die verbleibenden Restrisiko beruht nur auf menschlichem Versagen oder ist rein statistisch bedingt. Diese Gefahren können durch technisch vertretbare Maßnahmen nicht weiter reduziert werden.

Ritter Implants GmbH & Co. KG	SSCP Part B	I55-0526-2301
		Datum: 2023-02-28

Die Betrachtung der Usability Aspekte aus den Grundlegenden Sicherheits- und Leistungsanforderungen in Bezug auf die Sicherheit des Patienten sowie der vorgesehenen Anwendergruppen zeigt das die Produkte der dentalen Implantate und Implantataufbauten so ausgelegt und hergestellt sind, dass die Restrisiken so weit wie möglich minimiert sind und, dass die Produkte dentalen Implantate und Implantataufbauten den akzeptablen Nutzen im Vergleich, zu dem für den Patienten bei bestimmungsgemäßer Verwendung erzielten Leistungen der Produkte ermittelten Nutzen überwiegen.

Die Überwachung nach dem Inverkehrbringen beinhaltet Datenaufzeichnungen und Analysen zur Betrachtung der Restrisiken und unerwünschten Nebenwirkungsmeldungen durch das Feedback aus dem Markt. Es wurden neuen Meldungen auf den medizinischen Datenbanken zur Produktgruppe Abformkappen identifiziert. Die Erkenntnisse finden ihre Betrachtung im Riskmanagement und fließen als detaillierte Beschreibung in die Produktbegleitdokumente ein.

### **Klares Statement zu Übereinstimmung mit den Grundlegenden Anforderungen**

#### Sicherheitsanforderungen

Die Produkte der dentalen Implantate und Implantataufbauten erzielen ihre vorgesehene Leistung und werden so ausgelegt und hergestellt, dass sie sich unter normalen Verwendungsbedingungen für ihre Zweckbestimmung eignen. Sie sind sicher und wirksam und gefährden weder den klinischen Zustand und die Sicherheit der Patienten noch die Sicherheit und die Gesundheit der Anwender oder gegebenenfalls Dritter, wobei etwaige Restrisiken im Zusammenhang mit ihrer Anwendung gemessen am Nutzen für den Patienten vertretbar sind und mit einem hohen Maß an Gesundheitsschutz und Sicherheit vereinbar sind.

#### Nutzen – Risiko - Profil

Die gewonnenen Daten aus der Überwachung nach dem Inverkehrbringen, der Benefit des Patienten und die Bewertung der Produkte auf Basis der Grundlegenden Sicherheits- und Leistungsanforderungen belegen, dass die Produkte der dentalen Implantate und Implantataufbauten den akzeptablen Nutzen im Vergleich, zu dem für den Patienten bei bestimmungsgemäßer Anwendung erzielten Leistungen der Produkte ermittelten Nutzen überwiegen.

Die Merkmale und die Leistung der Produkte sind so ausgelegt, dass die Gesundheit oder die Sicherheit des Patienten oder Anwenders oder gegeben falls Dritter während der Lebensdauer der Produkte nicht gefährdet werden, wenn die Produkte Belastungen ausgesetzt werden, wie sie unter bestimmungsgemäßer Verwendungsbedingungen auftreten können, und die Produkte ordnungsgemäß entsprechend den Anweisungen der Produktbegleitdokumente instandgehalten werden.

#### Leistungspflicht

Die Produkte dentalen Implantate und Implantataufbauten werden so ausgelegt, hergestellt und verpackt, dass ihre Merkmale und Leistung während ihrer bestimmungsgemäßen Verwendung unter Berücksichtigung der Gebrauchsanweisung und der sonstigen Hinweise der Produktbegleitdokumente während des Transports und der Lagerung, nicht beeinträchtigt werden.

#### unerwünschte Nebenwirkungen

Alle bekannten und vorhersehbaren Risiken sowie unerwünschte Nebenwirkungen sind so weit wie möglich minimiert. Die Restrisiken sind im Vergleich zu dem für den Patienten und/oder Anwender bei normalen Verwendungsbedingungen aus erzielter Leistung der Produkte ermittelten Nutzen vertretbar.

**Aus den Aktivitäten der Überwachung nach dem Inverkehrbringen wurden neuen Meldungen auf den medizinischen Datenbanken zur Produktgruppe Abformkappen identifiziert. Die Erkenntnisse finden ihre Betrachtung im Riskmanagement und fließen als detaillierte Beschreibung in die Produktbegleitdokumente ein.**

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

**Es bestehen aktuell keine offenen Fragen bzw. Unsicherheiten im Bezug zur Zweckbestimmung und Maßnahmen zur Risikominimierung. Daher sind keine zusätzlichen PMS-Aktivitäten oder PMCF-Studien vorgesehen. Leistungsbewertungen der dental Implants und dental Implantats Accessories der Firma Ritter Implants GmbH & Co.KG befinden sich im Kapitel 5.3.3 klinische Daten aus Verifizierungs- und Validierungsdaten im klinischen Bewertungsbericht.**

#### 4.5.1 Klinischer Hintergrund des Produkts

Die Implantologie befasst sich mit dem Einbringen von Implantaten in den Mundraum des Patienten, um die fehlenden Zähne mit künstlichen Komponenten zu ersetzen. Im Laufe der Jahre hat sich die Implantation zur einer Routinebehandlung entwickelt, deren Ziel es ist, eine funktionsfähige und ästhetische Wiederherstellung der Kaufunktion zu erhalten.

Dentale Implantate bestehen aus einer Titanlegierung mit einer gestrahlten und geätzten Oberfläche. Die Implantate befinden sind in verschiedenen Längen und Durchmessern auf dem Markt.

Dentale Implantataufbauten bestehen aus metallischen und polymeren Werkstoffen. Die Implantataufbauten unterscheiden sich durch ihre Formgebung, Durchmesser, Länge und das entsprechende Material. Die Unterschiede zwischen den Implantataufbauten bestehen in der zahntechnischen Verarbeitung und der anatomischen Situation im Mund.

Die Herstellung der Produkte unterliegen strengen Qualitätsanforderungen und Qualitätskontrollen. Um die Sicherheit und Leistung (klinische Erfolg) der Produkte zugewährleisten werden unterschiedlichste Prüfverfahren durchgeführt.

#### 4.5.2 Der klinische Nachweis für die CE-Kennzeichnung

Der klinische Nachweis beruht auf Daten, die im Rahmen von Prüfungen am Produkt selbst erhoben wurden. Die Produkte der Ritter Implants GmbH & Co.KG haben Leistungstest zur biologischen Beurteilung von Medizinprodukten, Dynamischen Belastungsprüfungen, Prüfungen zur Sterilisation/ Desinfektion/ Reinigung und Prüfungen zur Lebensdauer, Transport und Lagerung durchlaufen.

#### 4.5.3 Sicherheit

##### **Nutzen-Risiko-Bewertung für jede angegebene Indikation**

Tabellarische Übersicht der Nutzen-Risiko-Bewertung für jede angegebene Indikation

*Tabelle 2 Nutzen-Risiko*

	<b>Produkt</b>	<b>Indikation</b>	<b>Nutzen-Risiko-Bewertung</b>
<b>1</b>	dentale Implantate	einfachen oder mehrfachen Ersatz von verlorenen Zähnen	<b>Nutzen:</b> ästhetischer und festsitzender Zahnersatz <b>Risiko:</b> Verlust des Implantates
<b>2</b>	dentale Implantate Ø 3.0 mm	Frontzähne Position 2er in beiden Kiefern	<b>Nutzen:</b> ästhetischer und festsitzender Zahnersatz bei wenig Knochenangebot bzw. Platzverhältnissen <b>Risiko:</b> Verlust des Implantates
<b>3</b>	dentale Implantate L 6 mm	- zusätzliches Implantat (Unterstützungsimplantat)	<b>Nutzen:</b> ästhetischer und festsitzender Zahnersatz bei geringen Knochenangebot

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

	Produkt	Indikation	Nutzen-Risiko-Bewertung
		- Hilfsimplantat für Stegkonstruktionen	<b>Risiko:</b> Verlust des Implantates
4	dentale Implantataufbauten gerade	alle Quadranten	<b>Nutzen:</b> ästhetischer und festsitzender Zahnersatz für Einzelzähne und/ oder implantatgetragenen Brücken <b>Risiko:</b> Verlust des Implantates
5	dentale Implantataufbauten 15°	vorderer Quadrant des OK	<b>Nutzen:</b> ästhetischer und festsitzender Zahnersatz (maximal mögliche Erweiterung der zuersetzenden Zahnreihen) <b>Risiko:</b> Verlust des Implantates
6	dentale Implantataufbauten 25°	Sektoren in denen vorhandene Defekte ein gerades Implantat unmöglich machen	<b>Nutzen:</b> ästhetischer und festsitzender Zahnersatz (maximal mögliche Erweiterung der zuersetzenden Zahnreihen) <b>Risiko:</b> Verlust des Implantates
7	Kugelköpfe/ Clicq Implantataufbauten	Frontzahnbereich jedoch in allen Bereichen des Mundes	<b>Nutzen:</b> Unterstützung der Haltefunktion bei wenig Knochensubstanz von zahnfleischgetragendem Zahnersatz <b>Risiko:</b> Verlust des Implantates

Die Nutzen-Risiko-Bewertung zeigt, dass eine prothetische Versorgung für jede angegebene Indikation eine festsitzende, ästhetische Lösung mit hoher Lebensqualität für den Patienten darstellt. Um möglichen Risiken, die durch unsachgemäße Anwendung hervorgerufen werden können vorzubeugen, sind in den Produktbegleitdokumenten entsprechende Maßnahmen definiert.

#### **Beschreibung der Art und Weise zur fortlaufenden Information über Sicherheit und Leistung**

Die Überwachung nach dem Inverkehrbringen (auf dem Markt erhältlich) dient der Sammlung fortlaufender Informationen über die Sicherheit und Leistung der auf dem Markt befindlichen Produkte. Die Erfassung der Daten erfolgt aus unterschiedlichen Quellen z.B. Kundenfeedbacks, wissenschaftlicher Fachliteratur und der Frühwarnsysteme aus medizinischen Datenbanken. Die erhobenen Daten werden analysiert, ausgewertet und entsprechende Maßnahmen durchgeführt. Der Zweck der fortlaufenden Informationssammlung ist es die kontinuierliche Bewertung der Sicherheit und Zuverlässigkeit des Medizinprodukts durchzuführen, sowie Bereiche und Möglichkeiten zur Verbesserung der Produktdesigns zu identifizieren.

In Übereinstimmung mit MEDDEV 2.12/2 ist eine PMCF-Studie nicht erforderlich, da die langfristige Sicherheit und die klinische Leistung bereits aus der früheren Verwendung der Produkte bekannt sind und die Überwachungsmaßnahmen nach dem Inverkehrbringen ausreichende Daten zur Behebung der Risiken liefern.

#### **4.6 Mögliche diagnostische oder therapeutische Alternativen**

Falls Sie alternative Behandlungen in Betracht ziehen, wird empfohlen, dass Sie Ihr Gesundheitspersonal kontaktieren, welches Ihre individuelle Situation berücksichtigen kann.

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

Die Tabelle stellt eine Übersicht der therapeutischen Alternativen zu implantatgetragem Zahnersatz dar.

*Tabelle 3 therapeutische Alternativen*

Nr.	Prothetiktypen	therapeutische Alternativen
1	Festsitzender Zahnersatz	Kronen, Brücken, Veneers, Onlays und Inlays
2	Herausnehmbarer Zahnersatz	Klammer- und Vollprothesen
3	Kombinierter Zahnersatz	Riegelprothesen, Geschiebeprothese, Teleskop- und Coverdenture-Prothesen

**Kronen und Brücken** ersetzen einen oder mehrere fehlende Zähne. Sie werden durch Zementieren oder Verkleben auf den beschliffenden Zahnstumpf/ Zahnstümpfen befestigt.

**Veneers** sind dünne Keramikschaalen, die Verfärbungen oder minimale Zahnfehlstellungen optisch korrigieren. Die Befestigung findet durch eine Verklebung auf dem natürlichen Zahn statt.

**Onlays** sind gegossenen Einlagefüllungen, die die gesamte Kaufläche und Zahnhöcker bedecken. Die Befestigung findet durch eine Verklebung auf dem präparierten Zahn statt.

**Inlays** sind gegossenen kleine Einlagefüllungen, die zum Einsatz kommen, wenn nur wenig Zahnhartsubstanz fehlt. Die Befestigung erfolgt durch eine Verklebung in dem präparierten Zahn.

**Klammerprothesen** bzw. Modellgussprothesen dienen den Ersatz von Einzelzähnen oder kleineren Zahnlücken. Die Versorgung wird mit Hilfe von Klammern an den Nachbarzähnen befestigt. Bei größeren Lücken im Oberkiefer wird zum besseren Halt der Gaumen als Prothesenbasis miteingeschlossen.

**Vollprothesen** ersetzen alle Zähne im Ober- und Unterkiefer. Eine konventionelle herausnehmbare Vollprothese findet ihren Halt mittels Unterdrucks auf dem Zahnfleisch des Kieferkammes.

**Riegelprothesen** dienen dem Ersatz von kleineren Zahnlücken mit natürlichen Restzahnbestand. Bei der Anwendung einer Riegelprothese werden an den Zähnen neben der Zahnücke Kronen mit Befestigungselementen (Schröderzapfen mit Aufnahmen für das Riegelblatt in geschlossenem Zustand) angebracht, an denen die Prothese beim Schließen des Riegels verankert wird.

**Geschiebeprothesen** bestehen aus einem festsitzenden und einem herausnehmbaren Prothesenanteil und dienen dem Ersatz von lückenhaften Ober-/ Unterkiefern. Das feinmechanische Geschiebe besteht aus zwei Teilen aus der Matrize und der Patrize. Die Restzähne an der die Verankerung stattfinden soll, werden beschliffen und mit einer Krone an der die Matrize (Teil 1 der Prothese) angebracht wird versorgt. Im herausnehmbarem Prothesenanteil befindet sich die Patrize, die das Gegenstück des Geschiebes darstellt. Über eine Gleitfunktion beim Einsetzen der Prothese wird die Verankerung des Zahnersatzes erreicht.

**Teleskopprothese** oder Doppelkronenprothese bestehen aus einen festsitzenden Aussenteleskopen und einem herausnehmbaren Prothesenanteil mit Innenteleskopen. Teleskopverankerte Prothesen finden ihre Anwendung bei lückenhaften Ober-/ Unterkiefern mit Restzahnbeständen. Dabei werden die natürlichen Zähne, die an die Lücken grenzen beschliffen und mit Aussenteleskopen (zylindrische Kappen) versorgt. Das Gegenstück sind die Innenteleskope die bebenfalls zylindrische Kappen sind und in dem herausnehmbaren Prothesenanteil zu finden sind. Über eine Gleitfunktion beim Einsetzen der Prothese wird die Verankerung des Zahnersatzes erreicht.

Die Ausdehnung und die Form einer **Coverdenture-Prothese** (Deckprothese) entspricht denen einer Totalprothese. Coverdentureverankerte Prothesen finden ihre Anwendung bei Ober- und Unterkiefern mit

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

geringen Restzähnen. Die letzten natürlichen Zähne werden mit Teleskopkronen (Doppelkronen) als Anker für den Prothesenkörper versehen. Die Coverdenture-Prothese liegt auf der Mundschleimhaut auf und wird hauptsächlich vom Kieferkamm getragen.

#### 4.7 Empfohlene Anwender-Schulungen

Die Handhabung der prothetischen Versorgung (Prothese) während des Einsetzens und herausnehmen der Prothese durch den Patienten sowie die Reinigungsvarianten bei herausnehmbaren oder festsitzenden prothetischen Versorgungen (verschraubter Zahnersatz auf Implantaten) sind dem Patienten vor der ersten Anwendung der Versorgung zu erläutern.

Bei herausnehmbaren prothetischen Versorgungen wird die Prothese von beiden Seiten mit einem Finger durch nach oben (zum Gaumen) drücken der Prothesenränder vorsichtig vom Implantataufbau gelöst und aus dem Mund entnommen werden. Die Prothese wird mit einer Zahnbürste oder einer Prothesenbürste mit weichen Borsten gründlich unter fließendem Wasser gereinigt.

Bei festsitzenden prothetischen Versorgungen (verschraubter Zahnersatz auf Implantaten) wird die Verwendung von Zahnseide, Interdentälbürstchen oder Munddusche zum Reinigen der Implantatregionen empfohlen. Der jährliche Zahnarztbesuch (Recall) dient der Reinigung der Implantate (durch Lösen einer Schraube am Zahnersatz und anschließenden abnehmen der Versorgung) und dient der Kontrolle des korrekten Sitzes des Zahnersatzes.


[DE Ritter Implantatausweis DE 21.00.08.pdf \(ritterimplants.de\)](#)

[Ritter ImplantCard EN 21.00.00.pdf \(shopify.com\)](#)


## 5 Anhang

### 5.1 Implantatsysteme


*Tabelle 4 QSI Ri-Quadro Spiral Implantate*

 QSI Ri-Quadro Spiral Implants		
No	REF/ Productname	Product Description
1	QSI-3.75-8	Ri-Quadro Spiral Implant D-3.75 mm / L-8 mm
2	QSI-3.75-10	Ri-Quadro Spiral Implant D-3.75 mm / L-10 mm
3	QSI-3.75-11.5	Ri-Quadro Spiral Implant D-3.75 mm / L-11.5 mm
4	QSI-3.75-13	Ri-Quadro Spiral Implant D-3.75 mm / L13 mm
5	QSI-3.75-16	Ri-Quadro Spiral Implant D-3.75 mm / L-16 mm
6	QSI-4.2-8	Ri-Quadro Spiral Implant D-4.2 mm / L-8 mm
7	QSI-4.2-10	Ri-Quadro Spiral Implant D-4.2 mm / L-10 mm
8	QSI-4.2-11.5	Ri-Quadro Spiral Implant D-4.2 mm / L-11.5 mm

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

 <p>QSI Ri-Quadro Spiral Implants</p>		
No	REF/ Productname	Product Description
9	QSI-4.2-13	Ri-Quadro Spiral Implant D-4.2 mm / L-13 mm
10	QSI-4.2-16	Ri-Quadro Spiral Implant D-4.2 mm / L-16 mm
11	QSI-5-6	Ri-Quadro Spiral Implant D-5.0 mm / L-6 mm
12	QSI-5-8	Ri-Quadro Spiral Implant D-5.0 mm / L-8 mm
13	QSI-5-10	Ri-Quadro Spiral Implant D-5.0 mm / L-10 mm
14	QSI-5-11.5	Ri-Quadro Spiral Implant D-5.0 mm / L-11.5 mm
15	QSI-5-13	Ri-Quadro Spiral Implant D-5.0 mm / L-13 mm
16	QSI-5-16	Ri-Quadro Spiral Implant D-5.0 mm / L-16 mm
17	QSI-6-6	Ri-Quadro Spiral Implant D-6.0 mm / L-6 mm
18	QSI-6-8	Ri-Quadro Spiral Implant D-6.0 mm / L-8 mm
19	QSI-6-10	Ri-Quadro Spiral Implant D-6.0 mm / L-10 mm
20	QSI-611.5	Ri-Quadro Spiral Implant D-6.0 mm / L-11.5 mm
21	QSI-6-13	Ri-Quadro Spiral Implant D-6.0 mm / L-13 mm


*Tabelle 5 NL-QSI Ri-Quadro Spiral Implantate*

 <p>NL-QSI Ri-Quadro Spiral Implant – Narrow Line</p>		
No	REF/ Productname	Product Description
1	NL-QSI-3.0-10	Ri-Quadro Spiral Implant D-3.0 mm/ L-10 mm
2	NL-QSI-3.0-11.5	Ri-Quadro Spiral Implant D-3.0 mm/ L-11.5 mm
3	NL-QSI-3.0-13	Ri-Quadro Spiral Implant D-3.0 mm/ L-13 mm
4	NL-QSI-3.0-16	Ri-Quadro Spiral Implant D-3.0 mm/ L-16 mm
5	NL-QSI-3.3-10	Ri-Quadro Spiral Implant D-3.3 mm/ L-10 mm
6	NL-QSI-3.3-11.5	Ri-Quadro Spiral Implant D-3.3 mm/ L-11.5 mm


Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

<b>7</b>	NL-QSI-3.3-13	Ri-Quadro Spiral Implant D-3.3 mm/ L-13 mm
<b>8</b>	NL-QSI-3.3-16	Ri-Quadro Spiral Implant D-3.3 mm/ L-16 mm


*Tabelle 6 SB/LA Spiral Implantate*

 SB/ LA Spiral Implants		
No	REF/ Productname	Product Description
1	SNAP-3.75-8	SB/LA Spiral Implant D-3.75 mm/ L-8 mm
2	SNAP-3.75-10	SB/LA Spiral Implant D-3.75 mm/ L-10 mm
3	SNAP-3.75-11.5	SB/LA Spiral Implant D-3.75 mm/ L-11.5 mm
4	SNAP-3.75-13	SB/LA Spiral Implant D-3.75 mm/ L-13 mm
5	SNAP-3.75-16	SB/LA Spiral Implant D-3.75 mm/ L-16 mm
6	SNAP-4.2-8	SB/LA Spiral Implant D-4.2 mm/ L-8 mm
7	SNAP-4.2-10	SB/LA Spiral Implant D-4.2 mm/ L-10 mm
8	SNAP-4.2-11.5	SB/LA Spiral Implant D-4.2 mm/ L-11.5 mm
9	SNAP-4.2-13	SB/LA Spiral Implant D-4.2 mm/ L-13 mm
10	SNAP-4.2-16	SB/LA Spiral Implant D-4.2 mm/ L-16 mm
11	SNAP-5-6	SB/LA Spiral Implant D-5.0 mm/ L-8 mm
12	SNAP-5-8	SB/LA Spiral Implant D-5.0 mm/ L-8 mm
13	SNAP-5-10	SB/LA Spiral Implant D-5.0 mm/ L-10 mm
14	SNAP-5-11.5	SB/LA Spiral Implant D-5.0 mm/ L-11.5 mm
15	SNAP-5-13	SB/LA Spiral Implant D-5.0 mm/ L-13 mm
16	SNAP-5-16	SB/LA Spiral Implant D-5.0 mm/ L-16 mm
17	SNAP-6-6	SB/LA Spiral Implant D-6.0 mm/ L-6 mm
18	SNAP-6-8	SB/LA Spiral Implant D-6.0 mm/ L-8 mm
19	SNAP-6-10	SB/LA Spiral Implant D-6.0 mm/ L-10 mm
20	SNAP-6-11.5	SB/LA Spiral Implant D-6.0 mm/ L-11.5 mm


Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

 SB/ LA Spiral Implants		
No	REF/ Productname	Product Description
21	SNAP-6-13	SB/LA Spiral Implant D-6.0 mm/ L-13 mm

*Tabelle 7 SB/LA Spiral Implantate (NL)*

 SB/ LA Spiral Implants – Narrow Line		
No	REF/ Productname	Product Description
1	NL-SNAP-3-10	SB/ LA Spiral Implant D-3.0 mm/ L-10 mm (Narrow Line)
2	NL-SNAP-3-11.5	SB/ LA Spiral Implant D-3.0 mm/ L-11.5 mm (Narrow Line)
3	NL-SNAP-3-13	SB/ LA Spiral Implant D-3.0 mm/ L-13 mm (Narrow Line)
4	NL-SNAP-3-16	SB/ LA Spiral Implant D-3.0 mm/ L-16 mm (Narrow Line)
5	NL-SNAP-3.3-10	SB/ LA Spiral Implant D-3.3 mm/ L-10 mm (Narrow Line)
6	NL-SNAP-3.3-11.5	SB/ LA Spiral Implant D-3.3 mm/ L-11.5 mm (Narrow Line)
7	NL-SNAP-3.3-13	SB/ LA Spiral Implant D-3.3 mm/ L-13 mm (Narrow Line)
8	NL-SNAP-3.3-16	SB/ LA Spiral Implant D-3.3 mm/ L-16 mm (Narrow Line)

*Tabelle 8 iOS3/ Ora3/ iT (EM)*


 iOS3/ Ora3/ iT3 (EM)		
No	REF/ Productname	Product Description
1	i3-OS-SP3010	Clicq™ iT3 or ios3 or Ora3 Dental Implant Ø3.1mm L 10mm
2	i3-OS-SP3011	Clicq™ iT3 or ios3 or Ora3 Dental Implant Ø3.1mm L 11mm
3	i3-OS-SP3013	Clicq™ iT3 or ios3 or Ora3 Dental Implant Ø3.1mm L 13mm
4	i3-OS-SP3015	Clicq™ iT3 or ios3 or Ora3 Dental Implant Ø3.1mm L 15mm

*Tabelle 9 Lamina/ iH (EM)*


Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

 Lamina/ iH (EM)		
No	REF/ Productname	Product Description
1	LM-SV-RP3310	Clicq™ iH or Lamina Dental Implant Ø3.3mm L 10mm
2	LM-SV-RP3311	Clicq™ iH or Lamina Dental Implant Ø3.3mm L 11mm
3	LM-SV-RP3313	Clicq™ iH or Lamina Dental Implant Ø3.3mm L 13mm
4	LM-SV-RP3315	Clicq™ iH or Lamina Dental Implant Ø3.3mm L 15mm
5	LM-SV-RP3508	Clicq™ iH or Lamina Dental Implant Ø3.5mm L 08mm
6	LM-SV-RP3510	Clicq™ iH or Lamina Dental Implant Ø3.5mm L 10mm
7	LM-SV-RP3511	Clicq™ iH or Lamina Dental Implant Ø3.5mm L 11mm
8	LM-SV-RP3513	Clicq™ iH or Lamina Dental Implant Ø3.5mm L 13mm
9	LM-SV-RP3515	Clicq™ iH or Lamina Dental Implant Ø3.5mm L 15mm
10	LM-SV-RP3808	Clicq™ iH or Lamina Dental Implant Ø3.8mm L 08mm
11	LM-SV-RP3810	Clicq™ iH or Lamina Dental Implant Ø3.8mm L 10mm
12	LM-SV-RP3811	Clicq™ iH or Lamina Dental Implant Ø3.8mm L 11mm
13	LM-SV-RP3813	Clicq™ iH or Lamina Dental Implant Ø3.8mm L 13mm
14	LM-SV-RP3815	Clicq™ iH or Lamina Dental Implant Ø3.8mm L 15mm
15	LM-SV-RP4206	Clicq™ iH or Lamina Dental Implant Ø4.2mm L 06mm
16	LM-SV-RP4208	Clicq™ iH or Lamina Dental Implant Ø4.2mm L 08mm
17	LM-SV-RP4210	Clicq™ iH or Lamina Dental Implant Ø4.2mm L 10mm
18	LM-SV-RP4211	Clicq™ iH or Lamina Dental Implant Ø4.2mm L 11mm
19	LM-SV-RP4213	Clicq™ iH or Lamina Dental Implant Ø4.2mm L 13mm
20	LM-SV-RP4215	Clicq™ iH or Lamina Dental Implant Ø4.2mm L 15mm
21	LM-SV-RP5006	Clicq™ iH or Lamina Dental Implant Ø5.0mm L 06mm
22	LM-SV-RP5008	Clicq™ iH or Lamina Dental Implant Ø5.0mm L 08mm
23	LM-SV-RP5010	Clicq™ iH or Lamina Dental Implant Ø5.0mm L 10mm
24	LM-SV-RP5011	Clicq™ iH or Lamina Dental Implant Ø5.0mm L 11mm
25	LM-SV-RP5013	Clicq™ iH or Lamina Dental Implant Ø5.0mm L 13mm


Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

 Lamina/ iH (EM)		
No	REF/ Productname	Product Description
26	LM-SV-RP5015	Clicq™ iH or Lamina Dental Implant Ø5.0mm L 15mm
27	LM-SV-RP6006	Clicq™ iH or Lamina Dental Implant Ø6.0mm L 06mm
28	LM-SV-RP6008	Clicq™ iH or Lamina Dental Implant Ø6.0mm L 08mm
29	LM-SV-RP6010	Clicq™ iH or Lamina Dental Implant Ø6.0mm L 10mm
30	LM-SV-RP6011	Clicq™ iH or Lamina Dental Implant Ø6.0mm L 11mm
31	LM-SV-RP6013	Clicq™ iH or Lamina Dental Implant Ø6.0mm L 13mm
32	LM-SV-RP6015	Clicq™ iH or Lamina Dental Implant Ø6.0mm L 15mm


*Tabelle 10 QSI (EM)*

 QSI (EM)		
No	REF/ Productname	Product Description
1	NL-QSI-3.0-15	NL-QSI Dental Implant Ø3.0mm L 15mm
2	NL-QSI-3.3-15	NL-QSI Dental Implant Ø3.3mm L 15mm
3	QSI-6-16	QSI Dental Implant Ø6.0mm L 16mm


*Tabelle 11 Augmentat iH/ Lamina Augmentat (EM)*

 Augmentat iH/ Lamina Augmentat (EM)		
No	REF/ Productname	Product Description
1	GM-SV-RP3508	Augmentat iH / Lamina Augmentat Dental Implant Ø3.5mm L 8mm
2	GM-SV-RP3510	Augmentat iH / Lamina Augmentat Dental Implant Ø3.5mm L 10mm
3	GM-SV-RP3511	Augmentat iH / Lamina Augmentat Dental Implant Ø3.5mm L 11mm
4	GM-SV-RP3513	Augmentat iH / Lamina Augmentat Dental Implant Ø3.5mm L 13mm
5	GM-SV-RP3515	Augmentat iH / Lamina Augmentat Dental Implant Ø3.5mm L 15mm


Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

 Augmentat iH/ Lamina Augmentat (EM)		
No	REF/ Productname	Product Description
6	GM-SV-RP4208	Augmentat iH / Lamina Augmentat Dental Implant Ø4.2mm L 8mm
7	GM-SV-RP4210	Augmentat iH / Lamina Augmentat Dental Implant Ø4.2mm L 10mm
8	GM-SV-RP4211	Augmentat iH / Lamina Augmentat Dental Implant Ø4.2mm L 11mm
9	GM-SV-RP4213	Augmentat iH / Lamina Augmentat Dental Implant Ø4.2mm L 13mm
10	GM-SV-RP4215	Augmentat iH / Lamina Augmentat Dental Implant Ø4.2mm L 15mm
11	GM-SV-RP5008	Augmentat iH / Lamina Augmentat Dental Implant Ø5.0mm L 08mm
12	GM-SV-RP5010	Augmentat iH / Lamina Augmentat Dental Implant Ø5.0mm L 10mm
13	GM-SV-RP5011	Augmentat iH / Lamina Augmentat Dental Implant Ø5.0mm L 11mm
14	GM-SV-RP5013	Augmentat iH / Lamina Augmentat Dental Implant Ø5.0mm L 13mm
15	GM-SV-RP5015	Augmentat iH / Lamina Augmentat Dental Implant Ø5.0mm L 15mm
16	GM-SV-RP6008	Augmentat iH / Lamina Augmentat Dental Implant Ø6.0mm L 8mm
17	GM-SV-RP6010	Augmentat iH / Lamina Augmentat Dental Implant Ø6.0mm L 10mm
18	GM-SV-RP6011	Augmentat iH / Lamina Augmentat Dental Implant Ø6.0mm L 11mm
19	GM-SV-RP6013	Augmentat iH / Lamina Augmentat Dental Implant Ø6.0mm L 13mm
20	GM-SV-RP6015	Augmentat iH / Lamina Augmentat Dental Implant Ø6.0mm L 15mm


*Tabelle 12 Artic/ Ora/ iV (EM)*

 Artic/ Ora/ iV (EM)		
No	REF/ Productname	Product Description
1	AR-NA-NP3508	Clicq™ iV or Artic or Ora Dental Implant Ø3.5mm L 08mm
2	AR-NA-NP3510	Clicq™ iV or Artic or Ora Dental Implant Ø3.5mm L 10mm
3	AR-NA-NP3511	Clicq™ iV or Artic or Ora Dental Implant Ø3.5mm L 11mm
4	AR-NA-NP3513	Clicq™ iV or Artic or Ora Dental Implant Ø3.5mm L 13mm
5	AR-NA-NP3515	Clicq™ iV or Artic or Ora Dental Implant Ø3.5mm L 15mm


Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

 Artic/ Ora/ iV (EM)		
No	REF/ Productname	Product Description
6	AR-NA-NP3808	Clicq™ iV or Artic or Ora Dental Implant Ø3.8mm L 08mm
7	AR-NA-NP3810	Clicq™ iV or Artic or Ora Dental Implant Ø3.8mm L 10mm
8	AR-NA-NP3811	Clicq™ iV or Artic or Ora Dental Implant Ø3.8mm L 11mm
9	AR-NA-NP3813	Clicq™ iV or Artic or Ora Dental Implant Ø3.8mm L 13mm
10	AR-NA-NP3815	Clicq™ iV or Artic or Ora Dental Implant Ø3.8mm L 15mm
11	AR-NA-RP4306	Clicq™ iV or Artic or Ora Dental Implant Ø4.2mm L 06mm
12	AR-NA-RP4308	Clicq™ iV or Artic or Ora Dental Implant Ø4.2mm L 08mm
13	AR-NA-RP4310	Clicq™ iV or Artic or Ora Dental Implant Ø4.2mm L 10mm
14	AR-NA-RP4311	Clicq™ iV or Artic or Ora Dental Implant Ø4.2mm L 11mm
15	AR-NA-RP4313	Clicq™ iV or Artic or Ora Dental Implant Ø4.2mm L 13mm
16	AR-NA-RP4315	Clicq™ iV or Artic or Ora Dental Implant Ø4.2mm L 15mm
17	AR-NA-RP5006	Clicq™ iV or Artic or Ora Dental Implant Ø5.0mm L 06mm
18	AR-NA-RP5008	Clicq™ iV or Artic or Ora Dental Implant Ø5.0mm L 08mm
19	AR-NA-RP5010	Clicq™ iV or Artic or Ora Dental Implant Ø5.0mm L 10mm
20	AR-NA-RP5011	Clicq™ iV or Artic or Ora Dental Implant Ø5.0mm L 11mm
21	AR-NA-RP5013	Clicq™ iV or Artic or Ora Dental Implant Ø5.0mm L 13mm
22	AR-NA-RP5015	Clicq™ iV or Artic or Ora Dental Implant Ø5.0mm L 15mm
23	AR-NA-RP5506	Clicq™ iV or Artic or Ora Dental Implant Ø5.5mm L 06mm
24	AR-NA-RP5508	Clicq™ iV or Artic or Ora Dental Implant Ø5.5mm L 08mm
25	AR-NA-RP5510	Clicq™ iV or Artic or Ora Dental Implant Ø5.5mm L 10mm
26	AR-NA-RP5511	Clicq™ iV or Artic or Ora Dental Implant Ø5.5mm L 11mm
27	AR-NA-RP5513	Clicq™ iV or Artic or Ora Dental Implant Ø5.5mm L 13mm
28	AR-NA-RP5515	Clicq™ iV or Artic or Ora Dental Implant Ø5.5mm L 15mm
29	AR-NA-RP6006	Clicq™ iV or Artic or Ora Dental Implant Ø6.0mm L 06mm
30	AR-NA-RP6008	Clicq™ iV or Artic or Ora Dental Implant Ø6.0mm L 08mm


Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

 Artic/ Ora/ iV (EM)		
No	REF/ Productname	Product Description
31	AR-NA-RP6010	Clicq™ iV or Artic or Ora Dental Implant Ø6.0mm L 10mm
32	AR-NA-RP6011	Clicq™ iV or Artic or Ora Dental Implant Ø6.0mm L 11mm
33	AR-NA-RP6013	Clicq™ iV or Artic or Ora Dental Implant Ø6.0mm L 13mm
34	AR-NA-RP6015	Clicq™ iV or Artic or Ora Dental Implant Ø6.0mm L 15mm


*Table 13 Augmentat iV/ Artic Augmentat (EM)*

 Augmentat iV/ Artic Augmentat (EM)		
No	REF/ Productname	Product Description
1	GM-NA-NP3508	Augmentat iV / Artic Augmentat Dental Implant Ø3.5mm L 8mm
2	GM-NA-NP3510	Augmentat iV / Artic Augmentat Dental Implant Ø3.5mm L 10mm
3	GM-NA-NP3511	Augmentat iV / Artic Augmentat Dental Implant Ø3.5mm L 11mm
4	GM-NA-NP3513	Augmentat iV / Artic Augmentat Dental Implant Ø3.5mm L 13mm
5	GM-NA-NP3515	Augmentat iV / Artic Augmentat Dental Implant Ø3.5mm L 15mm
6	GM-NA-RP4308	Augmentat iV / Artic Augmentat Dental Implant Ø4.2mm L 8mm
7	GM-NA-RP4310	Augmentat iV / Artic Augmentat Dental Implant Ø4.2mm L 10mm
8	GM-NA-RP4311	Augmentat iV / Artic Augmentat Dental Implant Ø4.2mm L 11mm
9	GM-NA-RP4313	Augmentat iV / Artic Augmentat Dental Implant Ø4.2mm L 13mm
10	GM-NA-RP4315	Augmentat iV / Artic Augmentat Dental Implant Ø4.2mm L 15mm
11	GM-NA-RP5008	Augmentat iV / Artic Augmentat Dental Implant Ø5.0mm L 8mm
12	GM-NA-RP5010	Augmentat iV / Artic Augmentat Dental Implant Ø5.0mm L 10mm
13	GM-NA-RP5011	Augmentat iV / Artic Augmentat Dental Implant Ø5.0mm L 11mm
14	GM-NA-RP5013	Augmentat iV / Artic Augmentat Dental Implant Ø5.0mm L 13mm
15	GM-NA-RP5015	Augmentat iV / Artic Augmentat Dental Implant Ø5.0mm L 15mm
16	GM-NA-RP6008	Augmentat iV / Artic Augmentat Dental Implant Ø6.0mm L 8mm


Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

 Augmentat iV/ Artic Augmentat (EM)		
No	REF/ Productname	Product Description
17	GM-NA-RP6010	Augmentat iV / Artic Augmentat Dental Implant Ø6.0mm L 10mm
18	GM-NA-RP6011	Augmentat iV / Artic Augmentat Dental Implant Ø6.0mm L 11mm
19	GM-NA-RP6013	Augmentat iV / Artic Augmentat Dental Implant Ø6.0mm L 13mm
20	GM-NA-RP6015	Augmentat iV / Artic Augmentat Dental Implant Ø6.0mm L 15mm


*Tabelle 14 Fortis/ iT (EM)*

 Fortis/ iT (EM)		
No	REF/ Productname	Product Description
1	FR-TS-RP3508	Clicq™ iT or Fortis Dental Implant Ø3.6mm L 08mm
2	FR-TS-RP3510	Clicq™ iT or Fortis Dental Implant Ø3.6mm L 10mm
3	FR-TS-RP3511	Clicq™ iT or Fortis Dental Implant Ø3.6mm L 11mm
4	FR-TS-RP3513	Clicq™ iT or Fortis Dental Implant Ø3.6mm L 13mm
5	FR-TS-RP3515	Clicq™ iT or Fortis Dental Implant Ø3.6mm L 15mm
6	FR-TS-RP3808	Clicq™ iT or Fortis Dental Implant Ø3.8mm L 08mm
7	FR-TS-RP3810	Clicq™ iT or Fortis Dental Implant Ø3.8mm L 10mm
8	FR-TS-RP3811	Clicq™ iT or Fortis Dental Implant Ø3.8mm L 11mm
9	FR-TS-RP3813	Clicq™ iT or Fortis Dental Implant Ø3.8mm L 13mm
10	FR-TS-RP3815	Clicq™ iT or Fortis Dental Implant Ø3.8mm L 15mm
11	FR-TS-RP4206	Clicq™ iT or Fortis Dental Implant Ø4.2mm L 06mm
12	FR-TS-RP4208	Clicq™ iT or Fortis Dental Implant Ø4.2mm L 08mm
13	FR-TS-RP4210	Clicq™ iT or Fortis Dental Implant Ø4.2mm L 10mm
14	FR-TS-RP4211	Clicq™ iT or Fortis Dental Implant Ø4.2mm L 11mm
15	FR-TS-RP4213	Clicq™ iT or Fortis Dental Implant Ø4.2mm L 13mm
16	FR-TS-RP4215	Clicq™ iT or Fortis Dental Implant Ø4.2mm L 15mm

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

 Fortis/ iT (EM)		
No	REF/ Productname	Product Description
17	FR-TS-RP5006	Clicq™ iT or Fortis Dental Implant Ø5.0mm L 06mm
18	FR-TS-RP5008	Clicq™ iT or Fortis Dental Implant Ø5.0mm L 08mm
19	FR-TS-RP5010	Clicq™ iT or Fortis Dental Implant Ø5.0mm L 10mm
20	FR-TS-RP5011	Clicq™ iT or Fortis Dental Implant Ø5.0mm L 11mm
21	FR-TS-RP5013	Clicq™ iT or Fortis Dental Implant Ø5.0mm L 13mm
22	FR-TS-RP5015	Clicq™ iT or Fortis Dental Implant Ø5.0mm L 15mm
23	FR-TS-RP5506	Clicq™ iT or Fortis Dental Implant Ø5.5mm L 06mm
24	FR-TS-RP5508	Clicq™ iT or Fortis Dental Implant Ø5.5mm L 08mm
25	FR-TS-RP5510	Clicq™ iT or Fortis Dental Implant Ø5.5mm L 10mm
26	FR-TS-RP5511	Clicq™ iT or Fortis Dental Implant Ø5.5mm L 11mm
27	FR-TS-RP5513	Clicq™ iT or Fortis Dental Implant Ø5.5mm L 13mm
28	FR-TS-RP5515	Clicq™ iT or Fortis Dental Implant Ø5.5mm L 15mm
29	FR-TS-RP6006	Clicq™ iT or Fortis Dental Implant Ø6.0mm L 06mm
30	FR-TS-RP6008	Clicq™ iT or Fortis Dental Implant Ø6.0mm L 08mm
31	FR-TS-RP6010	Clicq™ iT or Fortis Dental Implant Ø6.0mm L 10mm
32	FR-TS-RP6011	Clicq™ iT or Fortis Dental Implant Ø6.0mm L 11mm
33	FR-TS-RP6013	Clicq™ iT or Fortis Dental Implant Ø6.0mm L 13mm
34	FR-TS-RP6015	Clicq™ iT or Fortis Dental Implant Ø6.0mm L 15mm

*Tabelle 15 Augmentation iT/ Fortis Augmentat (EM)*

 Augmentat iT/ Fortis Augmentat (EM)		
No	REF/ Productname	Product Description
1	GM-TS-RP3508	Augmentat iT / Fortis Augmentat Dental Implant Ø3.6mm L 8mm
2	GM-TS-RP3510	Augmentat iT / Fortis Augmentat Dental Implant Ø3.6mm L 10mm

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28




 Augmentat iT/ Fortis Augmentat (EM)		
No	REF/ Productname	Product Description
3	GM-TS-RP3511	Augmentat iT / Fortis Augmentat Dental Implant Ø3.6mm L 11mm
4	GM-TS-RP3513	Augmentat iT / Fortis Augmentat Dental Implant Ø3.6mm L 13mm
5	GM-TS-RP3515	Augmentat iT / Fortis Augmentat Dental Implant Ø3.6mm L 15mm
6	GM-TS-RP4208	Augmentat iT / Fortis Augmentat Dental Implant Ø4.2mm L 8mm
7	GM-TS-RP4210	Augmentat iT / Fortis Augmentat Dental Implant Ø4.2mm L 10mm
8	GM-TS-RP4211	Augmentat iT / Fortis Augmentat Dental Implant Ø4.2mm L 11mm
9	GM-TS-RP4213	Augmentat iT / Fortis Augmentat Dental Implant Ø4.2mm L 13mm
10	GM-TS-RP4215	Augmentat iT / Fortis Augmentat Dental Implant Ø4.2mm L 15mm
11	GM-TS-RP5006	Augmentat iT / Fortis Augmentat Dental Implant Ø5.0mm L 6mm
12	GM-TS-RP5008	Augmentat iT / Fortis Augmentat Dental Implant Ø5.0mm L 8mm
13	GM-TS-RP5010	Augmentat iT / Fortis Augmentat Dental Implant Ø5.0mm L 10mm
14	GM-TS-RP5011	Augmentat iT / Fortis Augmentat Dental Implant Ø5.0mm L 11mm
15	GM-TS-RP5013	Augmentat iT / Fortis Augmentat Dental Implant Ø5.0mm L 13mm
16	GM-TS-RP5015	Augmentat iT / Fortis Augmentat Dental Implant Ø5.0mm L 15mm
17	GM-TS-RP6006	Augmentat iT / Fortis Augmentat Dental Implant Ø6.0mm L 6mm
18	GM-TS-RP6008	Augmentat iT / Fortis Augmentat Dental Implant Ø6.0mm L 8mm
19	GM-TS-RP6010	Augmentat iT / Fortis Augmentat Dental Implant Ø6.0mm L 10mm
20	GM-TS-RP6011	Augmentat iT / Fortis Augmentat Dental Implant Ø6.0mm L 11mm
21	GM-TS-RP6013	Augmentat iT / Fortis Augmentat Dental Implant Ø6.0mm L 13mm
22	GM-TS-RP6015	Augmentat iT / Fortis Augmentat Dental Implant Ø6.0mm L 15mm

Tabelle 16 i8 (EM)


 i8 (EM)		
No	REF/ Productname	Product Description
1	i8-OS-SP4208	i8 Dental Implant Ø4.2mm L 8mm

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

 i8 (EM)		
No	REF/ Productname	Product Description
2	i8-OS-SP4210	i8 Dental Implant Ø4.2mm L 10mm
3	i8-OS-SP4213	i8 Dental Implant Ø4.2mm L 13mm
4	i8-OS-SP4215	i8 Dental Implant Ø4.2mm L 15mm
5	i8-OS-SP5006	i8 Dental Implant Ø5.2mm L 6mm
6	i8-OS-SP5008	i8 Dental Implant Ø5.2mm L 8mm
7	i8-OS-SP5010	i8 Dental Implant Ø5.2mm L 10mm
8	i8-OS-SP5013	i8 Dental Implant Ø5.2mm L 13mm
9	i8-OS-SP5015	i8 Dental Implant Ø5.2mm L 15mm
10	i8-OS-SP6008	i8 Dental Implant Ø6.0mm L 8mm
11	i8-OS-SP6010	i8 Dental Implant Ø6.0mm L 10mm
12	i8-OS-SP6011	i8 Dental Implant Ø6.0mm L 11.5mm
13	i8-OS-SP6013	i8 Dental Implant Ø6.0mm L 13mm
14	i8-OS-SP6015	i8 Dental Implant Ø6.0mm L 15mm
15	i8-OS-SP6206	i8 Dental Implant Ø6.2mm L 6mm
16	i8-OS-SP6208	i8 Dental Implant Ø6.2mm L 8mm
17	i8-OS-SP6210	i8 Dental Implant Ø6.2mm L 10mm
18	i8-OS-SP6213	i8 Dental Implant Ø6.2mm L 13mm
19	i8-OS-SP6215	i8 Dental Implant Ø6.2mm L 15mm
20	i8-OS-SP6506	i8 Dental Implant Ø6.5mm L 6mm
21	i8-OS-SP6508	i8 Dental Implant Ø6.5mm L 8mm
22	i8-OS-SP6510	i8 Dental Implant Ø6.5mm L 10mm
23	i8-OS-SP6511	i8 Dental Implant Ø6.5mm L 11.5mm
24	i8-OS-SP6513	i8 Dental Implant Ø6.5mm L 13mm
25	i8-OS-SP6615	i8 Dental Implant Ø6.5mm L 15mm


*Tabelle 17 SB/LA Spiral (EM)*

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

 SB/ LA Spiral (EM)		
No	REF/ Productname	Product Description
1	SNAP-6-16	SB/LA Spiral Dental Implant Ø6.0mm L 16mm

## 5.2 Dentale Implantataufbauten

*Tabelle 18 Titan Abutments*

 Titanium Abutments		
No	REF/ Productname	Product Description
1	SSTA-1	Straight Titanium Abutment – Shoulder 1 mm
2	SSTA-2	Straight Titanium Abutment – Shoulder 2 mm
3	SSTA-3	Straight Titanium Abutment – Shoulder 3 mm
4	SSTA-4	Straight Titanium Abutment – Shoulder 4 mm
5	STA-5	Straight Titanium Abutment – 5 mm
6	STA-7	Straight Titanium Abutment – 7 mm
7	STA-9	Straight Titanium Abutment – 9 mm
8	STA-9W	Straight Titanium Abutment – 9 mm (Wide)
9	STA-12	Straight Titanium Abutment – 12 mm
10	STA-12W	Straight Titanium Abutment – 12 mm (Wide)
11	STA-15	Straight Titanium Abutment – 15 mm
12	ESPS-1	Esthetic Straight Abutment with Platform Shifting 1 mm
13	ESPS-2	Esthetic Straight Abutment with Platform Shifting 2 mm
14	ESPS-3	Esthetic Straight Abutment with Platform Shifting 3 mm
15	SAGA-1	Straight Anatomic Golden Titanium Abument 1 mm
16	SAGA-2	Straight Anatomic Golden Titanium Abument 2 mm
17	SAGA-3	Straight Anatomic Golden Titanium Abument 3 mm
18	SATA-1	Straight Anatomic Titanium Abutment 1 mm

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28




Titanium Abutments

No	REF/ Productname	Product Description
19	SATA-2	Straight Anatomic Titanium Abutment 2 mm
20	SATA-3	Straight Anatomic Titanium Abutment 3 mm
21	SLTA-5	Slim Titanium Abutment – 5 mm Smooth
22	SLTA-6	Slim Titanium Abutment – 6 mm Smooth
23	SLTA-7	Slim Titanium Abutment – 7 mm Smooth
24	SLTA-8	Slim Titanium Abutment – 8 mm Smooth
25	SLTA-9	Slim Titanium Abutment – 9 mm Smooth
26	SLTA-10	Slim Titanium Abutment – 10 mm Smooth
27	ATA-15	Angled Titanium Abutment-15
28	ATA-15L	Angled Titanium Abutment-15 (Long)
29	ATA-15N	Angled Titanium Abutment-15 (Narrow)
30	ATA-25	Angled Titanium Abutment-25
31	ATA-25N	Angled Titanium Abutment-25 (Narrow)
32	ATA-25L	Angled Titanium Abutment-25 (Long)
33	EATA-15-1	Esthetic Angled Titanium Abutment 15 degree 1 mm shoulder
34	EATA-15-2	Esthetic Angled Titanium Abutment 15 degree 2 mm shoulder
35	EATA-15-3	Esthetic Angled Titanium Abutment 15 degree 3 mm shoulder
36	EATA-25-1	Esthetic Angled Titanium Abutment 25 degree 1 mm shoulder
37	EATA-25-2	Esthetic Angled Titanium Abutment 25 degree 2 mm shoulder
38	EATA-25-3	Esthetic Angled Titanium Abutment 25 degree 3 mm shoulder
39	NL-SSTA-1	Straight Titanium Abumten – Shoulder 1 mm (Narrow Line)
40	NL-SSTA-2	Straight Titanium Abumten – Shoulder 2 mm (Narrow Line)
41	NL-SSTA-3	Straight Titanium Abumten – Shoulder 3 mm (Narrow Line)
42	NL-ATA-15	Angulated Titanium Abutment 15 Degree (Narrow Line)
43	NL-STA-10	Straight Titanium Abutment – 10 mm (Narrow Line)

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

Tabelle 19 Titan Abutments (EM)

 Titanium Abutments (EM)		
No	REF/ Productname	Product Description
1	SA-SV-RP5	Straight Ti-Abutment SV-RP Ø4.5 mm L 5 mm
2	SA-SV-RP7	Straight Ti-Abutment SV-RP Ø4.5 mm L 7 mm
3	SA-SV-RP9	Straight Ti-Abutment SV-RP Ø4.5 mm L 9 mm
4	SA-SV-RP12	Straight Ti-Abutment SV-RP Ø4.5 mm L 12 mm
5	SA-SV-RP15	Straight Ti-Abutment SV-RP Ø4.5 mm L 15 mm
6	SA-SV-RP9W	Straight Ti-Abutment SV-RP Ø5.5 mm L 9 mm Wide
7	SA-SV-RP12W	Straight Ti-Abutment SV-RP Ø5.5 mm L 12 mm Wide
8	SSA-SV-RP5	Straight Slim Ti-Abutment SV-RP Ø3.8 mm L 5 mm
9	SSA-SV-RP7	Straight Slim Ti-Abutment SV-RP Ø3.8 mm L 7 mm
10	SSA-SV-RP9	Straight Slim Ti-Abutment SV-RP Ø3.8 mm L 9 mm
11	TA-SV-RP1500N	Angled Standard Ti-Abutment SV-RP 15° Slim Body
12	TA-SV-RP1500	Angled Standard Ti-Abutment SV-RP 15°
13	TA-SV-RP1500L	Angled Standard Ti-Abutment SV-RP 15°/ Long
14	TA-SV-RP2500	Angled Standard Ti-Abutment SV-RP 25°
15	TA-SV-RP2500L	Angled Standard Ti-Abutment SV-RP 25°/ Long
16	TA-SV-RP3500	Angled Standard Ti-Abutment SV-RP 30°
17	TA-SV-RP10	Straight Ti-Abutment SV-RP Ø4.8 mm H 1 mm
18	TA-SV-RP20	Straight Ti-Abutment SV-RP Ø4.8 mm H 2 mm
19	TA-SV-RP30	Straight Ti-Abutment SV-RP Ø4.8 mm H 3 mm
20	TA-SV-RP40	Straight Ti-Abutment SV-RP Ø4.8 mm H 4 mm
21	TAC-SV-RP10	Straight ConcaveTi-Abutment SV-RP Ø4.8 mm H 1 mm
22	TAC-SV-RP20	Straight ConcaveTi-Abutment SV-RP Ø4.8 mm H 2 mm
23	TAC-SV-RP30	Straight ConcaveTi-Abutment SV-RP Ø4.8 mm H 3 mm
24	TAC-SV-RP40	Straight ConcaveTi-Abutment SV-RP Ø4.8 mm H 4 mm
25	TAA-SV-RP10	Straight AnatomicTi-Abutment SV-RP Ø5.0 mm H 1 mm
26	TAA-SV-RP20	Straight AnatomicTi-Abutment SV-RP Ø5.0 mm H 2 mm

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28



Titanium Abutments (EM)

No	REF/ Productname	Product Description
27	TAA-SV-RP30	Straight AnatomicTi-Abutment SV-RP Ø5.0 mm H 3 mm
28	TAA-SV-RP40	Straight AnatomicTi-Abutment SV-RP Ø5.0 mm H 4 mm
29	TAAC-SV-RP10	Straight Concave AnatomicTi-Abutment SV-RP Ø5.0 mm H 1 mm
30	TAAC-SV-RP20	Straight Concave AnatomicTi-Abutment SV-RP Ø5.0 mm H 2 mm
31	TAAC-SV-RP30	Straight Concave AnatomicTi-Abutment SV-RP Ø5.0 mm H 3 mm
32	TAAC-SV-RP40	Straight Concave AnatomicTi-Abutment SV-RP Ø5.0 mm H 4 mm
33	TAA-SV-RP1510	Angled Anatomic Ti-Abutment SV-RP 15°/ H 1 mm
34	TAA-SV-RP1520	Angled Anatomic Ti-Abutment SV-RP 15°/ H 2 mm
35	TAA-SV-RP1530	Angled Anatomic Ti-Abutment SV-RP 15°/ H 3 mm
36	TAA-SV-RP1540	Angled Anatomic Ti-Abutment SV-RP 15°/ H 4 mm
37	TAAC-SV- RP1510	Angled Concave Anatomic Ti-Abutment SV-RP 15°/ H 1 mm
38	TAAC-SV- RP1520	Angled Concave Anatomic Ti-Abutment SV-RP 15°/ H 2 mm
39	TAAC-SV- RP1530	Angled Concave Anatomic Ti-Abutment SV-RP 15°/ H 3 mm
40	TAAC-SV- RP1540	Angled Concave Anatomic Ti-Abutment SV-RP 15°/ H 4 mm
41	TAA-SV-RP2510	Angled Anatomic Ti-Abutment SV-RP 25°/ H 1 mm
42	TAA-SV-RP2520	Angled Anatomic Ti-Abutment SV-RP 25°/ H 2 mm
43	TAA-SV-RP2530	Angled Anatomic Ti-Abutment SV-RP 25°/ H 3 mm
44	TAA-SV-RP2540	Angled Anatomic Ti-Abutment SV-RP 25°/ H 4 mm
45	TAAC-SV- RP2510	Angled Concave Anatomic Ti-Abutment SV-RP 25°/ H 1 mm
46	TAAC-SV- RP2520	Angled Concave Anatomic Ti-Abutment SV-RP 25°/ H 2 mm
47	TAAC-SV- RP2530	Angled Concave Anatomic Ti-Abutment SV-RP 25°/ H 3 mm
48	TAAC-SV- RP2540	Angled Concave Anatomic Ti-Abutment SV-RP 25°/ H 4 mm

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28



Titanium Abutments (EM)

No	REF/ Productname	Product Description
49	SA-SV-WP5	Straight Ti-Abutment SV-WP Ø4.5 mm L 5 mm
50	SA-SV-WP7	Straight Ti-Abutment SV-WP Ø4.5 mm L 7 mm
51	SA-SV-WP9	Straight Ti-Abutment SV-WP Ø4.5 mm L 9 mm
52	SA-SV-WP12	Straight Ti-Abutment SV-WP Ø4.5 mm L 12 mm
53	SA-SV-WP15	Straight Ti-Abutment SV-WP Ø4.5 mm L 15mm
54	SA-SV-WP9W	Straight Ti-Abutment SV-WP Ø5.5 mm L 9 mm Wide
55	SA-SV-WP12W	Straight Ti-Abutment SV-WP Ø5.5 mm L 12 mm Wide
56	SSA-SV-WP5	Straight Slim Ti-Abutment SV-WP Ø3.8 mm L 5 mm
57	SSA-SV-WP7	Straight Slim Ti-Abutment SV-WP Ø3.8 mm L 7 mm
58	SSA-SV-WP9	Straight Slim Ti-Abutment SV-WP Ø3.8 mm L 9 mm
59	TA-SV-WP1500N	Angled Standard Ti-Abutment SV-WP 15° Slim Body
60	TA-SV-WP1500	Angled Standard Ti-Abutment SV-WP 15°
61	TA-SV-WP1500L	Angled Standard Ti-Abutment SV-WP 15°/ Long
62	TA-SV-WP2500	Angled Standard Ti-Abutment SV-WP 25°
63	TA-SV-WP2500L	Angled Standard Ti-Abutment SV-WP 25°/ Long
64	TA-SV-WP3500	Angled Standard Ti-Abutment SV-WP 30°
65	TA-SV-WP10	Straight Ti-Abutment SV-WP Ø4.8 mm H 1 mm
66	TA-SV-WP20	Straight Ti-Abutment SV-WP Ø4.8 mm H 2 mm
67	TA-SV-WP30	Straight Ti-Abutment SV-WP Ø4.8 mm H 3 mm
68	TA-SV-WP40	Straight Ti-Abutment SV-WP Ø4.8 mm H 4 mm
69	TAC-SV-WP10	Straight Concave Ti-Abutment SV-WP Ø4.8 mm H 1 mm
70	TAC-SV-WP20	Straight Concave Ti-Abutment SV-WP Ø4.8 mm H 2 mm
71	TAC-SV-WP30	Straight Concave Ti-Abutment SV-WP Ø4.8 mm H 3 mm
72	TAC-SV-WP40	Straight Concave Ti-Abutment SV-WP Ø4.8 mm H 4 mm
73	TAA-SV-WP10	Straight Anatomic Ti-Abutment SV-WP Ø5.0 mm H 1 mm
74	TAA-SV-WP20	Straight Anatomic Ti-Abutment SV-WP Ø5.0 mm H 2 mm
75	TAA-SV-WP30	Straight Anatomic Ti-Abutment SV-WP Ø5.0 mm H 3 mm

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

		
Titanium Abutments (EM)		
No	REF/ Productname	Product Description
76	TAA-SV-WP40	Straight Anatomic Ti-Abutment SV-WP Ø5.0 mm H 4 mm
77	TAAC-SV-WP10	Straight Concave Anatomic Ti-Abutment SV-WP Ø4.8 mm H 1 mm
78	TAAC-SV-WP20	Straight Concave Anatomic Ti-Abutment SV-WP Ø4.8 mm H 2 mm
79	TAAC-SV-WP30	Straight Concave Anatomic Ti-Abutment SV-WP Ø4.8 mm H 3 mm
80	TAAC-SV-WP40	Straight Concave Anatomic Ti-Abutment SV-WP Ø4.8 mm H 4 mm
81	TAA-SV-WP1510	Angled Anatomic Ti-Abutment SV-RP 15°/ H 1 mm
82	TAA-SV-WP1520	Angled Anatomic Ti-Abutment SV-RP 15°/ H 2 mm
83	TAA-SV-WP1530	Angled Anatomic Ti-Abutment SV-RP 15°/ H 3 mm
84	TAA-SV-WP1540	Angled Anatomic Ti-Abutment SV-RP 15°/ H 4 mm
85	TAAC-SV- WP1510	Angled Concave Anatomic Ti-Abutment SV-RP 15°/ H 1 mm
86	TAAC-SV- WP1520	Angled Concave Anatomic Ti-Abutment SV-RP 15°/ H 2 mm
87	TAAC-SV- WP1530	Angled Concave Anatomic Ti-Abutment SV-RP 15°/ H 3 mm
88	TAAC-SV- WP1540	Angled Concave Anatomic Ti-Abutment SV-RP 15°/ H 4 mm
89	TAA-SV-WP2510	Angled Anatomic Ti-Abutment SV-WP 25°/ H 1 mm
100	TAA-SV-WP2520	Angled Anatomic Ti-Abutment SV-WP 25°/ H 2 mm
101	TAA-SV-WP2530	Angled Anatomic Ti-Abutment SV-WP 25°/ H 3 mm
102	TAA-SV-WP2540	Angled Anatomic Ti-Abutment SV-WP 25°/ H 4 mm
103	TAAC-SV- WP2510	Angled Concave Anatomic Ti-Abutment SV-RP 25°/ H 1 mm
104	TAAC-SV- WP2520	Angled Concave Anatomic Ti-Abutment SV-RP 25°/ H 2 mm
105	TAAC-SV- WP2530	Angled Concave Anatomic Ti-Abutment SV-RP 25°/ H 3 mm
106	TAAC-SV- WP2540	Angled Concave Anatomic Ti-Abutment SV-RP 25°/ H 4 mm
107	SA-SV-NL5	Straight Ti-Abutment SV-NL Ø4.5 mm L 5 mm

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28



Titanium Abutments (EM)

No	REF/ Productname	Product Description
108	SA-SV-NL7	Straight Ti-Abutment SV-NL Ø4.5 mm L 7 mm
109	SA-SV-NL9	Straight Ti-Abutment SV-NL Ø4.5 mm L 9 mm
110	SA-SV-NL12	Straight Ti-Abutment SV-NL Ø4.5 mm L 12 mm
111	SA-SV-NL15	Straight Ti-Abutment SV-NL Ø4.5 mm L 15 mm
112	SA-SV-NL9W	Straight Ti-Abutment SV-NL Ø5.5 mm L 9 mm Wide
113	SA-SV-NL12W	Straight Ti-Abutment SV-NL Ø5.5 mm L 12 mm Wide
114	SSA-SV-NL5	Straight Slim Ti-Abutment SV-NL Ø3.8 mm L 5 mm
115	SSA-SV-NL7	Straight Slim Ti-Abutment SV-NL Ø3.8 mm L 7 mm
116	SSA-SV-NL9	Straight Slim Ti-Abutment SV-NL Ø3.8 mm L 9 mm
117	TA-SV-NL1500N	Angled Standard Ti-Abutment SV-NL 15° Slim Body
118	TA-SV-NL1500	Angled Standard Ti-Abutment SV-NL 15°
119	TA-SV-NL1500L	Angled Standard Ti-Abutment SV-NL 15° Long
120	TA-SV-NL2500	Angled Standard Ti-Abutment SV-NL 25°
121	TA-SV-NL2500L	Angled Standard Ti-Abutment SV-NL 25° Long
122	TA-SV-NL3500	Angled Standard Ti-Abutment SV-NL 30°
123	TA-SV-NL10	Straight Ti-Abutment SV-NL Ø4.8 mm H 1 mm
124	TA-SV-NL20	Straight Ti-Abutment SV-NL Ø4.8 mm H 2 mm
125	TA-SV-NL30	Straight Ti-Abutment SV-NL Ø4.8 mm H 3 mm
126	TA-SV-NL40	Straight Ti-Abutment SV-NL Ø4.8 mm H 4 mm
127	TAC-SV-NL10	Straight Concave Ti-Abutment SV-NL Ø4.8 mm H 1 mm
128	TAC-SV-NL20	Straight Concave Ti-Abutment SV-NL Ø4.8 mm H 2 mm
129	TAC-SV-NL30	Straight Concave Ti-Abutment SV-NL Ø4.8 mm H 3 mm
130	TAC-SV-NL40	Straight Concave Ti-Abutment SV-NL Ø4.8 mm H 4 mm
131	TAA-SV-NL10	Straight Anatomic Ti-Abutment SV-NL Ø5.0 mm H 1 mm
132	TAA-SV-NL20	Straight Anatomic Ti-Abutment SV-NL Ø5.0 mm H 2 mm
133	TAA-SV-NL30	Straight Anatomic Ti-Abutment SV-NL Ø5.0 mm H 3 mm
134	TAA-SV-NL40	Straight Anatomic Ti-Abutment SV-NL Ø5.0 mm H 4 mm

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28



Titanium Abutments (EM)

No	REF/ Productname	Product Description
135	TAAC-SV-NL10	Straight Concave Anatomic Ti-Abutment SV-NL Ø4.8 mm H 1 mm
136	TAAC-SV-NL20	Straight Concave Anatomic Ti-Abutment SV-NL Ø4.8 mm H 2 mm
137	TAAC-SV-NL30	Straight Concave Anatomic Ti-Abutment SV-NL Ø4.8 mm H 3 mm
138	TAAC-SV-NL40	Straight Concave Anatomic Ti-Abutment SV-NL Ø4.8 mm H 4 mm
139	TAA-SV-NL1510	Angled Anatomic Ti-Abutment SV-NL 15°/ H 1 mm
140	TAA-SV-NL1520	Angled Anatomic Ti-Abutment SV-NL 15°/ H 2 mm
141	TAA-SV-NL1530	Angled Anatomic Ti-Abutment SV-NL 15°/ H 3 mm
142	TAA-SV-NL1540	Angled Anatomic Ti-Abutment SV-NL 15°/ H 4 mm
143	TAAC-SV- NL1510	Angled Concave Anatomic Ti-Abutment SV-NL 15°/ H 1 mm
145	TAAC-SV- NL1520	Angled Concave Anatomic Ti-Abutment SV-NL 15°/ H 2 mm
146	TAAC-SV- NL1530	Angled Concave Anatomic Ti-Abutment SV-NL 15°/ H 3 mm
147	TAAC-SV- NL1540	Angled Concave Anatomic Ti-Abutment SV-NL 15°/ H 4 mm
148	TAA-SV-NL2510	Angled Anatomic Ti-Abutment SV-NL 25°/ H 1 mm
149	TAA-SV-NL2520	Angled Anatomic Ti-Abutment SV-NL 25°/ H 2 mm
150	TAA-SV-NL2530	Angled Anatomic Ti-Abutment SV-NL 25°/ H 3 mm
151	TAA-SV-NL2540	Angled Anatomic Ti-Abutment SV-NL 25°/ H 4 mm
152	TAAC-SV- NL2510	Angled Concave Anatomic Ti-Abutment SV-NL 25°/ H 1 mm
153	TAAC-SV- NL2520	Angled Concave Anatomic Ti-Abutment SV-NL 25°/ H 2 mm
154	TAAC-SV- NL2530	Angled Concave Anatomic Ti-Abutment SV-NL 25°/ H 3 mm
155	TAAC-SV- NL2540	Angled Concave Anatomic Ti-Abutment SV-NL 25°/ H 4 mm
156	SA-SV-NP5	Ti-Abutment SV-NP Ø4.5 mm L 5 mm
157	SA-SV-NP7	Ti-Abutment SV-NP Ø4.5 mm L 7 mm

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28



Titanium Abutments (EM)

No	REF/ Productname	Product Description
158	SA-SV-NP9	Ti-Abutment SV-NP Ø4.5 mm L 9 mm
159	SA-SV-NP12	Ti-Abutment SV-NP Ø4.5 mm L 12 mm
160	SA-SV-NP15	Ti-Abutment SV-NP Ø4.5 mm L 15 mm
161	SA-SV-NP9W	Ti-Abutment SV-NP Ø5.5 mm L 9 mm Wide
162	SA-SV-NP12W	Ti-Abutment SV-NP Ø5.5 mm L 12 mm Wide
163	SSA-SV-NP5	Slim Ti-Abutment SV-NPØ3.8 mm L 5 mm
164	SSA-SV-NP7	Slim Ti-Abutment SV-NPØ3.8 mm L 7 mm
165	SSA-SV-NP9	Slim Ti-Abutment SV-NPØ3.8 mm L 9 mm
166	TA-SV-NP1500N	Standard Ti-Abutment SV-NP 15° Slim Body
167	TA-SV-NP1500	Standard Ti-Abutment SV-NP 15°
168	TA-SV-NP1500L	Standard Ti-Abutment SV-NP 15°/ Long
169	TA-SV-NP2500	Standard Ti-Abutment SV-NP 25°
170	TA-SV-NP2500L	Standard Ti-Abutment SV-NP 25°/ Long
171	TA-SV-NP3500	Standard Ti-Abutment SV-NP 30°
172	TA-SV-NP10	Ti-Abument SV-NP Ø4.8 mm H 1 mm
173	TA-SV-NP20	Ti-Abument SV-NP Ø4.8 mm H 2 mm
174	TA-SV-NP30	Ti-Abument SV-NP Ø4.8 mm H 3 mm
175	TA-SV-NP40	Ti-Abument SV-NP Ø4.8 mm H 4 mm
176	TAC-SV-NP10	Concave Ti-Abument SV-NP Ø4.8 mm H 1 mm
177	TAC-SV-NP20	Concave Ti-Abument SV-NP Ø4.8 mm H 2 mm
178	TAC-SV-NP30	Concave Ti-Abument SV-NP Ø4.8 mm H 3 mm
179	TAC-SV-NP40	Concave Ti-Abument SV-NP Ø4.8 mm H 4 mm
180	TAA-SC-NP10	Anatomic Ti-Abument SV-NP Ø5.0 mm H 1 mm
181	TAA-SC-NP20	Anatomic Ti-Abument SV-NP Ø5.0 mm H 2 mm
182	TAA-SC-NP30	Anatomic Ti-Abument SV-NP Ø5.0 mm H 3 mm
183	TAA-SC-NP40	Anatomic Ti-Abument SV-NP Ø5.0 mm H 4 mm
184	TAAC-NP10	Concave Anatomic Ti-Abument SV-NP Ø4.8 mm H 1 mm

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28



Titanium Abutments (EM)

No	REF/ Productname	Product Description
185	TAAC-NP20	Concave Anatomic Ti-Abument SV-NP Ø4.8 mm H 2 mm
186	TAAC-NP30	Concave Anatomic Ti-Abument SV-NP Ø4.8 mm H 3 mm
187	TAAC-NP40	Concave Anatomic Ti-Abument SV-NP Ø4.8 mm H 4 mm
188	TAA-SV-NP1510	Anatomic Ti-Abument SV-NP 15°/ H 1 mm
189	TAA-SV-NP1520	Anatomic Ti-Abument SV-NP 15°/ H 2 mm
190	TAA-SV-NP1530	Anatomic Ti-Abument SV-NP 15°/ H 3 mm
191	TAA-SV-NP1540	Anatomic Ti-Abument SV-NP 15°/ H 4 mm
192	TAAC-SV- NP1510	Concave Anatomic Ti-Abument SV-NP 15°/ H 1 mm
193	TAAC-SV- NP1520	Concave Anatomic Ti-Abument SV-NP 15°/ H 2 mm
194	TAAC-SV- NP1530	Concave Anatomic Ti-Abument SV-NP 15°/ H 3 mm
195	TAAC-SV- NP1540	Concave Anatomic Ti-Abument SV-NP 15°/ H 4 mm
196	TAA-SV-NP2510	Anatomic Ti-Abument SV-NP 25°/ H 1 mm
197	TAA-SV-NP2520	Anatomic Ti-Abument SV-NP 25°/ H 2 mm
198	TAA-SV-NP2530	Anatomic Ti-Abument SV-NP 25°/ H 3 mm
199	TAA-SV-NP2540	Anatomic Ti-Abument SV-NP 25°/ H 4 mm
200	TAAC-SV- NP2510	Concave Anatomic Ti-Abument SV-NP 25°/ H 1 mm
201	TAAC-SV- NP2520	Concave Anatomic Ti-Abument SV-NP 25°/ H 2 mm
202	TAAC-SV- NP2530	Concave Anatomic Ti-Abument SV-NP 25°/ H 3 mm
203	TAAC-SV- NP2540	Concave Anatomic Ti-Abument SV-NP 25°/ H 4 mm
204	TA-NA-NP15	Ti-Abument NA-NP Ø4.8 mm H 1.5 mm
205	TA-NA-NP30	Ti-Abument NA-NP Ø4.8 mm H 3 mm
206	TA-NA-NP45	Ti-Abument NA-NP Ø4.8 mm H 4.5 mm

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28



Titanium Abutments (EM)

No	REF/ Productname	Product Description
207	SSA-NA-NP12	Slim Ti-Abutment NA-NP Ø3.8 mm L 12 mm
208	SA-NA-NPN	Ti-Abutment NA-NP Ø3.8 mm L 10 mm
209	SA-NA-NP	Ti-Abutment NA-NP Ø4.5 mm L 10 mm
210	TAA-NA-NP15	Anatomic Ti-Abumentent NA-NP Ø5.0 mm H 1.5 mm
211	TAA-NA-NP30	Anatomic Ti-Abumentent NA-NP Ø5.0 mm H 3 mm
212	TAA-NA-NP45	Anatomic Ti-Abumentent NA-NP Ø5.0 mm H 4.5 mm
213	TA-NA-NP1500	Standard Ti-Abutment NA-NP 15°
214	TAA-NA-NP1515	Anatomic Ti-Abumentent NA-NP 15°/ H 1.5 mm
215	TAA-NA-NP1530	Anatomic Ti-Abumentent NA-NP 15°/ H 3 mm
216	TAA-NA-NP1545	Anatomic Ti-Abumentent NA-NP 15°/ H 4.5 mm
217	TAA-NA-NP25	Standard Ti-Abutment NA-NP 25°
218	TAA-NA-NP2515	Anatomic Ti-Abumentent NA-NP 25°/ H 1.5 mm
219	TAA-NA-NP2530	Anatomic Ti-Abumentent NA-NP 25°/ H 3 mm
220	TAA-NA-NP2545	Anatomic Ti-Abumentent NA-NP 25°/ H 4.5 mm
221	TA-NA-RP15	Ti-Abumentent NA-RP Ø4.8 mm H 1.5 mm
223	TA-NA-RP30	Ti-Abumentent NA-RP Ø4.8 mm H 3 mm
224	TA-NA-RP45	Ti-Abumentent NA-RP Ø4.8 mm H 4.5 mm
225	SA-NA-RP	Ti-Abumentent NA-RP Ø4.5 mm L 10 mm
226	SA-NA-RPW	Ti-Abumentent NA-RP Ø5.5 mm L 10 mm
227	TAA-NA-RP15	Anatomic Ti-Abumentent NA-RP Ø5.0 mm H 1.5 mm
228	TAA-NA-RP30	Anatomic Ti-Abumentent NA-RP Ø5.0 mm H 3 mm
229	TAA-NA-RP45	Anatomic Ti-Abumentent NA-RP Ø5.0 mm H 4.5 mm
230	TA-NA-RP1500	Standard Ti-Abumentent NA-RP 15°
231	TA-NA-RP1515	Anatomic Ti-Abumentent NA-RP 15°/ H 1.5 mm
232	TA-NA-RP1530	Anatomic Ti-Abumentent NA-RP 15°/ H 3 mm
233	TA-NA-RP1545	Anatomic Ti-Abumentent NA-RP 15°/ H 4.5 mm
234	TA-NA-RP2500	Standard Ti-Abumentent NA-RP 25°

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28



Titanium Abutments (EM)

No	REF/ Productname	Product Description
235	TA-NA-RP2515	Anatomic Ti-Abumentent NA-RP 25°/ H 1.5 mm
236	TA-NA-RP2530	Anatomic Ti-Abumentent NA-RP 25°/ H 3 mm
237	TA-NA-RP2545	Anatomic Ti-Abumentent NA-RP 25°/ H 4.5 mm
238	TA-TS-NP15	Ti-Abumentent TS-NP/ Mini Ø4.8 mm H 1.5 mm
239	TA-TS-NP30	Ti-Abumentent TS-NP/ Mini Ø4.8 mm H 3 mm
240	TA-TS-NP45	Ti-Abumentent TS-NP/ Mini Ø4.8 mm H 4.5 mm
241	SSA-TS-NP12	Ti-Abumentent TS-NP/ Mini Ø3.8 mm L 12 mm
242	SA-TS-NPN	Ti-Abumentent TS-NP/ Mini Ø3.8 mm L 10 mm
243	SA-TS-NP	Ti-Abumentent TS-NP/ Mini Ø4.5 mm L 12 mm
244	TAA-TS-NP15	Anatomic Ti-Abumentent TS-NP/ Mini Ø5.0 mm H 1.5 mm
245	TAA-TS-NP30	Anatomic Ti-Abumentent TS-NP/ Mini Ø5.0 mm H 3 mm
246	TAA-TS-NP45	Anatomic Ti-Abumentent TS-NP/ Mini Ø5.0 mm H 4.5 mm
247	TA-TS-NP1500	Standard Ti-Abumentent TS-NP/ Mini 15°
248	TA-TS-NP1515	Anatomic Ti-Abumentent TS-NP/ Mini 15°/ H 1.5 mm
249	TAA-TS-NP1530	Anatomic Ti-Abumentent TS-NP/ Mini 15°/ H 3 mm
250	TAA-TS-NP1545	Anatomic Ti-Abumentent TS-NP/ Mini 15°/ H 4.5 mm
251	TAA-TS-NP2500	Standard Ti-Abumentent TS-NP/ Mini 25°
252	TAA-TS-NP2515	Anatomic Ti-Abumentent TS-NP/ Mini 25°/ H 1.5 mm
253	TAA-TS-NP2530	Anatomic Ti-Abumentent TS-NP/ Mini 25°/ H 3 mm
254	TAA-TS-NP2545	Anatomic Ti-Abumentent TS-NP/ Mini 25°/ H 4.5 mm
255	TA-TS-RP15	Ti-Abumentent TS-RP Ø4.8 mm H 1.5 mm
256	TA-TS-RP30	Ti-Abumentent TS-RP Ø4.8 mm H 3 mm
257	TA-TS-RP45	Ti-Abumentent TS-RP Ø4.8 mm H 4.5 mm
258	SSA-TS-RP12	Slim Ti-Abumentent TS-RP Ø3.8 mm L 12 mm
259	SA-TS-RP	Ti-Abumentent TS-RP Ø4.5 mm L 10 mm
260	SA-TS-RPW	Ti-Abumentent TS-RP Ø5.5 mm L 10 mm
261	TAA-TS-RP15	Anatomic Ti-Abumentent TS-RP Ø5.5 mm H 1.5 mm

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28



Titanium Abutments (EM)

No	REF/ Productname	Product Description
262	TAA-TS-RP30	Anatomic Ti-Abumentent TS-RP Ø5.5 mm H 3 mm
263	TAA-TS-RP45	Anatomic Ti-Abumentent TS-RP Ø5.5 mm H 4.5 mm
264	TA-TS-RP1500	Standard Ti-Abumentent TS-RP 15°
265	TAA-TS-RP1515	Anatomic Ti-Abumentent TS-RP 15°/ H 1.5 mm
266	TAA-TS-RP1530	Anatomic Ti-Abumentent TS-RP 15°/ H 3 mm
267	TAA-TS-RP1545	Anatomic Ti-Abumentent TS-RP 15°/ H 4.5 mm
268	TA-TS-RP2500	Standard Ti-Abumentent TS-RP 25°
269	TAA-TS-RP2515	Anatomic Ti-Abumentent TS-RP 25°/ H 1.5 mm
270	TAA-TS-RP2530	Anatomic Ti-Abumentent TS-RP 25°/ H 3 mm
271	TAA-TS-RP2545	Anatomic Ti-Abumentent TS-RP 25°/ H 4.5 mm
272	TA-OS-SP15	Ti-Abumentent OS-SP Ø4.8 mm H 1.5 mm
273	TA-OS-SP30	Ti-Abumentent OS-SP Ø4.8 mm H 3 mm
274	TA-OS-SP45	Ti-Abumentent OS-SP Ø4.8 mm H 4.5 mm
275	SSA-OS-SP12	Slim Ti-Abumentent OS-SP Ø3.8 mm L 12 mm
276	SA-OS-SPN	Ti-Abumentent OS-SP Ø3.8 mm L 10 mm
277	SA-OS-SP	Ti-Abumentent OS-SP Ø4.5 mm L 10 mm
278	TAA-OS-SP15	Anatomic Ti-Abumentent OS-SP Ø5.0 mm H 1.5 mm
279	TAA-OS-SP30	Anatomic Ti-Abumentent OS-SP Ø5.0 mm H 3 mm
280	TAA-OS-SP45	Anatomic Ti-Abumentent OS-SP Ø5.0 mm H 4.5 mm
281	TA-OS-SP1500	Standard Ti-Abumentent OS-SP 15°
282	TAA-OS-SP1515	Anatomic Ti-Abumentent OS-SP 15°/ H 1.5 mm
283	TAA-OS-SP1530	Anatomic Ti-Abumentent OS-SP 15°/ H 3 mm
284	TAA-OS-SP1545	Anatomic Ti-Abumentent OS-SP 15°/ H 4.5 mm
285	TA-OS-SP2500	Standard Ti-Abumentent OS-SP 25°
286	TAA-OS-SP2515	Anatomic Ti-Abumentent OS-SP 25°/ H 1.5 mm
287	TAA-OS-SP2530	Anatomic Ti-Abumentent OS-SP 25°/ H 3 mm
288	TAA-OS-SP2545	Anatomic Ti-Abumentent OS-SP 25°/ H 4.5 mm

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28



Titanium Abutments (EM)

No	REF/ Productname	Product Description
289	TA-OS-WP15	Ti-Abument OS-WP Ø4.8 mm H 1.5 mm
290	TA-OS-WP30	Ti-Abument OS-WP Ø4.8 mm H 3 mm
291	TA-OS-WP45	Ti-Abument OS-WP Ø4.8 mm H 4.5 mm
292	SA-OS-WP	Ti-Abument OS-WP Ø4.5 mm L 10 mm
293	SA-OS-WPW	Ti-Abument OS-WP Ø5.5 mm L 10 mm
294	TAA-OS-WP15	Anatomic Ti-Abument OS-WP Ø5.0 mm H 1.5 mm
295	TAA-OS-WP30	Anatomic Ti-Abument OS-WP Ø5.0 mm H 3 mm
296	TAA-OS-WP45	Anatomic Ti-Abument OS-WP Ø5.0 mm H 4.5 mm
297	TA-OS-WP1500	Standard Ti-Abument OS-WP 15°
298	TA-OS-WP1515	Anatomic Ti-Abument OS-WP 15°/ H 1.5 mm
299	TA-OS-WP1530	Anatomic Ti-Abument OS-WP 15°/ H 3 mm
300	TA-OS-WP1545	Anatomic Ti-Abument OS-WP 15°/ H 4.5 mm
301	TA-OS-WP2500	Standard Ti-Abument OS-WP 25°
302	TA-OS-WP2515	Anatomic Ti-Abument OS-WP 25°/ H 1.5 mm
303	TA-OS-WP2530	Anatomic Ti-Abument OS-WP 25°/ H 3 mm
304	TA-OS-WP2545	Anatomic Ti-Abument OS-WP 25°/ H 4.5 mm
305	TA-AR-RP15	Anatomic Ti-Abument AR-RP H 1.5 mm
306	TA-AR-RP30	Anatomic Ti-Abument AR-RP H 3 mm
307	TA-AR-RP45	Anatomic Ti-Abument AR-RP H 4.5 mm
308	SA-AR-RP	Ti-Abument AR-RP Ø4.5 mm L 10 mm
309	SA-AR-RPW	Ti-Abument AR-RP Ø5.5 mm L 10 mm
310	TAA-AR-RP15	Anatomic Ti-Abument AR-RP Ø5.0 mm H 1.5 mm
311	TAA-AR-RP30	Anatomic Ti-Abument AR-RP Ø5.0 mm H 3 mm
312	TAA-AR-RP45	Anatomic Ti-Abument AR-RP Ø5.0 mm H 4.5 mm
313	TA-AR-RP1500	Standard Ti-Abument AR-RP 15°
314	TA-AR-RP1515	Anatomic Ti-Abument AR-RP 15°/ H 1.5 mm
315	TA-AR-RP1530	Anatomic Ti-Abument AR-RP 15°/ H 3 mm


Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28



Titanium Abutments (EM)

No	REF/ Productname	Product Description
316	TA-AR-RP1545	Anatomic Ti-Abumentent AR-RP 15°/ H 4.5 mm
317	TA-AR-RP2500	Standard Ti-Abumentent AR-RP 25°
318	TA-AR-RP2515	Anatomic Ti-Abumentent AR-RP 25°/ H 1.5 mm
319	TA-AR-RP2530	Anatomic Ti-Abumentent AR-RP 25°/ H 3 mm
320	TA-AR-RP2545	Anatomic Ti-Abumentent AR-RP 25°/ H 4.5 mm
321	TA-GM-RP15	Ti-Abumentent GM-RP Ø4.8 mm H 1.5 mm
322	TA-GM-RP30	Ti-Abumentent GM-RP Ø4.8 mm H 3 mm
323	TA-GM-RP45	Ti-Abumentent GM-RP Ø4.8 mm H 4.5 mm
324	SA-GM-RP	Ti-Abumentent GM-RP Ø3.8 mm L 10 mm
325	SA-GM-RPW	Ti-Abumentent GM-RP Ø4.5 mm L 10 mm
326	TAA-GM-RP15	Anatomic Ti-Abumentent GM-RP Ø5.0 mm H 1.5 mm
327	TAA-GM-RP30	Anatomic Ti-Abumentent GM-RP Ø5.0 mm H 3 mm
328	TAA-GM-RP45	Anatomic Ti-Abumentent GM-RP Ø5.0 mm H 4.5 mm
329	TA-GM-RP1500	Standard Ti-Abumentent GM-RP 15°
330	TAA-GM-RP1515	Anatomic Ti-Abumentent GM-RP 15°/ H 1.5 mm
331	TAA-GM-RP1530	Anatomic Ti-Abumentent GM-RP 15°/ H 3 mm
332	TAA-GM-RP1545	Anatomic Ti-Abumentent GM-RP 15°/ H 4.5 mm
333	TA-GM-RP2500	Standard Ti-Abumentent GM-RP 25°
334	TAA-GM-RP2515	Anatomic Ti-Abumentent GM-RP 25°/ H 1.5 mm
335	TAA-GM-RP2530	Anatomic Ti-Abumentent GM-RP 25°/ H 3 mm
336	TAA-GM-RP2545	Anatomic Ti-Abumentent GM-RP 25°/ H 4.5 mm
337	TA-CF-RC15	Ti-Abumentent CF-RC Ø4.8 mm H 1.5 mm
338	TA-CF-RC30	Ti-Abumentent CF-RC Ø4.8 mm H 3 mm
339	TA-CF-RC45	Ti-Abumentent CF-RC Ø4.8 mm H 4.5 mm
340	SA-CF-RC	Ti-Abumentent CF-RC Ø4.5 mm L 10 mm
341	SA-CF-RCW	Ti-Abumentent CF-RC Ø5.5 mm L 10 mm
342	TAA-CF-RC15	Anatomic Ti-Abumentent CF-RC Ø5.0 mm H 1.5 mm

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

		
Titanium Abutments (EM)		
No	REF/ Productname	Product Description
343	TAA-CF-RC30	Anatomic Ti-Abument CF-RC Ø5.0 mm H 3 mm
344	TAA-CF-RC45	Anatomic Ti-Abument CF-RC Ø5.0 mm H 4.5 mm
345	TA-CF-RC1500A	Standard Ti-Abument CF-RC 15° A Type
346	TA-CF-RC1500B	Standard Ti-Abument CF-RC 15° B Type
347	TAA-CF-RC1515A	Anatomic Ti-Abument CF-RC 15°/ H 1.5 mm A Type
348	TAA-CF-RC1530A	Anatomic Ti-Abument CF-RC 15°/ H 3 mm A Type
349	TAA-CF-RC1545A	Anatomic Ti-Abument CF-RC 15°/ H 4.5 mm A Type
350	TAA-CF-RC1515B	Anatomic Ti-Abument CF-RC 15°/ H 1.5 mm B Type
351	TAA-CF-RC1530B	Anatomic Ti-Abument CF-RC 15°/ H 3 mm B Type
352	TAA-CF-RC1545B	Anatomic Ti-Abument CF-RC 15°/ H 4.5 mm B Type
353	TA-CF-RC2500A	Standard Ti-Abument CF-RC 25° A Type
354	TA-CF-RC2500B	Standard Ti-Abument CF-RC 25° B Type
355	TAA-CF-RC2515A	Anatomic Ti-Abument CF-RC 25°/ H 1.5 mm A Type
356	TAA-CF-RC2530A	Anatomic Ti-Abument CF-RC 25°/ H 3 mm A Type
357	TAA-CF-RC2545A	Anatomic Ti-Abument CF-RC 25°/ H 4.5 mm A Type
358	TAA-CF-RC2515B	Anatomic Ti-Abument CF-RC 25°/ H 1.5 mm B Type
359	TAA-CF-RC2530B	Anatomic Ti-Abument CF-RC 25°/ H 3 mm B Type
360	TAA-CF-RC2545B	Anatomic Ti-Abument CF-RC 25°/ H 4.5 mm B Type
361	TA-CF-NC15	Ti-Abument CF-NC Ø4.8 mm H 1.5 mm
362	TA-CF-NC30	Ti-Abument CF-NC Ø4.8 mm H 3 mm


Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28




Titanium Abutments (EM)

No	REF/ Productname	Product Description
363	TA-CF-NC45	Ti-Abument CF-NC Ø4.8 mm H 4.5 mm
364	SA-CF-NCN	Slim Ti-Abument CF-NC Ø3.8 mm L 10 mm
365	SA-CF-NC	Slim Ti-Abument CF-NC Ø4.5 mm L 10 mm
366	TAA-CF-NC15	Anatomic Ti-Abument CF-NC Ø5.0 mm H 1.5 mm
367	TAA-CF-NC30	Anatomic Ti-Abument CF-NC Ø5.0 mm H 3 mm
368	TAA-CF-NC45	Anatomic Ti-Abument CF-NC Ø5.0 mm H 4.5 mm
369	TA-CF-NC1500A	Standard Ti-Abument CF-NC 15° A Type
370	TA-CF-NC1500B	Standard Ti-Abument CF-NC 15° B Type
371	TAA-CF- NC1515A	Anatomic Ti-Abument CF-NC 15°/ H 1.5 mm A Type
372	TAA-CF- NC1530A	Anatomic Ti-Abument CF-NC 15°/ H 3 mm A Type
373	TAA-CF- NC1545A	Anatomic Ti-Abument CF-NC 15°/ H 4.5 mm A Type
374	TAA-CF- NC1515B	Anatomic Ti-Abument CF-NC 15°/ H 1.5 mm B Type
375	TAA-CF- NC1530B	Anatomic Ti-Abument CF-NC 15°/ H 3 mm B Type
376	TAA-CF- NC1545B	Anatomic Ti-Abument CF-NC 15°/ H 4.5 mm B Type
377	TA-CF-NC2500A	Standard Ti-Abument CF-NC 25° A Type
378	TA-CF-NC2500B	Standard Ti-Abument CF-NC 25° B Type
379	TAA-CF- NC2515A	Anatomic Ti-Abument CF-NC 25°/ H 1.5 mm A Type
380	TAA-CF- NC2530A	Anatomic Ti-Abument CF-NC 25°/ H 3 mm A Type
381	TAA-CF- NC2545A	Anatomic Ti-Abument CF-NC 25°/ H 4.5 mm A Type
382	TAA-CF- NC2515B	Anatomic Ti-Abument CF-NC 25°/ H 1.5 mm B Type
383	TAA-CF- NC2530B	Anatomic Ti-Abument CF-NC 25°/ H 3 mm B Type


Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

		
Titanium Abutments (EM)		
No	REF/ Productname	Product Description
384	TAA-CF- NC2545B	Anatomic Ti-Abument CF-NC 25°/ H 4.5 mm B Type


*Tabelle 20 Zirkon Abutments*

		
Zirconium Abutments		
No	REF/ Productname	Product Description
1	ZSTA-8	Zircorit Zirconium Straight Abutment with Metal Base – 8 mm
2	ZASA-05	Zircorit Zirconium Anatomic Straight Abutment – 0.5 mm, L = 10 mm
3	ZASA-1	Zircorit Zirconium Anatomic Straight Abutment – 1 mm, L = 10.5 mm
4	ZASA-2	Zircorit Zirconium Anatomic Straight Abutment – 2 mm, L = 10.9 mm
5	ZASA-3	Zircorit Zirconium Anatomic Straight Abutment – 3 mm, L = 12.5 mm
6	EAZA-15-05	Zircorit Esthetic Angled Zirconium 15 Degree 0.5 mm Shoulder
7	EAZA-15-1	Zircorit Esthetic Angled Zirconium 15 Degree 1 mm Shoulder
8	EAZA-15-2	Zircorit Esthetic Angled Zirconium 15 Degree 2 mm Shoulder
9	EAZA-15-3	Zircorit Esthetic Angled Zirconium 15 Degree 3 mm Shoulder
10	EAZA-25-05	Zircorit Esthetic Angled Zirconium 25 Degree 0.5 mm Shoulder
11	EAZA-25-1	Zircorit Esthetic Angled Zirconium 25 Degree 1 mm Shoulder
12	EAZA-25-2	Zircorit Esthetic Angled Zirconium 25 Degree 2 mm Shoulder
13	EAZA-25-3	Zircorit Esthetic Angled Zirconium 25 Degree 3 mm Shoulder
14	XSZA-05	Zircorit X-Shape Straight Zirconium Abutment 0.5 mm Shoulder
15	XSZA-1	Zircorit X-Shape Straight Zirconium Abutment 1 mm Shoulder L = 9.4 mm
16	XAZA-15-05	Zircorit X-Shape Angled Zirconium Abutment 15 Degree 0.5 mm Shoulder
17	XAZA-15-1W	Zircorit X-Shape Angled Zirconium Abutment 15 Degree 1 mm Shoulder – Wide

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

 Zirconium Abutments		
No	REF/ Productname	Product Description
18	XAZA-25-05	Zircorit X-Shape Angled Zirconium Abutment 25 Degree 0.5 mm Shoulder.
19	XAZA-25-1W	Zircorit X-Shape Angled Zirconium Abutment 25 Degree 1 mm Shoulder – Wide

*Tabelle 21 Zirkon Abutments (EM)*

 Zircocam Zirconia Abutments (EM)		
No	REF/ Productnamen	Product Description
1	ZAA-SV-RP10	Zircocam Adapter for Zirconia Abu. SV-PR H 1 mm
2	ZAA-SV-RP20	Zircocam Adapter for Zirconia Abu. SV-PR H 2 mm
3	ZAA-SV-RP30	Zircocam Adapter for Zirconia Abu. SV-PR H 3 mm
4	ZAA-SV-RP1510	Zircocam Adapter for Zirconia Abu. SV-PR 15°/ H 1 mm
5	ZAA-SV-RP1520	Zircocam Adapter for Zirconia Abu. SV-PR 15°/ H 2 mm
6	ZAA-SV-RP1530	Zircocam Adapter for Zirconia Abu. SV-PR 15°/ H 3 mm
7	ZAA-SV-RP2510	Zircocam Adapter for Zirconia Abu. SV-RP 25°/ H 1 mm
8	ZAA-SV-RP2520	Zircocam Adapter for Zirconia Abu. SV-RP 25°/ H 2 mm
9	ZAA-SV-RP2530	Zircocam Adapter for Zirconia Abu. SV-RP 25°/ H 3 mm
10	ZAAU-SV-RP15	Zircotec Adapter for Zirconia Abu. SV-RP H 1.5 mm Uni.
11	ZAAU-SV-RP1515	Zircotec Adapter for Zirconia Abu. SV-RP 15°/ H 1.5 mm Uni.
12	ZAA-SV-WP10	Zircocam Adapter for Zirconia Abu. SV-WP H 1 mm
13	ZAA-SV-WP20	Zircocam Adapter for Zirconia Abu. SV-WP H 2 mm
14	ZAA-SV-WP30	Zircocam Adapter for Zirconia Abu. SV-WP H 3 mm
15	ZAA-SV-WP1510	Zircocam Adapter for Zirconia Abu. SV-WP 15°/ H 1 mm
16	ZAA-SV-WP1520	Zircocam Adapter for Zirconia Abu. SV-WP 15°/ H 2 mm

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28



Zircocam Zirconia Abutments (EM)

No	REF/ Productnamen	Product Description
17	ZAA-SV-WP1530	Zircocam Adapter for Zirconia Abu. SV-WP 15°/ H 3 mm
18	ZAA-SV-WP2510	Zircocam Adapter for Zirconia Abu. SV-WP 25°/ H 1 mm
19	ZAA-SV-WP2520	Zircocam Adapter for Zirconia Abu. SV-WP 25°/ H 2 mm
20	ZAA-SV-WP2530	Zircocam Adapter for Zirconia Abu. SV-WP 25°/ H 3 mm
21	ZAAU-SV-WP15	Zircotec Adapter for Zirconia Abu. SV-WP H 1.5 mm Uni.
22	ZAAU-SV-WP1515	Zircotec Adapter for Zirconia Abu. SV-WP 15°/ H 1.5 mm Uni
23	ZAA-SV-NL10	Zircocam Adapter for Zirconia Abu. SV-NL H 1 mm
24	ZAA-SV-NL20	Zircocam Adapter for Zirconia Abu. SV-NL H 2 mm
25	ZAA-SV-NL30	Zircocam Adapter for Zirconia Abu. SV-NL H 3 mm
26	ZAA-SV-NL1510	Zircocam Adapter for Zirconia Abu. SV-NL 15°/ H 1 mm
27	ZAA-SV-NL1520	Zircocam Adapter for Zirconia Abu. SV-NL 15°/ H 2 mm
28	ZAA-SV-NL1530	Zircocam Adapter for Zirconia Abu. SV-NL 15°/ H 3 mm
29	ZAA-SV-NL2510	Zircocam Adapter for Zirconia Abu. SV-NL 25°/ H 1 mm
30	ZAA-SV-NL2520	Zircocam Adapter for Zirconia Abu. SV-NL 25°/ H 2 mm
31	ZAA-SV-NL2530	Zircocam Adapter for Zirconia Abu. SV-NL 25°/ H 3 mm
32	ZAAU-SV-NL15	Zircotec Adapter for Zirconia Abu. SV-NL H 1.5 mm Uni.
33	ZAAU-SV-NL1515	Zircotec Adapter for Zirconia Abu. SV-NL 15°/ H 1.5 mm Uni.
34	ZAA-SV-NP10	Zircocam Adapter for Zirconia Abu. SV-NP H 1 mm
35	ZAA-SV-NP20	Zircocam Adapter for Zirconia Abu. SV-NP H 2 mm
36	ZAA-SV-NP30	Zircocam Adapter for Zirconia Abu. SV-NP H 3 mm
37	ZAA-SV-NP1510	Zircocam Adapter for Zirconia Abu. SV-NP 15°/ H 1 mm
38	ZAA-SV-NP1520	Zircocam Adapter for Zirconia Abu. SV-NP 15°/ H 2 mm
39	ZAA-SV-NP2510	Zircocam Adapter for Zirconia Abu. SV-NP 15°/ H 3 mm
40	ZAA-SV-NP2520	Zircocam Adapter for Zirconia Abu. SV-NP 25°/ H 1 mm

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28



Zircocam Zirconia Abutments (EM)

No	REF/ Productnamen	Product Description
41	ZAA-SV-NP2530	Zircocam Adapter for Zirconia Abu. SV-NP 25°/ H 2 mm
42	ZAAU-SV-NP15	Zircocam Adapter for Zirconia Abu. SV-NP 25°/ H 3 mm
43	ZAAU-SV-NP1515	Zircotec Adapter for Zirconia Abu. SV-NP H 1.5 mm Uni.
44	ZAA-NA-NP15	Zircocam Adapter for Zirconia Abu. NA-NP H 1.5 mm Uni.
45	ZAA-NA-NP30	Zircocam Adapter for Zirconia Abu. NA-NP H 3 mm
46	ZAA-NA-NP45	Zircocam Adapter for Zirconia Abu. NA-NP H 4.5 mm
47	ZAA-NA-NP1515	Zircocam Adapter for Zirconia Abu. NA-NP 15°/ H 1.5 mm
48	ZAA-NA-NP1530	Zircocam Adapter for Zirconia Abu. NA-NP 15°/ H 3 mm
49	ZAA-NA-NP1545	Zircocam Adapter for Zirconia Abu. NA-NP 15°/ H 4.5 mm
50	ZAA-NA-NP2515	Zircocam Adapter for Zirconia Abu. NA-NP 25°/ H 1.5 mm
51	ZAA-NA-NP2530	Zircocam Adapter for Zirconia Abu. NA-NP 25°/ H 3 mm
52	ZAA-NA-NP2545	Zircocam Adapter for Zirconia Abu. NA-NP 25°/ H 4.5 mm
53	ZAAU-NA-NP15	Zircotec Adapter for Zirconia Abu. NA-NP H 1.5 mm Uni.
54	ZAAU-NA-NP1515	Zircotec Adapter for Zirconia Abu. NA-NP 15°/ H 1.5 mm Uni.
55	ZAA-NA-RP15	Zircocam Adapter for Zirconia Abu. NA-RP H 1.5 mm
56	ZAA-NA-RP30	Zircocam Adapter for Zirconia Abu. NA-RP H 3 mm
57	ZAA-NA-RP45	Zircocam Adapter for Zirconia Abu. NA-RP H 4.5 mm
58	ZAA-NA-RP1515	Zircocam Adapter for Zirconia Abu. NA-RP 15°/ H 1.5 mm
59	ZAA-NA-RP1530	Zircocam Adapter for Zirconia Abu. NA-RP 15°/ H 3 mm
60	ZAA-NA-RP1545	Zircocam Adapter for Zirconia Abu. NA-RP 15°/ H 4.5 mm
61	ZAA-NA-RP2515	Zircocam Adapter for Zirconia Abu. NA-RP 25°/ H 1.5 mm
62	ZAA-NA-RP2530	Zircocam Adapter for Zirconia Abu. NA-RP 25°/ H 3 mm
63	ZAA-NA-RP2545	Zircocam Adapter for Zirconia Abu. NA-RP 25°/ H 4.5 mm
64	ZAAU-NA-RP15	Zircotec Adapter for Zirconia Abu. NA-RP H 1.5 mm Uni.

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28



Zircocam Zirconia Abutments (EM)

No	REF/ Productnamen	Product Description
65	ZAAU-NA-RP1515	Zircotec Adapter for Zirconia Abu. NA-RP 15°/ H 1.5 mm Uni.
66	ZAA-TS-NP15	Zircocam Adapter for Zirconia Abu. TS-NP H 1.5 mm
67	ZAA-TS-NP30	Zircocam Adapter for Zirconia Abu. TS-NP H 3 mm
68	ZAA-TS-NP45	Zircocam Adapter for Zirconia Abu. TS-NP H 4.5 mm
69	ZAA-TS-NP1515	Zircocam Adapter for Zirconia Abu. TS-NP 15°/ H 1.5 mm
70	ZAA-TS-NP1530	Zircocam Adapter for Zirconia Abu. TS-NP 15°/ H 3 mm
71	ZAA-TS-NP1545	Zircocam Adapter for Zirconia Abu. TS-NP 15°/ H 4.5 mm
72	ZAA-TS-NP2515	Zircocam Adapter for Zirconia Abu. TS-NP 25°/ H 1.5 mm
73	ZAA-TS-NP2530	Zircocam Adapter for Zirconia Abu. TS-NP 25°/ H 3 mm
74	ZAA-TS-NP2545	Zircocam Adapter for Zirconia Abu. TS-NP 25°/ H 4.5 mm
75	ZAAU-TS-NP15	Zircotec Adapter for Zirconia Abu. TS-NP H 1.5 mm Uni.
76	ZAAU-TS-NP1515	Zircotec Adapter for Zirconia Abu. TS-NP 15°/ H 1.5 mm Uni.
77	ZAA-TS-RP15	Zircocam Adapter for Zirconia Abu. TS-RP H 1.5 mm
78	ZAA-TS-RP30	Zircocam Adapter for Zirconia Abu. TS-RP H 3 mm
79	ZAA-TS-RP45	Zircocam Adapter for Zirconia Abu. TS-RP H 4.5 mm
80	ZAA-TS-RP1515	Zircocam Adapter for Zirconia Abu. TS-RP 15°/ H 1.5 mm
81	ZAA-TS-RP1530	Zircocam Adapter for Zirconia Abu. TS-RP 15°/ H 3 mm
82	ZAA-TS-RP1545	Zircocam Adapter for Zirconia Abu. TS-RP 15°/ H 4.5 mm
83	ZAA-TS-RP2515	Zircocam Adapter for Zirconia Abu. TS-RP 25°/ H 1.5 mm
84	ZAA-TS-RP2530	Zircocam Adapter for Zirconia Abu. TS-RP 25°/ H 3 mm
85	ZAA-TS-RP2545	Zircocam Adapter for Zirconia Abu. TS-RP 25°/ H 4.5 mm
86	ZAAU-TS-RP15	Zircotec Adapter for Zirconia Abu. TS-RP H 1.5 mm Uni.
87	ZAAU-TS-RP1515	Zircotec Adapter for Zirconia Abu. TS-RP 15°/ H 1.5 mm Uni.
88	ZAA-OS-SP15	Zircocam Adapter for Zirconia Abu. OS-SP H 1.5 mm

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28



Zircocam Zirconia Abutments (EM)

No	REF/ Productnamen	Product Description
89	ZAA-OS-SP30	Zircocam Adapter for Zirconia Abu. OS-SP H 3 mm
90	ZAA-OS-SP45	Zircocam Adapter for Zirconia Abu. OS-SP H 4.5 mm
91	ZAA-OS-SP1515	Zircocam Adapter for Zirconia Abu. OS-SP 15°/ H 1.5 mm
92	ZAA-OS-SP1530	Zircocam Adapter for Zirconia Abu. OS-SP 15°/ H 3 mm
93	ZAA-OS-SP1545	Zircocam Adapter for Zirconia Abu. OS-SP 15°/ H 4.5 mm
94	ZAA-OS-SP2515	Zircocam Adapter for Zirconia Abu. OS-SP 25°/ H 1.5 mm
95	ZAA-OS-SP2530	Zircocam Adapter for Zirconia Abu. OS-SP 25°/ H 3 mm
96	ZAA-OS-SP2545	Zircocam Adapter for Zirconia Abu. OS-SP 25°/ H 4.5 mm
97	ZAAU-OS-SP15	Zircotec Adapter for Zirconia Abu. OS-SP H 1.5 mm Uni.
98	ZAAU-OS-SP1515	Zircotec Adapter for Zirconia Abu. OS-SP 15°/ H 1.5 mm Uni.
99	ZAA-OS-WP15	Zircocam Adapter for Zirconia Abu. OS-SP H 1.5 mm
101	ZAA-OS-WP30	Zircocam Adapter for Zirconia Abu. OS-SP H 3 mm
102	ZAA-OS-WP45	Zircocam Adapter for Zirconia Abu. OS-SP H 4.5 mm
103	ZAA-OS-WP1515	Zircocam Adapter for Zirconia Abu. OS-SP 15°/ H 1.5 mm
104	ZAA-OS-WP1530	Zircocam Adapter for Zirconia Abu. OS-SP 15°/ H 3 mm
105	ZAA-OS-WP1545	Zircocam Adapter for Zirconia Abu. OS-SP 15°/ H 4.5 mm
106	ZAA-OS-WP2515	Zircocam Adapter for Zirconia Abu. OS-SP 25°/ H 1.5 mm
107	ZAA-OS-WP2530	Zircocam Adapter for Zirconia Abu. OS-SP 25°/ H 3 mm
108	ZAA-OS-WP2545	Zircocam Adapter for Zirconia Abu. OS-SP 25°/ H 4.5 mm
109	ZAAU-OS-WP15	Zircotec Adapter for Zirconia Abu. OS-SP H 1.5 mm Uni.
110	ZAAU-OS-WP1515	Zircotec Adapter for Zirconia Abu. OS-SP 15°/ H 1.5 mm Uni.
111	ZAA-AR-RP15	Zircocam Adapter for Zirconia Abu. AR-RP H 1.5 mm
112	ZAA-AR-RP30	Zircocam Adapter for Zirconia Abu. AR-RP H 3 mm
113	ZAA-AR-RP45	Zircocam Adapter for Zirconia Abu. AR-RP H 4.5 mm

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28



Zircocam Zirconia Abutments (EM)

No	REF/ Productnamen	Product Description
114	ZAA-AR-RP1515	Zircocam Adapter for Zirconia Abu. AR-RP 15°/ H 1.5 mm
115	ZAA-AR-RP1530	Zircocam Adapter for Zirconia Abu. AR-RP 15°/ H 3 mm
116	ZAA-AR-RP1545	Zircocam Adapter for Zirconia Abu. AR-RP 15°/ H 4.5 mm
117	ZAA-AR-RP2515	Zircocam Adapter for Zirconia Abu. AR-RP 25°/ H 1.5 mm
118	ZAA-AR-RP2530	Zircocam Adapter for Zirconia Abu. AR-RP 25°/ H 3 mm
119	ZAA-AR-RP2545	Zircocam Adapter for Zirconia Abu. AR-RP 25°/ H 4.5 mm
120	ZAAU-AR-RP15	Zircotec Adapter for Zirconia Abu. AR-RP H 1.5 mm Uni.
121	ZAAU-AR-RP1515	Zircotec Adapter for Zirconia Abu. AR-RP 15°/ H 1.5 mm Uni.
122	ZAA-GM-RP15	Zircocam Adapter for Zirconia Abu. GM-RP H 1.5 mm
123	ZAA-GM-RP30	Zircocam Adapter for Zirconia Abu. GM-RP H 3 mm
124	ZAA-GM-RP45	Zircocam Adapter for Zirconia Abu. GM-RP H 4.5 mm
125	ZAA-GM-RP1515	Zircocam Adapter for Zirconia Abu. GM-RP 15°/ H 1.5 mm
126	ZAA-GM-RP1530	Zircocam Adapter for Zirconia Abu. GM-RP 15°/ H 3 mm
127	ZAA-GM-RP1545	Zircocam Adapter for Zirconia Abu. GM-RP 15°/ H 4.5 mm
128	ZAA-GM-RP2515	Zircocam Adapter for Zirconia Abu. GM-RP 25°/ H 1.5 mm
129	ZAA-GM-RP2530	Zircocam Adapter for Zirconia Abu. GM-RP 25°/ H 3 mm
130	ZAA-GM-RP2545	Zircocam Adapter for Zirconia Abu. GM-RP 25°/ H 4.5 mm
131	ZAAU-GM-RP15	Zircotec Adapter for Zirconia Abu. GM-RP H 1.5 mm Uni.
132	ZAAU-GM-RP1515	Zircotec Adapter for Zirconia Abu. GM-RP 15°/ H 1.5 mm Uni.
133	ZAA-CF-RC15	Zircocam Adapter for Zirconia Abu. CF-RC H 1.5 mm
134	ZAA-CF-RC30	Zircocam Adapter for Zirconia Abu. CF-RC H 3 mm
135	ZAA-CF-RC45	Zircocam Adapter for Zirconia Abu. CF-RC H 4.5 mm
136	ZAA-CF-RC1515A	Zircocam Adapter for Zirconia Abu. CF-RC 15°/ H 1.5 mm A Type
137	ZAA-CF-RC1530A	Zircocam Adapter for Zirconia Abu. CF-RC 15°/ H 3 mm A Type


Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28




Zircocam Zirconia Abutments (EM)

No	REF/ Productnamen	Product Description
138	ZAA-CF-RC1545A	Zircocam Adapter for Zirconia Abu. CF-RC 15°/ H 4.5 mm A Type
139	ZAA-CF-RC1515B	Zircocam Adapter for Zirconia Abu. CF-RC 15°/ H 1.5 mm B Type
140	ZAA-CF-RC1530B	Zircocam Adapter for Zirconia Abu. CF-RC 15°/ H 3 mm B Type
141	ZAA-CF-RC1545B	Zircocam Adapter for Zirconia Abu. CF-RC 15°/ H 4.5 mm B Type
142	ZAA-CF-RC2515A	Zircocam Adapter for Zirconia Abu. CF-RC 25°/ H 1.5 mm A Type
143	ZAA-CF-RC2530A	Zircocam Adapter for Zirconia Abu. CF-RC 25°/ H 3 mm A Type
144	ZAA-CF-RC2545A	Zircocam Adapter for Zirconia Abu. CF-RC 25°/ H 4.5 mm A Type
145	ZAA-CF-RC2515B	Zircocam Adapter for Zirconia Abu. CF-RC 25°/ H 1.5 mm B Type
146	ZAA-CF-RC2530B	Zircocam Adapter for Zirconia Abu. CF-RC 25°/ H 3 mm B Type
147	ZAA-CF-RC2545B	Zircocam Adapter for Zirconia Abu. CF-RC 25°/ H 4.5 mm B Type
148	ZAA-CF-RC15	Zircotec Adapter for Zirconia Abu. CF-RC H 1.5 mm Uni.
149	ZAAU-CF-RC1515A	Zircotec Adapter for Zirconia Abu. CF-RC 15°/ H 1.5 mm Uni.
150	ZAAU-CF-RC1515B	Zircotec Adapter for Zirconia Abu. CF-RC 15°/ H 1.5 mm B Type
151	ZAA-CF-NC15	Zircocam Adapter for Zirconia Abu. CF-RC H 1.5 mm
152	ZAA-CF-NC30	Zircocam Adapter for Zirconia Abu. CF-RC H 3 mm
153	ZAA-CF-NC45	Zircocam Adapter for Zirconia Abu. CF-RC H 4.5 mm
154	ZAA-CF-NC1515A	Zircocam Adapter for Zirconia Abu. CF-RC 15°/ H 1.5 mm A Type
155	ZAA-CF-NC1530A	Zircocam Adapter for Zirconia Abu. CF-RC 15°/ H 3 mm A Type
156	ZAA-CF-NC1545A	Zircocam Adapter for Zirconia Abu. CF-RC 15°/ H 4.5 mm A Type
157	ZAA-CF-NC1515B	Zircocam Adapter for Zirconia Abu. CF-RC 15°/ H 1.5 mm B Type
158	ZAA-CF-NC1530B	Zircocam Adapter for Zirconia Abu. CF-RC 15°/ H 3 mm B Type
159	ZAA-CF-NC1545B	Zircocam Adapter for Zirconia Abu. CF-RC 15°/ H 4.5 mm B Type
160	ZAA-CF-NC2515A	Zircocam Adapter for Zirconia Abu. CF-RC 25°/ H 1.5 mm A Type


Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

 <p style="text-align: center;">Zircocam Zirconia Abutments (EM)</p>		
No	REF/ Productnamen	Product Description
161	ZAA-CF-NC2530A	Zircocam Adapter for Zirconia Abu. CF-RC 25°/ H 3 mm A Type
162	ZAA-CF-2545A	Zircocam Adapter for Zirconia Abu. CF-RC 25°/ H 4.5 mm A Type
163	ZAA-CF-NC2515B	Zircocam Adapter for Zirconia Abu. CF-RC 25°/ H 1.5 mm B Type
164	ZAA-CF-NC2545B	Zircocam Adapter for Zirconia Abu. CF-RC 25°/ H 3 mm B Type
165	ZAAU-CF-NC15	Zircocam Adapter for Zirconia Abu. CF-RC 25°/ H 4.5 mm B Type
166	ZAAU-CF-NC15	Zircotec Adapter for Zirconia Abu. CF-RC H 1.5 mm Uni.
167	ZAAU-CF-NC1515A	Zircotec Adapter for Zirconia Abu. CF-RC 15°/ H 1.5 mm A Type
168	ZAAU-CF-NC1515B	Zircotec Adapter for Zirconia Abu. CF-RC 15°/ H 1.5 mm B Type


*Tabelle 22 Pop Up Transfer Abutments (PUT)*

 <p style="text-align: center;">Pop Up Transfer Abutment (PUT)</p>		
No	REF/ Productname	Product Description
1	PUT-1S	Pop Up Straight Titanium Transfer Abutment 1 mm Shoulder, 4 mm Cone/ Short
2	PUT-1SC	Pop Up Straight Titanium Transfer Abutment 1 mm Shoulder / Short with 0.5 mm Collar
3	PUT-2S	Pop Up Sraight Titanium Transfer Abutment 2 mm Shoulder, 4 mm Cone/ Short
4	PUT-2SC	Pop Up Straight Titanium Transfer Abutment 2 mm Shoulder/ Short with 1 mm Collar
5	PUT-3S	Pop Up Straight Titanium Transfer Abutment 3 mm Shoulder, 4 mm Cone/ Short
6	PUT-3SC	Pop Up Straight Titanium Transfer Abutment 3 mm Shoulder/ Short with 2 mm Collar
7	PUT-5S	Pop Up Transfer Abutment 5 mm Shoulder, 4 mm Cone


Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

		
Pop Up Transfer Abutment (PUT)		
No	REF/ Productname	Product Description
8	PUT-1M	Pop Up Straight Titanium Transfer Abutment 1 mm Shoulder, 6 mm Cone/ Medium
9	PUT-1MC	Pop Up Straight Titanium Transfer Abutment 0.5 mm with Shoulder, 0.5 mm Collar
10	PUT-2M	Pop Up Straight Titanium Transfer Abutment 2 mm Shoulder, 6 mm Cone/ Medium
11	PUT-2MC	Pop Up Straight Titanium Transfer Abutment 1 mm Shoulder with 1 mm Collar
12	PUT-3M	Pop Up Straight Titanium Transfer Abutment 3 mm Shoulder, 6 mm Cone/ Medium
13	PUT-3MC	Pop Up Straight Titanium Transfer Abutment 1 mm Shoulder with 2 mm Collar
14	PUT-1L	Pop Up Straight Titanium Transfer Abutment 1 mm Shoulder, 8 mm Cone/ Long
15	PUT-1LC	Pop Up Straight Titanium Transfer Abutment 0.5 mm Shoulder/ Long with 0.5 mm Collar
16	PUT-2L	Pop Up Straight Transfer Titanium Abutment 2 mm Shoulder, 8 mm Cone/ Long
17	PUT-2LC	Pop Up Straight Titanium Transfer Abutment 1 mm Shoulder/ Long with 1 mm Collar
18	PUT-3L	Pop Up Straight Titanium Transfer Abutment 3 mm Shoulder, 8 mm Cone/ Long
19	PUT-3LC	Pop Up Straight Titanium Transfer Abutment 1 mm Shoulder/ Long with 2 mm Collar
20	PUT-4L	Pop Up Straight Titanium Transfer Abutment 4 mm Shoulder, 8 mm Cone/ Long
21	PUT-5L	Pop Up Straight Titanium Transfer Abutment 5 mm Long
22	NL-PUT-1L	Pop Up Straight Titanium Transfer Abutment NL-Narrow Line 1 mm Shoulder 8 mm Cone/ Long
23	NL-PUT-1LC	Pop Up Straight Titanium Transfer Abutment NL-Narrow Line 1 mm Shoulder/ Long with 0.5 mm Collar
24	NL-PUT-1M	Pop Up Straight Titanium Transfer Abutment NL-Narrow Line 1 mm Shoulder 6 mm Cone/ Short


Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

		
Pop Up Transfer Abutment (PUT)		
No	REF/ Productname	Product Description
25	NL-PUT-1MC	Pop Up Straight Titanium Transfer Abutment NL-Narrow Line 0.5 mm Shoulder Medium with 0.5 mm Collar
26	NL-PUT-1S	Pop Up Straight Titanium Transfer Abutment NL-Narrow Line 1 mm Shoulder, 4 mm Cone/ Short
27	NL-PUT-2L	Pop Up Straight Titanium Transfer Abutment NL-Narrow Line 1 mm Shoulder, 8 mm Cone/ Long
28	NL-PUT-2LC	Pop Up Straight Titanium Transfer Abutment NL-Narrow Line 1 mm Shoulder/ Long with 1 mm Collar
29	NL-PUT-2M	Pop Up Straight Titanium Transfer Abutment NL-Narrow Line 2 mm Shoulder, 6 mm Cone/ Short
30	NL-PUT-2MC	Pop Up Straight Titanium Transfer Abutment NL-Narrow Line 1 mm Shoulder Medium with 1 mm Collar
31	NL-PUT-2S	Pop Up Straight Titanium Transfer Abutment NL-Narrow Line 2 mm Shoulder, 4 mm Cone/ Short
32	NL-PUT-3L	Pop Up Straight Titanium Transfer Abutment NL-Narrow Line 2 mm Shoulder, 8 mm Cone/ Long
33	NL-PUT-3LC	Pop Up Straight Titanium Transfer Abutment NL-Narrow Line 2 mm Shoulder/ Long with 2 mm Collar
34	NL-PUT-3M	Pop Up Straight Titanium Transfer Abutment NL-Narrow Line 3 mm Shoulder, 6 mm Cone/ Short
35	NL-PUT-3MC	Pop Up Straight Titanium Transfer Abutment NL-Narrow Line 1 mm Shoulder Medium with 2 mm Collar
36	NL-PUT-3S	Pop Up Straight Titanium Transfer Abutment NL-Narrow Line 3 mm Shoulder, 4 mm Cone/ Short
37	NL-PUT-4L	Pop Up Straight Titanium Transfer Abutment NL-Narrow Line 3 mm Shoulder, 8 mm Cone/ Long
38*	NL-PUT-2SC	Pop Up Straight Titanium Transfer Abutment 2 mm Shoulder/ Short with 1 mm Collar – Narrow Line
39*	NL-PUT-3SC	Pop Up Straight Titanium Transfer Abutment 3 mm Shoulder/ Short with 2 mm Collar - Narrow Line
40*	PUT-15-1M	Pop Up Angled Transfer Abutment 15 Degree, 1 mm Shoulder, 8 mm Cone
41*	NL-PUT-15-1M	Pop Up Angled Transfer Abutment 15 Degree, 1 mm Shoulder, 8 mm Cone, Narrow Line

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

 Pop Up Transfer Abutment (PUT)		
No	REF/ Productname	Product Description
42*	PUT-15-2M	Pop Up Angled Transfer Abutment 15 Degree, 2 mm Shoulder, 6 mm Cone
43*	NL-PUT-15-2M	Pop Up Angled Transfer Abutment 15 Degree, 2 mm Shoulder, 6 mm Cone, Narrow Line
44*	PUT-15-3M	Pop Up Angled Transfer Abutment 15 Degree, 3 mm Shoulder, 6 mm Cone
45*	NL-PUT-15-3M	Pop Up Angled Transfer Abutment 15 Degree, 3 mm Shoulder, 6 mm Cone, Narrow Line
46*	PUT-25-1M	Pop Up Angled Transfer Abutment 25 Degree, 1 mm Shoulder, 8 mm Cone
47*	NL-PUT-25-1M	Pop Up Angled Transfer Abutment 25 Degree, 1 mm Shoulder, 8 mm Cone, Narrow Line
48*	PUT-25-2M	Pop Up Angled Transfer Abutment 25 Degree, 2 mm Shoulder, 6 mm Cone
49*	NL-PUT-25-2M	Pop Up Angled Transfer Abutment 25 Degree, 2 mm Shoulder, 6 mm Cone, Narrow Line
50*	PUT-25-3M	Pop Up Angled Transfer Abutment 25 Degree, 3 mm Shoulder, 6 mm Cone
51*	NL-PUT-25-3M	Pop Up Angled Transfer Abutment 25 Degree, 3 mm Shoulder, 6 mm Cone, Narrow Line

*Table 23 Clicq-Pression Transfer Abutments (EM)*

 Clicq-Pression Transfer Abutments (EM)		
No	REF/ Productname	Product Description
1	TAP-SV-RP10S	Clicq-Pression Transfer Ti-Abutment SV-RP H 1 mm Short
2	TAP-SV-RP20S	Clicq-Pression Transfer Ti-Abutment SV-RP H 2 mm Short
3	TAP-SV-RP30S	Clicq-Pression Transfer Ti-Abutment SV-RP H 3 mm Short
4	TAP-SV-RP10M	Clicq-Pression Transfer Ti-Abutment SV-RP H 1 mm Medium
5	TAP-SV-RP20M	Clicq-Pression Transfer Ti-Abutment SV-RP H 2 mm Medium
6	TAP-SV-RP30M	Clicq-Pression Transfer Ti-Abutment SV-RP H 3 mm Medium
7	TAP-SV-RP10L	Clicq-Pression Transfer Ti-Abutment SV-RP H 1 mm Long
8	TAP-SV-RP20L	Clicq-Pression Transfer Ti-Abutment SV-RP H 2 mm Long

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28



Clicq-Pression Transfer Abutments (EM)

No	REF/ Productname	Product Description
9	TAP-SV-RP30L	Clicq-Pression Transfer Ti-Abutment SV-RP H 3 mm Long
10	TAP-SV-RP1510	Clicq-Pression Transfer Ti-Abutment SV-RP 15°/ H 1 mm
11	TAP-SV-RP1520	Clicq-Pression Transfer Ti-Abutment SV-RP 15°/ H 2 mm
12	TAP-SV-RP1530	Clicq-Pression Transfer Ti-Abutment SV-RP 15°/ H 3 mm
13	TAP-SV-RP2510	Clicq-Pression Transfer Ti-Abutment SV-RP 25°/ H 1 mm
14	TAP-SV-RP2520	Clicq-Pression Transfer Ti-Abutment SV-RP 25°/ H 2 mm
15	TAP-SV-RP2530	Clicq-Pression Transfer Ti-Abutment SV-RP 25°/ H 3 mm
16	TAP-SV-WP10S	Clicq-Pression Transfer Ti-Abutment SV-WP H 1 mm Short
17	TAP-SV-WP20S	Clicq-Pression Transfer Ti-Abutment SV-WP H 2 mm Short
18	TAP-SV-WP30S	Clicq-Pression Transfer Ti-Abutment SV-WP H 3 mm Short
19	TAP-SV-WP10M	Clicq-Pression Transfer Ti-Abutment SV-WP H 1 mm Medium
20	TAP-SV-WP20M	Clicq-Pression Transfer Ti-Abutment SV-WP H 2 mm Medium
21	TAP-SV-WP30M	Clicq-Pression Transfer Ti-Abutment SV-WP H 3 mm Medium
22	TAP-SV-WP10L	Clicq-Pression Transfer Ti-Abutment SV-WP H 1 mm Long
23	TAP-SV-WP20L	Clicq-Pression Transfer Ti-Abutment SV-WP H 2 mm Long
24	TAP-SV-WP30L	Clicq-Pression Transfer Ti-Abutment SV-WP H 3 mm Long
25	TAP-SV-WP1510	Clicq-Pression Transfer Ti-Abutment SV-WP 15°/ H 1 mm
26	TAP-SV-WP1520	Clicq-Pression Transfer Ti-Abutment SV-WP 15°/ H 2 mm
27	TAP-SV-WP1530	Clicq-Pression Transfer Ti-Abutment SV-WP 15°/ H 3 mm
28	TAP-SV-WP2510	Clicq-Pression Transfer Ti-Abutment SV-WP 25°/ H 1 mm
29	TAP-SV-WP2520	Clicq-Pression Transfer Ti-Abutment SV-WP 25°/ H 2 mm
30	TAP-SV-WP2530	Clicq-Pression Transfer Ti-Abutment SV-WP 25°/ H 3 mm
31	TAP-SV-NL10S	Clicq-Pression Transfer Ti-Abutment SV-NL H 1 mm Short
32	TAP-SV-NL20S	Clicq-Pression Transfer Ti-Abutment SV-NL H 2 mm Short
33	TAP-SV-NL30S	Clicq-Pression Transfer Ti-Abutment SV-NL H 3 mm Short
34	TAP-SV-NL10M	Clicq-Pression Transfer Ti-Abutment SV-NL H 1 mm Medium
35	TAP-SV-NL20M	Clicq-Pression Transfer Ti-Abutment SV-NL H 2 mm Medium

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28



Clicq-Pression Transfer Abutments (EM)

No	REF/ Productname	Product Description
36	TAP-SV-NL30M	Clicq-Pression Transfer Ti-Abutment SV-NL H 3 mm Medium
37	TAP-SV-NL10L	Clicq-Pression Transfer Ti-Abutment SV-NL H 1 mm Long
38	TAP-SV-NL20L	Clicq-Pression Transfer Ti-Abutment SV-NL H 2 mm Long
39	TAP-SV-NL30L	Clicq-Pression Transfer Ti-Abutment SV-NL H 3 mm Long
40	TAP-SV-NL1510	Clicq-Pression Transfer Ti-Abutment SV-NL 15°/ H 1 mm
41	TAP-SV-NL1520	Clicq-Pression Transfer Ti-Abutment SV-NL 15°/ H 2 mm
42	TAP-SV-NL1530	Clicq-Pression Transfer Ti-Abutment SV-NL 15°/ H 3 mm
43	TAP-SV-NL2510	Clicq-Pression Transfer Ti-Abutment SV-NL 25°/ H 1 mm
44	TAP-SV-NL2520	Clicq-Pression Transfer Ti-Abutment SV-NL 25°/ H 2 mm
45	TAP-SV-NL2530	Clicq-Pression Transfer Ti-Abutment SV-NL 25°/ H 3 mm
46	TAP-SV-NP10S	Clicq-Pression Transfer Ti-Abutment SV-NP H 1 mm Short
47	TAP-SV-NP20S	Clicq-Pression Transfer Ti-Abutment SV-NP H 2 mm Short
48	TAP-SV-NP30S	Clicq-Pression Transfer Ti-Abutment SV-NP H 3 mm Short
50	TAP-SV-NP10M	Clicq-Pression Transfer Ti-Abutment SV-NP H 1 mm Medium
51	TAP-SV-NP20M	Clicq-Pression Transfer Ti-Abutment SV-NP H 2 mm Medium
52	TAP-SV-NP30M	Clicq-Pression Transfer Ti-Abutment SV-NP H 3 mm Medium
53	TAP-SV-NP10L	Clicq-Pression Transfer Ti-Abutment SV-NP H 1 mm Long
54	TAP-SV-NP20L	Clicq-Pression Transfer Ti-Abutment SV-NP H 2 mm Long
55	TAP-SV-NP30L	Clicq-Pression Transfer Ti-Abutment SV-NP H 3 mm Long
56	TAP-SV-NP1510	Clicq-Pression Transfer Ti-Abutment SV-NP 15°/ H 1 mm
57	TAP-SV-NP1520	Clicq-Pression Transfer Ti-Abutment SV-NP 15°/ H 2 mm
58	TAP-SV-NP1530	Clicq-Pression Transfer Ti-Abutment SV-NP 15°/ H 3 mm
59	TAP-SV-NP2510	Clicq-Pression Transfer Ti-Abutment SV-NP 25°/ H 1 mm
60	TAP-SV-NP2520	Clicq-Pression Transfer Ti-Abutment SV-NP 25°/ H 2 mm
61	TAP-SV-NP2530	Clicq-Pression Transfer Ti-Abutment SV-NP 25°/ H 3 mm
62	TAP-NA-NP15S	Clicq-Pression Transfer Ti-Abutment NA-NP H 1.5 mm Short
63	TAP-NA-NP30S	Clicq-Pression Transfer Ti-Abutment NA-NP H 3 mm Short

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28



Clicq-Pression Transfer Abutments (EM)

No	REF/ Productname	Product Description
64	TAP-NA-NP45S	Clicq-Pression Transfer Ti-Abutment NA-NP H 4.5 mm Short
65	TAP-NA-NP15M	Clicq-Pression Transfer Ti-Abutment NA-NP H 1.5 mm Medium
66	TAP-NA-NP30M	Clicq-Pression Transfer Ti-Abutment NA-NP H 3 mm Medium
67	TAP-NA-NP45M	Clicq-Pression Transfer Ti-Abutment NA-NP H 4.5 mm Medium
68	TAP-NA-NP15L	Clicq-Pression Transfer Ti-Abutment NA-NP H 1.5 mm Long
69	TAP-NA-NP30L	Clicq-Pression Transfer Ti-Abutment NA-NP H 3 mm Long
70	TAP-NA-NP45L	Clicq-Pression Transfer Ti-Abutment NA-NP H 4.5 mm Long
71	TAP-NA-NP1515	Clicq-Pression Transfer Ti-Abutment NA-NP 15°/ H 1.5 mm
72	TAP-NA-NP1530	Clicq-Pression Transfer Ti-Abutment NA-NP 15°/ H 3 mm
73	TAP-NA-NP2515	Clicq-Pression Transfer Ti-Abutment NA-NP 25°/ H 1.5 mm
74	TAP-NA-NP2530	Clicq-Pression Transfer Ti-Abutment NA-NP 25°/ H 3 mm
75	TAP-NA-RP15S	Clicq-Pression Transfer Ti-Abutment NA-RP H 1.5 mm Short
76	TAP-NA-RP30S	Clicq-Pression Transfer Ti-Abutment NA-RP H 3 mm Short
77	TAP-NA-RP45S	Clicq-Pression Transfer Ti-Abutment NA-RP H 4.5 mm Short
78	TAP-NA-RP15M	Clicq-Pression Transfer Ti-Abutment NA-RP H 1.5 mm Medium
79	TAP-NA-RP30M	Clicq-Pression Transfer Ti-Abutment NA-RP H 3 mm Medium
80	TAP-NA-RP45M	Clicq-Pression Transfer Ti-Abutment NA-RP H 4.5 mm Medium
81	TAP-NA-RP15L	Clicq-Pression Transfer Ti-Abutment NA-RP H 1.5 mm Long
82	TAP-NA-RP30L	Clicq-Pression Transfer Ti-Abutment NA-RP H 3 mm Long
83	TAP-NA-RP45L	Clicq-Pression Transfer Ti-Abutment NA-RP H 4.5 mm Long
84	TAP-NA-RP1515	Clicq-Pression Transfer Ti-Abutment NA-RP 15°/ H 1.5 mm
85	TAP-NA-RP1530	Clicq-Pression Transfer Ti-Abutment NA-RP 15°/ H 3 mm
86	TAP-NA-RP2515	Clicq-Pression Transfer Ti-Abutment NA-RP 25°/ H 1.5 mm
87	TAP-NA-RP2530	Clicq-Pression Transfer Ti-Abutment NA-RP 25°/ H 3 mm
88	TAP-TS-NP15S	Clicq-Pression Transfer Ti-Abutment TS-NP H 1.5 mm Short
89	TAP-TS-NP30S	Clicq-Pression Transfer Ti-Abutment TS-NP H 3 mm Short
90	TAP-TS-NP45S	Clicq-Pression Transfer Ti-Abutment TS-NP H 4.5 mm Short

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28



Clicq-Pression Transfer Abutments (EM)

No	REF/ Productname	Product Description
91	TAP-TS-NP15M	Clicq-Pression Transfer Ti-Abutment TS-NP H 1.5 mm Medium
92	TAP-TS-NP30M	Clicq-Pression Transfer Ti-Abutment TS-NP H 3 mm Medium
93	TAP-TS-NP45M	Clicq-Pression Transfer Ti-Abutment TS-NP H 4.5 mm Medium
94	TAP-TS-NP15L	Clicq-Pression Transfer Ti-Abutment TS-NP H 1.5 mm Long
95	TAP-TS-NP30L	Clicq-Pression Transfer Ti-Abutment TS-NP H 3 mm Long
96	TAP-TS-NP45L	Clicq-Pression Transfer Ti-Abutment TS-NP H 4.5 mm Long
97	TAP-TS-NP1515	Clicq-Pression Transfer Ti-Abutment TS-NP 15°/ H 1.5 mm
98	TAP-TS-NP1530	Clicq-Pression Transfer Ti-Abutment TS-NP 15°/ H 3 mm
99	TAP-TS-NP2515	Clicq-Pression Transfer Ti-Abutment TS-NP 25°/ H 1.5 mm
100	TAP-TS-NP2530	Clicq-Pression Transfer Ti-Abutment TS-NP 25°/ H 3 mm
101	TAP-TS-RP15S	Clicq-Pression Transfer Ti-Abutment TS-RP H 1.5 mm Short
102	TAP-TS-RP30S	Clicq-Pression Transfer Ti-Abutment TS-RP H 3 mm Short
103	TAP-TS-RP45S	Clicq-Pression Transfer Ti-Abutment TS-RP H 4.5 mm Short
104	TAP-TS-RP15M	Clicq-Pression Transfer Ti-Abutment TS-RP H 1.5 mm Medium
105	TAP-TS-RP30M	Clicq-Pression Transfer Ti-Abutment TS-RP H 3 mm Medium
106	TAP-TS-RP45M	Clicq-Pression Transfer Ti-Abutment TS-RP H 4.5 mm Medium
107	TAP-TS-RP15L	Clicq-Pression Transfer Ti-Abutment TS-RP H 1.5 mm Long
108	TAP-TS-RP30L	Clicq-Pression Transfer Ti-Abutment TS-RP H 3 mm Long
109	TAP-TS-RP45L	Clicq-Pression Transfer Ti-Abutment TS-RP H 4.5 mm Long
110	TAP-TS-NP1515	Clicq-Pression Transfer Ti-Abutment TS-NP 15°/ H 1.5 mm
111	TAP-TS-NP1530	Clicq-Pression Transfer Ti-Abutment TS-NP 15°/ H 3 mm
112	TAP-TS-NP2515	Clicq-Pression Transfer Ti-Abutment TS-NP 25°/ H 1.5 mm
113	TAP-TS-NP2530	Clicq-Pression Transfer Ti-Abutment TS-NP 25°/ H 3 mm
114	TAP-OS-SP15S	Clicq-Pression Transfer Ti-Abutment OS-SP H 1.5 mm Short
115	TAP-OS-SP30S	Clicq-Pression Transfer Ti-Abutment OS-SP H 3 mm Short
116	TAP-OS-SP45S	Clicq-Pression Transfer Ti-Abutment OS-SP H 4.5 mm Short
117	TAP-OS-SP15M	Clicq-Pression Transfer Ti-Abutment OS-SP H 1.5 mm Medium

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28



Clicq-Pression Transfer Abutments (EM)

No	REF/ Productname	Product Description
118	TAP-OS-SP30M	Clicq-Pression Transfer Ti-Abutment OS-SP H 3 mm Medium
119	TAP-OS-SP45M	Clicq-Pression Transfer Ti-Abutment OS-SP H 4.5 mm Medium
120	TAP-OS-SP15L	Clicq-Pression Transfer Ti-Abutment OS-SP H 1.5 mm Long
121	TAP-OS-SP30L	Clicq-Pression Transfer Ti-Abutment OS-SP H 3 mm Long
122	TAP-OS-SP45L	Clicq-Pression Transfer Ti-Abutment OS-SP H 4.5 mm Long
123	TAP-OS-SP1515	Clicq-Pression Transfer Ti-Abutment OS-SP 15°/ H 1.5 mm
124	TAP-OS-SP1530	Clicq-Pression Transfer Ti-Abutment OS-SP 15°/ H 3 mm
125	TAP-OS-SP2515	Clicq-Pression Transfer Ti-Abutment OS-SP 25°/ H 1.5 mm
126	TAP-OS-SP2530	Clicq-Pression Transfer Ti-Abutment OS-SP 25°/ H 3 mm
127	TAP-OS-WP15S	Clicq-Pression Transfer Ti-Abutment OS-WP H 1.5 mm Short
128	TAP-OS-WP30S	Clicq-Pression Transfer Ti-Abutment OS-WP H 3 mm Short
129	TAP-OS-WP45S	Clicq-Pression Transfer Ti-Abutment OS-WP H 4.5 mm Short
130	TAP-OS-WP15M	Clicq-Pression Transfer Ti-Abutment OS-WP H 1.5 mm Medium
131	TAP-OS-WP30M	Clicq-Pression Transfer Ti-Abutment OS-WP H 3 mm Medium
132	TAP-OS-WP45M	Clicq-Pression Transfer Ti-Abutment OS-WP H 4.5 mm Medium
133	TAP-OS-WP15L	Clicq-Pression Transfer Ti-Abutment OS-WP H 1.5 mm Long
134	TAP-OS-WP30L	Clicq-Pression Transfer Ti-Abutment OS-WP H 3 mm Long
135	TAP-OS-WP45L	Clicq-Pression Transfer Ti-Abutment OS-WP H 4.5 mm Long
136	TAP-OS-WP1515	Clicq-Pression Transfer Ti-Abutment OS-WP 15°/ H 1.5 mm
137	TAP-OS-WP1530	Clicq-Pression Transfer Ti-Abutment OS-WP 15°/ H 3 mm
138	TAP-OS-WP2515	Clicq-Pression Transfer Ti-Abutment OS-WP 25°/ H 1.5 mm
139	TAP-OS-WP2530	Clicq-Pression Transfer Ti-Abutment OS-WP 25°/ H 3 mm
140	TAP-AR-RP15S	Clicq-Pression Transfer Ti-Abutment AR-RP H 1.5 mm Short
141	TAP-AR-RP30S	Clicq-Pression Transfer Ti-Abutment AR-RP H 3 mm Short
142	TAP-AR-RP45S	Clicq-Pression Transfer Ti-Abutment AR-RP H 4.5 mm Short
143	TAP-AR-RP15M	Clicq-Pression Transfer Ti-Abutment AR-RP H 1.5 mm Medium
144	TAP-AR-RP30M	Clicq-Pression Transfer Ti-Abutment AR-RP H 3 mm Medium

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28



Clicq-Pression Transfer Abutments (EM)

No	REF/ Productname	Product Description
145	TAP-AR-RP45M	Clicq-Pression Transfer Ti-Abutment AR-RP H 4.5 mm Medium
146	TAP-AR-RP15L	Clicq-Pression Transfer Ti-Abutment AR-RP H 1.5 mm Long
147	TAP-AR-RP30L	Clicq-Pression Transfer Ti-Abutment AR-RP H 3 mm Long
148	TAP-AR-RP45L	Clicq-Pression Transfer Ti-Abutment AR-RP H 4.5 mm Long
149	TAP-AR-RP1515	Clicq-Pression Transfer Ti-Abutment AR-RP 15°/ H 1.5 mm
150	TAP-AR-RP1530	Clicq-Pression Transfer Ti-Abutment AR-RP 15°/ H 3 mm
151	TAP-AR-RP2515	Clicq-Pression Transfer Ti-Abutment AR-RP 25°/ H 1.5 mm
152	TAP-AR-RP2530	Clicq-Pression Transfer Ti-Abutment AR-RP 25°/ H 3 mm
153	TAP-GM-RP15S	Clicq-Pression Transfer Ti-Abutment GM-RP H 1.5 mm Short
154	TAP-GM-RP30S	Clicq-Pression Transfer Ti-Abutment GM-RP H 3 mm Short
155	TAP-GM-RP45S	Clicq-Pression Transfer Ti-Abutment GM-RP H 4.5 mm Short
156	TAP-GM-RP15M	Clicq-Pression Transfer Ti-Abutment GM-RP H 1.5 mm Medium
157	TAP-GM-RP30M	Clicq-Pression Transfer Ti-Abutment GM-RP H 3 mm Medium
158	TAP-GM-RP45M	Clicq-Pression Transfer Ti-Abutment GM-RP H 4.5 mm Medium
159	TAP-GM-RP15L	Clicq-Pression Transfer Ti-Abutment GM-RP H 1.5 mm Long
160	TAP-GM-RP30L	Clicq-Pression Transfer Ti-Abutment GM-RP H 3 mm Long
161	TAP-GM-RP45L	Clicq-Pression Transfer Ti-Abutment GM-RP H 4.5 mm Long
162	TAP-GM-RP1515	Clicq-Pression Transfer Ti-Abutment GM-RP 15°/ H 1.5 mm
163	TAP-GM-RP1530	Clicq-Pression Transfer Ti-Abutment GM-RP 15°/ H 3 mm
164	TAP-GM-RP2515	Clicq-Pression Transfer Ti-Abutment GM-RP 25°/ H 1.5 mm
165	TAP-GM-RP2530	Clicq-Pression Transfer Ti-Abutment GM-RP 25°/ H 3 mm
166	TAP-CF-RC15S	Clicq-Pression Transfer Ti-Abutment CF-RC H 1.5 mm Short
167	TAP-CF-RC30S	Clicq-Pression Transfer Ti-Abutment CF-RC H 3 mm Short
168	TAP-CF-RC45S	Clicq-Pression Transfer Ti-Abutment CF-RC H 4.5 mm Short
169	TAP-CF-RC15M	Clicq-Pression Transfer Ti-Abutment CF-RC H 1.5 mm Medium
170	TAP-CF-RC30M	Clicq-Pression Transfer Ti-Abutment CF-RC H 3 mm Medium
171	TAP-CF-RC45M	Clicq-Pression Transfer Ti-Abutment CF-RC H 4.5 mm Medium


Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28



Clicq-Pression Transfer Abutments (EM)

No	REF/ Productname	Product Description
172	TAP-CF-RC15L	Clicq-Pression Transfer Ti-Abutment CF-RC H 1.5 mm Long
173	TAP-CF-RC30L	Clicq-Pression Transfer Ti-Abutment CF-RC H 3 mm Long
174	TAP-CF-RC45L	Clicq-Pression Transfer Ti-Abutment CF-RC H 4.5 mm Long
175	TAP-CF-RC1515A	Clicq-Pression Transfer Ti-Abutment CF-RC 15°/ H 1.5 mm Type A
176	TAP-CF-RC1530A	Clicq-Pression Transfer Ti-Abutment CF-RC 15°/ H 3 mm Type A
177	TAP-CF-RC1515B	Clicq-Pression Transfer Ti-Abutment CF-RC 15°/ H 1.5 mm Type B
178	TAP-CF-RC1530B	Clicq-Pression Transfer Ti-Abutment CF-RC 15°/ H 3 mm Type B
179	TAP-CF-RC2515A	Clicq-Pression Transfer Ti-Abutment CF-RC 25°/ H 1.5 mm Type A
180	TAP-CF-RC2530A	Clicq-Pression Transfer Ti-Abutment CF-RC 25°/ H 3 mm Type A
181	TAP-CF-RC2515B	Clicq-Pression Transfer Ti-Abutment CF-RC 25°/ H 1.5 mm Type B
182	TAP-CF-RC2530B	Clicq-Pression Transfer Ti-Abutment CF-RC 25°/ H 3 mm Type B
183	TAP-CF-NC15S	Clicq-Pression Transfer Ti-Abutment CF-NC H 1.5 mm Short
184	TAP-CF-NC30S	Clicq-Pression Transfer Ti-Abutment CF-NC H 3 mm Short
185	TAP-CF-NC45S	Clicq-Pression Transfer Ti-Abutment CF-NC H 4.5 mm Short
186	TAP-CF-NC15M	Clicq-Pression Transfer Ti-Abutment CF-NC H 1.5 mm Medium
187	TAP-CF-NC30M	Clicq-Pression Transfer Ti-Abutment CF-NC H 3 mm Medium
188	TAP-CF-NC45M	Clicq-Pression Transfer Ti-Abutment CF-NC H 4.5 mm Medium
189	TAP-CF-NC15L	Clicq-Pression Transfer Ti-Abutment CF-NC H 1.5 mm Long
190	TAP-CF-NC30L	Clicq-Pression Transfer Ti-Abutment CF-NC H 3 mm Long
191	TAP-CF-NC45L	Clicq-Pression Transfer Ti-Abutment CF-NC H 4.5 mm Long
192	TAP-CF-NC1515A	Clicq-Pression Transfer Ti-Abutment CF-NC 15°/ H 1.5 mm Type A

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

 Clicq-Pression Transfer Abutments (EM)		
No	REF/ Productname	Product Description
193	TAP-CF-NC1530A	Clicq-Pression Transfer Ti-Abutment CF-NC 15°/ H 3 mm Type A
194	TAP-CF-NC1515B	Clicq-Pression Transfer Ti-Abutment CF-NC 15°/ H 1.5 mm Type B
195	TAP-CF-NC1530B	Clicq-Pression Transfer Ti-Abutment CF-NC 15°/ H 3 mm Type B
196	TAP-CF-NC2515A	Clicq-Pression Transfer Ti-Abutment CF-NC 25°/ H 1.5 mm Type A
197	TAP-CF-NC2530A	Clicq-Pression Transfer Ti-Abutment CF-NC 25°/ H 3 mm Type A
198	TAP-CF-NC2515B	Clicq-Pression Transfer Ti-Abutment CF-NC 25°/ H 1.5 mm Type B
199	TAP-CF-NC2530B	Clicq-Pression Transfer Ti-Abutment CF-NC 25°/ H 3 mm Type B

*Tabelle 24 Clicq-Pression Kits (EM)*


Clicq-Pression Convencience Kits (EM)		
No	REF/ Productname	Product Description
1	TAP-SV-RPSK	Clicq-Pression Transfer Ti-Abutment SV-RP Starter KIT
2	TAP-SV-RPPK	Clicq-Pression Transfer Ti-Abutment SV-RP Professional KIT
3	TAP-SV-WPSK	Clicq-Pression Transfer Ti-Abutment SV-WP Starter KIT
4	TAP-SV-WPPK	Clicq-Pression Transfer Ti-Abutment SV-WP Professional KIT
5	TAP-SV-NLSK	Clicq-Pression Transfer Ti-Abutment SV-NL Starter KIT
6	TAP-SV-NLPK	Clicq-Pression Transfer Ti-Abutment SV-NL Professional KIT
7	TAP-SV-NPSK	Clicq-Pression Transfer Ti-Abutment SV-NP Starter KIT
8	TAP-SV-NPPK	Clicq-Pression Transfer Ti-Abutment SV-NP Professional KIT
9	TAP-NA-NPSK	Clicq-Pression Transfer Ti-Abutment NA-NP Starter KIT

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

Clicq-Pression Convencence Kits (EM)		
No	REF/ Productname	Product Description
10	TAP-NA-NPPK	Clicq-Pression Transfer Ti-Abutment NA-NP Professional KIT
11	TAP-NA-RPSK	Clicq-Pression Transfer Ti-Abutment NA-RP Starter KIT
12	TAP-NA-RPPK	Clicq-Pression Transfer Ti-Abutment NA-RP Professional KIT
13	TAP-TS-NPSK	Clicq-Pression Transfer Ti-Abutment TS-NP/ Mini Starter KIT
14	TAP-TS-NPPK	Clicq-Pression Transfer Ti-Abutment TS-NP/ Mini Professional KIT
15	TAP-TS-RPSK	Clicq-Pression Transfer Ti-Abutment TS-RP Starter KIT
16	TAP-TS-RPPK	Clicq-Pression Transfer Ti-Abutment TS-RP Professional KIT
17	TAP-OS-SPSK	Clicq-Pression Transfer Ti-Abutment OS-SP/ Mini Starter KIT
18	TAP-OS-SPPK	Clicq-Pression Transfer Ti-Abutment OS-SP Professional KIT
19	TAP-OS-WPSK	Clicq-Pression Transfer Ti-Abutment OS-WP/ Mini Starter KIT
20	TAP-OS-WPPK	Clicq-Pression Transfer Ti-Abutment OS-WP Professional KIT
21	TAP-AR-RPSK	Clicq-Pression Transfer Ti-Abutment AR-RP Starter KIT
22	TAP-AR-RPPK	Clicq-Pression Transfer Ti-Abutment AR-RP Professional KIT
23	TAP-GM-RPSK	Clicq-Pression Transfer Ti-Abutment GM-RP/ Mini Starter KIT
24	TAP-GM-RPPK	Clicq-Pression Transfer Ti-Abutment GM-RP Professional KIT
25	TAP-CF-RCASK	Clicq-Pression Transfer Ti-Abutment CF-RC Starter KIT - A Type
26	TAP-CF-RCBSK	Clicq-Pression Transfer Ti-Abutment CF-RC Professional KIT - B Type
27	TAP-CF-RCAPK	Clicq-Pression Transfer Ti-Abutment CF-RC Starter KIT - A Type
28	TAP-CF-RCBPK	Clicq-Pression Transfer Ti-Abutment CF-RC Professional KIT - B Type
29	TAP-CF-NCASK	Clicq-Pression Transfer Ti-Abutment CF-NC Starter KIT - A Type
30	TAP-CF-NCBSK	Clicq-Pression Transfer Ti-Abutment CF-NC Professional KIT - B Type
31	TAP-CF-NCAPK	Clicq-Pression Transfer Ti-Abutment CF-RC Starter KIT - A Type
32	TAP-CF-NCBPK	Clicq-Pression Transfer Ti-Abutment CF-RC Professional KIT - B Type


Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

*Tabelle 25 Ball Attachments*


		
Ball Attachments & Caps		
No	REF/ Productname	Product Description
1	BA-1	Ball Attachment 1 mm Kit
2	BA-2	Ball Attachment 2 mm Kit
3	BA-3	Ball Attachment 3 mm Kit
4	BA-4	Ball Attachment 4 mm Kit
5	BA-5	Ball Attachment 5 mm Kit
6	BA-6	Ball Attachment 6 mm Kit
7	BA-7	Ball Attachment 7 mm Kit
8	SCB-B	Silicone Cap for Ball Attachment – Black (4pcs.) Lab Use Only
9	SCB-G	Silicone Cap for Ball Attachment – Green (4pcs.) Extremely Elastic
10	SCB-P	Silicone Cap for Ball Attachment – Pink (4pcs.) Elastic
11	SCB-T	Silicone Cap for Ball Attachment – Transparent (4pcs.)
12	SCB-Y	Silicone Cap for Ball Attachment – Yellow (4pcs.) Very Elastic
13	BA-SP	Separators O_Rings for Ball Attachment and Clicq Overdenture FOR LAB
14	MCB	Metal Cap for Ball Attachment
15	NL-BA-1	Ball Attachment & Cap-1 (Kit f 3pcs) (Narrow Line) with SCB, BA SP, MCB
16	NL-BA-2	Ball Attachment & Cap-2 (Kit f 3pcs) (Narrow Line) with SCB, BA SP, MCB
17	NL-BA-3	Ball Attachment & Cap-3 (Kit f 3pcs) (Narrow Line) with SCB, BA SP, MCB
18	NL-BA-4	Ball Attachment & Cap-4 (Kit f 3pcs) (Narrow Line) with SCB, BA SP, MCB

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28


*Tabelle 26 Ball Attachments (EM)*

		
Ball Attachments & Caps (EM)		
No	REF/ Productname	Product Description
1	BAT-SV-RP10	Ball Attachment SV-RP H 1 mm
2	BAT-SV-RP20	Ball Attachment SV-RP H 2 mm
3	BAT-SV-RP30	Ball Attachment SV-RP H 3 mm
4	BAT-SV-RP40	Ball Attachment SV-RP H 4 mm
5	BAT-SV-RP50	Ball Attachment SV-RP H 5 mm
6	BAT-SV-RP60	Ball Attachment SV-RP H 6 mm
7	BAT-SV-RP10T	Ball Attachment SV-RP H 1 mm Incl. RCT/ Basic
8	BAT-SV-RP20T	Ball Attachment SV-RP H 2 mm Incl. RCT/ Basic
9	BAT-SV-RP30T	Ball Attachment SV-RP H 3 mm Incl. RCT/ Basic
10	BAT-SV-RP40T	Ball Attachment SV-RP H 4 mm Incl. RCT/ Basic
11	BAT-SV-RP50T	Ball Attachment SV-RP H 5 mm Incl. RCT/ Basic
12	BAT-SV-RP60T	Ball Attachment SV-RP H 6 mm Incl. RCT/ Basic
13	BAT-SV-RP10P	Ball Attachment SV-RP H 1 mm Incl. RCT/ Basic
14	BAT-SV-RP20P	Ball Attachment SV-RP H 2 mm Incl. RCT/ Basic
15	BAT-SV-RP30P	Ball Attachment SV-RP H 3 mm Incl. RCT/ Basic
16	BAT-SV-RP40P	Ball Attachment SV-RP H 4 mm Incl. RCT/ Basic
17	BAT-SV-RP50P	Ball Attachment SV-RP H 5 mm Incl. RCT/ Basic
18	BAT-SV-RP60P	Ball Attachment SV-RP H 6 mm Incl. RCT/ Basic
19	BAT-SV-NL10	Ball Attachment SV-NL H 1 mm
20	BAT-SV-NL20	Ball Attachment SV-NL H 2 mm
21	BAT-SV-NL30	Ball Attachment SV-NL H 3 mm
22	BAT-SV-NL40	Ball Attachment SV-NL H 4 mm
23	BAT-SV-NL50	Ball Attachment SV-NL H 5 mm
24	BAT-SV-NL60	Ball Attachment SV-NL H 6 mm
25	BAT-SV-NL10T	Ball Attachment SV-NL H 1 mm Incl. RCT/ Basic
26	BAT-SV-NL20T	Ball Attachment SV-NL H 2 mm Incl. RCT/ Basic

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

		
Ball Attachments & Caps (EM)		
No	REF/ Productname	Product Description
27	BAT-SV-NL30T	Ball Attachment SV-NL H 3 mm Incl. RCT/ Basic
28	BAT-SV-NL40T	Ball Attachment SV-NL H 4 mm Incl. RCT/ Basic
29	BAT-SV-NL50T	Ball Attachment SV-NL H 5 mm Incl. RCT/ Basic
30	BAT-SV-NL60T	Ball Attachment SV-NL H 6 mm Incl. RCT/ Basic
31	BAT-SV-NL10P	Ball Attachment SV-NL H 1 mm Incl. RCT/ Basic
32	BAT-SV-NL20P	Ball Attachment SV-NL H 2 mm Incl. RCT/ Basic
33	BAT-SV-NL30P	Ball Attachment SV-NL H 3 mm Incl. RCT/ Basic
34	BAT-SV-NL40P	Ball Attachment SV-NL H 4 mm Incl. RCT/ Basic
35	BAT-SV-NL50P	Ball Attachment SV-NL H 5 mm Incl. RCT/ Basic
36	BAT-SV-NL60P	Ball Attachment SV-NL H 6 mm Incl. RCT/ Basic
37	BAT-SV-NP10	Ball Attachment SV-NP H 1 mm
38	BAT-SV-NP20	Ball Attachment SV-NP H 2 mm
39	BAT-SV-NP30	Ball Attachment SV-NP H 3 mm
40	BAT-SV-NP40	Ball Attachment SV-NP H 4 mm
41	BAT-SV-NP50	Ball Attachment SV-NP H 5 mm
42	BAT-SV-NP60	Ball Attachment SV-NP H 6 mm
43	BAT-SV-NL10T	Ball Attachment SV-NP H 1 mm Incl. RCT/ Basic
44	BAT-SV-NL20T	Ball Attachment SV-NP H 2 mm Incl. RCT/ Basic
45	BAT-SV-NL30T	Ball Attachment SV-NP H 3 mm Incl. RCT/ Basic
46	BAT-SV-NL40T	Ball Attachment SV-NP H 4 mm Incl. RCT/ Basic
47	BAT-SV-NL50T	Ball Attachment SV-NP H 5 mm Incl. RCT/ Basic
48	BAT-SV-NL60T	Ball Attachment SV-NP H 6 mm Incl. RCT/ Basic
49	BAT-SV-NL10P	Ball Attachment SV-NP H 1 mm Incl. RCT/ Basic
50	BAT-SV-NL20P	Ball Attachment SV-NP H 2 mm Incl. RCT/ Basic
51	BAT-SV-NL30P	Ball Attachment SV-NP H 3 mm Incl. RCT/ Basic
52	BAT-SV-NL40P	Ball Attachment SV-NP H 4 mm Incl. RCT/ Basic
53	BAT-SV-NL50P	Ball Attachment SV-NP H 5 mm Incl. RCT/ Basic

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

		
Ball Attachments & Caps (EM)		
No	REF/ Productname	Product Description
54	BAT-SV-NL60P	Ball Attachment SV-NP H 6 mm Incl. RCT/ Basic
55	BAT-MH	Ball Attachment Metal Housing 2 Pcs
56	LOCN-MH	Metal Housing 2 Pcs
57	LOCN-PD	Protective Disc 4 Pcs
58	BAT-RCB	Retentive Cap 4 Pcs Black Lab use only
59	BAT-RCG	Retentive Cap 4 Pcs Green Ex-Soft (0.6 kg)
60	BAT-RCY	Retentive Cap 4 Pcs Yellow Soft (1.2 kg)
61	BAT-RCP	Retentive Cap 4 Pcs Violet Standard (1.8 kg)
62	BAT-RCT	Retentive Cap 4 Pcs Transparent Strong (2.7 Kg)

*Tabelle 27 Ball Attachments Kits (EM)*


Ball Attachments Convenience Kits (EM)		
No	REF/ Productname	Product Description
1	BAT-SV-RP10T	Ball Attachment SV-RP H 1 mm Incl. RCT/ Basic KIT
2	BAT-SV-RP20T	Ball Attachment SV-RP H 2 mm Incl. RCT/ Basic KIT
3	BAT-SV-RP30T	Ball Attachment SV-RP H 3 mm Incl. RCT/ Basic KIT
4	BAT-SV-RP40T	Ball Attachment SV-RP H 4 mm Incl. RCT/ Basic KIT
5	BAT-SV-RP50T	Ball Attachment SV-RP H 5 mm Incl. RCT/ Basic KIT
6	BAT-SV-RP60T	Ball Attachment SV-RP H 6 mm Incl. RCT/ Basic KIT
7	BAT-SV-RP10P	Ball Attachment SV-RP H 1 mm Incl. RCP/ Basic KIT
8	BAT-SV-RP20P	Ball Attachment SV-RP H 2 mm Incl. RCP/ Basic KIT
9	BAT-SV-RP30P	Ball Attachment SV-RP H 3 mm Incl. RCP/ Basic KIT
10	BAT-SV-RP40P	Ball Attachment SV-RP H 4 mm Incl. RCP/ Basic KIT
11	BAT-SV-RP50P	Ball Attachment SV-RP H 5 mm Incl. RCP/ Basic KIT
12	BAT-SV-RP60P	Ball Attachment SV-RP H 6 mm Incl. RCP/ Basic KIT

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28


Ball Attachments Convenience Kits (EM)		
No	REF/ Productname	Product Description
13	BAT-SV-NL10T	Ball Attachment SV-NL H 1 mm Incl. RCT/ Basic KIT
14	BAT-SV-NL20T	Ball Attachment SV-NL H 2 mm Incl. RCT/ Basic KIT
15	BAT-SV-NL30T	Ball Attachment SV-NL H 3 mm Incl. RCT/ Basic KIT
16	BAT-SV-NL40T	Ball Attachment SV-NL H 4 mm Incl. RCT/ Basic KIT
17	BAT-SV-NL50T	Ball Attachment SV-NL H 5 mm Incl. RCT/ Basic KIT
18	BAT-SV-NL60T	Ball Attachment SV-NL H 6 mm Incl. RCT/ Basic KIT
19	BAT-SV-NL10P	Ball Attachment SV-NL H 1 mm Incl. RCP/ Basic KIT
20	BAT-SV-NL20P	Ball Attachment SV-NL H 2 mm Incl. RCP/ Basic KIT
21	BAT-SV-NL30P	Ball Attachment SV-NL H 3 mm Incl. RCP/ Basic KIT
22	BAT-SV-NL40P	Ball Attachment SV-NL H 4 mm Incl. RCP/ Basic KIT
23	BAT-SV-NL50P	Ball Attachment SV-NL H 5 mm Incl. RCP/ Basic KIT
24	BAT-SV-NL60P	Ball Attachment SV-NL H 6 mm Incl. RCP/ Basic KIT
25	BAT-SV-NP10T	Ball Attachment SV-NP H 1 mm Incl. RCT/ Basic KIT
26	BAT-SV-NP20T	Ball Attachment SV-NP H 2 mm Incl. RCT/ Basic KIT
27	BAT-SV-NP30T	Ball Attachment SV-NP H 3 mm Incl. RCT/ Basic KIT
28	BAT-SV-NP40T	Ball Attachment SV-NP H 4 mm Incl. RCT/ Basic KIT
29	BAT-SV-NP50T	Ball Attachment SV-NP H 5 mm Incl. RCT/ Basic KIT
30	BAT-SV-NP60T	Ball Attachment SV-NP H 6 mm Incl. RCT/ Basic KIT
31	BAT-SV-NP10P	Ball Attachment SV-NP H 1 mm Incl. RCP/ Basic KIT
32	BAT-SV-NP20P	Ball Attachment SV-NP H 2 mm Incl. RCP/ Basic KIT
33	BAT-SV-NP30P	Ball Attachment SV-NP H 3 mm Incl. RCP/ Basic KIT
34	BAT-SV-NP40P	Ball Attachment SV-NP H 4 mm Incl. RCP/ Basic KIT
35	BAT-SV-NP50P	Ball Attachment SV-NP H 5 mm Incl. RCP/ Basic KIT
36	BAT-SV-NP60P	Ball Attachment SV-NP H 6 mm Incl. RCP/ Basic KIT

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28


Tabelle 28 Clicq Overdenture

		
Clicq Overdenture & Accessories		
No	REF/ Productname	Product Description
1	COD-0.5	Clicq Overdenture Kit 0.5 mm Shoulder Kit
2	COD-1	Clicq Overdenture Kit 1 mm Shoulder Kit
3	COD-2	Clicq Overdenture Kit 2 mm Shoulder Kit
4	COD-3	Clicq Overdenture Kit 3 mm Shoulder Kit
5	COD-4	Clicq Overdenture Kit 4 mm Shoulder Kit
6	COD-5	Clicq Overdenture Kit 5 mm Shoulder Kit
7	COD-6	Clicq Overdenture Kit 6 mm Shoulder Kit
8	COD-7	Clicq Overdenture Kit 7 mm Shoulder Kit
9	SCL-V	Silicone Cap for Clicq Overdenture Violet (4 pcs) Strong (2.7 Kg)
10	SCL-T	Silicone Cap for Clicq Overdenture (4 pcs) Standard (1.8Kg)
11	SCL-P	Silicone Cap for Clicq Overdenture Pink (4pcs) Soft (1.2 Kg)
12	SCL-Y	Silicone Cap for Clicq Overdenture Yellow (4pcs) Extra Soft (0.6 Kg)
13	SCL-B	Silicone Cap for Clicq Overdenture Black (4pcs) Lab Use Only
14	SCL-G	Silicone Cap for Clicq Overdenture Grey (4pcs) Hard
15	MC-COD	Metal Cap for Clicq Overdenture
16	NL-COD-0.5	Clicq Overdenture 0.5 mm KIT (Narrow Line)
17	NL-COD-1	Clicq Overdenture 1 mm KIT (Narrow Line)
18	NL-COD-2	Clicq Overdenture 2 mm KIT (Narrow Line)
19	NL-COD-3	Clicq Overdenture 3 mm KIT (Narrow Line)
20	NL-COD-4	Clicq Overdenture 4 mm KIT (Narrow Line)
21	NL-COD-5	Clicq Overdenture 5 mm KIT (Narrow Line)
22	NL-COD-6	Clicq Overdenture 6 mm KIT (Narrow Line)
Clicq Overdenture Straight		


Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

		
Clicq Overdenture & Accessories		
No	REF/ Productname	Product Description
23*	COD-0.5P	Clicq Overdenture Plus 0.5 mm Shoulder Kit
24*	NL-COD-0.5P	Clicq Overdenture Plus 0.5 mm Shoulder Kit, Narrow Line
25*	COD-1P	Clicq Overdenture Plus 1 mm Shoulder Kit
26*	NL-COD-1P	Clicq Overdenture Plus 1 mm Shoulder Kit, Narrow Line
27*	COD-2P	Clicq Overdenture Plus 2 mm Shoulder Kit
28*	NL-COD-2P	Clicq Overdenture Plus 2 mm Shoulder Kit, Narrow Line
29*	COD-3P	Clicq Overdenture Plus 3 mm Shoulder Kit
30*	NL-COD-3P	Clicq Overdenture Plus 3 mm Shoulder Kit, Narrow Line
31*	COD-4P	Clicq Overdenture Plus 4 mm Shoulder Kit
32*	NL-COD-4P	Clicq Overdenture Plus 4 mm Shoulder Kit, Narrow Line
33*	COD-5P	Clicq Overdenture Plus 5 mm Shoulder Kit
34*	NL-COD-5P	Clicq Overdenture Plus 5 mm Shoulder Kit, Narrow Line
35*	COD-6P	Clicq Overdenture Plus 6 mm Shoulder Kit
36*	NL-COD-6P	Clicq Overdenture Plus 6 mm Shoulder Kit, Narrow Line
37*	COD-7P	Clicq Overdenture Plus 7 mm Shoulder Kit
38*	NL-COD-7P	Clicq Overdenture Plus 7 mm Shoulder Kit, Narrow Line
Clicq Overdenture Angled		
39*	COD-18-0.5	Clicq Overdenture 18° 0.5 mm Shoulder Kit
40*	COD-18-0.5P	Clicq Overdenture Plus 18° 0.5 mm Shoulder Kit
41*	NL-COD-18-0.5	Clicq Overdenture 18° 0.5 mm Shoulder Kit Narrow Line
42*	NL-COD-18-0.5P	Clicq Overdenture Plus 18° 0.5 mm Shoulder Kit Narrow Line
43*	COD-18-1	Clicq Overdenture 18° 1 mm Shoulder Kit
44*	COD-18-1P	Clicq Overdenture Plus 18° 1 mm Shoulder Kit
45*	NL-COD-18-1	Clicq Overdenture 18° 1 mm Shoulder Kit Narrow Line
46*	NL-COD-18-1P	Clicq Overdenture Plus 18° 1 mm Shoulder Kit Narrow Line


Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

		
Clicq Overdenture & Accessories		
No	REF/ Productname	Product Description
47*	COD-18-2	Clicq Overdenture 18° 2 mm Shoulder Kit
48*	COD-18-2P	Clicq Overdenture Plus 18° 2 mm Shoulder
49*	NL-COD-18-2	Clicq Overdenture 18° 2 mm Shoulder Kit Narrow Line
50*	NL-COD-18-2P	Clicq Overdenture Plus 18° 2 mm Shoulder Narrow Line
51*	COD-18-3	Clicq Overdenture 18° 3 mm Shoulder
52*	COD-18-3P	Clicq Overdenture Plus 18° 3 mm Shoulder
53*	NL-COD-18-3	Clicq Overdenture 18° 3 mm Shoulder Narrow Line
54*	NL-COD-18-3P	Clicq Overdenture Plus 18° 3 mm Shoulder Narrow Line
55*	COD-18-4	Clicq Overdenture 18° 4 mm Shoulder
56*	COD-18-4P	Clicq Overdenture Plus 18° 4 mm Shoulder
57*	NL-COD-18-4	Clicq Overdenture 18° 4 mm Shoulder Narrow Line
58*	NL-COD-18-4	Clicq Overdenture Plus 18° 4 mm Shoulder Narrow Line
59*	COD-18-5	Clicq Overdenture 18° 5 mm Shoulder
60*	COD-18-5P	Clicq Overdenture Plus 18° 5 mm Shoulder
61*	NL-COD-18-5	Clicq Overdenture 18° 5 mm Shoulder Narrow Line
62*	NL-COD-18-5P	Clicq Overdenture Plus 18° 5 mm Shoulder Narrow Line
63*	COD-18-6	Clicq Overdenture 18° 6 mm Shoulder
64*	COD-18-6P	Clicq Overdenture Plus 18° 5 mm Shoulder
65*	NL-COD-18-6	Clicq Overdenture Plus 18° 5 mm Shoulder Narrow Line
66*	NL-COD-18-6P	Clicq Overdenture Plus 18° 5 mm Shoulder Narrow Line
67*	COD-18-7	Clicq Overdenture 18° 7 mm Shoulder
68*	COD-18-7P	Clicq Overdenture Plus 18° 7 mm Shoulder
69*	NL-COD-18-7	Clicq Overdenture 18° 7 mm Shoulder Narrow Line
70*	NL-COD-18-7P	Clicq Overdenture Plus 18° 7 mm Shoulder Narrow Line
71*	COD-30-0.5	Clicq Overdenture 30° 0.5 mm Shoulder

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

		
Clicq Overdenture & Accessories		
No	REF/ Productname	Product Description
72*	COD-30-0.5P	Clicq Overdenture Plus 30° 0.5 mm Shoulder
73*	NL-COD-30-0.5	Clicq Overdenture 30° 0.5 mm Shoulder Narrow Line
74*	NL-COD-30-0.5P	Clicq Overdenture Plus 30° 0.5 mm Narrow Line
75*	COD-30-1	Clicq Overdenture 30° 1 mm Shoulder
76*	COD-30-1P	Clicq Overdenture Plus 30° 1 mm Shoulder
77*	NL-COD-30-1	Clicq Overdenture 30° 1 mm Shoulder Narrow Line
78*	NL-COD-30-1P	Clicq Overdenture Plus 30° 1 mm Shoulder Narrow Line
79*	COD-30-2	Clicq Overdenture 30° 2 mm Shoulder
80*	COD-30-2P	Clicq Overdenture Plus 30° 2 mm Shoulder
81*	NL-COD-30-2	Clicq Overdenture 30° 2 mm Shoulder Narrow Line
82*	NL-COD-30-2P	Clicq Overdenture Plus 30° 2 mm Shoulder Narrow Line
83*	COD-30-3	Clicq Overdenture 30° 3 mm Shoulder
84*	COD-30-3P	Clicq Overdenture Plus 30° 3 mm Shoulder
85*	NL-COD-30-3	Clicq Overdenture 30° 3 mm Shoulder Narrow Line
86*	NL-COD-30-3P	Clicq Overdenture Plus 30° 3 mm Shoulder Narrow Line
87*	COD-30-4	Clicq Overdenture 30° 4 mm Shoulder
88*	COD-30-4P	Clicq Overdenture Plus 30° 4 mm Shoulder
89*	NL-COD-30-4	Clicq Overdenture 30° 4 mm Shoulder Narrow Line
90*	NL-COD-30-4P	Clicq Overdenture Plus 30° 4 mm Shoulder Narrow Line
91*	COD-30-5	Clicq Overdenture 30° 5 mm Shoulder
92*	COD-30-5P	Clicq Overdenture Plus 30° 5 mm Shoulder
93*	NL-COD-30-5	Clicq Overdenture 30° 5 mm Shoulder Narrow Line
94*	NL-COD-30-5P	Clicq Overdenture Plus 30° 5 mm Shoulder Narrow Line
95*	COD-30-6	Clicq Overdenture 30° 6 mm Shoulder
96*	COD-30-6P	Clicq Overdenture Plus 30° 6 mm Shoulder

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

		
Clicq Overdenture & Accessories		
No	REF/ Productname	Product Description
97*	NL-COD-30-6	Clicq Overdenture 30° 6 mm Shoulder Narrow Line
98*	NL-COD-30-6P	Clicq Overdenture Plus 30° 6 mm Shoulder Narrow Line
99*	COD-30-7	Clicq Overdenture 30° 7 mm Shoulder
100*	COD-30-7P	Clicq Overdenture Plus 30° 7 mm Shoulder
101*	NL-COD-30-7	Clicq Overdenture 30° 7 mm Shoulder Narrow Line
102*	NL-COD-30-7P	Clicq Overdenture Plus 30° 7 mm Shoulder Narrow Line

*Tabelle 29 Pop-Clicq Overdenture (EM)*

Pop-Clicq Convenience KIT (EM)		
No	REF/ Product name	Product Description
1	LOCW-SV-RPSK	Pop-Clicq Attachment SV-RP Starter KIT
2	LOCW-SV-RPPK	Pop-Clicq Attachment SV-RP Professional KIT
3	LOCW-SV- RP05K	Pop-Clicq Straight Attachment SV-RP H 0.5 mm KIT
4	LOCW-SV- RP10K	Pop-Clicq Straight Attachment SV-RP H 1 mm KIT
5	LOCW-SV- RP20K	Pop-Clicq Straight Attachment SV-RP H 2 mm KIT
6	LOCW-SV- RP30K	Pop-Clicq Straight Attachment SV-RP H 3 mm KIT
7	LOCW-SV- RP40K	Pop-Clicq Straight Attachment SV-RP H 4 mm KIT
8	LOCW-SV- RP50K	Pop-Clicq Straight Attachment SV-RP H 5 mm KIT
9	LOCW-SV- RP60K	Pop-Clicq Straight Attachment SV-RP H 6 mm KIT
10	LOCW-SV- RP1805K	Pop-Clicq Angled Attachment SV-RP 18°/ H 0.5 mm KIT
11	LOCW-SV- RP1810K	Pop-Clicq Angled Attachment SV-RP 18°/ H 1 mm KIT

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

Pop-Clicq Convenience KIT (EM)		
No	REF/ Product name	Product Description
12	LOCW-SV-RP1820K	Pop-Clicq Angled Attachment SV-RP 18°/ H 2 mm KIT
13	LOCW-SV-RP1830K	Pop-Clicq Angled Attachment SV-RP 18°/ H 3 mm KIT
14	LOCW-SV-RP3005K	Pop-Clicq Angled Attachment SV-RP 30°/ H 0.5 mm KIT
15	LOCW-SV-RP3010K	Pop-Clicq Angled Attachment SV-RP 30°/ H 1 mm KIT
16	LOCW-SV-RP3020K	Pop-Clicq Angled Attachment SV-RP 30°/ H 2 mm KIT
17	LOCW-SV-RP3030K	Pop-Clicq Angled Attachment SV-RP 30°/ H 3 mm KIT
18	LOCW-SV-WPSK	Pop-Clicq Attachment SV-WP Starter Kit
19	LOCW-SV-WPPK	Pop-Clicq Attachment SV-WP Professional Kit
20	LOCW-SV-WP05K	Pop-Clicq Straight Attachment SV-WP H 0.5 mm KIT
21	LOCW-SV-WP10K	Pop-Clicq Straight Attachment SV-WP H 1 mm KIT
22	LOCW-SV-WP20K	Pop-Clicq Straight Attachment SV-WP H 2 mm KIT
23	LOCW-SV-WP30K	Pop-Clicq Straight Attachment SV-WP H 3 mm KIT
24	LOCW-SV-WP40K	Pop-Clicq Straight Attachment SV-WP H 4 mm KIT
25	LOCW-SV-WP50K	Pop-Clicq Straight Attachment SV-WP H 5 mm KIT
26	LOCW-SV-WP60K	Pop-Clicq Straight Attachment SV-WP H 6 mm KIT
27	LOCW-SV-WP1805K	Pop-Clicq Angled Attachment SV-WP 18°/ H 0.5 mm KIT
28	LOCW-SV-WP1810K	Pop-Clicq Angled Attachment SV-WP 18°/ H 1 mm KIT
29	LOCW-SV-WP1820K	Pop-Clicq Angled Attachment SV-WP 18°/ H 2 mm KIT

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

Pop-Clicq Convenience KIT (EM)		
No	REF/ Product name	Product Description
30	LOCW-SV-WP1830K	Pop-Clicq Angled Attachment SV-WP 18°/ H 3 mm KIT
31	LOCW-SV-WP3005K	Pop-Clicq Angled Attachment SV-WP 30°/ H 0.5 mm KIT
32	LOCW-SV-WP3010K	Pop-Clicq Angled Attachment SV-WP 30°/ H 1 mm KIT
33	LOCW-SV-WP3020K	Pop-Clicq Angled Attachment SV-WP 30°/ H 2 mm KIT
34	LOCW-SV-WP3030K	Pop-Clicq Angled Attachment SV-WP 30°/ H 3 mm KIT
35	LOCW-SV-NLSK	Pop-Clicq Attachment SV-NL Starter KIT
36	LOCW-SV-NLPK	Pop-Clicq Attachment SV-NL Professional KIT
37	LOCW-SV-NL05K	Pop-Clicq Straight Attachment SV-NL H 0.5 mm KIT
38	LOCW-SV-NL10K	Pop-Clicq Straight Attachment SV-NL H 1 mm KIT
39	LOCW-SV-NL20K	Pop-Clicq Straight Attachment SV-NL H 2 mm KIT
40	LOCW-SV-NL30K	Pop-Clicq Straight Attachment SV-NL H 3 mm KIT
41	LOCW-SV-NL40K	Pop-Clicq Straight Attachment SV-NL H 4 mm KIT
42	LOCW-SV-NL50K	Pop-Clicq Straight Attachment SV-NL H 5 mm KIT
43	LOCW-SV-NL60K	Pop-Clicq Straight Attachment SV-NL H 6 mm KIT
44	LOCW-SV-NL1805K	Pop-Clicq Angled Attachment SV-NL 18°/ H 0.5 mm KIT
45	LOCW-SV-NL1810K	Pop-Clicq Angled Attachment SV-NL 18°/ H 1 mm KIT
46	LOCW-SV-NL1820K	Pop-Clicq Angled Attachment SV-NL 18°/ H 2 mm KIT
47	LOCW-SV-NL1830K	Pop-Clicq Angled Attachment SV-NL 18°/ H 3 mm KIT

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

Pop-Clicq Convenience KIT (EM)		
No	REF/ Product name	Product Description
48	LOCW-SV-NL3005K	Pop-Clicq Angled Attachment SV-NL 30°/ H 0.5 mm KIT
49	LOCW-SV-NL3010K	Pop-Clicq Angled Attachment SV-NL 30°/ H 1 mm KIT
50	LOCW-SV-NL3020K	Pop-Clicq Angled Attachment SV-NL 30°/ H 2 mm KIT
51	LOCW-SV-NL3030K	Pop-Clicq Angled Attachment SV-NL 30°/ H 3 mm KIT
52	LOCW-SV-NPSK	Pop-Clicq Attachment SV-NP Starter KIT
53	LOCW-SV-NPPK	Pop-Clicq Attachment SV-NP Professional KIT
54	LOCW-SV-NP05K	Pop-Clicq Straight Attachment SV-NP H 0.5 mm KIT
55	LOCW-SV-NP10K	Pop-Clicq Straight Attachment SV-NP H 1 mm KIT
56	LOCW-SV-NP20K	Pop-Clicq Straight Attachment SV-NP H 2 mm KIT
57	LOCW-SV-NP30K	Pop-Clicq Straight Attachment SV-NP H 3 mm KIT
58	LOCW-SV-NP40K	Pop-Clicq Straight Attachment SV-NP H 4 mm KIT
59	LOCW-SV-NP50K	Pop-Clicq Straight Attachment SV-NP H 5 mm KIT
60	LOCW-SV-NP60K	Pop-Clicq Straight Attachment SV-NP H 6 mm KIT
61	LOCW-SV-NP1805K	Pop-Clicq Angled Attachment SV-NP 18°/ H 0.5 mm KIT
62	LOCW-SV-NP1810K	Pop-Clicq Angled Attachment SV-NP 18°/ H 1 mm KIT
63	LOCW-SV-NP1820K	Pop-Clicq Angled Attachment SV-NP 18°/ H 2 mm KIT
64	LOCW-SV-NP1830K	Pop-Clicq Angled Attachment SV-NP 18°/ H 3 mm KIT
65	LOCW-SV-NP3005K	Pop-Clicq Angled Attachment SV-NP 30°/ H 0.5 mm KIT

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

Pop-Clicq Convenience KIT (EM)		
No	REF/ Product name	Product Description
66	LOCW-SV-NP3010K	Pop-Clicq Angled Attachment SV-NP 30°/ H 1 mm KIT
67	LOCW-SV-NP3020K	Pop-Clicq Angled Attachment SV-NP 30°/ H 2 mm KIT
68	LOCW-SV-NP3030K	Pop-Clicq Angled Attachment SV-NP 30°/ H 3 mm KIT
69	LOCW-NA-NPSK	Pop-Clicq Attachment NA-NP Starter KIT
70	LOCW-NA-NPPK	Pop-Clicq Attachment NA-NP Professional KIT
71	LOCW-NA-NP10K	Pop-Clicq Straight Attachment NA-NP H 1 mm KIT
72	LOCW-NA-NP20K	Pop-Clicq Straight Attachment NA-NP H 2 mm KIT
73	LOCW-NA-NP30K	Pop-Clicq Straight Attachment NA-NP H 3 mm KIT
74	LOCW-NA-NP40K	Pop-Clicq Straight Attachment NA-NP H 4 mm KIT
75	LOCW-NA-NP50K	Pop-Clicq Straight Attachment NA-NP H 5 mm KIT
76	LOCW-NA-NP60K	Pop-Clicq Straight Attachment NA-NP H 6 mm KIT
77	LOCW-NA-NP1810K	Pop-Clicq Angled Attachment NA-NP 18°/ H 1 mm KIT
78	LOCW-NA-NP1820K	Pop-Clicq Angled Attachment NA-NP 18°/ H 2 mm KIT
79	LOCW-NA-NP1830K	Pop-Clicq Angled Attachment NA-NP 18°/ H 3 mm KIT
80	LOCW-NA-NP3010K	Pop-Clicq Angled Attachment NA-NP 30°/ H 1 mm KIT
81	LOCW-NA-NP3020K	Pop-Clicq Angled Attachment NA-NP 30°/ H 2 mm KIT
82	LOCW-NA-NP3030K	Pop-Clicq Angled Attachment NA-NP 30°/ H 3 mm KIT
83	LOCW-NA-RPSK	Pop-Clicq Attachment NA-RP Starter KIT

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

Pop-Clicq Convenience KIT (EM)		
No	REF/ Product name	Product Description
84	LOCW-NA-RPPK	Pop-Clicq Attachment NA-RP Professional KIT
85	LOCW-NA-RP10K	Pop-Clicq Straight Attachment NA-RP H 1 mm KIT
86	LOCW-NA-RP20K	Pop-Clicq Straight Attachment NA-RP H 2 mm KIT
87	LOCW-NA-RP30K	Pop-Clicq Straight Attachment NA-RP H 3 mm KIT
88	LOCW-NA-RP40K	Pop-Clicq Straight Attachment NA-RP H 4 mm KIT
89	LOCW-NA-RP50K	Pop-Clicq Straight Attachment NA-RP H 5 mm KIT
90	LOCW-NA-RP60K	Pop-Clicq Straight Attachment NA-RP H 6 mm KIT
91	LOCW-NA-RP1810K	Pop-Clicq Angled Attachment NA-RP 18°/ H 1 mm KIT
92	LOCW-NA-RP1810K	Pop-Clicq Angled Attachment NA-RP 18°/ H 2 mm KIT
93	LOCW-NA-RP1810K	Pop-Clicq Angled Attachment NA-RP 18°/ H 3 mm KIT
94	LOCW-NA-RP3010K	Pop-Clicq Angled Attachment NA-RP 30°/ H 1 mm KIT
95	LOCW-NA-RP3020K	Pop-Clicq Angled Attachment NA-RP 30°/ H 2 mm KIT
96	LOCW-NA-RP3030K	Pop-Clicq Angled Attachment NA-RP 30°/ H 3 mm KIT
97	LOCW-TS-NPSK	Pop-Clicq Attachment TS-NP/Mini Starter KIT
98	LOCW-TS-NPPK	Pop-Clicq Attachment TS-NP/Mini Professional KIT
99	LOCW-TS-NP10K	Pop-Clicq Straight Attachment TS-NP/Mini H 1 mm KIT
100	LOCW-TS-NP20K	Pop-Clicq Straight Attachment TS-NP/Mini H 2 mm KIT
101	LOCW-TS-NP30K	Pop-Clicq Straight Attachment TS-NP/Mini H 3 mm KIT

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

Pop-Clicq Convenience KIT (EM)		
No	REF/ Product name	Product Description
102	LOCW-TS-NP40K	Pop-Clicq Straight Attachment TS-NP/Mini H 4 mm KIT
103	LOCW-TS-NP50K	Pop-Clicq Straight Attachment TS-NP/Mini H 5 mm KIT
104	LOCW-TS-NP60K	Pop-Clicq Straight Attachment TS-NP/Mini H 6 mm KIT
105	LOCW-TS-NP1810K	Pop-Clicq Angled Attachment TS-NP/Mini 18°/ H 1 mm KIT
106	LOCW-TS-NP1820K	Pop-Clicq Angled Attachment TS-NP/Mini 18°/ H 2 mm KIT
107	LOCW-TS-NP1830K	Pop-Clicq Angled Attachment TS-NP/Mini 18°/ H 3 mm KIT
108	LOCW-TS-NP3010K	Pop-Clicq Angled Attachment TS-NP/Mini 30°/ H 1 mm KIT
109	LOCW-TS-NP3020K	Pop-Clicq Angled Attachment TS-NP/Mini 30°/ H 2 mm KIT
110	LOCW-TS-NP3030K	Pop-Clicq Angled Attachment TS-NP/Mini 30°/ H 3 mm KIT
111	LOCW-TS-RPSK	Pop-Clicq Attachment TS-RP Starter KIT
112	LOCW-TS-RPPK	Pop-Clicq Attachment TS-RP Professional KIT
113	LOCW-TS-RP10K	Pop-Clicq Straight Attachment TS-RP H 1 mm KIT
114	LOCW-TS-RP20K	Pop-Clicq Straight Attachment TS-RP H 2 mm KIT
115	LOCW-TS-RP30K	Pop-Clicq Straight Attachment TS-RP H 3 mm KIT
116	LOCW-TS-RP40K	Pop-Clicq Straight Attachment TS-RP H 4 mm KIT
117	LOCW-TS-RP50K	Pop-Clicq Straight Attachment TS-RP H 5 mm KIT
118	LOCW-TS-RP60K	Pop-Clicq Straight Attachment TS-RP H 6 mm KIT
119	LOCW-TS-RP1810K	Pop-Clicq Straight Attachment TS-RP 18°/ H 1 mm KIT

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

Pop-Clicq Convenience KIT (EM)		
No	REF/ Product name	Product Description
120	LOCW-TS-RP1820K	Pop-Clicq Straight Attachment TS-RP 18°/ H 2 mm KIT
121	LOCW-TS-RP1830K	Pop-Clicq Straight Attachment TS-RP 18°/ H 3 mm KIT
122	LOCW-TS-RP3010K	Pop-Clicq Straight Attachment TS-RP 30°/ H 1 mm KIT
123	LOCW-TS-RP3020K	Pop-Clicq Straight Attachment TS-RP 30°/ H 2 mm KIT
124	LOCW-TS-RP3030K	Pop-Clicq Straight Attachment TS-RP 30°/ H 3 mm KIT
125	LOCW-OS-SPSK	Pop-Clicq Attachment OS-SP/ Mini Starter KIT
126	LOCW-OS-SPPK	Pop-Clicq Attachment OS-SP/ Mini Professional KIT
127	LOCW-OS-SP10K	Pop-Clicq Straight Attachment OS-SP H 1 mm KIT
128	LOCW-OS-SP20K	Pop-Clicq Straight Attachment OS-SP H 2 mm KIT
129	LOCW-OS-SP30K	Pop-Clicq Straight Attachment OS-SP H 3 mm KIT
130	LOCW-OS-SP40K	Pop-Clicq Straight Attachment OS-SP H 4 mm KIT
131	LOCW-OS-SP50K	Pop-Clicq Straight Attachment OS-SP H 5 mm KIT
132	LOCW-OS-SP60K	Pop-Clicq Straight Attachment OS-SP H 6 mm KIT
133	LOCW-OS-SP1810K	Pop-Clicq Angled Attachment OS-SP 18°/ H 1 mm KIT
134	LOCW-OS-SP1820K	Pop-Clicq Angled Attachment OS-SP 18°/ H 2 mm KIT
135	LOCW-OS-SP1830K	Pop-Clicq Angled Attachment OS-SP 18°/ H 3 mm KIT
136	LOCW-OS-SP3010K	Pop-Clicq Angled Attachment OS-SP 30°/ H 1 mm KIT
137	LOCW-OS-SP3020K	Pop-Clicq Angled Attachment OS-SP 30°/ H 2 mm KIT

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

Pop-Clicq Convenience KIT (EM)		
No	REF/ Product name	Product Description
138	LOCW-OS-SP3030K	Pop-Clicq Angled Attachment OS-SP 30°/ H 3 mm KIT
139	LOCW-OS-WPSK	Pop-Clicq Attachment OS-WP Starter KIT
140	LOCW-OS-WPPK	Pop-Clicq Attachment OS-WP Professional KIT
141	LOCW-OS-WP10K	Pop-Clicq Straight Attachment OS-WP H 1 mm KIT
142	LOCW-OS-WP20K	Pop-Clicq Straight Attachment OS-WP H 2 mm KIT
143	LOCW-OS-WP30K	Pop-Clicq Straight Attachment OS-WP H 3 mm KIT
144	LOCW-OS-WP40K	Pop-Clicq Straight Attachment OS-WP H 4 mm KIT
145	LOCW-OS-WP50K	Pop-Clicq Straight Attachment OS-WP H 5 mm KIT
146	LOCW-OS-WP60K	Pop-Clicq Straight Attachment OS-WP H 6 mm KIT
147	LOCW-OS-WP1810K	Pop-Clicq Straight Attachment OS-WP 18°/ H 1 mm KIT
148	LOCW-OS-WP1820K	Pop-Clicq Straight Attachment OS-WP 18°/ H 2 mm KIT
149	LOCW-OS-WP1830K	Pop-Clicq Straight Attachment OS-WP 18°/ H 3 mm KIT
150	LOCW-OS-WP3010K	Pop-Clicq Straight Attachment OS-WP 30°/ H 1 mm KIT
151	LOCW-OS-WP3020K	Pop-Clicq Straight Attachment OS-WP 30°/ H 2 mm KIT
152	LOCW-OS-WP3030K	Pop-Clicq Straight Attachment OS-WP 30°/ H 3 mm KIT
153	LOCW-AR-RPSK	Pop-Clicq Attachment AR-RP Starter KIT
154	LOCW-AR-RPPK	Pop-Clicq Attachment AR-RP Professional KIT
155	LOCW-AR-RP10K	Pop-Clicq Attachment AR-RP H 1 mm KIT

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

Pop-Clicq Convenience KIT (EM)		
No	REF/ Product name	Product Description
156	LOCW-AR-RP20K	Pop-Clicq Attachment AR-RP H 2 mm KIT
157	LOW-AR-RP30K	Pop-Clicq Attachment AR-RP H 3 mm KIT
158	LOCW-AR-RP40K	Pop-Clicq Attachment AR-RP H 4 mm KIT
159	LOCW-AR-RP50K	Pop-Clicq Attachment AR-RP H 5 mm KIT
160	LOCW-AR-RP60K	Pop-Clicq Attachment AR-RP H 6 mm KIT
161	LOCW-AR-RP1810K	Pop-Clicq Attachment AR-RP 18°/ H 1 mm KIT
162	LOCW-AR-RP1820K	Pop-Clicq Attachment AR-RP 18°/ H 2 mm KIT
163	LOCW-AR-RP1830K	Pop-Clicq Attachment AR-RP 18°/ H 3 mm KIT
164	LOCW-AR-RP3010K	Pop-Clicq Attachment AR-RP 30°/ H 1 mm KIT
165	LOCW-AR-RP3020K	Pop-Clicq Attachment AR-RP 30°/ H 2 mm KIT
166	LOCW-AR-RP3030K	Pop-Clicq Attachment AR-RP 30°/ H 3 mm KIT
167	LOCW-GM-RPSK	Pop-Clicq Attachment GM-RP Starter KIT
168	LOCW-GM-RPPK	Pop-Clicq Attachment GM-RP Professional KIT
169	LOCW-GM-RP10K	Pop-Clicq Attachment GM-RP H 1 mm KIT
170	LOCW-GM-RP20K	Pop-Clicq Attachment GM-RP H 2 mm KIT
171	LOCW-GM-RP30K	Pop-Clicq Attachment GM-RP H 3 mm KIT
172	LOCW-GM-RP40K	Pop-Clicq Attachment GM-RP H 4 mm KIT
173	LOCW-GM-RP50K	Pop-Clicq Attachment GM-RP H 5 mm KIT

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

Pop-Clicq Convenience KIT (EM)		
No	REF/ Product name	Product Description
174	LOCW-GM-RP60K	Pop-Clicq Attachment GM-RP H 6 mm KIT
175	LOCW-GM-RP1810K	Pop-Clicq Attachment GM-RP 18°/ H 1 mm KIT
176	LOCW-GM-RP1820K	Pop-Clicq Attachment GM-RP 18°/ H 2 mm KIT
177	LOCW-GM-RP1830K	Pop-Clicq Attachment GM-RP 18°/ H 3 mm KIT
178	LOCW-GM-RP3010K	Pop-Clicq Attachment GM-RP 30°/ H 1 mm KIT
179	LOCW-GM-RP3020K	Pop-Clicq Attachment GM-RP 30°/ H 2 mm KIT
180	LOCW-GM-RP3030K	Pop-Clicq Attachment GM-RP 30°/ H 3 mm KIT
181	LOCW-CF-RCASK	Pop-Clicq Attachment CF-RC Starter KIT – A Type
182	LOCW-CF-RCBSK	Pop-Clicq Attachment CF-RC Starter KIT – B Type
183	LOCW-CF-RCAPK	Pop-Clicq Attachment CF-RC Professional KIT – A Type
184	LOCW-CF-RCBPK	Pop-Clicq Attachment CF-RC Professional KIT – B Type
185	LOCW-CF-RC10K	Pop-Clicq Straight Attachment CF-RC H 1 mm KIT
186	LOCW-CF-RC20K	Pop-Clicq Straight Attachment CF-RC H 2 mm KIT
187	LOCW-CF-RC30K	Pop-Clicq Straight Attachment CF-RC H 3 mm KIT
188	LOCW-CF-RC40K	Pop-Clicq Straight Attachment CF-RC H 4 mm KIT
189	LOCW-CF-RC50K	Pop-Clicq Straight Attachment CF-RC H 5 mm KIT
190	LOCW-CF-RC60K	Pop-Clicq Straight Attachment CF-RC H 6 mm KIT

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

Pop-Clicq Convenience KIT (EM)		
No	REF/ Product name	Product Description
191	LOCW-CF-RC1810AK	Pop-Clicq Angled Attachment CF-RC 18°/ H 1 mm KIT Type A
192	LOCW-CF-RC1820AK	Pop-Clicq Angled Attachment CF-RC 18°/ H 2 mm KIT Type A
193	LOCW-CF-RC1830AK	Pop-Clicq Angled Attachment CF-RC 18°/ H 3 mm KIT Type A
194	LOCW-CF-RC1810BK	Pop-Clicq Angled Attachment CF-RC 18°/ H 1 mm KIT Type B
195	LOCW-CF-RC1820BK	Pop-Clicq Angled Attachment CF-RC 18°/ H 2 mm KIT Type B
196	LOCW-CF-RC1830BK	Pop-Clicq Angled Attachment CF-RC 18°/ H 3 mm KIT Type B
197	LOCW-CF-RC3010AK	Pop-Clicq Angled Attachment CF-RC 30°/ H 1 mm KIT Type A
198	LOCW-CF-RC3020AK	Pop-Clicq Angled Attachment CF-RC 30°/ H 2 mm KIT Type A
199	LOCW-CF-RC3030AK	Pop-Clicq Angled Attachment CF-RC 30°/ H 3 mm KIT Type A
200	LOCW-CF-RC3010BK	Pop-Clicq Angled Attachment CF-RC 30°/ H 1 mm KIT Type B
201	LOCW-CF-RC3020BK	Pop-Clicq Angled Attachment CF-RC 30°/ H 2 mm KIT Type B
202	LOCW-CF-RC3030BK	Pop-Clicq Angled Attachment CF-RC 30°/ H 3 mm KIT Type B
203	LOCW-CF-NCASK	Pop-Clicq Attachment CF-NC Starter KIT Type A
204	LOCW-CF-NCBSK	Pop-Clicq Attachment CF-NC Starter KIT Type B
205	LOCW-CF-NCAPK	Pop-Clicq Attachment CF-NC Professional KIT Type A
206	LOCW-CF-NCBPK	Pop-Clicq Attachment CF-NC Professional KIT Type B
207	LOCW-CF-NC10K	Pop-Clicq Straight Attachment CF-NC H 1 mm KIT

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

Pop-Clicq Convenience KIT (EM)		
No	REF/ Product name	Product Description
208	LOCW-CF-NC20K	Pop-Clicq Straight Attachment CF-NC H 2 mm KIT
209	LOCW-CF-NC30K	Pop-Clicq Straight Attachment CF-NC H 3 mm KIT
210	LOCW-CF-NC40K	Pop-Clicq Straight Attachment CF-NC H 4 mm KIT
211	LOCW-CF-NC50K	Pop-Clicq Straight Attachment CF-NC H 5 mm KIT
212	LOCW-CF-NC60K	Pop-Clicq Straight Attachment CF-NC H 6 mm KIT
213	LOCW-CF-NC1810AK	Pop-Clicq Angled Attachment CF-NC 18°/ H 1 mm KIT A Type
214	LOCW-CF-NC1820AK	Pop-Clicq Angled Attachment CF-NC 18°/ H 2 mm KIT A Type
215	LOCW-CF-NC1830AK	Pop-Clicq Angled Attachment CF-NC 18°/ H 3 mm KIT A Type
216	LOCW-CF-NC1810BK	Pop-Clicq Angled Attachment CF-NC 18°/ H 1 mm KIT B Type
217	LOCW-CF-NC1820BK	Pop-Clicq Angled Attachment CF-NC 18°/ H 2 mm KIT B Type
218	LOCW-CF-NC1830BK	Pop-Clicq Angled Attachment CF-NC 18°/ H 3 mm KIT B Type
219	LOCW-CF-NC3010AK	Pop-Clicq Angled Attachment CF-NC 30°/ H 1 mm KIT A Type
220	LOCW-CF-NC3020AK	Pop-Clicq Angled Attachment CF-NC 30°/ H 2 mm KIT A Type
221	LOCW-CF-NC3030AK	Pop-Clicq Angled Attachment CF-NC 30°/ H 3 mm KIT A Type
222	LOCW-CF-NC3010BK	Pop-Clicq Angled Attachment CF-NC 30°/ H 1 mm KIT B Type
223	LOCW-CF-NC3020BK	Pop-Clicq Angled Attachment CF-NC 30°/ H 2 mm KIT B Type
224	LOCW-CF-NC3030BK	Pop-Clicq Angled Attachment CF-NC 30°/ H 3 mm KIT B Type

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

Pop-Clicq Convenience KIT (EM)		
No	REF/ Product name	Product Description
225	LOCW-BK2	Pop-Clicq Bar Pre-Fabricated Bar on 2
226	LOCW-BK2D	Pop-Clicq Bar Pre-Fabricated Bar on 2 +
227	LOCW-BK4	Pop-Clicq Bar Pre-Fabricated Bar on 4
228	LOCW-BK4D	Pop-Clicq Bar Pre-Fabricated Bar on 4 + Driver


*Tabelle 30 Q-Clicq (EM)*

		
No	REF/ Productname	Product Description
1	LOCN-SV-RP05	Q-Clicq Straight Attachment SV-RP H 0.5 mm
2	LOCN-SV-RP10	Q-Clicq Straight Attachment SV-RP H 1 mm
3	LOCN-SV-RP20	Q-Clicq Straight Attachment SV-RP H 2 mm
4	LOCN-SV-RP30	Q-Clicq Straight Attachment SV-RP H 3 mm
5	LOCN-SV-RP40	Q-Clicq Straight Attachment SV-RP H 4 mm
6	LOCN-SV-RP50	Q-Clicq Straight Attachment SV-RP H 5 mm
7	LOCN-SV-RP60	Q-Clicq Straight Attachment SV-RP H 6 mm
8	LOCN-SV-RP70	Q-Clicq Straight Attachment SV-RP H 7 mm
9	LOCN-SV-RP1810	Q-Clicq Angled Attachment SV-RP 18° H 1 mm
10	LOCN-SV-RP1820	Q-Clicq Angled Attachment SV-RP 18° H 2 mm
11	LOCN-SV-RP1830	Q-Clicq Angled Attachment SV-RP 18° H 3 mm
12	LOCN-SV-RP1840	Q-Clicq Angled Attachment SV-RP 18° H 4 mm
13	LOCN-SV-RP3010	Q-Clicq Angled Attachment SV-RP 30° H 1 mm
14	LOCN-SV-RP3020	Q-Clicq Angled Attachment SV-RP 30° H 2 mm

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

		
Q-Clcq Attachment & Caps (EM)		
No	REF/ Productname	Product Description
15	LOCN-SV-RP3030	Q-Clcq Angled Attachment SV-RP 30°/ H 3 mm
16	LOCN-SV-RP3040	Q-Clcq Angled Attachment SV-RP 30°/ H 4 mm
17	LOCN-SV-NL05	Q-Clcq Straight Attachment SV-NL H 0.5 mm
18	LOCN-SV-NL10	Q-Clcq Straight Attachment SV-NL H 1 mm
19	LOCN-SV-NL20	Q-Clcq Straight Attachment SV-NL H 2 mm
20	LOCN-SV-NL30	Q-Clcq Straight Attachment SV-NL H 3 mm
21	LOCN-SV-NL40	Q-Clcq Straight Attachment SV-NL H 4 mm
22	LOCN-SV-NL50	Q-Clcq Straight Attachment SV-NL H 5 mm
23	LOCN-SV-NL60	Q-Clcq Straight Attachment SV-NL H 6 mm
24	LOCN-SV-NL1810	Q-Clcq Angled Attachment SV-NL 18°/ H 1 mm
25	LOCN-SV-NL1820	Q-Clcq Angled Attachment SV-NL 18°/ H 2 mm
26	LOCN-SV-NL1830	Q-Clcq Angled Attachment SV-NL 18°/ H 3 mm
27	LOCN-SV-NL1840	Q-Clcq Angled Attachment SV-NL 18°/ H 4 mm
28	LOCN-SV-NL3010	Q-Clcq Angled Attachment SV-NL 30°/ H 1 mm
29	LOCN-SV-NL3020	Q-Clcq Angled Attachment SV-NL 30°/ H 2 mm
30	LOCN-SV-NL3030	Q-Clcq Angled Attachment SV-NL 30°/ H 3 mm
31	LOCN-SV-NL3040	Q-Clcq Angled Attachment SV-NL 30°/ H 4 mm
32	LOCN-SV-NL05	Q-Clcq Straight Attachment SV-NL H 0.5 mm
33	LOCN-SV-NL10	Q-Clcq Straight Attachment SV-NL H 1 mm
34	LOCN-SV-NL20	Q-Clcq Straight Attachment SV-NL H 2 mm
35	LOCN-SV-NL30	Q-Clcq Straight Attachment SV-NL H 3 mm

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

		
Q-Clicq Attachment & Caps (EM)		
No	REF/ Productname	Product Description
36	LOCN-SV-NL40	Q-Clicq Straight Attachment SV-NL H 4 mm
37	LOCN-SV-NL50	Q-Clicq Straight Attachment SV-NL H 5 mm
38	LOCN-SV-NL60	Q-Clicq Straight Attachment SV-NL H 6 mm
39	LOCN-SV-NL70	Q-Clicq Straight Attachment SV-NL H 7 mm
40	LOCN-SV- NP1810	Q-Clicq Angled Attachment SV-NL 18°/ H 1 mm
41	LOCN-SV- NP1820	Q-Clicq Angled Attachment SV-NL 18°/ H 2 mm
42	LOCN-SV- NP1830	Q-Clicq Angled Attachment SV-NL 18°/ H 3 mm
43	LOCN-SV- NP1840	Q-Clicq Angled Attachment SV-NL 18°/ H 4 mm
44	LOCN-SV- NP3010	Q-Clicq Angled Attachment SV-NL 30°/ H 1 mm
45	LOCN-SV- NP3020	Q-Clicq Angled Attachment SV-NL 30°/ H 2 mm
46	LOCN-SV- NP3030	Q-Clicq Angled Attachment SV-NL 30°/ H 3 mm
47	LOCN-SV- NP3040	Q-Clicq Angled Attachment SV-NL 30°/ H 4 mm
48	LOCN-MH	Q-Clicq Metal Housing
49	LOCN-PD	Protective Disc 4 Pcs
50	LOCN-RCB	Q-Clicq Retentive Cap 4 Pcs Black Lab use only
51	LOCN-RCY	Q-Clicq Retentive Cap 4 Pcs Yellow Ex-Soft (0.6 Kg)
52	LOCN-RCP	Q-Clicq Retentive Cap 4 Pcs Pink Soft (1.2 Kg)
53	LOCN-RCT	Q-Clicq Retentive Cap 4 Pcs Transparent Standard (1.8 Kg)
54	LOCN-RCV	Q-Clicq Retentive Cap 4 Pcs Violet Strong (2.7 Kg)

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

*Tabelle 31 Q-Clicq Kits (EM)*

Q-Clicq Attachment Convenience Kits (EM)		
No	REF/ Product name	Product Description
1	LOCN-SV-RPPK	Q-Clicq Attachment SV-RP Professional KIT
2	LOCN-SV-RP05K	Q-Clicq Straight Attachment SV-RP H 0.5 mm KIT
3	LOCN-SV-RP10K	Q-Clicq Straight Attachment SV-RP H 1 mm KIT
4	LOCN-SV-RP20K	Q-Clicq Straight Attachment SV-RP H 2 mm KIT
5	LOCN-SV-RP30K	Q-Clicq Straight Attachment SV-RP H 3 mm KIT
6	LOCN-SV-RP40K	Q-Clicq Straight Attachment SV-RP H 4 mm KIT
7	LOCN-SV-RP50K	Q-Clicq Straight Attachment SV-RP H 5 mm KIT
8	LOCN-SV-RP60K	Q-Clicq Straight Attachment SV-RP H 6 mm KIT
9	LOCN-SV-RP70K	Q-Clicq Straight Attachment SV-RP H 7 mm KIT
10	LOCN-SV-RP1810K	Q-Clicq Angled Attachment SV-RP 18°/ H 1 mm KIT
11	LOCN-SV-RP1820K	Q-Clicq Angled Attachment SV-RP 18°/ H 2 mm KIT
12	LOCN-SV-RP1830K	Q-Clicq Angled Attachment SV-RP 18°/ H 3 mm KIT
13	LOCN-SV-RP1840K	Q-Clicq Angled Attachment SV-RP 18°/ H 4 mm KIT
14	LOCN-SV-RP3010K	Q-Clicq Angled Attachment SV-RP 30°/ H 1 mm KIT
15	LOCN-SV-RP3020K	Q-Clicq Angled Attachment SV-RP 30°/ H 2 mm KIT
16	LOCN-SV-RP3030K	Q-Clicq Angled Attachment SV-RP 30°/ H 3 mm KIT
17	LOCN-SV-RP3040K	Q-Clicq Angled Attachment SV-RP 30°/ H 4 mm KIT
18	LOCN-SV-NLPK	Q-Clicq Attachment SV-NL Professional KIT
19	LOCN-SV-NL05K	Q-Clicq Straight Attachment SV-NL H 0.5 mm KIT
20	LOCN-SV-NL10K	Q-Clicq Straight Attachment SV-NL H 1 mm KIT
21	LOCN-SV-NL20K	Q-Clicq Straight Attachment SV-NL H 2 mm KIT


Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

Q-Clicq Attachment Convenience Kits (EM)		
No	REF/ Product name	Product Description
22	LOCN-SV-NL30K	Q-Clicq Straight Attachment SV-NL H 3 mm KIT
23	LOCN-SV-NL40K	Q-Clicq Straight Attachment SV-NL H 4 mm KIT
24	LOCN-SV-NL50K	Q-Clicq Straight Attachment SV-NL H 5 mm KIT
25	LOCN-SV-NL60K	Q-Clicq Straight Attachment SV-NL H 6 mm KIT
26	LOCN-SV-NL70K	Q-Clicq Straight Attachment SV-NL H 7 mm KIT
27	LOCN-SV-NL1810K	Q-Clicq Angled Attachment SV-NL 18°/ H 1 mm KIT
28	LOCN-SV-NL1820K	Q-Clicq Angled Attachment SV-NL 18°/ H 2 mm KIT
29	LOCN-SV-NL1830K	Q-Clicq Angled Attachment SV-NL 18°/ H 3 mm KIT
30	LOCN-SV-NL1840K	Q-Clicq Angled Attachment SV-NL 18°/ H 4 mm KIT
31	LOCN-SV-NL3010K	Q-Clicq Angled Attachment SV-NL 30°/ H 1 mm KIT
32	LOCN-SV-NL3020K	Q-Clicq Angled Attachment SV-NL 30°/ H 2 mm KIT
33	LOCN-SV-NL3030K	Q-Clicq Angled Attachment SV-NL 30°/ H 3 mm KIT
34	LOCN-SV-NL3040K	Q-Clicq Angled Attachment SV-NL 30°/ H 4 mm KIT
35	LOCN-SV-NPPK	Q-Clicq Attachment SV-RP Professional KIT
36	LOCN-SV-NP05K	Q-Clicq Straight Attachment SV-RP H 0.5 mm KIT
37	LOCN-SV-NP10K	Q-Clicq Straight Attachment SV-RP H 1 mm KIT
38	LOCN-SV-NP20K	Q-Clicq Straight Attachment SV-RP H 2 mm KIT
39	LOCN-SV-NP30K	Q-Clicq Straight Attachment SV-RP H 3 mm KIT
40	LOCN-SV-NP40K	Q-Clicq Straight Attachment SV-RP H 4 mm KIT
41	LOCN-SV-NP50K	Q-Clicq Straight Attachment SV-RP H 5 mm KIT
42	LOCN-SV-NP60K	Q-Clicq Straight Attachment SV-RP H 6 mm KIT
43	LOCN-SV-NP70K	Q-Clicq Straight Attachment SV-RP H 7 mm KIT



Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

Q-Clicq Attachment Convenience Kits (EM)		
No	REF/ Product name	Product Description
44	LOCN-SV-NP1810K	Q-Clicq Angled Attachment SV-RP 18°/ H 1 mm KIT
45	LOCN-SV-NP1820K	Q-Clicq Angled Attachment SV-RP 18°/ H 2 mm KIT
46	LOCN-SV-NP1830K	Q-Clicq Angled Attachment SV-RP 18°/ H 3 mm KIT
47	LOCN-SV-NP1840K	Q-Clicq Angled Attachment SV-RP 18°/ H 4 mm KIT
48	LOCN-SV-NP3010K	Q-Clicq Angled Attachment SV-RP 30°/ H 1 mm KIT
49	LOCN-SV-NP3020K	Q-Clicq Angled Attachment SV-RP 30°/ H 2 mm KIT
50	LOCN-SV-NP3030K	Q-Clicq Angled Attachment SV-RP 30°/ H 3 mm KIT
51	LOCN-SV-NP3040K	Q-Clicq Angled Attachment SV-RP 30°/ H 4 mm KIT


*Tabelle 32 One Piece Multi-Unit*

 <p style="text-align: center;">One Piece Multi-Unit Abutments &amp; Multi Unit Parts</p>		
No	REF/ Productname	Product Description
Straight Multi-Unit/ Straight Multi Unit KITS		
1*	MU-KS10	Straight Multi Unit 1 mm Shoulder
2*	NL-MU-KS10	Straight Multi Unit 1 mm Shoulder Narrow Line
3*	MU-KS20	Straight Multi Unit 2 mm Shoulder
4*	NL-MU-KS20	Straight Multi Unit 2 mm Shoulder Narrow Line
5*	MU-KS30	Straight Multi Unit 3 mm Shoulder
6*	NL-MU-KS30	Straight Multi Unit 3 mm Shoulder Narrow Line
7*	MU-KS40	Straight Multi Unit 4 mm Shoulder
8*	NL-MU-KS40	Straight Multi Unit 4 mm Shoulder Narrow Line


Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

		
One Piece Multi-Unit Abutments & Multi Unit Parts		
No	REF/ Productname	Product Description
9*	MU-KS50	Straight Multi Unit 5 mm Shoulder
10*	NL-MU-KS50	Straight Multi Unit 5 mm Shoulder Narrow Line
11*	MU-KS10K	Straight Multi Unit 1 mm Shoulder-Kit
12*	NL-MU-KS10K	Straight Multi Unit 1 mm Shoulder-Kit Narrow Line
13*	MU-KS20K	Straight Multi Unit 2 mm Shoulder-Kit
14*	NL-MU-KS20K	Straight Multi Unit 2 mm Shoulder-Kit Narrow Line
15*	MU-KS30K	Straight Multi Unit 3 mm Shoulder-Kit
16*	NL-MU-KS30K	Straight Multi Unit 3 mm Shoulder-Kit Narrow Line
17*	MU-KS40K	Straight Multi Unit 4 mm Shoulder-Kit
18*	NL-MU-KS40K	Straight Multi Unit 4 mm Shoulder-Kit Narrow Line
19*	MU-KS50K	Straight Multi Unit 5 mm Shoulder-Kit
20*	NL-MU-KS50K	Straight Multi Unit 5 mm Shoulder-Kit Narrow Line
		
Esthetic and Overdenture Abutments		
1	ACA-05	Anatomic Connection Abutment 0.5 mm Shoulder
2	ACA-15	Anatomic Connection Abutment 1.5 mm Shoulder
3	ACA-25	Anatomic Connection Abutment 2.5 mm Shoulder
	NL-ACA-05	Anatomic Connection Abutment 0.5 mm Shoulder Narrow Line
	NL-ACA-15	Anatomic Connection Abutment 1.5 mm Shoulder Narrow Line
	NL-ACA-25	Anatomic Connection Abutment 2.5 mm Shoulder Narrow Line
4	OCA-05	Overdenture Connection Abutment 0.5 mm Shoulder with SF-O
5	OCA15	Overdenture Connection Abutment 1.5 mm Shoulder with SF-O
6	OCA-25	Overdenture Connection Abutment 2.5 mm Shoulder with SF-O
Angled Multi Unit 17°		
1*	MU-KS1710	Angled Multi Unit One Piece 17°/ 1 mm Shoulder
2*	NL-MU-KS1710	Angled Multi Unit One Piece 17°/ 1 mm Shoulder Narrow Line
3*	MU-KS1720	Angled Multi Unit One Piece 17°/ 2 mm Shoulder
4*	NL-MU-KS1720	Angled Multi Unit One Piece 17°/ 2 mm Shoulder Narrow Line

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28


		
One Piece Multi-Unit Abutments & Multi Unit Parts		
No	REF/ Productname	Product Description
5*	MU-KS1730	Angled Multi Unit One Piece 17°/ 3 mm Shoulder
6*	NL-MU-KS1730	Angled Multi Unit One Piece 17°/ 3 mm Shoulder Narrow Line
7*	MU-KS1740	Angled Multi Unit One Piece 17°/ 4 mm Shoulder
8*	NL-MU-KS1740	Angled Multi Unit One Piece 17°/ 4 mm Shoulder Narrow Line
9*	MU-KS1710H	Angled Multi Unit One Piece 17°/ 1 mm Shoulder Anti Rotational
10*	NL-MU-KS1710H	Angled Multi Unit One Piece 17°/ 1 mm Shoulder Anti Rotational Narrow Line
11*	MU-KS1720H	Angled Multi Unit One Piece 17°/ 2 mm Shoulder Anti Rotational
12*	NL-MU-KS1720H	Angled Multi Unit One Piece 17°/ 2 mm Shoulder Anti Rotational Narrow Line
13*	MU-KS1730H	Angled Multi Unit One Piece 17°/ 3 mm Shoulder Anti Rotational
14*	NL-MU-KS1730H	Angled Multi Unit One Piece 17°/ 3 mm Shoulder Anti Rotational Narrow Line
15*	MU-KS1740	Angled Multi Unit One Piece 17°/ 4 mm Shoulder Anti Rotational
16*	NL-MU-KS1740	Angled Multi Unit One Piece 17°/ 4 mm Shoulder Anti Rotational Narrow Line
Angled Multi Unit 30°/ Angled Multi Unit KITS		
1*	MU-KS3010	Angled Multi Unit One Piece 30°/ 1 mm Shoulder
2*	NL-MU-KS3010	Angled Multi Unit One Piece 30°/ 1 mm Shoulder Narrow Line
3*	MU-KS3020	Angled Multi Unit One Piece 30°/ 2 mm Shoulder
4*	NL-MU-KS3020	Angled Multi Unit One Piece 30°/ 2 mm Shoulder Narrow Line
5*	MU-KS3030	Angled Multi Unit One Piece 30°/ 3 mm Shoulder
6*	NL-MU-KS3030	Angled Multi Unit One Piece 30°/ 3 mm Shoulder Narrow Line
7*	MU-KS3040	Angled Multi Unit One Piece 30°/ 4 mm Shoulder
8*	NL-MU-KS3040	Angled Multi Unit One Piece 30°/ 4 mm Shoulder Narrow Line
9*	MU-KS3010H	Angled Multi Unit One Piece 30°/ 1 mm Shoulder Anti Rotational
10*	NL-MU-KS3010H	Angled Multi Unit One Piece 30°/ 1 mm Shoulder Anti Rotational Narrow Line
11*	MU-KS3020H	Angled Multi Unit One Piece 30°/ 2 mm Shoulder Anti Rotational
12*	NL-MU-KS3020H	Angled Multi Unit One Piece 30°/ 2 mm Shoulder Anti Rotational Narrow Line
13*	MU-KS3030H	Angled Multi Unit One Piece 30°/ 3 mm Shoulder Anti Rotational
14*	NL-MU-KS3030H	Angled Multi Unit One Piece 30°/ 3 mm Shoulder Anti Rotational Narrow Line
15*	MU-KS3040H	Angled Multi Unit One Piece 30°/ 4 mm Shoulder Anti Rotational

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28


 <p style="text-align: center;">One Piece Multi-Unit Abutments &amp; Multi Unit Parts</p>		
No	REF/ Productname	Product Description
16*	NL-MU-KS3040H	Angled Multi Unit One Piece 30°/ 4 mm Shoulder Anti Rotational Narrow Line
17*	MU-KS1710K	Angled Multi Unit One Piece 17°/ 1 mm Shoulder-Kit
18*	NL-MU-KS1710K	Angled Multi Unit One Piece 17°/ 1 mm Shoulder-Kit Narrow Line
19*	MU-KS1720K	Angled Multi Unit One Piece 17°/ 2 mm Shoulder-Kit
20*	NL-MU-KS1720K	Angled Multi Unit One Piece 17°/ 2 mm Shoulder-Kit Narrow Line
21*	MU-KS1730K	Angled Multi Unit One Piece 17°/ 3 mm Shoulder-Kit
22*	NL-MU-KS1730K	Angled Multi Unit One Piece 17°/ 2 mm Shoulder-Kit Narrow Line
23*	MU-KS1740K	Angled Multi Unit One Piece 17°/ 2 mm Shoulder-Kit
24*	NL-MU-KS1740K	Angled Multi Unit One Piece 17°/ 2 mm Shoulder-Kit Narrow Line
25*	MU-KS3010K	Angled Multi Unit One Piece 30°/ 1 mm Shoulder- Kit
26*	NL-MU-KS3010K	Angled Multi Unit One Piece 30°/ 1 mm Shoulder- Kit Narrow Line
27*	MU-KS-3020K	Angled Multi Unit One Piece 30°/ 2 mm Shoulder- Kit
28*	NL-MU-KS3020K	Angled Multi Unit One Piece 30°/ 2 mm Shoulder- Kit Narrow Line
29*	MU-KS3030K	Angled Multi Unit One Piece 30°/ 3 mm Shoulder- Kit
30*	NL-MU-KS3030K	Angled Multi Unit One Piece 30°/ 3 mm Shoulder- Kit Narrow Line
31*	MU-KS3040K	Angled Multi Unit One Piece 30°/ 4 mm Shoulder- Kit
32*	NL-MU-KS3040K	Angled Multi Unit One Piece 30°/ 4 mm Shoulder- Kit Narrow Line
33*	MU-KSSL	Titanium Sleeve for KS Multi Unit System
34*	MU-KSSLP	Plastic Sleeve for KS Multi Unit System
35*	MU-KSAC-R	Adhesive Cap for KS Multi Unit System Rotational
36*	MU-KSAC-AR	Adhesive Cap for KS Multi Unit System Anti Rotational

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

Tabelle 33 Multi-Clicq On-X (EM)

 Multi-Clicq On-X (EM)		
No	REF/ Productname	Product Description
1	MUL-SV-RPNB10	"Classic" Straight Multi-Unit SV-RP H 1 mm (M 1.4)
2	MUL-SV-RPNB20	"Classic" Straight Multi-Unit SV-RP H 2 mm (M 1.4)
3	MUL-SV-RPNB30	"Classic" Straight Multi-Unit SV-RP H 3 mm (M 1.4)
4	MUL-SV-RPNB40	Classic" Straight Multi-Unit SV-RP H 4 mm (M 1.4)
5	MUL-SV-RPNB50	Classic" Straight Multi-Unit SV-RP H 5 mm (M 1.4)
6	MUL-SV-RPNB60	Classic" Straight Multi-Unit SV-RP H 6 mm (M 1.4)
7	MUL-SV-RP1710	"Classic" Angled Multi-Unit SV-RP 17°/ H 1 mm (M 1.4)
8	MUL-SV-RP1720	"Classic" Angled Multi-Unit SV-RP 17°/ H 2 mm (M 1.4)
9	MUL-SV-RP1730	"Classic" Angled Multi-Unit SV-RP 17°/ H 3 mm (M 1.4)
10	MUL-SV-RP3010	"Classic" Angled Multi-Unit SV-RP 30°/ H 1 mm (M 1.4)
11	MUL-SV-RP3020	"Classic" Angled Multi-Unit SV-RP 30°/ H 2 mm (M 1.4)
12	MUL-SV-RP3030	"Classic" Angled Multi-Unit SV-RP 30°/ H 3 mm (M 1.4)
13	MUL-SV-RPKS10	Multi-Clicq Straight Multi-Unit SV-RP H 1 mm
14	MUL-SV-RPKS20	Multi-Clicq Straight Multi-Unit SV-RP H 2 mm
15	MUL-SV-RPKS30	Multi-Clicq Straight Multi-Unit SV-RP H 3 mm
16	MUL-SV-RPKS40	Multi-Clicq Straight Multi-Unit SV-RP H 4 mm
17	MUL-SV-RPKS50	Multi-Clicq Straight Multi-Unit SV-RP H 5 mm
18	MUL-SV-RPKS60	Multi-Clicq Straight Multi-Unit SV-RP H 6 mm
19	MUL-SVRPKS1710	Multi-Clicq Angled Multi-Unit SV-RP 17°/ H 1 mm A Type
20	MUL-SVRPKS1720	Multi-Clicq Angled Multi-Unit SV-RP 17°/ H 2 mm A Type
21	MUL-SVRPKS1730	Multi-Clicq Angled Multi-Unit SV-RP 17°/ H 3 mm A Type
22	MUL-SVRPKS1740	Multi-Clicq Angled Multi-Unit SV-RP 17°/ H 4 mm A Type
23	MUL-SVRPKS1710B	Multi-Clicq Angled Multi-Unit SV-RP 17°/ H 1 mm B Type


Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

		
Multi-Clicq On-X (EM)		
No	REF/ Productname	Product Description
24	MUL-SVRPKS1720B	Multi-Clicq Angled Multi-Unit SV-RP 17°/ H 2 mm B Type
25	MUL-SVRPKS1730B	Multi-Clicq Angled Multi-Unit SV-RP 17°/ H 3 mm B Type
26	MUL-SV-RPKS1740B	Multi-Clicq Angled Multi-Unit SV-RP 17°/ H 4 mm B Type
27	MUL-SV-RPKS3010	Multi-Clicq Angled Multi-Unit SV-RP 30°/ H 1 mm A Type
28	MUL-SV-RPKS3020	Multi-Clicq Angled Multi-Unit SV-RP 30°/ H 2 mm A Type
29	MUL-SV-RPKS3030	Multi-Clicq Angled Multi-Unit SV-RP 30°/ H 3 mm A Type
30	MUL-SV-RPKS3040	Multi-Clicq Angled Multi-Unit SV-RP 30°/ H 4 mm A Type
31	MUL-SV-RPKS3010B	Multi-Clicq Angled Multi-Unit SV-RP 30°/ H 1 mm B Type
32	MUL-SV-RPKS3020B	Multi-Clicq Angled Multi-Unit SV-RP 30°/ H 2 mm B Type
33	MUL-SV-RPKS3030B	Multi-Clicq Angled Multi-Unit SV-RP 30°/ H 3 mm B Type
34	MUL-SV-RPKS3040B	Multi-Clicq Angled Multi-Unit SV-RP 30°/ H 4 mm B Type
35	MUL-SV-RPKS1710H	Multi-Clicq Angled Multi-Unit SV-RP 17°/ H 1 mm A.R
36	MUL-SV-RPKS1720H	Multi-Clicq Angled Multi-Unit SV-RP 17°/ H 2 mm A.R
37	MUL-SV-RPKS1730H	Multi-Clicq Angled Multi-Unit SV-RP 17°/ H 3 mm A.R
38	MUL-SV-RPKS1740H	Multi-Clicq Angled Multi-Unit SV-RP 17°/ H 4 mm A.R
39	MUL-SV-RPKS3010H	Multi-Clicq Angled Multi-Unit SV-RP 30°/ H 1 mm A.R
40	MUL-SV-RPKS3020H	Multi-Clicq Angled Multi-Unit SV-RP 30°/ H 2 mm A.R


Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

 Multi-Clicq On-X (EM)		
No	REF/ Productname	Product Description
41	MUL-SV- RPKS3030H	Multi-Clicq Angled Multi-Unit SV-RP 30°/ H 3 mm A.R
42	MUL-SV- RPKS3040H	Multi-Clicq Angled Multi-Unit SV-RP 30°/ H 4 mm A.R
43	MUL-SV- WPNB10	"Classic" Straight Multi-Unit SV-WP H 1 mm (M 1.4)
44	MUL-SV- WPNB20	"Classic" Straight Multi-Unit SV-WP H 2 mm (M1.4)
45	MUL-SV- WPNB30	"Classic" Straight Multi-Unit SV-WP H 3 mm (M1.4)
46	MUL-SV- WPNB40	"Classic" Straight Multi-Unit SV-WP H 4 mm (M1.4)
47	MUL-SV- WPNB50	"Classic" Straight Multi-Unit SV-WP H 5 mm (M1.4)
48	MUL-SV- WPNB60	"Classic" Straight Multi-Unit SV-WP H 6 mm (M 1.4)
49	MUL-SV- WPNB1710	"Classic" Angled Multi-Unit SV-WP 17°/ H 1 mm (M 1.4)
50	MUL-SV- WPNB1720	"Classic" Angled Multi-Unit SV-WP 17°/ H 2 mm (M1.4)
51	MUL-SV- WPNB1730	"Classic" Angled Multi-Unit SV-WP 17°/ H 3 mm (M 1.4)
52	MUL-SV- WPNB3010	"Classic" Angled Multi-Unit SV-WP 30°/ H 1 mm (M 1.4)
53	MUL-SV- WPNB3020	"Classic" Angled Multi-Unit SV-WP 30°/ H 2 mm (M 1.4)
54	MUL-SV- WPNB3030	"Classic" Angled Multi-Unit SV-WP 30°/ H 3 mm (M 1.4)
55	MUL-SV- WPKS10	Multi-Clicq Straight Multi-Unit SV-WP H 1 mm
56	MUL-SV- WPKS20	Multi-Clicq Straight Multi-Unit SV-WP H 2 mm
57	MUL-SV- WPKS30	Multi-Clicq Straight Multi-Unit SV-WP H 3 mm

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

		
Multi-Clicq On-X (EM)		
No	REF/ Productname	Product Description
58	MUL-SV-WPKS40	Multi-Clicq Straight Multi-Unit SV-WP H 4 mm
59	MUL-SV-WPKS50	Multi-Clicq Straight Multi-Unit SV-WP H 5 mm
60	MUL-SV-WPKS60	Multi-Clicq Straight Multi-Unit SV-WP H 6 mm
61	MUL-SV-WPKS1710	Multi-Clicq Angled Multi-Unit SV-WP 17°/ H 1 mm
62	MUL-SV-WPKS1720	Multi-Clicq Angled Multi-Unit SV-WP 17°/ H 2 mm
63	MUL-SV-WPKS3010	Multi-Clicq Angled Multi-Unit SV-WP 30°/ H 1 mm
64	MUL-SV-WPKS3010	Multi-Clicq Angled Multi-Unit SV-WP 30°/ H 2 mm
65	MUL-SV-WPKS1710H	Multi-Clicq Angled Multi-Unit SV-WP 17°/ H 1 mm A.R
66	MUL-SV-WPKS1720H	Multi-Clicq Angled Multi-Unit SV-WP 17°/ H 2 mm A.R
67	MUL-SV-WPKS3010H	Multi-Clicq Angled Multi-Unit SV-WP 30°/ H 1 mm A.R
68	MUL-SV-WPKS3020H	Multi-Clicq Angled Multi-Unit SV-WP 30°/ H 2 mm A.R
69	MUL-SV-NLNB10	"Classic" Straight Multi-Unit SV-NL H 1 mm (M1.4)
70	MUL-SV-NLNB20	"Classic" Straight Multi-Unit SV-NL H 2 mm (M1.4)
71	MUL-SV-NLNB30	"Classic" Straight Multi-Unit SV-NL H 3 mm (M1.4)
72	MUL-SV-NLNB40	"Classic" Straight Multi-Unit SV-NL H 4 mm (M1.4)
73	MUL-SV-NLNB50	"Classic" Straight Multi-Unit SV-NL H 5 mm (M1.4)
74	MUL-SV-NLNB60	"Classic" Straight Multi-Unit SV-NL H 6 mm (M1.4)
75	MUL-SV-NLNB1710	"Classic" Angled Multi-Unit SV-NL 17°/ H 1 mm (M1.4)
76	MUL-SV-NLNB1720	"Classic" Angled Multi-Unit SV-NL 17°/ H 2 mm (M1.4)

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

		
Multi-Clicq On-X (EM)		
No	REF/ Productname	Product Description
77	MUL-SV-NLNB1730	"Classic" Angled Multi-Unit SV-NL 17°/ H 3 mm (M1.4)
78	MUL-SV-NLNB3010	"Classic" Angled Multi-Unit SV-NL 30°/ H 1 mm (M1.4)
79	MUL-SV-NLNB3020	"Classic" Angled Multi-Unit SV-NL 30°/ H 2 mm (M1.4)
80	MUL-SV-NLNB3030	"Classic" Angled Multi-Unit SV-NL 30°/ H 3 mm (M1.4)
81	MUL-SV-NLKS10	Multi-Clicq Straight Multi-Unit SV-NL H 1 mm
82	MUL-SV-NLKS20	Multi-Clicq Straight Multi-Unit SV-NL H 2 mm
83	MUL-SV-NLKS30	Multi-Clicq Straight Multi-Unit SV-NL H 3 mm
84	MUL-SV-NLKS40	Multi-Clicq Straight Multi-Unit SV-NL H 4 mm
85	MUL-SV-NLKS50	Multi-Clicq Straight Multi-Unit SV-NL H 5 mm
86	MUL-SV-NLKS60	Multi-Clicq Straight Multi-Unit SV-NL H 6 mm
87	MUL-SV-NLKS1710	Multi-Clicq Angled Multi-Unit SV-NL 17°/ H 1 mm A Type
88	MUL-SV-NLKS1720	Multi-Clicq Angled Multi-Unit SV-NL 17°/ H 2 mm A Type
89	MUL-SV-NLKS1730	Multi-Clicq Angled Multi-Unit SV-NL 17°/ H 3 mm A Type
90	MUL-SV-NLKS1740	Multi-Clicq Angled Multi-Unit SV-NL 17°/ H 4 mm A Type
91	MUL-SV-NLKS1710B	Multi-Clicq Angled Multi-Unit SV-NL 17°/ H 1 mm B Type
92	MUL-SV-NLKS1720B	Multi-Clicq Angled Multi-Unit SV-NL 17°/ H 2 mm B Type
93	MUL-SV-NLKS1730B	Multi-Clicq Angled Multi-Unit SV-NL 17°/ H 3 mm B Type
94	MUL-SV-NLKS1740B	Multi-Clicq Angled Multi-Unit SV-NL 17°/ H 4 mm B Type
95	MUL-SV-NLKS3010	Multi-Clicq Angled Multi-Unit SV-NL 30°/ H 1 mm A Type

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

 Multi-Clicq On-X (EM)		
No	REF/ Productname	Product Description
96	MUL-SV-NLKS3020	Multi-Clicq Angled Multi-Unit SV-NL 30°/ H 2 mm A Type
97	MUL-SV-NLKS3030	Multi-Clicq Angled Multi-Unit SV-NL 30°/ H 3 mm A Type
98	MUL-SV-NLKS3040	Multi-Clicq Angled Multi-Unit SV-NL 30°/ H 4 mm A Type
99	MUL-SV-NLKS3010B	Multi-Clicq Angled Multi-Unit SV-NL 30°/ H 1 mm B Type
100	MUL-SV-NLKS3020B	Multi-Clicq Angled Multi-Unit SV-NL 30°/ H 2 mm B Type
101	MUL-SV-NLKS3030B	Multi-Clicq Angled Multi-Unit SV-NL 30°/ H 3 mm B Type
102	MUL-SV-NLKS3040B	Multi-Clicq Angled Multi-Unit SV-NL 30°/ H 4 mm B Type
103	MUL-SV-NLKS1710H	Multi-Clicq Angled Multi-Unit SV-NL 17°/ H 1 mm A.R
104	MUL-SV-NLKS1720H	Multi-Clicq Angled Multi-Unit SV-NL 17°/ H 2 mm A.R
105	MUL-SV-NLKS1730H	Multi-Clicq Angled Multi-Unit SV-NL 17°/ H 3 mm A.R
106	MUL-SV-NLKS1740H	Multi-Clicq Angled Multi-Unit SV-NL 17°/ H 4 mm A.R
107	MUL-SV-NLKS3010H	Multi-Clicq Angled Multi-Unit SV-NL 30°/ H 1 mm A.R
108	MUL-SV-NLKS3020H	Multi-Clicq Angled Multi-Unit SV-NL 30°/ H 2 mm A.R
109	MUL-SV-NLKS3030H	Multi-Clicq Angled Multi-Unit SV-NL 30°/ H 3 mm A.R
110	MUL-SV-NLKS3040H	Multi-Clicq Angled Multi-Unit SV-NL 30°/ H 4 mm A.R
111	MUL-SV-NPNB10	"Classic" Straight Multi-Unit SV-NP H 1 mm (M1.4)
112	MUL-SV-NPNB20	"Classic" Straight Multi-Unit SV-NP H 2 mm (M1.4)
113	MUL-SV-NPNB30	"Classic" Straight Multi-Unit SV-NP H 3 mm (M1.4)


Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

		
Multi-Clicq On-X (EM)		
No	REF/ Productname	Product Description
114	MUL-SV-NPNB40	"Classic" Straight Multi-Unit SV-NP H 4 mm (M1.4)
115	MUL-SV-NPNB50	"Classic" Straight Multi-Unit SV-NP H 5 mm (M1.4)
116	MUL-SV-NPNB60	"Classic" Straight Multi-Unit SV-NP H 6 mm (M1.4)
117	MUL-SV-NPNB1710	"Classic" Angled Multi-Unit SV-NP 17°/ H 1 mm
118	MUL-SV-NPNB1720	"Classic" Angled Multi-Unit SV-NP 17°/ H 2 mm
119	MUL-SV-NPNB1730	"Classic" Angled Multi-Unit SV-NP 17°/ H 3 mm
120	MUL-SV-NPNB3010	"Classic" Angled Multi-Unit SV-NP 30°/ H 1 mm
121	MUL-SV-NPNB3020	"Classic" Angled Multi-Unit SV-NP 30°/ H 2 mm
122	MUL-SV-NPNB3030	"Classic" Angled Multi-Unit SV-NP 30°/ H 3 mm
123	MUL-SV-NPKS10	Multi-Clicq Straight Multi-Unit SV-NP H 1 mm
124	MUL-SV-NPKS20	Multi-Clicq Straight Multi-Unit SV-NP H 2 mm
125	MUL-SV-NPKS30	Multi-Clicq Straight Multi-Unit SV-NP H 3 mm
126	MUL-SV-NPKS40	Multi-Clicq Straight Multi-Unit SV-NP H 4 mm
127	MUL-SV-NPKS50	Multi-Clicq Straight Multi-Unit SV-NP H 5 mm
128	MUL-SV-NPKS60	Multi-Clicq Straight Multi-Unit SV-NP H 6 mm
129	MUL-SV-NPKS1710	Multi-Clicq Angled Multi-Unit SV-NP 17°/ H 1 mm A Type
130	MUL-SV-NPKS1720	Multi-Clicq Angled Multi-Unit SV-NP 17°/ H 2 mm A Type
131	MUL-SV-NPKS1730	Multi-Clicq Angled Multi-Unit SV-NP 17°/ H 3 mm A Type
132	MUL-SV-NPKS1740	Multi-Clicq Angled Multi-Unit SV-NP 17°/ H 4 mm A Type
133	MUL-SV-NPKS1710B	Multi-Clicq Angled Multi-Unit SV-NP 17°/ H 1 mm B Type

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

 Multi-Clicq On-X (EM)		
No	REF/ Productname	Product Description
134	MUL-SV- NPKS1720B	Multi-Clicq Angled Multi-Unit SV-NP 17°/ H 2 mm B Type
135	MUL-SV- NPKS1730B	Multi-Clicq Angled Multi-Unit SV-NP 17°/ H 3 mm B Type
136	MUL-SV- NPKS1740B	Multi-Clicq Angled Multi-Unit SV-NP 17°/ H 4 mm B Type
137	MUL-SV- NPKS3010	Multi-Clicq Angled Multi-Unit SV-NP 30°/ H 1 mm A Type
138	MUL-SV- NPKS3020	Multi-Clicq Angled Multi-Unit SV-NP 30°/ H 2 mm A Type
139	MUL-SV- NPKS3030	Multi-Clicq Angled Multi-Unit SV-NP 30°/ H 3 mm A Type
140	MUL-SV- NPKS3040	Multi-Clicq Angled Multi-Unit SV-NP 30°/ H 4 mm A Type
141	MUL-SV- NPKS3010B	Multi-Clicq Angled Multi-Unit SV-NP 30°/ H 1 mm B Type
142	MUL-SV- NPKS3020B	Multi-Clicq Angled Multi-Unit SV-NP 30°/ H 2 mm B Type
143	MUL-SV- NPKS3030B	Multi-Clicq Angled Multi-Unit SV-RP 30°/ H 3 mm B Type
144	MUL-SV- NPKS3040B	Multi-Clicq Angled Multi-Unit SV-NP 30°/ H 4 mm B Type
145	MUL-SV- NPKS1710H	Multi-Clicq Angled Multi-Unit SV-NP 17°/ H 1 mm A.R
146	MUL-SV- NPKS1720H	Multi-Clicq Angled Multi-Unit SV-NP 17°/ H 2 mm A.R
147	MUL-SV- NPKS1730H	Multi-Clicq Angled Multi-Unit SV-NP 17°/ H 3 mm A.R
148	MUL-SV- NPKS1740H	Multi-Clicq Angled Multi-Unit SV-NP 17°/ H 4 mm A.R
149	MUL-SV- NPKS3010H	Multi-Clicq Angled Multi-Unit SV-NP 30°/ H 1 mm A.R
150	MUL-SV- NPKS3020H	Multi-Clicq Angled Multi-Unit SV-NP 30°/ H 2 mm A.R


Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

		
Multi-Clicq On-X (EM)		
No	REF/ Productname	Product Description
151	MUL-SV- NPKS3030H	Multi-Clicq Angled Multi-Unit SV-NP 30°/ H 3 mm A.R
152	MUL-SV- NPKS3040H	Multi-Clicq Angled Multi-Unit SV-NP 30°/ H 4 mm A.R
153	MUL-NA-NPNB15	"Classic" Straight Multi-Unit NA-NP H 1.5 mm (M 1.4)
154	MUL-NA-NPNB25	"Classic" Straight Multi-Unit NA-NP H 2.5 mm (M 1.4)
155	MUL-NA-NPNB35	"Classic" Straight Multi-Unit NA-NP H 3.5 mm (M 1.4)
156	MUL-NA-NPNB45	"Classic" Straight Multi-Unit NA-NP H 4.5 mm (M 1.4)
157	MUL-NA-NPNB55	"Classic" Straight Multi-Unit NA-NP H 5.5 mm (M 1.4)
158	MUL-NA- NPNB1710	"Classic" Angled Multi-Unit NA-NP 17°/ H 1 mm (M 1.4)
159	MUL-NA- NPNB1720	"Classic" Angled Multi-Unit NA-NP 17°/ H 2 mm (M 1.4)
160	MUL-NA- NPNB1730	"Classic" Angled Multi-Unit NA-NP 17°/ H 3 mm (M 1.4)
161	MUL-NA- NPNB3010	"Classic" Angled Multi-Unit NA-NP 30°/ H 1 mm (M 1.4)
162	MUL-NA- NPNB3020	"Classic" Angled Multi-Unit NA-NP 30°/ H 2 mm (M 1.4)
163	MUL-NA- NPNB3030	"Classic" Angled Multi-Unit NA-NP 30°/ H 3 mm (M 1.4)
164	MUL-NA-NPKS15	Multi-Clicq Straight Multi-Unit NA-NP H 1.5 mm
165	MUL-NA-NPKS25	Multi-Clicq Straight Multi-Unit NA-NP H 2.5 mm
166	MUL-NA-NPKS35	Multi-Clicq Straight Multi-Unit NA-NP H 3.5 mm
167	MUL-NA-NPKS45	Multi-Clicq Straight Multi-Unit NA-NP H 4.5 mm
168	MUL-NA-NPKS55	Multi-Clicq Straight Multi-Unit NA-NP H 5.5 mm
169	MUL-NA- NPKS1710	Multi-Clicq Angled Multi-Unit NA-NP 17°/ H 1 mm A Type
170	MUL-NA- NPKS1720	Multi-Clicq Angled Multi-Unit NA-NP 17°/ H 2 mm A Type
171	MUL-NA- NPKS1730	Multi-Clicq Angled Multi-Unit NA-NP 17°/ H 3 mm A Type

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

 Multi-Clicq On-X (EM)		
No	REF/ Productname	Product Description
172	MUL-NA-NPKS1740	Multi-Clicq Angled Multi-Unit NA-NP 17°/ H 4 mm A Type
173	MUL-NA-NPKS1710B	Multi-Clicq Angled Multi-Unit NA-NP 17°/ H 1 mm B Type
174	MUL-NA-NPKS1720B	Multi-Clicq Angled Multi-Unit NA-NP 17°/ H 2 mm B Type
175	MUL-NA-NPKS1730B	Multi-Clicq Angled Multi-Unit NA-NP 17°/ H 3 mm B Type
176	MUL-NA-NPKS1740B	Multi-Clicq Angled Multi-Unit NA-NP 17°/ H 4 mm B Type
177	MUL-NA-NPKS3010	Multi-Clicq Angled Multi-Unit NA-NP 30°/ H 1 mm A Type
178	MUL-NA-NPKS3020	Multi-Clicq Angled Multi-Unit NA-NP 30°/ H 2 mm A Type
179	MUL-NA-NPKS3030	Multi-Clicq Angled Multi-Unit NA-NP 30°/ H 3 mm A Type
180	MUL-NA-NPKS3040	Multi-Clicq Angled Multi-Unit NA-NP 30°/ H 4 mm A Type
181	MUL-NA-NPKS3010B	Multi-Clicq Angled Multi-Unit NA-NP 30°/ H 1 mm B Type
182	MUL-NA-NPKS3020B	Multi-Clicq Angled Multi-Unit NA-NP 30°/ H 2 mm B Type
183	MUL-NA-NPKS3030B	Multi-Clicq Angled Multi-Unit NA-NP 30°/ H 3 mm B Type
184	MUL-NA-NPKS3040B	Multi-Clicq Angled Multi-Unit NA-NP 30°/ H 4 mm B Type
185	MUL-NA-NPKS1710H	Multi-Clicq Angled Multi-Unit NA-NP 17°/ H 1 mm A.R
186	MUL-NA-NPKS1720H	Multi-Clicq Angled Multi-Unit NA-NP 17°/ H 2 mm A.R
187	MUL-NA-NPKS1730H	Multi-Clicq Angled Multi-Unit NA-NP 17°/ H 3 mm A.R
188	MUL-NA-NPKS1740H	Multi-Clicq Angled Multi-Unit NA-NP 17°/ H 4 mm A.R

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

		
Multi-Clicq On-X (EM)		
No	REF/ Productname	Product Description
189	MUL-NA-NPKS3010H	Multi-Clicq Angled Multi-Unit NA-NP 30°/ H 1 mm A.R
190	MUL-NA-NPKS3020H	Multi-Clicq Angled Multi-Unit NA-NP 30°/ H 2 mm A.R
191	MUL-NA-NPKS3030H	Multi-Clicq Angled Multi-Unit NA-NP 30°/ H 3 mm A.R
192	MUL-NA-NPKS3040H	Multi-Clicq Angled Multi-Unit NA-NP 30°/ H 4 mm A.R
193	MUL-NA-RPNB15	"Classic" Straight Multi-Unit NA-RP H 1.5 mm
194	MUL-NA-RPNB25	"Classic" Straight Multi-Unit NA-RP H 2.5 mm
195	MUL-NA-RPNB35	"Classic" Straight Multi-Unit NA-RP H 3.5 mm
196	MUL-NA-RPNB45	"Classic" Straight Multi-Unit NA-RP H 4.5 mm
197	MUL-NA-RPNB55	"Classic" Straight Multi-Unit NA-RP H 5.5 mm
198	MUL-NA-RPNB1710	"Classic" Angled Multi-Unit NA-RP 17°/ H 1 mm (M 1.4)
199	MUL-NA-RPNB1720	"Classic" Angled Multi-Unit NA-RP 17°/ H 2 mm (M 1.4)
200	MUL-NA-RPNB1730	"Classic" Angled Multi-Unit NA-RP 17°/ H 3 mm (M 1.4)
201	MUL-NA-RPNB3010	"Classic" Angled Multi-Unit NA-RP 30°/ H 1 mm (M 1.4)
202	MUL-NA-RPNB3020	"Classic" Angled Multi-Unit NA-RP 30°/ H 2 mm (M 1.4)
203	MUL-NA-RPNB3030	"Classic" Angled Multi-Unit NA-RP 30°/ H 3 mm (M 1.4)
204	MUL-NA-RPKS15	Multi-Clicq Straight Multi-Unit NA-RP H 1.5 mm
205	MUL-NA-RPKS25	Multi-Clicq Straight Multi-Unit NA-RP H 2.5 mm
206	MUL-NA-RPKS35	Multi-Clicq Straight Multi-Unit NA-RP H 3.5 mm
207	MUL-NA-RPKS45	Multi-Clicq Straight Multi-Unit NA-RP H 4.5 mm
208	MUL-NA-RPKS55	Multi-Clicq Straight Multi-Unit NA-RP H 5.5 mm
209	MUL-NA-RPKS1710	Multi-Clicq Angled Multi-Unit NA-RP 17°/ H 1 mm A Type

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

		
Multi-Clicq On-X (EM)		
No	REF/ Productname	Product Description
210	MUL-NA-RPKS1720	Multi-Clicq Angled Multi-Unit NA-RP 17°/ H 2 mm A Type
211	MUL-NA-RPKS1730	Multi-Clicq Angled Multi-Unit NA-RP 17°/ H 3 mm A Type
212	MUL-NA-RPKS1740	Multi-Clicq Angled Multi-Unit NA-RP 17°/ H 4 mm A Type
213	MUL-NA-RPKS1710B	Multi-Clicq Angled Multi-Unit NA-RP 17°/ H 1 mm B Type
214	MUL-NA-RPKS1720B	Multi-Clicq Angled Multi-Unit NA-RP 17°/ H 2 mm B Type
215	MUL-NA-RPKS1730B	Multi-Clicq Angled Multi-Unit NA-RP 17°/ H 3 mm B Type
216	MUL-NA-RPKS1740B	Multi-Clicq Angled Multi-Unit NA-RP 17°/ H 4 mm B Type
217	MUL-NA-RPKS3010	Multi-Clicq Angled Multi-Unit NA-RP 30°/ H 1 mm A Type
218	MUL-NA-RPKS3020	Multi-Clicq Angled Multi-Unit NA-RP 30°/ H 2 mm A Type
219	MUL-NA-RPKS3030	Multi-Clicq Angled Multi-Unit NA-RP 30°/ H 3 mm A Type
220	MUL-NA-RPKS3040	Multi-Clicq Angled Multi-Unit NA-RP 30°/ H 4 mm A Type
221	MUL-NA-RPKS3010B	Multi-Clicq Angled Multi-Unit NA-RP 30°/ H 1 mm B Type
222	MUL-NA-RPKS3020B	Multi-Clicq Angled Multi-Unit NA-RP 30°/ H 2 mm B Type
223	MUL-NA-RPKS3030B	Multi-Clicq Angled Multi-Unit NA-RP 30°/ H 3 mm B Type
224	MUL-NA-RPKS3040B	Multi-Clicq Angled Multi-Unit NA-RP 30°/ H 4 mm B Type
225	MUL-NA-RPKS1710H	Multi-Clicq Angled Multi-Unit NA-RP 17°/ H 1 mm A.R
226	MUL-NA-RPKS1720H	Multi-Clicq Angled Multi-Unit NA-RP 17°/ H 2 mm A.R

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

		
Multi-Clicq On-X (EM)		
No	REF/ Productname	Product Description
227	MUL-NA- RPKS1730H	Multi-Clicq Angled Multi-Unit NA-RP 17°/ H 3 mm A.R
228	MUL-NA- RPKS1740H	Multi-Clicq Angled Multi-Unit NA-RP 17°/ H 4 mm A.R
229	MUL-NA- RPKS3010H	Multi-Clicq Angled Multi-Unit NA-RP 30°/ H 1 mm A.R
230	MUL-NA- RPKS3020H	Multi-Clicq Angled Multi-Unit NA-RP 30°/ H 2 mm A.R
231	MUL-NA- RPKS3030H	Multi-Clicq Angled Multi-Unit NA-RP 30°/ H 3 mm A.R
232	MUL-NA- RPKS3040H	Multi-Clicq Angled Multi-Unit NA-RP 30°/ H 4 mm A.R
233	MUL-TS-NPNB15	"Classic" Straight Multi-Unit TS-NP H 1.5 mm (M1.4)
234	MUL-TS-NPNB25	"Classic" Straight Multi-Unit TS-NP H 2.5 mm (M1.4)
235	MUL-TS-NPNB35	"Classic" Straight Multi-Unit TS-NP H 3.5 mm (M1.4)
236	MUL-TS-NPNB45	"Classic" Straight Multi-Unit TS-NP H 4.5 mm (M1.4)
237	MUL-TS-NPNB5	"Classic" Straight Multi-Unit TS-NP H 5.5 mm (M1.4)
238	MUL-TS- NPNB1710	"Classic" Straight Multi-Unit TS-NP 17°/ H 1 mm (M1.4)
239	MUL-TS- NPNB1720	"Classic" Straight Multi-Unit TS-NP 17°/ H 2 mm (M1.4)
240	MUL-TS- NPNB1730	"Classic" Straight Multi-Unit TS-NP 17°/ H 3 mm (M1.4)
241	MUL-TS- NPNB3010	"Classic" Straight Multi-Unit TS-NP 30°/ H 1 mm (M1.4)
242	MUL-TS- NPNB3020	"Classic" Straight Multi-Unit TS-NP 30°/ H 2 mm (M1.4)
243	MUL-TS- NPNB3030	"Classic" Straight Multi-Unit TS-NP 30°/ H 3 mm (M1.4)
244	MUL-TS-NPKS15	Multi-Clicq Straight Multi-Unit TS-NP/ Mini H 1.5 mm
245	MUL-TS-NPKS25	Multi-Clicq Straight Multi-Unit TS-NP/ Mini H 2.5 mm
246	MUL-TS-NPKS35	Multi-Clicq Straight Multi-Unit TS-NP/ Mini H 3.5 mm

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

		
Multi-Clicq On-X (EM)		
No	REF/ Productname	Product Description
247	MUL-TS-NPKS45	Multi-Clicq Straight Multi-Unit TS-NP/ Mini H 4.5 mm
248	MUL-TS-NPKS55	Multi-Clicq Straight Multi-Unit TS-NP/ Mini H 5.5 mm
249	MUL-TS-NPKS1710	Multi-Clicq Angled Multi-Unit TS-NP/ Mini 17°/ H 1 mm A Type
250	MUL-TS-NPKS1720	Multi-Clicq Angled Multi-Unit TS-NP/ Mini 17°/ H 2 mm A Type
251	MUL-TS-NPKS1730	Multi-Clicq Angled Multi-Unit TS-NP/ Mini 17°/ H 3 mm A Type
252	MUL-TS-NPKS1740	Multi-Clicq Angled Multi-Unit TS-NP/ Mini 17°/ H 4 mm A Type
253	MUL-TS-NPKS1710B	Multi-Clicq Angled Multi-Unit TS-NP/ Mini 17°/ H 1 mm B Type
254	MUL-TS-NPKS1720B	Multi-Clicq Angled Multi-Unit TS-NP/ Mini 17°/ H 2 mm B Type
255	MUL-TS-NPKS1730B	Multi-Clicq Angled Multi-Unit TS-NP/ Mini 17°/ H 3 mm B Type
256	MUL-TS-NPKS1740B	Multi-Clicq Angled Multi-Unit TS-NP/ Mini 17°/ H 4 mm B Type
257	MUL-TS-NPKS3010	Multi-Clicq Angled Multi-Unit TS-NP/ Mini 30°/ H 1 mm A Type
258	MUL-TS-NPKS3020	Multi-Clicq Angled Multi-Unit TS-NP/ Mini 30°/ H 2 mm A Type
259	MUL-TS-NPKS3030	Multi-Clicq Angled Multi-Unit TS-NP/ Mini 30°/ H 3 mm A Type
260	MUL-TS-NPKS3040	Multi-Clicq Angled Multi-Unit TS-NP/ Mini 30°/ H 4 mm A Type
261	MUL-TS-NPKS3010B	Multi-Clicq Angled Multi-Unit TS-NP/ Mini 30°/ H 1 mm B Type
262	MUL-TS-NPKS3020B	Multi-Clicq Angled Multi-Unit TS-NP/ Mini 30°/ H 2 mm B Type
263	MUL-TS-NPKS3030B	Multi-Clicq Angled Multi-Unit TS-NP/ Mini 30°/ H 3 mm B Type
264	MUL-TS-NPKS3040B	Multi-Clicq Angled Multi-Unit TS-NP/ Mini 30°/ H 4 mm B Type


Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

		
Multi-Clicq On-X (EM)		
No	REF/ Productname	Product Description
265	MUL-TS-NPKS1710H	Multi-Clicq Angled Multi-Unit TS-NP/ Mini 17°/ H 1 mm A.R
266	MUL-TS-NPKS1720H	Multi-Clicq Angled Multi-Unit TS-NP/ Mini 17°/ H 2 mm A.R
267	MUL-TS-NPKS1730H	Multi-Clicq Angled Multi-Unit TS-NP/ Mini 17°/ H 3 mm A.R
268	MUL-TS-NPKS1740H	Multi-Clicq Angled Multi-Unit TS-NP/ Mini 17°/ H 4 mm A.R
269	MUL-TS-NPKS3010H	Multi-Clicq Angled Multi-Unit TS-NP/ Mini 30°/ H 1 mm A.R
270	MUL-TS-NPKS3020H	Multi-Clicq Angled Multi-Unit TS-NP/ Mini 30°/ H 2 mm A.R
271	MUL-TS-NPKS3030H	Multi-Clicq Angled Multi-Unit TS-NP/ Mini 30°/ H 3 mm A.R
272	MUL-TS-NPKS3040H	Multi-Clicq Angled Multi-Unit TS-NP/ Mini 30°/ H 4 mm A.R
273	MUL-TS-RPNB15	"Classic" Straight Multi-Unit TS-RP H 1.5 mm (M 1.4)
274	MUL-TS-RPNB25	"Classic" Straight Multi-Unit TS-RP H 2.5 mm (M 1.4)
275	MUL-TS-RPNB35	"Classic" Straight Multi-Unit TS-RP H 3.5 mm (M 1.4)
276	MUL-TS-RPNB45	"Classic" Straight Multi-Unit TS-RP H 4.5 mm (M 1.4)
277	MUL-TS-RPNB55	"Classic" Straight Multi-Unit TS-RP H 5.5 mm (M 1.4)
278	MUL-TS-RPNB1710	"Classic" Angled Multi-Unit TS-RP 17°/ H 1 mm (M 1.4)
279	MUL-TS-RPNB1720	"Classic" Angled Multi-Unit TS-RP 17°/ H 2 mm (M 1.4)
280	MUL-TS-RPNB1730	"Classic" Angled Multi-Unit TS-RP 17°/ H 3 mm (M 1.4)
281	MUL-TS-RPNB3010	"Classic" Angled Multi-Unit TS-RP 30°/ H 1 mm (M 1.4)
282	MUL-TS-RPNB3020	"Classic" Angled Multi-Unit TS-RP 30°/ H 2 mm (M 1.4)
283	MUL-TS-RPNB3030	"Classic" Angled Multi-Unit TS-RP 30°/ H 3 mm (M 1.4)


Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

 Multi-Clicq On-X (EM)		
No	REF/ Productname	Product Description
284	MUL-TS-RPKS15	Multi-Clicq Straight Multi-Unit TS-RP H 1.5 mm
285	MUL-TS-RPKS25	Multi-Clicq Straight Multi-Unit TS-RP H 2.5 mm
286	MUL-TS-RPKS35	Multi-Clicq Straight Multi-Unit TS-RP H 3.5 mm
287	MUL-TS-RPKS45	Multi-Clicq Straight Multi-Unit TS-RP H 4.5 mm
288	MUL-TS-RPKS55	Multi-Clicq Straight Multi-Unit TS-RP H 5.5 mm
289	MUL-TS-RPKS1710	Multi-Clicq Straight Multi-Unit TS-RP 17°/ H 1 mm A Type
290	MUL-TS-RPKS1720	Multi-Clicq Angled Multi-Unit TS-RP 17°/ H 2 mm A Type
291	MUL-TS-RPKS1730	Multi-Clicq Angled Multi-Unit TS-RP 17°/ H 3 mm A Type
292	MUL-TS-RPKS1740	Multi-Clicq Angled Multi-Unit TS-RP 17°/ H 4 mm A Type
293	MUL-TS-RPKS1710B	Multi-Clicq Angled Multi-Unit TS-RP 17°/ H 1 mm B Type
294	MUL-TS-RPKS1720B	Multi-Clicq Angled Multi-Unit TS-RP 17°/ H 2 mm B Type
295	MUL-TS-RPKS1730B	Multi-Clicq Angled Multi-Unit TS-RP 17°/ H 3 mm B Type
296	MUL-TS-RPKS1740B	Multi-Clicq Angled Multi-Unit TS-RP 17°/ H 4 mm B Type
297	MUL-TS-RPKS3010	Multi-Clicq Angled Multi-Unit TS-RP 30°/ H 1 mm A Type
298	MUL-TS-RPKS3020	Multi-Clicq Angled Multi-Unit TS-RP 30°/ H 2 mm A Type
299	MUL-TS-RPKS3030	Multi-Clicq Angled Multi-Unit TS-RP 30°/ H 3 mm A Type
300	MUL-TS-RPKS3040	Multi-Clicq Angled Multi-Unit TS-RP 30°/ H 4 mm A Type
301	MUL-TS-RPKS3010B	Multi-Clicq Angled Multi-Unit TS-RP 30°/ H 1 mm B Type
302	MUL-TS-RPKS3020B	Multi-Clicq Angled Multi-Unit TS-RP 30°/ H 2 mm B Type


Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

		
Multi-Clicq On-X (EM)		
No	REF/ Productname	Product Description
303	MUL-TS-RPKS3030B	Multi-Clicq Angled Multi-Unit TS-RP 30°/ H 3 mm B Type
304	MUL-TS-RPKS3040B	Multi-Clicq Angled Multi-Unit TS-RP 30°/ H 4 mm B Type
305	MUL-TS-RPKS1710H	Multi-Clicq Angled Multi-Unit TS-RP 17°/ H 1 mm A.Rotational
306	MUL-TS-RPKS1720H	Multi-Clicq Angled Multi-Unit TS-RP 17°/ H 2 mm A.Rotational
307	MUL-TS-RPKS1730H	Multi-Clicq Angled Multi-Unit TS-RP 17°/ H 3 mm A.Rotational
308	MUL-TS-RPKS1740H	Multi-Clicq Angled Multi-Unit TS-RP 17°/ H 4 mm A.Rotational
309	MUL-TS-RPKS3010H	Multi-Clicq Angled Multi-Unit TS-RP 30°/ H 1 mm A.Rotational
310	MUL-TS-RPKS3020H	Multi-Clicq Angled Multi-Unit TS-RP 30°/ H 2 mm A.Rotational
311	MUL-TS-RPKS3030H	Multi-Clicq Angled Multi-Unit TS-RP 30°/ H 3 mm A.Rotational
312	MUL-TS-RPKS3040H	Multi-Clicq Angled Multi-Unit TS-RP 30°/ H 4 mm A.Rotational
313	MUL-OS-SPNB15	"Classic" Straight Multi-Unit OS-SP H 1.5 mm (M 1.4)
314	MUL-OS-SPNB25	"Classic" Straight Multi-Unit OS-SP H 2.5 mm (M 1.4)
315	MUL-OS-SPNB35	"Classic" Straight Multi-Unit OS-SP H 3.5 mm (M 1.4)
316	MUL-OS-SPNB45	"Classic" Straight Multi-Unit OS-SP H 4.5 mm (M 1.4)
317	MUL-OS-SPNB55	"Classic" Straight Multi-Unit OS-SP H 5.5 mm (M 1.4)
318	MUL-OS-SPNB1710	"Classic" Angled Multi-Unit OS-SP 17° H 1 mm (M 1.4)
319	MUL-OS-SPNB1710	"Classic" Angled Multi-Unit OS-SP 17° H 2 mm (M 1.4)
320	MUL-OS-SPNB1720	"Classic" Angled Multi-Unit OS-SP 17° H 3 mm (M 1.4)
321	MUL-OS-SPNB3010	"Classic" Angled Multi-Unit OS-SP 30° H 1 mm (M 1.4)

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

 Multi-Clicq On-X (EM)		
No	REF/ Productname	Product Description
322	MUL-OS-SPNB3020	"Classic" Angled Multi-Unit OS-SP 30° H 2 mm (M 1.4)
323	MUL-OS-SPNB3030	"Classic" Angled Multi-Unit OS-SP 30° H 3 mm (M 1.4)
324	MUL-OS-SPKS15	Multi-Clicq Straight Multi-Unit OS-SP H 1.5 mm
325	MUL-OS-SPKS25	Multi-Clicq Straight Multi-Unit OS-SP H 2.5 mm
326	MUL-OS-SPKS35	Multi-Clicq Straight Multi-Unit OS-SP H 3.5 mm
327	MUL-OS-SPKS45	Multi-Clicq Straight Multi-Unit OS-SP H 4.5 mm
328	MUL-OS-SPKS55	Multi-Clicq Straight Multi-Unit OS-SP H 5.5 mm
329	MUL-OS-SPKS1710	Multi-Clicq Angled Multi-Unit OS-SP 17°/ H 1 mm Type A
330	MUL-OS-SPKS1720	Multi-Clicq Angled Multi-Unit OS-SP 17°/ H 2 mm Type A
331	MUL-OS-SPKS1730	Multi-Clicq Angled Multi-Unit OS-SP 17°/ H 3 mm Type A
332	MUL-OS-SPKS1740	Multi-Clicq Angled Multi-Unit OS-SP 17°/ H 4 mm Type A
333	MUL-OS-SPKS1710B	Multi-Clicq Angled Multi-Unit OS-SP 17°/ H 1 mm Type B
334	MUL-OS-SPKS1720B	Multi-Clicq Angled Multi-Unit OS-SP 17°/ H 2 mm Type B
335	MUL-OS-SPKS1730B	Multi-Clicq Angled Multi-Unit OS-SP 17°/ H 3 mm Type B
336	MUL-OS-SPKS1740B	Multi-Clicq Angled Multi-Unit OS-SP 17°/ H 4 mm Type B
337	MUL-OS-SPKS3010	Multi-Clicq Angled Multi-Unit OS-SP 30°/ H 1 mm Type A
338	MUL-OS-SPKS3020	Multi-Clicq Angled Multi-Unit OS-SP 30°/ H 2 mm Type A
339	MUL-OS-SPKS3030	Multi-Clicq Angled Multi-Unit OS-SP 30°/ H 3 mm Type A
340	MUL-OS-SPKS3040	Multi-Clicq Angled Multi-Unit OS-SP 30°/ H 4 mm Type A


Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

		
Multi-Clicq On-X (EM)		
No	REF/ Productname	Product Description
341	MUL-OS- SPKS3010B	Multi-Clicq Angled Multi-Unit OS-SP 30°/ H 1 mm Type B
342	MUL-OS- SPKS3020B	Multi-Clicq Angled Multi-Unit OS-SP 30°/ H 2 mm Type B
343	MUL-OS- SPKS3030B	Multi-Clicq Angled Multi-Unit OS-SP 30°/ H 3 mm Type B
344	MUL-OS- SPKS3040B	Multi-Clicq Angled Multi-Unit OS-SP 30°/ H 4 mm Type B
345	MUL-OS- SPKS1710H	Multi-Clicq Angled Multi-Unit OS-SP 17°/ H 1 mm A.Rotational
346	MUL-OS- SPKS1720H	Multi-Clicq Angled Multi-Unit OS-SP 17°/ H 2 mm A.Rotational
347	MUL-OS- SPKS1730H	Multi-Clicq Angled Multi-Unit OS-SP 17°/ H 3 mm A.Rotational
348	MUL-OS- SPKS1740H	Multi-Clicq Angled Multi-Unit OS-SP 17°/ H 4 mm A.Rotational
349	MUL-OS- SPKS3010H	Multi-Clicq Angled Multi-Unit OS-SP 30°/ H 1 mm A.Rotational
350	MUL-OS- SPKS3020H	Multi-Clicq Angled Multi-Unit OS-SP 30°/ H 2 mm A.Rotational
351	MUL-OS- SPKS3030H	Multi-Clicq Angled Multi-Unit OS-SP 30°/ H 3 mm A.Rotational
352	MUL-OS- SPKS3040H	Multi-Clicq Angled Multi-Unit OS-SP 30°/ H 4 mm A.Rotational
353	MUL-OS- WPNB15	"Classic" Straight Multi-Unit OS-WP H 1.5 mm (M 1.4)
354	MUL-OS- WPNB25	"Classic" Straight Multi-Unit OS-WP H 2.5 mm (M 1.4)
355	MUL-OS- WPNB35	"Classic" Straight Multi-Unit OS-WP H 3.5 mm (M 1.4)
356	MUL-OS- WPNB45	"Classic" Straight Multi-Unit OS-WP H 4.5 mm (M 1.4)
357	MUL-OS- WPNB55	"Classic" Straight Multi-Unit OS-WP H 5.5 mm (M 1.4)

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

		
Multi-Clicq On-X (EM)		
No	REF/ Productname	Product Description
358	MUL-OS- WPNB1710	"Classic" Angled Multi-Unit OS-WP 17°/ H 1 mm (M 1.4)
359	MUL-OS- WPNB1720	"Classic" Angled Multi-Unit OS-WP 17°/ H 2 mm (M 1.4)
360	MUL-OS- WPNB1730	"Classic" Angled Multi-Unit OS-WP 17°/ H 3 mm (M 1.4)
361	MUL-OS- WPNB3010	"Classic" Angled Multi-Unit OS-WP 30°/ H 1 mm (M 1.4)
362	MUL-OS- WPNB3020	"Classic" Angled Multi-Unit OS-WP 30°/ H 2 mm (M 1.4)
363	MUL-OS- WPNB3030	"Classic" Angled Multi-Unit OS-WP 30°/ H 3 mm (M 1.4)
364	MUL-OS- WPKS15	Multi-Clicq Straight Multi-Unit OS-WP H 1.5 mm
365	MUL-OS- WPKS25	Multi-Clicq Straight Multi-Unit OS-WP H 2.5 mm
366	MUL-OS- WPKS35	Multi-Clicq Straight Multi-Unit OS-WP H 3.5 mm
367	MUL-OS- WPKS45	Multi-Clicq Straight Multi-Unit OS-WP H 4.5 mm
368	MUL-OS- WPKS55	Multi-Clicq Straight Multi-Unit OS-WP H 5.5 mm
369	MUL-OS- WPKS1710	Multi-Clicq Angled Multi-Unit OS-WP 17°/ H 1 mm
370	MUL-OS- WPKS1720	Multi-Clicq Angled Multi-Unit OS-WP 17°/ H 2 mm
371	MUL-OS- WPKS3010	Multi-Clicq Angled Multi-Unit OS-WP 30°/ H 1 mm
372	MUL-OS- WPKS3020	Multi-Clicq Angled Multi-Unit OS-WP 30°/ H 2 mm
373	MUL-OS- WPKS1710H	Multi-Clicq Angled Multi-Unit OS-WP 17°/ H 1 mm A.R
374	MUL-OS- WPKS1720H	Multi-Clicq Angled Multi-Unit OS-WP 17°/ H 2 mm A.R

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

		
Multi-Clicq On-X (EM)		
No	REF/ Productname	Product Description
375	MUL-OS- WPKS3010H	Multi-Clicq Angled Multi-Unit OS-WP 30°/ H 1 mm A.R
376	MUL-OS- WPKS3020H	Multi-Clicq Angled Multi-Unit OS-WP 30°/ H 2 mm A.R
377	MUL-AR-RPNB15	"Classic" Angled Multi-Unit AR-RP H 1.5 mm ( M 1.4)
378	MUL-AR-RPNB25	"Classic" Angled Multi-Unit AR-RP H 2.5 mm ( M 1.4)
379	MUL-AR-RPNB35	"Classic" Angled Multi-Unit AR-RP H 3.5 mm ( M 1.4)
380	MUL-AR-RPNB45	"Classic" Angled Multi-Unit AR-RP H 4.5 mm ( M 1.4)
381	MUL-AR-RPNB55	"Classic" Angled Multi-Unit AR-RP H 5.5 mm ( M 1.4)
382	MUL-AR- RPNB1710	"Classic" Angled Multi-Unit AR-RP 17°/ H 1 mm ( M 1.4)
383	MUL-AR- RPNB1720	"Classic" Angled Multi-Unit AR-RP 17°/ H 2 mm ( M 1.4)
384	MUL-AR- RPNB1730	"Classic" Angled Multi-Unit AR-RP 17°/ H 3 mm ( M 1.4)
385	MUL-AR- RPNB3010	"Classic" Angled Multi-Unit AR-RP 30°/ H 1 mm ( M 1.4)
386	MUL-AR- RPNB3020	"Classic" Angled Multi-Unit AR-RP 30°/ H 2 mm ( M 1.4)
387	MUL-AR- RPNB3030	"Classic" Angled Multi-Unit AR-RP 30°/ H 3 mm ( M 1.4)
388	MUL-AR-RPKS15	Multi-Clicq Straight Multi-Unit AR-RP H 1.5 mm
389	MUL-AR-RPKS25	Multi-Clicq Straight Multi-Unit AR-RP H 2.5 mm
390	MUL-AR-RPKS35	Multi-Clicq Straight Multi-Unit AR-RP H 3.5 mm
391	MUL-AR-RPKS45	Multi-Clicq Straight Multi-Unit AR-RP H 4.5 mm
392	MUL-AR-RPKS55	Multi-Clicq Straight Multi-Unit AR-RP H 5.5 mm
393	MUL-AR- RPKS1710	Multi-Clicq Angled Multi-Unit AR-RP 17°/ H 1 mm A Type
394	MUL-AR- RPKS1720	Multi-Clicq Angled Multi-Unit AR-RP 17°/ H 2 mm A Type
395	MUL-AR- RPKS1730	Multi-Clicq Angled Multi-Unit AR-RP 17°/ H 3 mm A Type


Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

 Multi-Clicq On-X (EM)		
No	REF/ Productname	Product Description
396	MUL-AR-RPKS1740	Multi-Clicq Angled Multi-Unit AR-RP 17°/ H 4 mm A Type
397	MUL-AR-RPKS1710B	Multi-Clicq Angled Multi-Unit AR-RP 17°/ H 1 mm B Type
398	MUL-AR-RPKS1720B	Multi-Clicq Angled Multi-Unit AR-RP 17°/ H 2 mm B Type
399	MUL-AR-RPKS1730B	Multi-Clicq Angled Multi-Unit AR-RP 17°/ H 3 mm B Type
400	MUL-AR-RPKS1740B	Multi-Clicq Angled Multi-Unit AR-RP 17°/ H 4 mm B Type
401	MUL-AR-RPKS3010	Multi-Clicq Angled Multi-Unit AR-RP 30°/ H 1 mm A Type
402	MUL-AR-RPKS3020	Multi-Clicq Angled Multi-Unit AR-RP 30°/ H 2 mm A Type
403	MUL-AR-RPKS3030	Multi-Clicq Angled Multi-Unit AR-RP 30°/ H 3 mm A Type
404	MUL-AR-RPKS3040	Multi-Clicq Angled Multi-Unit AR-RP 30°/ H 4 mm A Type
405	MUL-AR-RPKS3010B	Multi-Clicq Angled Multi-Unit AR-RP 30°/ H 1 mm B Type
406	MUL-AR-RPKS3020B	Multi-Clicq Angled Multi-Unit AR-RP 30°/ H 2 mm B Type
407	MUL-AR-RPKS3030B	Multi-Clicq Angled Multi-Unit AR-RP 30°/ H 3 mm B Type
408	MUL-AR-RPKS3040B	Multi-Clicq Angled Multi-Unit AR-RP 30°/ H 4 mm B Type
409	MUL-AR-RPKS1710H	Multi-Clicq Angled Multi-Unit AR-RP 17°/ H 1 mm A.Rotational
410	MUL-AR-RPKS1720H	Multi-Clicq Angled Multi-Unit AR-RP 17°/ H 2 mm A.Rotational
411	MUL-AR-RPKS1730H	Multi-Clicq Angled Multi-Unit AR-RP 17°/ H 3 mm A.Rotational
412	MUL-AR-RPKS1740H	Multi-Clicq Angled Multi-Unit AR-RP 17°/ H 4 mm A.Rotational

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

		
Multi-Clicq On-X (EM)		
No	REF/ Productname	Product Description
413	MUL-AR- RPKS3010H	Multi-Clicq Angled Multi-Unit AR-RP 30°/ H 1 mm A.Rotational
414	MUL-AR- RPKS3020H	Multi-Clicq Angled Multi-Unit AR-RP 30°/ H 2 mm A.Rotational
415	MUL-AR- RPKS3030H	Multi-Clicq Angled Multi-Unit AR-RP 30°/ H 3 mm A.Rotational
416	MUL-AR- RPKS3040H	Multi-Clicq Angled Multi-Unit AR-RP 30°/ H 4 mm A.Rotational
417	MUL-GM- RPNB15	"Classic" Straight Multi-Unit GM-RP H 1.5 mm (M1.4)
418	MUL-GM- RPNB25	"Classic" Straight Multi-Unit GM-RP H 2.5 mm (M1.4)
419	MUL-GM- RPNB35	"Classic" Straight Multi-Unit GM-RP H 3.5 mm (M1.4)
420	MUL-GM- RPNB45	"Classic" Straight Multi-Unit GM-RP H 4.5 mm (M1.4)
421	MUL-GM- RPNB55	"Classic" Straight Multi-Unit GM-RP H 5.5 mm (M1.4)
422	MUL-GM- RPNB1710	"Classic" Angled Multi-Unit GM-RP 17°/ H 1 mm (M1.4)
423	MUL-GM- RPNB1720	"Classic" Angled Multi-Unit GM-RP 17°/ H 2 mm (M1.4)
424	MUL-GM- RPNB1730	"Classic" Angled Multi-Unit GM-RP 17°/ H 3 mm (M1.4)
425	MUL-GM- RPNB3010	"Classic" Angled Multi-Unit GM-RP 30°/ H 1 mm (M1.4)
426	MUL-GM- RPNB3020	"Classic" Angled Multi-Unit GM-RP 30°/ H 2 mm (M1.4)
427	MUL-GM- RPNB3030	"Classic" Angled Multi-Unit GM-RP 30°/ H 3 mm (M1.4)
428	MUL-GM- RPKS15	Multi-Clicq Straight Multi-Unit GM-RP H 1.5 mm
429	MUL-GM- RPKS25	Multi-Clicq Straight Multi-Unit GM-RP H 2.5 mm

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

		
Multi-Clicq On-X (EM)		
No	REF/ Productname	Product Description
430	MUL-GM-RPKS35	Multi-Clicq Straight Multi-Unit GM-RP H 3.5 mm
431	MUL-GM-RPKS45	Multi-Clicq Straight Multi-Unit GM-RP H 4.5 mm
432	MUL-GM-RPKS55	Multi-Clicq Straight Multi-Unit GM-RP H 5.5 mm
433	MUL-GM-RPKS1710	Multi-Clicq Straight Multi-Unit GM-RP 17°/ H 1 mm
434	MUL-GM-RPKS1720	Multi-Clicq Straight Multi-Unit GM-RP 17°/ H 2 mm
435	MUL-GM-RPKS3010	Multi-Clicq Straight Multi-Unit GM-RP 30°/ H 1 mm
436	MUL-GM-RPKS3020	Multi-Clicq Straight Multi-Unit GM-RP 30°/ H 2 mm
437	MUL-GM-RPKS1710H	Multi-Clicq Straight Multi-Unit GM-RP 17°/ H 1 mm A.R
438	MUL-GM-RPKS1720H	Multi-Clicq Straight Multi-Unit GM-RP 17°/ H 2 mm A.R
439	MUL-GM-RPKS3010H	Multi-Clicq Straight Multi-Unit GM-RP 30°/ H 1 mm A.R
440	MUL-GM-RPKS3020H	Multi-Clicq Straight Multi-Unit GM-RP 30°/ H 2 mm A.R
441	MUL-CF-RCKS15	Multi-Clicq Straight Multi-Unit CF-RC H 1.5 mm
442	MUL-CF-RCKS25	Multi-Clicq Straight Multi-Unit CF-RC H 2.5 mm
443	MUL-CF-RCKS35	Multi-Clicq Straight Multi-Unit CF-RC H 3.5 mm
444	MUL-CF-RCKS45	Multi-Clicq Straight Multi-Unit CF-RC H 4.5 mm
445	MUL-CF-RCKS55	Multi-Clicq Straight Multi-Unit CF-RC H 5.5 mm
446	MUL-CF-RCKS1710A	Multi-Clicq Angled Multi-Unit CF-RC 17°/ H 1 mm A Type
447	MUL-CF-RCKS1720A	Multi-Clicq Angled Multi-Unit CF-RC 17°/ H 2 mm A Type
448	MUL-CF-RCKS3010A	Multi-Clicq Angled Multi-Unit CF-RC 30°/ H 1 mm A Type


Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

 Multi-Clicq On-X (EM)		
No	REF/ Productname	Product Description
449	MUL-CF-RCKS3020A	Multi-Clicq Angled Multi-Unit CF-RC 30°/ H 2 mm A Type
450	MUL-CF-RCKS1710B	Multi-Clicq Angled Multi-Unit CF-RC 17°/ H 1 mm B Type
451	MUL-CF-RCKS1720B	Multi-Clicq Angled Multi-Unit CF-RC 17°/ H 2 mm B Type
452	MUL-CF-RCKS3010B	Multi-Clicq Angled Multi-Unit CF-RC 30°/ H 1 mm B Type
453	MUL-CF-RCKS3020B	Multi-Clicq Angled Multi-Unit CF-RC 30°/ H 2 mm B Type
454	MUL-CF-RCKS1710HA	Multi-Clicq Angled Multi-Unit CF-RC 17°/ H 1 mm A.R A Type
455	MUL-CF-RCKS1720HA	Multi-Clicq Angled Multi-Unit CF-RC 17°/ H 2 mm A.R A Type
456	MUL-CF-RCKS3010HA	Multi-Clicq Angled Multi-Unit CF-RC 30°/ H 1 mm A.R A Type
457	MUL-CF-RCKS3020HA	Multi-Clicq Angled Multi-Unit CF-RC 30°/ H 2 mm A.R A Type
458	MUL-CF-RCKS1710HB	Multi-Clicq Angled Multi-Unit CF-RC 17°/ H 1 mm A.R B Type
459	MUL-CF-RCKS1720HB	Multi-Clicq Angled Multi-Unit CF-RC 17°/ H 2 mm A.R B Type
460	MUL-CF-RCKS3010HB	Multi-Clicq Angled Multi-Unit CF-RC 30°/ H 1 mm A.R B Type
461	MUL-CF-RCKS3020HB	Multi-Clicq Angled Multi-Unit CF-RC 30°/ H 2 mm A.R B Type
462	MUL-CF-NCKS15	Multi-Clicq Straight Multi-Unit CF-NC H 1.5 mm
463	MUL-CF-NCKS25	Multi-Clicq Straight Multi-Unit CF-NC H 2.5 mm
464	MUL-CF-NCKS35	Multi-Clicq Straight Multi-Unit CF-NC H 3.5 mm
465	MUL-CF-NCKS45	Multi-Clicq Straight Multi-Unit CF-NC H 4.5 mm
466	MUL-CF-NCKS55	Multi-Clicq Straight Multi-Unit CF-NC H 5.5 mm
467	MUL-CF-NCKS1710A	Multi-Clicq Angled Multi-Unit CF-NC 17°/ H 1 mm A Type


Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

 Multi-Clicq On-X (EM)		
No	REF/ Productname	Product Description
468	MUL-CF-NCKS1720A	Multi-Clicq Angled Multi-Unit CF-NC 17°/ H 2 mm A Type
469	MUL-CF-NCKS3010A	Multi-Clicq Angled Multi-Unit CF-NC 30°/ H 1 mm A Type
470	MUL-CF-NCKS3020A	Multi-Clicq Angled Multi-Unit CF-NC 30°/ H 2 mm A Type
471	MUL-CF-NCKS1710B	Multi-Clicq Angled Multi-Unit CF-NC 17°/ H 1 mm B Type
472	MUL-CF-NCKS1720B	Multi-Clicq Angled Multi-Unit CF-NC 17°/ H 2 mm B Type
473	MUL-CF-NCKS3010B	Multi-Clicq Angled Multi-Unit CF-NC 30°/ H 1 mm B Type
474	MUL-CF-NCKS3020B	Multi-Clicq Angled Multi-Unit CF-NC 30°/ H 2 mm B Type
475	MUL-CF-NCKS1710HA	Multi-Clicq Angled Multi-Unit CF-NC 17°/ H 1 mm A.R A Type
476	MUL-CF-NCKS1720HA	Multi-Clicq Angled Multi-Unit CF-NC 17°/ H 2 mm A.R A Type
477	MUL-CF-NCKS3010HA	Multi-Clicq Angled Multi-Unit CF-NC 30°/ H 1 mm A.R A Type
478	MUL-CF-NCKS3020HA	Multi-Clicq Angled Multi-Unit CF-NC 30°/ H 2 mm A.R A Type
479	MUL-CF-NCKS1710HB	Multi-Clicq Angled Multi-Unit CF-NC 17°/ H 1 mm A.R B Type
480	MUL-CF-NCKS1720HB	Multi-Clicq Angled Multi-Unit CF-NC 17°/ H 2 mm A.R B Type
481	MUL-CF-NCKS3010HB	Multi-Clicq Angled Multi-Unit CF-NC 30°/ H 1 mm A.R B Type
482	MUL-CF-NCKS3020HB	Multi-Clicq Angled Multi-Unit CF-NC 30°/ H 2 mm A.R B Type
483	MUL-KS-ADUG	Multi-Clicq Adhesive Cap (Dual Hex/ Unigrip)
484	MUL-KS-ADRUG	Multi-Clicq Adhesive Cap Rotational (Dual/ Hex/ Unigrip)
485	MUL-KS-SLVCCUG	Multi-Clicq Chrome Cobalt Sleeve/ UCLA (Dual Hex/ Unigrip)

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

 Multi-Clicq On-X (EM)		
No	REF/ Productname	Product Description
486	MUL-KS-BSLVUG	Clicq Multi Bar – Pre-Fabricated Titanium Adjustable Sleeve
487	MUL-KS-ADAR	Multi-Clicq Adhesive Cap Hex 1.2 mm
488	MUL-KS-ADRAR	Multi-Clicq Adhesive Cap Rotational Hex 1.2 mm
489	MUL-KS-SLVCCAR	Multi-Clicq Chrome Cobalt Sleeve/ ULCA Hex 1.2 mm
490	MUL-KS-BSLVAR	Clicq Multi Bar – Pre-Fabricated Titanium Adjustable Sleeve
491	MUL-KS-ADTX	Multi-Clicq Adhesive Cap (Torx)
492	MUL-KS-ADRTX	Multi-Clicq Adhesive Cap Rotational (Torx)
493	MUL-KS-SLVCCTX	Multi-Clicq Chrome Cobalt Sleeve/ ULCA (Torx)
494	MUL-KS-BSLVTX	Clicq Multi Bar – Pre-Fabricated Titanium Adjustable Sleeve
495	MUL-NB-ADUG	“Classic” Multi Unit Adhesive Cap (Dual Hex/ Unigrip)
496	MUL-NB-ADRUG	“Classic” Multi Unit Adhesive Cap Rotational (Dual)
497	MUL-NB-SLVCCUG	“Classic” Multi Unit Adhesive Cap (Dual Hex/ Unigrip)
498	MUL-NB-BSLVUG	Clicq Multi Bar – Pre-Fabricated Titanium Adjustable Sleeve
499	MUL-NB-ADTX	“Classic” Multi Unit Adhesive Cap (Torx)
500	MUL-NB-ADRTX	“Classic” Multi Unit Adhesive Cap Rotational (Torx)
501	MUL-NB-SLVCCTX	“Classic” Multi Unit Chrome Cobalt Sleeve/ ULCA (Torx)
502	MUL-NB-BSLVTX	Clicq Multi Bar – Pre-Fabricated Titanium Adjustable Sleeve
503	MUL-NB-ADAR	“Classic” Multi Unit Adhesive Cap Hex 1.2 mm
504	MUL-NB-ADRAR	“Classic” Multi Unit Adhesive Cap Rotational Hex 1.2 mm
505	MUL-NB-SLVCCAR	“Classic” Multi Unit Chrome Cobalt Sleeve/ ULCA Hex 1.2 mm
506	MUL-NB-BSLVAR	Clicq Multi Bar – Pre-Fabricated Titanium Adjustable Sleeve
		

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

		
Multi-Clicq On-X (EM)		
No	REF/ Productname	Product Description
507	MUL-KS-TBUG	Multi-Clicq Titanium Base (Dual Hex/ Unigrip)
508	MUL-KS-TBRUG	Multi-Clicq Titanium Base Rotational (Dual Hex/ Unigrip)
509	MUL-KS-TBAR	Multi-Clicq Titanium Base Hex 1.2 mm
510	MUL-KS-TBRAR	Multi-Clicq Titanium Base Rotational Hex 1.2 mm
511	MUL-KS-TBTX	Multi-Clicq Titanium Base (Torx)
512	MUL-KS-TBRTX	Multi-Clicq Titanium Base Rotational (Torx)
513	MUL-NB-TBUG	“Classic” Multi-Clicq Titanium Base (Dual Hex/ Unigrip)
514	MUL-NB-TBRUG	“Classic” Multi-Clicq Titanium Base Rotational (Dual)
515	MUL-NB-TBTX	“Classic” Multi-Clicq Titanium Base (Torx)
516	MUL-NB-TBRTX	“Classic” Multi-Clicq Titanium Base Rotational (Torx)
517	MUL-NB-TBAR	“Classic” Multi-Clicq Titanium Base Hex 1.2 mm
518	MUL-NB-TBRAR	“Classic” Multi-Clicq Titanium Base Rotational Hex 1.2 mm

*Tabelle 34 Multi-Clicq Kits (EM)*

Multi-Clicq Convenience Kits (EM)		
No	REF/ Product name	Product Description
1	MUL-SV-RPSK	Multi-Clicq Multi-Unit SV-RP Starter KIT
2	MUL-SV-RPPK	Multi-Clicq Multi-Unit SV-RP Professional KIT
3	MUL-SV- RPKS10K	Multi-Clicq Straight Multi-Unit SV-RP H 1 mm KIT
4	MUL-SV- RPKS20K	Multi-Clicq Straight Multi-Unit SV-RP H 2 mm KIT
5	MUL-SV- RPKS30K	Multi-Clicq Straight Multi-Unit SV-RP H 3 mm KIT
6	MUL-SV- RPKS40K	Multi-Clicq Straight Multi-Unit SV-RP H 4 mm KIT
7	MUL-SV- RPKS50K	Multi-Clicq Straight Multi-Unit SV-RP H 5 mm KIT

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

Multi-Clicq Convenience Kits (EM)		
No	REF/ Product name	Product Description
8	MUL-SV-RPKS60K	Multi-Clicq Straight Multi-Unit SV-RP H 6 mm KIT
9	MUL-SV-RPKS1710K	Multi-Clicq Angled Multi-Unit SV-RP 17°/ H 1 mm KIT A Type
10	MUL-SV-RPKS1720K	Multi-Clicq Angled Multi-Unit SV-RP 17°/ H 2 mm KIT A Type
11	MUL-SV-RPKS1730K	Multi-Clicq Angled Multi-Unit SV-RP 17°/ H 3 mm KIT A Type
12	MUL-SV-RPKS1740K	Multi-Clicq Angled Multi-Unit SV-RP 17°/ H 4 mm KIT A Type
13	MUL-SV-RPKS1710BK	Multi-Clicq Angled Multi-Unit SV-RP 17°/ H 1 mm KIT B Type
14	MUL-SV-RPKS1720K	Multi-Clicq Angled Multi-Unit SV-RP 17°/ H 2 mm KIT B Type
15	MUL-SV-RPKS1730K	Multi-Clicq Angled Multi-Unit SV-RP 17°/ H 3 mm KIT B Type
16	MUL-SV-RPKS1740K	Multi-Clicq Angled Multi-Unit SV-RP 17°/ H 4 mm KIT B Type
17	MUL-SV-RPKS3010K	Multi-Clicq Angled Multi-Unit SV-RP 30°/ H 1 mm KIT A Type
18	MUL-SV-RPKS3020K	Multi-Clicq Angled Multi-Unit SV-RP 30°/ H 2 mm KIT A Type
19	MUL-SV-RPKS3030K	Multi-Clicq Angled Multi-Unit SV-RP 30°/ H 3 mm KIT A Type
20	MUL-SV-RPKS3040K	Multi-Clicq Angled Multi-Unit SV-RP 30°/ H 4 mm KIT A Type
21	MUL-SV-RPKS3010BK	Multi-Clicq Angled Multi-Unit SV-RP 30°/ H 1 mm KIT B Type
22	MUL-SV-RPKS3020BK	Multi-Clicq Angled Multi-Unit SV-RP 30°/ H 2 mm KIT B Type
23	MUL-SV-RPKS3030BK	Multi-Clicq Angled Multi-Unit SV-RP 30°/ H 3 mm KIT B Type
24	MUL-SV-RPKS3040BK	Multi-Clicq Angled Multi-Unit SV-RP 30°/ H 4 mm KIT B Type

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

Multi-Clicq Convenience Kits (EM)		
No	REF/ Product name	Product Description
25	MUL-SV-WPSK	Multi-Clicq Multi-Unit SV-WP Starter KIT
26	MUL-SV-WPPK	Multi-Clicq Multi-Unit SV-WP Professional KIT
27	MUL-SV-WPKS10K	Multi-Clicq Straight Multi-Unit SV-WP SV-WP H 1 mm KIT
28	MUL-SV-WPKS20K	Multi-Clicq Straight Multi-Unit SV-WP SV-WP H 2 mm KIT
29	MUL-SV-WPKS30K	Multi-Clicq Straight Multi-Unit SV-WP SV-WP H 3 mm KIT
30	MUL-SV-WPKS40K	Multi-Clicq Straight Multi-Unit SV-WP SV-WP H 4 mm KIT
31	MUL-SV-WPKS50K	Multi-Clicq Straight Multi-Unit SV-WP SV-WP H 5 mm KIT
32	MUL-SV-WPKS60K	Multi-Clicq Straight Multi-Unit SV-WP SV-WP H 6 mm KIT
33	MUL-SV-WPKS1710K	Multi-Clicq Angled Multi-Unit SV-WP 17°/ H 1 mm KIT
34	MUL-SV-WPKS1720K	Multi-Clicq Angled Multi-Unit SV-WP 17°/ H 2 mm KIT
35	MUL-SV-WPKS3010K	Multi-Clicq Angled Multi-Unit SV-WP 30°/ H 1 mm KIT
36	MUL-SV-WPKS3020K	Multi-Clicq Angled Multi-Unit SV-WP 30°/ H 2 mm KIT
37	MUL-SV-NLSK	Multi-Clicq Multi-Unit SV-NL Starter KIT
38	MUL-SV-NLPK	Multi-Clicq Multi-Unit SV-NL Professional KIT
39	MUL-SV-NLKS10K	Multi-Clicq Straight Multi-Unit SV-NL H 1 mm KIT
40	MUL-SV-NLKS20K	Multi-Clicq Straight Multi-Unit SV-NL H 2 mm KIT
41	MUL-SV-NLKS30K	Multi-Clicq Straight Multi-Unit SV-NL H 3 mm KIT
42	MUL-SV-NLKS40K	Multi-Clicq Straight Multi-Unit SV-NL H 4 mm KIT
43	MUL-SV-NLKS50K	Multi-Clicq Straight Multi-Unit SV-NL H 5 mm KIT

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

Multi-Clicq Convenience Kits (EM)		
No	REF/ Product name	Product Description
44	MUL-SV-NLKS60K	Multi-Clicq Straight Multi-Unit SV-NL H 6 mm KIT
45	MUL-SV-NLKS1710K	Multi-Clicq Angled Multi-Unit SV-NL 17°/ H 1 mm KIT A Type
46	MUL-SV-NLKS1720K	Multi-Clicq Angled Multi-Unit SV-NL 17°/ H 2 mm KIT A Type
47	MUL-SV-NLKS1730K	Multi-Clicq Angled Multi-Unit SV-NL 17°/ H 3 mm KIT A Type
48	MUL-SV-NLKS1740K	Multi-Clicq Angled Multi-Unit SV-NL 17°/ H 4 mm KIT A Type
49	MUL-SV-NLKS1710BK	Multi-Clicq Angled Multi-Unit SV-NL 17°/ H 1 mm KIT B Type
50	MUL-SV-NLKS1720BK	Multi-Clicq Angled Multi-Unit SV-NL 17°/ H 2 mm KIT B Type
51	MUL-SV-NLKS1730BK	Multi-Clicq Angled Multi-Unit SV-NL 17°/ H 3 mm KIT B Type
52	MUL-SV-NLKS1740BK	Multi-Clicq Angled Multi-Unit SV-NL 17°/ H 4 mm KIT B Type
53	MUL-SV-NLKS3010K	Multi-Clicq Angled Multi-Unit SV-NL 30°/ H 1 mm KIT A Type
54	MUL-SV-NLKS3020K	Multi-Clicq Angled Multi-Unit SV-NL 30°/ H 2 mm KIT A Type
55	MUL-SV-NLKS3030K	Multi-Clicq Angled Multi-Unit SV-NL 30°/ H 3 mm KIT A Type
56	MUL-SV-NLKS3040K	Multi-Clicq Angled Multi-Unit SV-NL 30°/ H 4 mm KIT A Type
57	MUL-SV-NLKS3010B	Multi-Clicq Angled Multi-Unit SV-NL 30°/ H 1 mm KIT B Type
58	MUL-SV-NLKS3020B	Multi-Clicq Angled Multi-Unit SV-NL 30°/ H 2 mm KIT B Type
59	MUL-SV-NLKS3030B	Multi-Clicq Angled Multi-Unit SV-NL 30°/ H 3 mm KIT B Type
60	MUL-SV-NLKS3040B	Multi-Clicq Angled Multi-Unit SV-NL 30°/ H 4 mm KIT B Type

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

Multi-Clicq Convenience Kits (EM)		
No	REF/ Product name	Product Description
61	MUL-SV-NPSK	Multi-Clicq Multi-Unit SV-NP Starter KIT
62	MUL-SV-NPPK	Multi-Clicq Multi-Unit SV-NP Professional KIT
63	MUL-SV-NPKS10K	Multi-Clicq Straight Multi-Unit SV-NP H 1 mm KIT
64	MUL-SV-NPKS20K	Multi-Clicq Straight Multi-Unit SV-NP H 2 mm KIT
65	MUL-SV-NPKS30K	Multi-Clicq Straight Multi-Unit SV-NP H 3 mm KIT
66	MUL-SV-NPKS40K	Multi-Clicq Straight Multi-Unit SV-NP H 4 mm KIT
67	MUL-SV-NPKS50K	Multi-Clicq Straight Multi-Unit SV-NP H 5 mm KIT
68	MUL-SV-NPKS60K	Multi-Clicq Straight Multi-Unit SV-NP H 6 mm KIT
69	MUL-SV-NPKS1710K	Multi-Clicq Angled Multi-Unit SV-NP 17°/ H 1 mm KIT A Type
70	MUL-SV-NPKS1720K	Multi-Clicq Angled Multi-Unit SV-NP 17°/ H 2 mm KIT A Type
71	MUL-SV-NPKS1730K	Multi-Clicq Angled Multi-Unit SV-NP 17°/ H 3 mm KIT A Type
72	MUL-SV-NPKS1740K	Multi-Clicq Angled Multi-Unit SV-NP 17°/ H 4 mm KIT A Type
73	MUL-SV-NPKS1710BK	Multi-Clicq Angled Multi-Unit SV-NP 17°/ H 1 mm KIT B Type
74	MUL-SV-NPKS1720BK	Multi-Clicq Angled Multi-Unit SV-NP 17°/ H 2 mm KIT B Type
75	MUL-SV-NPKS1730BK	Multi-Clicq Angled Multi-Unit SV-NP 17°/ H 3 mm KIT B Type
76	MUL-SV-NPKS1740BK	Multi-Clicq Angled Multi-Unit SV-NP 17°/ H 4 mm KIT B Type
77	MUL-SV-NPKS3010K	Multi-Clicq Angled Multi-Unit SV-NP 30°/ H 1 mm KIT A Type
78	MUL-SV-NPKS3020K	Multi-Clicq Angled Multi-Unit SV-NP 30°/ H 2 mm KIT A Type

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

Multi-Clicq Convenience Kits (EM)		
No	REF/ Product name	Product Description
79	MUL-SV-NPKS3030K	Multi-Clicq Angled Multi-Unit SV-NP 30°/ H 3 mm KIT A Type
80	MUL-SV-NPKS3040K	Multi-Clicq Angled Multi-Unit SV-NP 30°/ H 4 mm KIT A Type
81	MUL-SV-NPKS3010BK	Multi-Clicq Angled Multi-Unit SV-NP 30°/ H 1 mm KIT B Type
82	MUL-SV-NPKS3020BK	Multi-Clicq Angled Multi-Unit SV-NP 30°/ H 2 mm KIT B Type
83	MUL-SV-NPKS3030BK	Multi-Clicq Angled Multi-Unit SV-NP 30°/ H 3 mm KIT B Type
84	MUL-SV-NPKS3040BK	Multi-Clicq Angled Multi-Unit SV-NP 30°/ H 4 mm KIT B Type
85	MUL-NA-NPSK	Multi-Clicq Multi-Unit NA-NP Starter KIT
86	MUL-NPPK	Multi-Clicq Multi-Unit NA-NP Professional KIT
87	MUL-NPKS15K	Multi-Clicq Straight Multi-Unit NA-NP H 1.5 mm KIT
88	MUL-NPKS25K	Multi-Clicq Straight Multi-Unit NA-NP H 2.5 mm KIT
89	MUL-NPKS35K	Multi-Clicq Straight Multi-Unit NA-NP H 3.5 mm KIT
90	MUL-NPKS45K	Multi-Clicq Straight Multi-Unit NA-NP H 4.5 mm KIT
91	MUL-NPKS55K	Multi-Clicq Straight Multi-Unit NA-NP H 5.5 mm KIT
92	MUL-NA-NPKS1710K	Multi-Clicq Angled Multi-Unit NA-NP 17°/ H 1 mm KIT A Type
93	MUL-NA-NPKS1720K	Multi-Clicq Angled Multi-Unit NA-NP 17°/ H 2 mm KIT A Type
94	MUL-NA-NPKS1730K	Multi-Clicq Angled Multi-Unit NA-NP 17°/ H 3 mm KIT A Type
95	MUL-NA-NPKS1740K	Multi-Clicq Angled Multi-Unit NA-NP 17°/ H 4 mm KIT A Type
96	MUL-NA-NPKS1710BK	Multi-Clicq Angled Multi-Unit NA-NP 17°/ H 1 mm KIT B Type
97	MUL-NA-NPKS1720BK	Multi-Clicq Angled Multi-Unit NA-NP 17°/ H 2 mm KIT B Type
98	MUL-NA-NPKS1730BK	Multi-Clicq Angled Multi-Unit NA-NP 17°/ H 3 mm KIT B Type

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

Multi-Clicq Convenience Kits (EM)		
No	REF/ Product name	Product Description
99	MUL-NA-NPKS1740BK	Multi-Clicq Angled Multi-Unit NA-NP 17°/ H 4 mm KIT B Type
100	MUL-NA-NPKS3010K	Multi-Clicq Angled Multi-Unit NA-NP 30°/ H 1 mm KIT A Type
101	MUL-NA-NPKS3020K	Multi-Clicq Angled Multi-Unit NA-NP 30°/ H 2 mm KIT A Type
102	MUL-NA-NPKS3030K	Multi-Clicq Angled Multi-Unit NA-NP 30°/ H 3 mm KIT A Type
103	MUL-NA-NPKS3040K	Multi-Clicq Angled Multi-Unit NA-NP 30°/ H 4 mm KIT A Type
104	MUL-NA-NPKS3010BK	Multi-Clicq Angled Multi-Unit NA-NP 30°/ H 1 mm KIT B Type
105	MUL-NA-NPKS3020BK	Multi-Clicq Angled Multi-Unit NA-NP 30°/ H 2 mm KIT B Type
106	MUL-NA-NPKS3030BK	Multi-Clicq Angled Multi-Unit NA-NP 30°/ H 3 mm KIT B Type
107	MUL-NA-NPKS3040BK	Multi-Clicq Angled Multi-Unit NA-NP 30°/ H 4 mm KIT B Type
108	MUL-NA-RPSK	Multi-Clicq Multi-Unit NA-RP Starter KIT
109	MUL-NA-RPPK	Multi-Clicq Multi-Unit NA-RP Professional KIT
110	MUL-NA-RP15K	Multi-Clicq Multi-Unit NA-RP H 1.5 mm KIT
111	MUL-NA-RP25K	Multi-Clicq Multi-Unit NA-RP H 2.5 mm KIT
112	MUL-NA-RP35K	Multi-Clicq Multi-Unit NA-RP H 3.5 mm KIT
113	MUL-NA-RP45K	Multi-Clicq Multi-Unit NA-RP H 4.5 mm KIT
114	MUL-NA-RP55K	Multi-Clicq Multi-Unit NA-RP H 5.5 mm KIT
115	MUL-NA-RPKS1710K	Multi-Clicq Straight Multi-Unit NA-RP 17°/ H 1 mm KIT A Type
116	MUL-NA-RPKS1720K	Multi-Clicq Straight Multi-Unit NA-RP 17°/ H 2 mm KIT A Type
117	MUL-NA-RPKS1730K	Multi-Clicq Straight Multi-Unit NA-RP 17°/ H 3 mm KIT A Type

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

Multi-Clicq Convenience Kits (EM)		
No	REF/ Product name	Product Description
118	MUL-NA-RPKS1740K	Multi-Clicq Straight Multi-Unit NA-RP 17°/ H 1 mm KIT A Type
119	MUL-NA-RPKS1710BK	Multi-Clicq Straight Multi-Unit NA-RP 17°/ H 1 mm KIT B Type
120	MUL-NA-RPKS1720BK	Multi-Clicq Straight Multi-Unit NA-RP 17°/ H 2 mm KIT B Type
121	MUL-NA-RPKS1730BK	Multi-Clicq Straight Multi-Unit NA-RP 17°/ H 3 mm KIT B Type
122	MUL-NA-RPKS1740BK	Multi-Clicq Straight Multi-Unit NA-RP 17°/ H 4 mm KIT B Type
123	MUL-NA-RPKS3010K	Multi-Clicq Angled Multi-Unit NA-RP 30°/ H 1 mm KIT A Type
124	MUL-NA-RPKS3020K	Multi-Clicq Angled Multi-Unit NA-RP 30°/ H 2 mm KIT A Type
125	MUL-NA-RPKS3030K	Multi-Clicq Angled Multi-Unit NA-RP 30°/ H 3 mm KIT A Type
126	MUL-NA-RPKS3040K	Multi-Clicq Angled Multi-Unit NA-RP 30°/ H 4 mm KIT A Type
127	MUL-NA-RPKS3010BK	Multi-Clicq Angled Multi-Unit NA-RP 30°/ H 1 mm KIT B Type
128	MUL-NA-RPKS3020BK	Multi-Clicq Angled Multi-Unit NA-RP 30°/ H 2 mm KIT B Type
129	MUL-NA-RPKS3030BK	Multi-Clicq Angled Multi-Unit NA-RP 30°/ H 3 mm KIT B Type
130	MUL-NA-RPKS3040BK	Multi-Clicq Angled Multi-Unit NA-RP 30°/ H 4 mm KIT B Type
131	MUL-TS-NPSK	Multi-Clicq Multi-Unit TS-NP/ Mini Starter KIT
132	MUL-TS-NPKSPK	Multi-Clicq Multi-Unit TS-NP/ Mini Professional KIT
133	MUL-TS-NPKS15K	Multi-Clicq Multi-Unit TS-NP/ Mini H 1.5 mm KIT
134	MUL-TS-NPKS25K	Multi-Clicq Multi-Unit TS-NP/ Mini H 2.5 mm KIT
135	MUL-TS-NPKS35K	Multi-Clicq Multi-Unit TS-NP/ Mini H 3.5 mm KIT

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

Multi-Clicq Convenience Kits (EM)		
No	REF/ Product name	Product Description
136	MUL-TS-NPKS45K	Multi-Clicq Multi-Unit TS-NP/ Mini H 4.5 mm KIT
137	MUL-TS-NPKS55K	Multi-Clicq Multi-Unit TS-NP/ Mini H 5.5 mm KIT
138	MUL-TS-NPKS1710K	Multi-Clicq Angled Multi-Unit TS-NP 17°/ H 1 mm KIT Type A
139	MUL-TS-NPKS1720K	Multi-Clicq Angled Multi-Unit TS-NP 17°/ H 2 mm KIT Type A
140	MUL-TS-NPKS1730K	Multi-Clicq Angled Multi-Unit TS-NP 17°/ H 3 mm KIT Type A
141	MUL-TS-NPKS1740K	Multi-Clicq Angled Multi-Unit TS-NP 17°/ H 4 mm KIT Type A
142	MUL-TS-NPKS1710BK	Multi-Clicq Angled Multi-Unit TS-NP 17°/ H 1 mm KIT Type B
143	MUL-TS-NPKS1720BK	Multi-Clicq Angled Multi-Unit TS-NP 17°/ H 2 mm KIT Type B
144	MUL-TS-NPKS1730BK	Multi-Clicq Angled Multi-Unit TS-NP 17°/ H 3 mm KIT Type B
145	MUL-TS-NPKS1740BK	Multi-Clicq Angled Multi-Unit TS-NP 17°/ H 4 mm KIT Type B
146	MUL-TS-NPKS3010K	Multi-Clicq Angled Multi-Unit TS-NP 30°/ H 1 mm KIT Type A
147	MUL-TS-NPKS3020K	Multi-Clicq Angled Multi-Unit TS-NP 30°/ H 2 mm KIT Type A
148	MUL-TS-NPKS3030K	Multi-Clicq Angled Multi-Unit TS-NP 30°/ H 3 mm KIT Type A
149	MUL-TS-NPKS3040K	Multi-Clicq Angled Multi-Unit TS-NP 30°/ H 4 mm KIT Type A
150	MUL-TS-NPKS3010BK	Multi-Clicq Angled Multi-Unit TS-NP 30°/ H 1 mm KIT Type B
151	MUL-TS-NPKS3020BK	Multi-Clicq Angled Multi-Unit TS-NP 30°/ H 2 mm KIT Type B
152	MUL-TS-NPKS3030BK	Multi-Clicq Angled Multi-Unit TS-NP 30°/ H 3 mm KIT Type B
153	MUL-TS-NPKS3040BK	Multi-Clicq Angled Multi-Unit TS-NP 30°/ H 4 mm KIT Type B

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

Multi-Clicq Convenience Kits (EM)		
No	REF/ Product name	Product Description
154	MUL-TS-RPSK	Multi-Clicq Multi-Unit TS-RP Starter KIT
155	MUL-TS-RPPK	Multi-Clicq Multi-Unit TS-RP Professional KIT
156	MUL-TS-RPKS15K	Multi-Clicq Straight Multi-Unit TS-RP H 1.5 mm KIT
157	MUL-TS-RPKS25K	Multi-Clicq Straight Multi-Unit TS-RP H 2.5 mm KIT
158	MUL-TS-RPKS35K	Multi-Clicq Straight Multi-Unit TS-RP H 3.5 mm KIT
159	MUL-TS-RPKS45K	Multi-Clicq Straight Multi-Unit TS-RP H 4.5 mm KIT
160	MUL-TS-RPKS55K	Multi-Clicq Straight Multi-Unit TS-RP H 5.5 mm KIT
161	MUL-TS-RPKS1710K	Multi-Clicq Angled Multi-Unit TS-RP 17°/ H 1 mm KIT Type A
162	MUL-TS-RPKS1720K	Multi-Clicq Angled Multi-Unit TS-RP 17°/ H 2 mm KIT Type A
163	MUL-TS-RPKS1730K	Multi-Clicq Angled Multi-Unit TS-RP 17°/ H 3 mm KIT Type A
164	MUL-TS-RPKS1740K	Multi-Clicq Angled Multi-Unit TS-RP 17°/ H 4 mm KIT Type A
165	MUL-TS-RPKS1710BK	Multi-Clicq Angled Multi-Unit TS-RP 17°/ H 1 mm KIT Type B
166	MUL-TS-RPKS1720BK	Multi-Clicq Angled Multi-Unit TS-RP 17°/ H 2 mm KIT Type B
167	MUL-TS-RPKS1730BK	Multi-Clicq Angled Multi-Unit TS-RP 17°/ H 3 mm KIT Type B
168	MUL-TS-RPKS1740BK	Multi-Clicq Angled Multi-Unit TS-RP 17°/ H 4 mm KIT Type B
169	MUL-TS-RPKS3010K	Multi-Clicq Angled Multi-Unit TS-RP 30°/ H 1 mm KIT Type A
170	MUL-TS-RPKS3020K	Multi-Clicq Angled Multi-Unit TS-RP 30°/ H 2 mm KIT Type A
171	MUL-TS-RPKS3030K	Multi-Clicq Angled Multi-Unit TS-RP 30°/ H 3 mm KIT Type A

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

Multi-Clicq Convenience Kits (EM)		
No	REF/ Product name	Product Description
172	MUL-TS-RPKS3040K	Multi-Clicq Angled Multi-Unit TS-RP 30°/ H 4 mm KIT Type A
173	MUL-TS-RPKS3010BK	Multi-Clicq Angled Multi-Unit TS-RP 30°/ H 1 mm KIT Type B
174	MUL-TS-RPKS3020BK	Multi-Clicq Angled Multi-Unit TS-RP 30°/ H 2 mm KIT Type B
175	MUL-TS-RPKS3030BK	Multi-Clicq Angled Multi-Unit TS-RP 30°/ H 3 mm KIT Type B
176	MUL-TS-RPKS3040BK	Multi-Clicq Angled Multi-Unit TS-RP 30°/ H 4 mm KIT Type B
177	MUL-OS-SPSK	Multi-Clicq Multi-Unit OS-SP/ Mini Starter KIT
178	MUL-OS-SPPK	Multi-Clicq Multi-Unit OS-SP/ Mini Professional KIT
179	MUL-OS-SPKS15K	Multi-Clicq Straight Multi-Unit OS-SP/ Mini H 1.5 mm KIT
180	MUL-OS-SPKS25K	Multi-Clicq Straight Multi-Unit OS-SP/ Mini H 2.5 mm KIT
181	MUL-OS-SPKS35K	Multi-Clicq Straight Multi-Unit OS-SP/ Mini H 3.5 mm KIT
182	MUL-OS-SPKS45K	Multi-Clicq Straight Multi-Unit OS-SP/ Mini H 4.5 mm KIT
183	MUL-OS-SPKS55K	Multi-Clicq Straight Multi-Unit OS-SP/ Mini H 5.5 mm KIT
184	MUL-OS-SPKS1710K	Multi-Clicq Angled Multi-Unit OS-SP 17°/ H 1 mm KIT Type A
185	MUL-OS-SPKS1720K	Multi-Clicq Angled Multi-Unit OS-SP 17°/ H 2 mm KIT Type A
186	MUL-OS-SPKS1730K	Multi-Clicq Angled Multi-Unit OS-SP 17°/ H 3 mm KIT Type A
187	MUL-OS-SPKS1740K	Multi-Clicq Angled Multi-Unit OS-SP 17°/ H 4 mm KIT Type A
188	MUL-OS-SPKS1710BK	Multi-Clicq Angled Multi-Unit OS-SP 17°/ H 1 mm KIT Type B
189	MUL-OS-SPKS1720BK	Multi-Clicq Angled Multi-Unit OS-SP 17°/ H 2 mm KIT Type B

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

Multi-Clicq Convenience Kits (EM)		
No	REF/ Product name	Product Description
190	MUL-OS-SPKS1730BK	Multi-Clicq Angled Multi-Unit OS-SP 17°/ H 3 mm KIT Type B
191	MUL-OS-SPKS1740BK	Multi-Clicq Angled Multi-Unit OS-SP 17°/ H 4 mm KIT Type B
192	MUL-OS-SPKS3010K	Multi-Clicq Angled Multi-Unit OS-SP 30°/ H 1 mm KIT Type A
193	MUL-OS-SPKS3020K	Multi-Clicq Angled Multi-Unit OS-SP 30°/ H 2 mm KIT Type A
194	MUL-OS-SPKS3030K	Multi-Clicq Angled Multi-Unit OS-SP 30°/ H 3 mm KIT Type A
195	MUL-OS-SPKS3040K	Multi-Clicq Angled Multi-Unit OS-SP 30°/ H 4 mm KIT Type A
196	MUL-OS-SPKS3010BK	Multi-Clicq Angled Multi-Unit OS-SP 30°/ H 1 mm KIT Type B
197	MUL-OS-SPKS3020BK	Multi-Clicq Angled Multi-Unit OS-SP 30°/ H 2 mm KIT Type B
198	MUL-OS-SPKS3030BK	Multi-Clicq Angled Multi-Unit OS-SP 30°/ H 3 mm KIT Type B
199	MUL-OS-SPKS3040BK	Multi-Clicq Angled Multi-Unit OS-SP 30°/ H 4 mm KIT Type B
200	MUL-OS-WPSK	Multi-Clicq Multi-Unit OS-WP Starter KIT
201	MUL-OS-WPPK	Multi-Clicq Multi-Unit OS-WP Professional KIT
202	MUL-OS-WPKS15K	Multi-Clicq Straight Multi-Unit OS-WP H 1.5 mm KIT
203	MUL-OS-WPKS25K	Multi-Clicq Straight Multi-Unit OS-WP H 2.5 mm KIT
204	MUL-OS-WPKS35K	Multi-Clicq Straight Multi-Unit OS-WP H 3.5 mm KIT
205	MUL-OS-WPKS45K	Multi-Clicq Straight Multi-Unit OS-WP H 4.5 mm KIT
206	MUL-OS-WPKS55K	Multi-Clicq Straight Multi-Unit OS-WP H 5.5 mm KIT
207	MUL-OS-WPKS1710K	Multi-Clicq Angled Multi-Unit OS-WP 17°/ H 1 mm KIT

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

Multi-Clicq Convenience Kits (EM)		
No	REF/ Product name	Product Description
208	MUL-OS-WPKS1720K	Multi-Clicq Angled Multi-Unit OS-WP 17°/ H 2 mm KIT
209	MUL-OS-WPKS3010K	Multi-Clicq Angled Multi-Unit OS-WP 30°/ H 1 mm KIT
210	MUL-OS-WPKS3020K	Multi-Clicq Angled Multi-Unit OS-WP 30°/ H 2 mm KIT
211	MUL-AR-RPSK	Multi-Cliq Multi-Unit AR-RP Starter KIT
212	MUL-AR-RPPK	Multi-Cliq Multi-Unit AR-RP Professional KIT
213	MUL-AR-RPKS15K	Multi-Cliq Straight Multi-Unit AR-RP H 1.5mm KIT
214	MUL-AR-RPKS25K	Multi-Cliq Straight Multi-Unit AR-RP H 2.5mm KIT
215	MUL-AR-RPKS35K	Multi-Cliq Straight Multi-Unit AR-RP H 3.5mm KIT
216	MUL-AR-RPKS45K	Multi-Cliq Straight Multi-Unit AR-RP H 4.5mm KIT
217	MUL-AR-RPKS55K	Multi-Cliq Straight Multi-Unit AR-RP H 5.5mm KIT
218	MUL-AR-RPKS1710K	Multi-Cliq Angled Multi-Unit AR-RP 17°/ H 1 mm KIT Type A
219	MUL-AR-RPKS1720K	Multi-Cliq Angled Multi-Unit AR-RP 17°/ H 2 mm KIT Type A
220	MUL-AR-RPKS1730K	Multi-Cliq Angled Multi-Unit AR-RP 17°/ H 3 mm KIT Type A
221	MUL-AR-RPKS1740K	Multi-Cliq Angled Multi-Unit AR-RP 17°/ H 4 mm KIT Type A
222	MUL-AR-RPKS1710BK	Multi-Cliq Angled Multi-Unit AR-RP 17°/ H 1 mm KIT Type B
223	MUL-AR-RPKS1720BK	Multi-Cliq Angled Multi-Unit AR-RP 17°/ H 2 mm KIT Type B
224	MUL-AR-RPKS1730BK	Multi-Cliq Angled Multi-Unit AR-RP 17°/ H 3 mm KIT Type B
225	MUL-AR-RPKS1740BK	Multi-Cliq Angled Multi-Unit AR-RP 17°/ H 4 mm KIT Type B

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

Multi-Clicq Convenience Kits (EM)		
No	REF/ Product name	Product Description
226	MUL-AR-RPKS3010K	Multi-Clicq Angled Multi-Unit AR-RP 30°/ H 1 mm KIT Type A
227	MUL-AR-RPKS3020K	Multi-Clicq Angled Multi-Unit AR-RP 30°/ H 2 mm KIT Type A
228	MUL-AR-RPKS3030K	Multi-Clicq Angled Multi-Unit AR-RP 30°/ H 3 mm KIT Type A
229	MUL-AR-RPKS3040K	Multi-Clicq Angled Multi-Unit AR-RP 30°/ H 4 mm KIT Type A
230	MUL-AR-RPKS3010BK	Multi-Clicq Angled Multi-Unit AR-RP 30°/ H 1 mm KIT Type B
231	MUL-AR-RPKS3020BK	Multi-Clicq Angled Multi-Unit AR-RP 30°/ H 2 mm KIT Type B
232	MUL-AR-RPKS3030BK	Multi-Clicq Angled Multi-Unit AR-RP 30°/ H 3 mm KIT Type B
233	MUL-AR-RPKS3040BK	Multi-Clicq Angled Multi-Unit AR-RP 30°/ H 4 mm KIT Type B
234	MUL-GM-RPSK	Multi-Clicq Attachment GM-RP Starter KIT
235	MUL-GM-RPPK	Multi-Clicq Attachment GM-RP Professional KIT
236	MUL-GM-RPKS15K	Multi-Clicq Straight Attachment GM-RP H 1.5 mm KIT
237	MUL-GM-RPKS25K	Multi-Clicq Straight Attachment GM-RP H 2.5 mm KIT
238	MUL-GM-RPKS35K	Multi-Clicq Straight Attachment GM-RP H 3.5 mm KIT
239	MUL-GM-RPKS45K	Multi-Clicq Straight Attachment GM-RP H 4.5 mm KIT
240	MUL-GM-RPKS55K	Multi-Clicq Straight Attachment GM-RP H 5.5 mm KIT
241	MUL-GM-RPKS1710K	Multi-Clicq Angled Attachment GM-RP 17°/ H 1 mm KIT
242	MUL-GM-RPKS1720K	Multi-Clicq Angled Attachment GM-RP 17°/ H 2 mm KIT
243	MUL-GM-RPKS3010K	Multi-Clicq Angled Attachment GM-RP 30°/ H 1 mm KIT

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

Multi-Clicq Convenience Kits (EM)		
No	REF/ Product name	Product Description
244	MUL-GM-RPKS3020K	Multi-Clicq Angled Attachment GM-RP 30°/ H 2 mm KIT
245	MUL-CF-RCASK	Multi-Clicq Multi-Unit CF-RC Starter KIT A Type
246	MUL-CF-RCBSK	Multi-Clicq Multi-Unit CF-RC Starter KIT B Type
247	MUL-CF-RCAPK	Multi-Clicq Multi-Unit CF-RC Professional KIT A Type
248	MUL-CF-RCBPK	Multi-Clicq Multi-Unit CF-RC Professional KIT B Type
249	MUL-CF-RCKS15K	Multi-Clicq Straight Multi-Unit CF-RC H 1.5 mm KIT
250	MUL-CF-RCKS25K	Multi-Clicq Straight Multi-Unit CF-RC H 2.5 mm KIT
251	MUL-CF-RCKS35K	Multi-Clicq Straight Multi-Unit CF-RC H 3.5 mm KIT
252	MUL-CF-RCKS45K	Multi-Clicq Straight Multi-Unit CF-RC H 4.5 mm KIT
253	MUL-CF-RCKS55K	Multi-Clicq Straight Multi-Unit CF-RC H 5.5 mm KIT
254	MUL-CF-RCKS1710AK	Multi-Clicq Angled Multi-Unit CF-RC 17°/ H 1 mm KIT A Type
255	MUL-CF-RCKS1720AK	Multi-Clicq Angled Multi-Unit CF-RC 17°/ H 2 mm KIT A Type
256	MUL-CF-RCKS3010AK	Multi-Clicq Angled Multi-Unit CF-RC 30°/ H 1 mm KIT A Type
257	MUL-CF-RCKS3020AK	Multi-Clicq Angled Multi-Unit CF-RC 30°/ H 2 mm KIT A Type
258	MUL-CF-RCKS1710BK	Multi-Clicq Angled Multi-Unit CF-RC 17°/ H 1 mm KIT B Type
259	MUL-CF-RCKS1720BK	Multi-Clicq Angled Multi-Unit CF-RC 17°/ H 2 mm KIT B Type
260	MUL-CF-RCKS3010BK	Multi-Clicq Angled Multi-Unit CF-RC 30°/ H 1 mm KIT B Type
261	MUL-CF-RCKS3020BK	Multi-Clicq Angled Multi-Unit CF-RC 30°/ H 2 mm KIT B Type
262	MUL-CF-NCASK	Multi-Clicq Multi-Unit CF-NC Starter KIT A Type

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28


Multi-Clicq Convenience Kits (EM)		
No	REF/ Product name	Product Description
263	MUL-CF-NCBSK	Multi-Clicq Multi-Unit CF-NC Starter KIT B Type
264	MUL-CF-NCAPK	Multi-Clicq Multi-Unit CF-NC Professional KIT A Type
265	MUL-CF-NCBPK	Multi-Clicq Multi-Unit CF-NC Professional KIT B Type
266	MUL-CF-NCKS15K	Multi-Clicq Straight Multi-Unit CF-NC H 1.5 mm KIT
267	MUL-CF-NCKS25K	Multi-Clicq Straight Multi-Unit CF-NC H 2.5 mm KIT
268	MUL-CF-NCKS35K	Multi-Clicq Straight Multi-Unit CF-NC H 3.5 mm KIT
269	MUL-CF-NCKS45K	Multi-Clicq Straight Multi-Unit CF-NC H 4.5 mm KIT
270	MUL-CF-NCKS55K	Multi-Clicq Straight Multi-Unit CF-NC H 5.5 mm KIT
271	MUL-CF-NCKS1710AK	Multi-Clicq Angle Multi-Unit CF-NC 17°/ H 1 mm KIT A Type
272	MUL-CF-NCKS1720AK	Multi-Clicq Angle Multi-Unit CF-NC 17°/ H 2 mm KIT A Type
273	MUL-CF-NCKS3010AK	Multi-Clicq Angle Multi-Unit CF-NC 30°/ H 1 mm KIT A Type
274	MUL-CF-NCKS3020AK	Multi-Clicq Angle Multi-Unit CF-NC 30°/ H 2 mm KIT A Type
275	MUL-CF-NCKS1710BK	Multi-Clicq Angle Multi-Unit CF-NC 17°/ H 1 mm KIT B Type
276	MUL-CF-NCKS1720BK	Multi-Clicq Angle Multi-Unit CF-NC 17°/ H 2 mm KIT B Type
277	MUL-CF-NCKS3010BK	Multi-Clicq Angle Multi-Unit CF-NC 30°/ H 1 mm KIT B Type
278	MUL-CF-NCKS3020BK	Multi-Clicq Angle Multi-Unit CF-NC 30°/ H 2 mm KIT B Type
279	MUL-KS-BK4UG	Clicq Multi Bar Pre-Fabricated Bar on 4 for KS
280	MUL-KS-BK4DUG	Clicq Multi Bar Pre-Fabricated Bar on 4 for KS + Driver
281	MUL-KS-BK6UG	Clicq Multi Bar Pre-Fabricated Bar on 6 for KS

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28


Multi-Clicq Convenience Kits (EM)		
No	REF/ Product name	Product Description
282	MUL-KS-BK6DUG	Clicq Multi Bar Pre-Fabricated Bar on 6 for KS + Driver
283	MUL-KS-BK4AR	Clicq Multi Bar Pre-Fabricated Bar on 4 for KS
284	MUL-KS-BK4DAR	Clicq Multi Bar Pre-Fabricated Bar on 4 for KS + Driver
285	MUL-KS-BK6AR	Clicq Multi Bar Pre-Fabricated Bar on 6 for KS
286	MUL-KS-BK6DAR	Clicq Multi Bar Pre-Fabricated Bar on 6 for KS + Driver
287	MUL-KS-BK4TX	Clicq Multi Bar Pre-Fabricated Bar on 4 for KS
288	MUL-KS-BK4DTX	Clicq Multi Bar Pre-Fabricated Bar on 4 for KS + Driver
289	MUL-KS-BK6TX	Clicq Multi Bar Pre-Fabricated Bar on 6 for KS
290	MUL-KS-BK6DTX	Clicq Multi Bar Pre-Fabricated Bar on 6 for KS + Driver
291	MUL-NB-BK4UG	Clicq Multi Bar Pre-Fabricated Bar on 4 for Classic
292	MUL-NB-BK4DUG	Clicq Multi Bar Pre-Fabricated Bar on 4 for Classic + Driver
293	MUL-NB-BK6UG	Clicq Multi Bar Pre-Fabricated Bar on 6 for Classic
294	MUL-NB-BK6DUG	Clicq Multi Bar Pre-Fabricated Bar on 6 for Classic + Driver
295	MUL-NB-BK4TX	Clicq Multi Bar Pre-Fabricated Bar on 4 for Classic
296	MUL-NB-BK4DTX	Clicq Multi Bar Pre-Fabricated Bar on 4 for Classic + Driver
297	MUL-NB-BK6TX	Clicq Multi Bar Pre-Fabricated Bar on 6 for Classic
298	MUL-NB-BK6DTX	Clicq Multi Bar Pre-Fabricated Bar on 6 for Classic + Driver
299	MUL-NB-BK4AR	Clicq Multi Bar Pre-Fabricated Bar on 4 for Classic
300	MUL-NB-BK4DAR	Clicq Multi Bar Pre-Fabricated Bar on 4 for Classic + Driver
301	MUL-NB-BK6AR	Clicq Multi Bar Pre-Fabricated Bar on 6 for Classic
302	MUL-NB-BK6DAR	Clicq Multi Bar Pre-Fabricated Bar on 6 for Classic + Driver

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28


Tabelle 35 2 Piece Multi-Unit

		
2-Piece Multi-Unit Abutments		
No	REF/ Productname	Product Description
All On 4/ All ON 6 Multi Unit - Angled		
1	MUL-17-1	IVORY ON 4/ ON 6 MULTI UNIT Golden 17 Degree 1 mm
2	MUL-17-2	IVORY ON 4/ ON 6 MULTI UNIT Golden 17 Degree 2 mm
3	MUL-17-3	IVORY ON 4/ ON 6 MULTI UNIT Golden 17 Degree 3 mm
4	MUL-30-1	IVORY ON 4/ ON 6 MULTI UNIT Golden 30 Degree 1 mm
5	MUL-30-2	IVORY ON 4/ ON 6 MULTI UNIT Golden 30 Degree 2 mm
6	MUL-30-3	IVORY ON 4/ ON 6 MULTI UNIT Golden 30 Degree 3 mm
Straight Multi Unit/ KITS		
1	MUL-BA-1	Ball Attachment for IVORY ON 4/ ON 6 1mm
2	MUL-BA-2	Ball Attachment for IVORY ON 4/ ON 6 2mm
3	MUL-COD-0.5	Clicq Overdenture for IVORY ON 4/ ON 6 0.5 mm
4	MUL-COD-1	Clicq Overdenture for IVORY ON 4/ ON 6 1 mm
5	MUL-COD-2	Clicq Overdenture for IVORY ON 4/ ON 6 2 mm
6	MUL-SAM-1	Straight Connection Abutment for Multi Unit MUL 17/ 30 – 1 mm
7	MUL-SAM-2	Straight Connection Abutment for Multi Unit MUL 17/ 30 – 2 mm
8	MUL-TTA-9	Temporary Abutment 9 mm for Multi Unit
9	MUL-TTA-12	Temporary Abutment 12 mm for Multi Unit
10	MUL-STA-9	Straight Abutment for Multi Unit
11	MUL-OCA-05	Overdenture Connection Abutment for MULTI UNIT – 0.5 mm Shoulder
12	MUL-OCA-15	Overdenture Connection Abutment for MULTI UNIT – 1.5 mm Shoulder
13	MUL-PSCA	Plastic Sleeve for Connection Abutment for MULTI UNIT with MUL-SCS
14	MUL-TSCA	Titanium Sleeve for Connection Abutment for MULTI UNIT with MUL-SCS
15	MUL-SCA-1	Straight Connection Abument for MULTI UNIT – 1 mm
16	MUL-SCA-1BK	Straight Connection Abument for MULTI UNIT – 1 mm Basic Kit
17	MUL-SCA-1FK	Straight Connection Abument for MULTI UNIT – 1 mm Full Kit
18	MUL-SCA-2	Straight Connection Abument for MULTI UNIT – 2 mm

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

 <p style="text-align: center;">2-Piece Multi-Unit Abutments</p>		
No	REF/ Productname	Product Description
19	MUL-SCA-2BK	Straight Connection Abument for MULTI UNIT – 2 mm Basic Kit
20	MUL-SCA-2FK	Straight Connection Abument for MULTI UNIT – 2 mm Full Kit
21	MUL-SCA-3	Straight Connection Abument for MULTI UNIT – 3 mm
22	MUL-SCA-3BK	Straight Connection Abument for MULTI UNIT – 3 mm Basic Kit
23	MUL-SCA-3FK	Straight Connection Abument for MULTI UNIT – 3 mm Full Kit
24	MUL-SCA-4	Straight Connection Abument for MULTI UNIT – 4 mm
25	MUL-SCA-4BK	Straight Connection Abument for MULTI UNIT – 4 mm Basic Kit
26	MUL-SCA-4FK	Straight Connection Abument for MULTI UNIT – 4 mm Full Kit

*Tabelle 36 Clicq-Base (EM)*

 <p style="text-align: center;">Clicq-Base 2 Piece Ti-Base (EM)</p>		
No	REF/ Product name	Product Description
1	TBK-SV-RPONE	Clicq-Base SV-RP Straight One Piece for Adapter
2	TBK-SV-RP15	Clicq-Base SV-RP H 1.5 mm
3	TBK-SV-RP30	Clicq-Base SV-RP H 3 mm
4	TBK-SV-RP45	Clicq-Base SV-RP H 4.5 mm
5	TBK-SV-RPR	Clicq-Base SV-RP Rotational
6	TBK-SV-RP15R	Clicq-Base SV-RP H 1.5 mm Rotational
7	TBK-SV-RP30R	Clicq-Base SV-RP H 3 mm Rotational
8	TBK-SV-RP45R	Clicq-Base SV-RP H 4.5 mm Rotational
9	TBK-SV-RP1715A	Clicq-Base SV-RP 17°/ H 1.5 mm A Type
10	TBK-SV-RP1730A	Clicq-Base SV-RP 17°/ H 3 mm A Type
11	TBK-SV- RP1715UA	Clicq-Base SV-RP 17°/ H 1.5 mm A Type Anterior

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28



Clicq-Base 2 Piece Ti-Base (EM)

No	REF/ Product name	Product Description
12	TBK-SV-RP1715B	Clicq-Base SV-RP 17°/ H 1.5 mm B Type
13	TBK-SV-RP1730B	Clicq-Base SV-RP 17°/ H 3 mm B Type
14	TBK-SV-RP1715UB	Clicq-Base SV-RP 17°/ H 1.5 mm B Type Anterior
15	TBK-SV-RP1715R	Clicq-Base SV-RP 17°/ H 1.5 mm Rotational
16	TBK-SV-RP1730R	Clicq-Base SV-RP 17°/ H 3 mm Rotational
17	TBK-SV-RP1715UR	Clicq-Base SV-RP 17°/ H 1.5 mm Rotational Anterior
18	TBK-SV-RP3015A	Clicq-Base SV-RP 30°/ H 1.5 mm A Type
19	TBK-SV-RP3030A	Clicq-Base SV-RP 30°/ H 3 mm A Type
20	TBK-SV-RP3015UA	Clicq-Base SV-RP 30°/ H 1.5 mm A Type Anterior
21	TBK-SV-RP3015B	Clicq-Base SV-RP 30°/ H 1.5 mm B Type
22	TBK-SV-RP3030B	Clicq-Base SV-RP 30°/ H 3 mm B Type
23	TBK-SV-RP3015UB	Clicq-Base SV-RP 30°/ H 1.5 mm B Type Anterior
24	TBK-SV-RP3015R	Clicq-Base SV-RP 30°/ H 1.5 mm Rotational
25	TBK-SV-RP3030R	Clicq-Base SV-RP 30°/ H 3 mm Rotational
26	TBK-SV-RP3015UR	Clicq-Base SV-RP 30°/ H 1.5 mm Rotational Anterior
27	TBK-SV-WONE	Clicq-Base SV-WP Straight One Piece for Adapter
28	TBK-SV-WP15	Clicq-Base SV-WP H 1.5 mm
29	TBK-SV-WP30	Clicq-Base SV-WP H 3 mm
30	TBK-SV-WP45	Clicq-Base SV-WP H 4.5 mm
31	TBK-SV-WPR	Clicq-Base SV-WP Rotational
32	TBK-SV-WP15R	Clicq-Base SV-WP H 1.5 mm Rotational
33	TBK-SV-WP30R	Clicq-Base SV-WP H 3 mm Rotational

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28



Clicq-Base 2 Piece Ti-Base (EM)

No	REF/ Product name	Product Description
34	TBK-SV-WP1715A	Clicq-Base SV-WP 17°/ H 1.5 mm A Type
35	TBK-SV-WP1730A	Clicq-Base SV-WP 17°/ H 3 mm A Type
36	TBK-SV-WP1715B	Clicq-Base SV-WP 17°/ H 1.5 mm B Type
37	TBK-SV-WP1730B	Clicq-Base SV-WP 17°/ H 3 mm B Type
38	TBK-SV-WP1715R	Clicq-Base SV-WP 17°/ H 1.5 mm Rotational
39	TBK-SV-WP1730R	Clicq-Base SV-WP 17°/ H 3 mm Rotational
40	TBK-SV-WP3015A	Clicq-Base SV-WP 30°/ H 1.5 mm A Type
41	TBK-SV-WP3030A	Clicq-Base SV-WP 30°/ H 3 mm A Type
42	TBK-SV-WP3015B	Clicq-Base SV-WP 30°/ H 1.5 mm B Type
43	TBK-SV-WP3030B	Clicq-Base SV-WP 30°/ H 3 mm B Type
44	TBK-SV-WP3015R	Clicq-Base SV-WP 30°/ H 1.5 mm Rotational
45	TBK-SV-WP3030R	Clicq-Base SV-WP 30°/ H 3 mm Rotational
46	TBK-SV-NLONE	Clicq-Base SV-NL Straight One Piece for Adapter
47	TBK-SV-NL15	Clicq-Base SV-NL H 1.5 mm
48	TBK-SV-NL30	Clicq-Base SV-NL H 3 mm
49	TBK-SV-NL45	Clicq-Base SV-NL H 4.5 mm
50	TBK-SV-NLR	Clicq-Base SV-NL Rotational
51	TBK-SV-NL15R	Clicq-Base SV-NL H 1.5 mm Rotational
52	TBK-SV-NL30R	Clicq-Base SV-NL H 3 mm Rotational
53	TBK-SV-NL45R	Clicq-Base SV-NL H 4.5 mm Rotational
54	TBK-SV-NL1715A	Clicq-Base SV-NL 17°/ H 1.5 mm A Type
55	TBK-SV-NL1730A	Clicq-Base SV-NL 17°/ H 3 mm A Type
56	TBK-SV-NL1715UA	Clicq-Base SV-NL 17°/ H 1.5 mm A Type Antrior
57	TBK-SV-NL1715B	Clicq-Base SV-NL 17°/ H 1.5 mm B Type

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28



Clicq-Base 2 Piece Ti-Base (EM)

No	REF/ Product name	Product Description
58	TBK-SV-NL1730B	Clicq-Base SV-NL 17°/ H 3 mm B Type
59	TBK-SV-NL1715UB	Clicq-Base SV-NL 17°/ H 1.5 mm B Type Antrior
60	TBK-SV-NL1715R	Clicq-Base SV-NL 17 / H 1.5 mm Rotational
61	TBK-SV-NL1730R	Clicq-Base SV-NL 17 / H 3 mm Rotational
62	TBK-SV-NL1715UR	Clicq-Base SV-NL 17 / H 1.5 mm Rotational Antrior
63	TBK-SV-NL3015A	Clicq-Base SV-NL 30°/ H 1.5 mm A Type
64	TBK-SV-NL3030A	Clicq-Base SV-NL 30°/ H 3 mm A Type
65	TBK-SV-NL3015UA	Clicq-Base SV-NL 30 / H 1.5 mm A Type Antrior
66	TBK-SV-NL3015B	Clicq-Base SV-NL 30°/ H 1.5 mm B Type
67	TBK-SV-NL3030B	Clicq-Base SV-NL 30°/ H 3 mm B Type
68	TBK-SV-NL3015UB	Clicq-Base SV-NL 30 / H 1.5 mm B Type Antrior
69	TBK-SV-NL3015R	Clicq-Base SV-NL 30°/ H 1.5 mm Rotational
70	TBK-SV-NL3030R	Clicq-Base SV-NL 30°/ H 3 mm Rotational
71	TBK-SV-NL3015UR	Clicq-Base SV-NL 30°/ H 1.5 mm Rotational Antrior
72	TBK-SV-NPONE	Clicq-Base SV-NP Straight One Piece for Adapter
73	TBK-SV-NP15	Clicq-Base SV-NP H 1.5 mm
74	TBK-SV-NP30	Clicq-Base SV-NP H 3 mm
75	TBK-SV-NP45	Clicq-Base SV-NP H 4.5 mm
76	TBK-SV-NPR	Clicq-Base SV-NP Rotational
77	TBK-SV-NP15R	Clicq-Base SV-NP H 1.5 mm Rotational
78	TBK-SV-NP30R	Clicq-Base SV-NP H 3 mm Rotational
79	TBK-SV-NP45R	Clicq-Base SV-NP H 4.5 mm Rotational

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28



Clicq-Base 2 Piece Ti-Base (EM)

No	REF/ Product name	Product Description
80	TBK-SV-NP1715A	Clicq-Base SV-NP 17° H 1.5 mm A Type
81	TBK-SV-NP1730A	Clicq-Base SV-NP 17° H 3 mm A Type
82	TBK-SV-NP1715UA	Clicq-Base SV-NP 17° H 1.5 mm A Type Anterior
83	TBK-SV-NP1715B	Clicq-Base SV-NP 17° H 1.5 mm B Type
84	TBK-SV-NP1730B	Clicq-Base SV-NP 17° H 3 mm B Type
85	TBK-SV-NP1715UB	Clicq-Base SV-NP 17° H 1.5 mm B Type Anterior
86	TBK-SV-NP1715R	Clicq-Base SV-NP 17° H 1.5 mm Rotational
87	TBK-SV-NP1730R	Clicq-Base SV-NP 17° H 3 mm Rotational
88	TBK-SV-NP1715UR	Clicq-Base SV-NP 17° H 1.5 mm Rotational Anterior
89	TBK-SV-NP3015A	Clicq-Base SV-NP 30° H 1.5 mm A Type
90	TBK-SV-NP3030A	Clicq-Base SV-NP 30° H 3 mm A Type
91	TBK-SV-NP3015UA	Clicq-Base SV-NP 30° H 1.5 mm A Type Anterior
92	TBK-SV-NP3015B	Clicq-Base SV-NP 30° H 1.5 mm B Type
93	TBK-SV-NP3030B	Clicq-Base SV-NP 30° H 3 mm B Type
94	TBK-SV-NP3015UB	Clicq-Base SV-NP 30° H 1.5 mm B Type Anterior
95	TBK-SV-NP3015R	Clicq-Base SV-NP 30° H 1.5 mm Rotational
96	TBK-SV-NP3030R	Clicq-Base SV-NP 30° H 3 mm Rotational
97	TBK-SV-NP3015UR	Clicq-Base SV-NP 30° H 1.5 mm Rotational Anterior
98	TBK-NA-NPONE	Clicq-Base NA-NP Straight One Piece for Adapter
98	TBK-NA-NP15	Clicq-Base NA-NP H 1.5 mm
99	TBK-NA-NP30	Clicq-Base NA-NP H 3 mm

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28



Clicq-Base 2 Piece Ti-Base (EM)

No	REF/ Product name	Product Description
100	TBK-NA-NP45	Clicq-Base NA-NP H 4.5 mm
101	TBK-NA-NPR	Clicq-Base NA-NP Rotational
102	TBK-NA-NP15R	Clicq-Base NA-NP H 1.5 mm Rotational
103	TBK-NA-NP30R	Clicq-Base NA-NP H 3 mm Rotational
104	TBK-NA-NP45R	Clicq-Base NA-NP H 4.5 mm Rotational
105	TBK-NA-NP1715A	Clicq-Base NA-NP 17°/ H 1.5 mm A Type
106	TBK-NA-NP1730A	Clicq-Base NA-NP 17°/ H 3 mm A Type
107	TBK-NA-NP1715B	Clicq-Base NA-NP 17°/ H 1.5 mm B Type
108	TBK-NA-NP1730B	Clicq-Base NA-NP 17°/ H 3 mm B Type
109	TBK-NA-NP1715R	Clicq-Base NA-NP 17°/ H 1.5 mm Rotational
110	TBK-NA-NP1730R	Clicq-Base NA-NP 17°/ H 3 mm Rotational
111	TBK-NA-NP3015A	Clicq-Base NA-NP 30°/ H 1.5 mm A Type
112	TBK-NA-NP3030A	Clicq-Base NA-NP 30°/ H 3 mm A Type
113	TBK-NA-NP3015B	Clicq-Base NA-NP 30°/ H 1.5 mm B Type
114	TBK-NA-NP3030B	Clicq-Base NA-NP 30°/ H 3 mm B Type
115	TBK-NA-NP3015R	Clicq-Base NA-NP 30°/ H 1.5 mm Rotational
116	TBK-NA-NP3030R	Clicq-Base NA-NP 30°/ H 3 mm Rotational
117	TBK-NA-RPONE	Clicq-Base NA-RP Straight One Piece for Adapter
118	TBK-NA-RP15	Clicq-Base NA-RP H 1.5 mm
119	TBK-NA-RP30	Clicq-Base NA-RP H 13 mm
120	TBK-NA-RP45	Clicq-Base NA-RP H 4.5 mm
121	TBK-NA-RPR	Clicq-Base NA-RP Rotational
122	TBK-NA-RP15R	Clicq-Base NA-RP H 1.5 mm Rotational
123	TBK-NA-RP30R	Clicq-Base NA-RP H 3 mm Rotational

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28



Clicq-Base 2 Piece Ti-Base (EM)

No	REF/ Product name	Product Description
124	TBK-NA-RP45R	Clicq-Base NA-RP H 4.5 mm Rotational
125	TBK-NA-RP1715A	Clicq-Base NA-RP 17°/ H 1.5 mm A Type
126	TBK-NA-RP1730A	Clicq-Base NA-RP 17°/ H 3 mm A Type
127	TBK-NA-RP1715B	Clicq-Base NA-RP 17°/ H 1.5 mm B Type
128	TBK-NA-RP1730B	Clicq-Base NA-RP 17°/ H 3 mm B Type
129	TBK-NA-RP1715R	Clicq-Base NA-RP 17°/ H 1.5 mm Rotational
130	TBK-NA-RP1730R	Clicq-Base NA-RP 17°/ H 3 mm Rotational
131	TBK-NA-RP3015A	Clicq-Base NA-RP 30°/ H 1.5 mm A Type
132	TBK-NA-RP3030A	Clicq-Base NA-RP 30°/ H 3 mm A Type
133	TBK-NA-RP3015B	Clicq-Base NA-RP 30°/ H 1.5 mm B Type
134	TBK-NA-RP3030B	Clicq-Base NA-RP 30°/ H 3 mm B Type
135	TBK-NA-RP3015R	Clicq-Base NA-RP 30°/ H 1.5 mm Rotational
136	TBK-NA-RP3030R	Clicq-Base NA-RP 30°/ H 3 mm Rotational
137	TBK-TS-NPONE	Clicq-Base TS-NP/ Mini Straight One Piece for Adapter
138	TBK-TS-NP15	Clicq-Base TS-NP/ Mini H 1.5 mm
139	TBK-TS-NP30	Clicq-Base TS-NP/ Mini H 3 mm
140	TBK-TS-NP45	Clicq-Base TS-NP/ Mini H 4.5 mm
141	TBK-TS-NPR	Clicq-Base TS-NP/ Mini Rotational
142	TBK-TS-NP15R	Clicq-Base TS-NP/ Mini H 1.5 mm Rotational
143	TBK-TS-NP30R	Clicq-Base TS-NP/ Mini H 3 mm Rotational
144	TBK-TS-NP45R	Clicq-Base TS-NP/ Mini H 4.5 mm Rotational
145	TBK-TS-NP1715A	Clicq-Base TS-NP/ Mini 17°/ H 1.5 mm A Type
146	TBK-TS-NP1730A	Clicq-Base TS-NP/ Mini 17°/ H 3 mm A Type
147	TBK-TS-NP1715B	Clicq-Base TS-NP/ Mini 17°/ H 1.5 mm B Type

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28



Clicq-Base 2 Piece Ti-Base (EM)

No	REF/ Product name	Product Description
148	TBK-TS-NP1730B	Clicq-Base TS-NP/ Mini 17°/ H 3 mm B Type
149	TBK-TS-NP1715R	Clicq-Base TS-NP/ Mini 17°/ H 1.5 mm Rotational
150	TBK-TS-NP1730R	Clicq-Base TS-NP/ Mini 17°/ H 3 mm Rotational
151	TBK-TS-NP3015A	Clicq-Base TS-NP/ Mini 30°/ H 1.5 mm A Type
152	TBK-TS-NP3030A	Clicq-Base TS-NP/ Mini 30°/ H 3 mm A Type
153	TBK-TS-NP3015B	Clicq-Base TS-NP/ Mini 30°/ H 1.5 mm B Type
154	TBK-TS-NP3030B	Clicq-Base TS-NP/ Mini 30°/ H 3 mm B Type
155	TBK-TS-NP3015R	Clicq-Base TS-NP/ Mini 30°/ H 1.5 mm Rotational
156	TBK-TS-NP3030R	Clicq-Base TS-NP/ Mini 30°/ H 3 mm Rotational
157	TBK-TS-RPONE	Clicq-Base TS-RP Straight One Piece for Adapter
158	TBK-TS-RP15	Clicq-Base TS-RP H 1.5 mm
159	TBK-TS-RP30	Clicq-Base TS-RP H 3 mm
160	TBK-TS-RP45	Clicq-Base TS-RP H 4.5 mm
161	TBK-TS-RPR	Clicq-Base TS-RP Rotational
162	TBK-TS-RP15R	Clicq-Base TS-RP H 1.5 mm Rotational
163	TBK-TS-RP30R	Clicq-Base TS-RP H 3 mm Rotational
164	TBK-TS-RP45R	Clicq-Base TS-RP H 4.5 mm Rotational
165	TBK-TS-RP1715A	Clicq-Base TS-RP 17°/ H 1.5 mm A Type
166	TBK-TS-RP1730A	Clicq-Base TS-RP 17°/ H 3 mm A Type
167	TBK-TS-RP1715UA	Clicq-Base TS-RP 17°/ H 1.5 mm A Type Anterior
168	TBK-TS-RP1715B	Clicq-Base TS-RP 17°/ H 1.5 mm B Type
169	TBK-TS-RP1730B	Clicq-Base TS-RP 17°/ H 3 mm B Type
170	TBK-TS-RP1715UB	Clicq-Base TS-RP 17°/ H 1.5 mm B Type Anterior

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28



Clicq-Base 2 Piece Ti-Base (EM)

No	REF/ Product name	Product Description
171	TBK-TS-RP1715R	Clicq-Base TS-RP 17°/ H 1.5 mm Rotational
172	TBK-TS-RP1730R	Clicq-Base TS-RP 17°/ H 3 mm Rotational
173	TBK-TS-RP1715UR	Clicq-Base TS-RP 17°/ H 1.5 mm Rotational Anterior
174	TBK-TS-RP3015A	Clicq-Base TS-RP 30°/ H 1.5 mm A Type
175	TBK-TS-RP3030A	Clicq-Base TS-RP 30°/ H 3 mm A Type
176	TBK-TS-RP3015UA	Clicq-Base TS-RP 30°/ H 1.5 mm A Type Anterior
177	TBK-TS-RP3015B	Clicq-Base TS-RP 30°/ H 1.5 mm B Type
178	TBK-TS-RP3030B	Clicq-Base TS-RP 30°/ H 3 mm B Type
179	TBK-TS-RP3015UB	Clicq-Base TS-RP 30°/ H 1.5 mm B Type Anterior
180	TBK-TS-RP3015R	Clicq-Base TS-RP 30°/ H 1.5 mm Rotational
181	TBK-TS-RP3030R	Clicq-Base TS-RP 30°/ H 3 mm Rotational
182	TBK-TS-RP3015UR	Clicq-Base TS-RP 30°/ H 1.5 mm Rotational Anterior
183	TBK-OS-SPONE	Clicq-Base OS-SP Straight One Piece for Adapter
184	TBK-OS-SP15	Clicq-Base OS-SP H 1.5 mm
185	TBK-OS-SP30	Clicq-Base OS-SP H 3 mm
186	TBK-OS-SP45	Clicq-Base OS-SP H 4.5 mm
187	TBK-OS-SPR	Clicq-Base OS-SP Rotational
188	TBK-OS-SP15R	Clicq-Base OS-SP H 1.5 mm Rotational
189	TBK-OS-SP30R	Clicq-Base OS-SP H 3 mm Rotational
190	TBK-OS-SP45R	Clicq-Base OS-SP H 4.5 mm Rotational
191	TBK-OS-SP1715A	Clicq-Base OS-SP 17°/ H 1.5 mm A Type
192	TBK-OS-SP1730A	Clicq-Base OS-SP 17°/ H 3 mm A Type

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28



Clicq-Base 2 Piece Ti-Base (EM)

No	REF/ Product name	Product Description
193	TBK-OS-SP1715UA	Clicq-Base OS-SP 17°/ H 1.5 mm A Type Anterior
194	TBK-OS-SP1715B	Clicq-Base OS-SP 17°/ H 1.5 mm B Type
195	TBK-OS-SP1730B	Clicq-Base OS-SP 17°/ H 3 mm B Type
196	TBK-OS-SP1715UB	Clicq-Base OS-SP 17°/ H 1.5 mm B Type Anterior
197	TBK-OS-SP1715R	Clicq-Base OS-SP 17°/ H 1.5 mm Rotational
198	TBK-OS-SP1730R	Clicq-Base OS-SP 17°/ H 3 mm Rotational
199	TBK-OS-SP1715UR	Clicq-Base OS-SP 17°/ H 1.5 mm Rotational Anterior
200	TBK-OS-SP3015A	Clicq-Base OS-SP 30°/ H 1.5 mm A Type
201	TBK-OS-SP3030A	Clicq-Base OS-SP 30°/ H 3 mm A Type
202	TBK-OS-SP3015UA	Clicq-Base OS-SP 30°/ H 1.5 mm A Type Anterior
203	TBK-OS-SP3015B	Clicq-Base OS-SP 30°/ H 1.5 mm B Type
204	TBK-OS-SP3030B	Clicq-Base OS-SP 30°/ H 1.5 mm B Type
205	TBK-OS-SP3015UB	Clicq-Base OS-SP 30°/ H 1.5 mm B Type Anterior
206	TBK-OS-SP3015R	Clicq-Base OS-SP 30°/ H 1.5 mm Rotational
207	TBK-OS-SP3030R	Clicq-Base OS-SP 30°/ H 3 mm Rotational
208	TBK-OS-SP3015UR	Clicq-Base OS-SP 30°/ H 1.5 mm Rotational Anterior
209	TBK-OS-WPONE	Clicq-Base OS-WP Straight One Piece for Adapter
210	TBK-OS-WP15	Clicq-Base OS-WP H 1.5 mm
211	TBK-OS-WP30	Clicq-Base OS-WP H 3 mm
212	TBK-OS-WP45	Clicq-Base OS-WP H 4.5 mm
213	TBK-OS-WPR	Clicq-Base OS-WP Rotational

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28



Clicq-Base 2 Piece Ti-Base (EM)

No	REF/ Product name	Product Description
214	TBK-OS-WP15R	Clicq-Base OS-WP H 1.5 mm Rotational
215	TBK-OS-WP30R	Clicq-Base OS-WP H 3 mm Rotational
216	TBK-OS-WP45R	Clicq-Base OS-WP H 4.5 mm Rotational
217	TBK-OS-WP1715A	Clicq-Base OS-WP 17 / H 1.5 mm A Type
218	TBK-OS-WP1730A	Clicq-Base OS-WP 17 / H 3 mm A Type
219	TBK-OS-WP1715B	Clicq-Base OS-WP 17 / H 1.5 mm B Type
220	TBK-OS-WP1730B	Clicq-Base OS-WP 17 / H 3 mm B Type
221	TBK-OS-WP1715R	Clicq-Base OS-WP 17 / H 1.5 mm Rotational
222	TBK-OS-WP1730R	Clicq-Base OS-WP 17 / H 3 mm Rotational
223	TBK-OS-WP3015A	Clicq-Base OS-WP 30 / H 1.5 mm A Type
224	TBK-OS-WP3030A	Clicq-Base OS-WP 30 / H 3 mm A Type
225	TBK-OS-WP3015B	Clicq-Base OS-WP 30 / H 1.5 mm B Type
226	TBK-OS-WP3030B	Clicq-Base OS-WP 30 / H 3 mm B Type
227	TBK-OS-WP3015R	Clicq-Base OS-WP 30 / H 1.5 mm Rotational
228	TBK-OS-WP3030R	Clicq-Base OS-WP 30 / H 3 mm Rotational
229	TBK-AR-RPONE	Clicq-Base AR-RP Straight One Piece for Adapter
230	TBK-AR-RP15	Clicq-Base AR-RP H 1.5 mm
231	TBK-AR-RP30	Clicq-Base AR-RP H 3 mm
232	TBK-AR-RP45	Clicq-Base AR-RP H 4.5 mm
233	TBK-AR-RPR	Clicq-Base AR-RP Rotational
234	TBK-AR-RP15R	Clicq-Base AR-RP H 1.5 mm Rotational
235	TBK-AR-RP30R	Clicq-Base AR-RP H 3 mm Rotational
236	TBK-AR-RP45R	Clicq-Base AR-RP H 4.5 mm Rotational
237	TBK-AR-RP1715A	Clicq-Base AR-RP 17° / H 1.5 mm A Type

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28



Clicq-Base 2 Piece Ti-Base (EM)

No	REF/ Product name	Product Description
238	TBK-AR-RP1730A	Clicq-Base AR-RP 17°/ H 3 mm A Type
239	TBK-AR-RP1715B	Clicq-Base AR-RP 17°/ H 1.5 mm B Type
240	TBK-AR-RP1730B	Clicq-Base AR-RP 17°/ H 3 mm B Type
241	TBK-AR-RP1715R	Clicq-Base AR-RP 17°/ H 1.5 mm Rotational
242	TBK-AR-RP1730R	Clicq-Base AR-RP 17°/ H 3 mm Rotational
243	TBK-AR-RP3015A	Clicq-Base AR-RP 30°/ H 1.5 mm A Type
244	TBK-AR-RP3030A	Clicq-Base AR-RP 30°/ H 3 mm A Type
245	TBK-AR-RP3015B	Clicq-Base AR-RP 30°/ H 1.5 mm B Type
246	TBK-AR-RP3030B	Clicq-Base AR-RP 30°/ H 3 mm B Type
247	TBK-AR-RP3015R	Clicq-Base AR-RP 30°/ H 1.5 mm Rotational
248	TBK-AR-RP3030R	Clicq-Base AR-RP 30°/ H 3 mm Rotational
249	TBK-GM-RPONE	Clicq-Base GM-RP Straight One Piece for Adapter
250	TBK-GM-RP15	Clicq-Base GM-RP GM-RP H 1.5 mm
251	TBK-GM-RP30	Clicq-Base GM-RP GM-RP H 3 mm
252	TBK-GM-RP45	Clicq-Base GM-RP GM-RP H 4.5 mm
253	TBK-GM-RPR	Clicq-Base GM-RP GM-RP Rotational
254	TBK-GM-RP15	Clicq-Base GM-RP GM-RP H 1.5 mm Rotational
255	TBK-GM-RP30	Clicq-Base GM-RP GM-RP H 3 mm Rotational
256	TBK-GM-RP45	Clicq-Base GM-RP GM-RP H 4.5 mm Rotational
257	TBK-GM-RP1715A	Clicq-Base GM-RP GM-RP 17°/ H 1.5 mm A Type
258	TBK-GM-RP1730A	Clicq-Base GM-RP GM-RP 17°/ H 3 mm A Type
259	TBK-GM-RP1715B	Clicq-Base GM-RP GM-RP 17°/ H 1.5 mm B Type
260	TBK-GM-RP1730B	Clicq-Base GM-RP GM-RP 17°/ H 3 mm B Type
261	TBK-GM-RP1715R	Clicq-Base GM-RP GM-RP 17°/ H 1.5 mm Rotational

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28



Clicq-Base 2 Piece Ti-Base (EM)

No	REF/ Product name	Product Description
262	TBK-GM-RP1730R	Clicq-Base GM-RP GM-RP 17°/ H 3 mm Rotational
263	TBK-GM-RP3015A	Clicq-Base GM-RP GM-RP 30°/ H 1.5 mm A Type
264	TBK-GM-RP3030A	Clicq-Base GM-RP GM-RP 30°/ H 3 mm A Type
265	TBK-GM-RP3015B	Clicq-Base GM-RP GM-RP 30°/ H 1.5 mm B Type
266	TBK-GM-RP3030B	Clicq-Base GM-RP GM-RP 30°/ H 3 mm B Type
267	TBK-GM-RP3015R	Clicq-Base GM-RP GM-RP 30°/ H 1.5 mm Rotational
268	TBK-GM-RP3030R	Clicq-Base GM-RP GM-RP 30°/ H 3 mm Rotational
269	TBK-CF-RCONE	Clicq-Base CF-RC Straight One Piece for Adapter
270	TBK-CF-RC15	Clicq-Base CF-RC H 1.5 mm
271	TBK-CF-RC30	Clicq-Base CF-RC H 3 mm
272	TBK-CF-RC45	Clicq-Base CF-RC H 4.5 mm
273	TBK-CF-RCR	Clicq-Base CF-RC Rotational
274	TBK-CF-RC15R	Clicq-Base CF-RC H 1.5 mm Rotational
275	TBK-CF-RC30R	Clicq-Base CF-RC H 3 mm Rotational
276	TBK-CF-RC45R	Clicq-Base CF-RC H 4.5 mm Rotational
277	TBK-CF-RC1715A	Clicq-Base CF-RC 17°/ H 1.5 mm A Type
278	TBK-CF-RC1730A	Clicq-Base CF-RC 17°/ H 3 mm A Type
279	TBK-CF-RC1715B	Clicq-Base CF-RC 17°/ H 1.5 mm B Type
280	TBK-CF-RC1730B	Clicq-Base CF-RC 17°/ H 3 mm B Type
281	TBK-CF-RC1715R	Clicq-Base CF-RC 17°/ H 1.5 mm Rotational
282	TBK-CF-RC1730R	Clicq-Base CF-RC 17°/ H 3 mm Rotational
283	TBK-CF-RC3015A	Clicq-Base CF-RC 30°/ H 1.5 mm A Type
284	TBK-CF-RC3030A	Clicq-Base CF-RC 30°/ H 3 mm A Type
285	TBK-CF-RC3015B	Clicq-Base CF-RC 30°/ H 1.5 mm B Type


Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28




Clicq-Base 2 Piece Ti-Base (EM)

No	REF/ Product name	Product Description
286	TBK-CF-RC3030B	Clicq-Base CF-RC 30°/ H 3 mm B Type
287	TBK-CF-RC3015R	Clicq-Base CF-RC 30°/ H 1.5 mm Rotational
288	TBK-CF-RC3030R	Clicq-Base CF-RC 30°/ H 3 mm Rotational
289	TBK-CF-NCONE	Clicq-Base CF-NC Straight ONE Piece for Adapter
290	TBK-CF-NC15	Clicq-Base CF-NC H 1.5 mm
291	TBK-CF-NC30	Clicq-Base CF-NC H 3 mm
292	TBK-CF-NC45	Clicq-Base CF-NC H 4.5 mm
293	TBK-CF-NCR	Clicq-Base CF-NC Rotational
294	TBK-CF-NC15R	Clicq-Base CF-NC H 1.5 mm Rotational
295	TBK-CF-NC30R	Clicq-Base CF-NC H 3 mm Rotational
296	TBK-CF-NC45R	Clicq-Base CF-NC H 4.5 mm Rotational
297	TBK-CF-NC1715A	Clicq-Base CF-NC 17°/ H 1.5 mm A Type
298	TBK-CF-NC1730A	Clicq-Base CF-NC 17°/ H 3 mm A Type
299	TBK-CF-NC1715B	Clicq-Base CF-NC 17°/ H 1.5 mm B Type
300	TBK-CF-NC1730B	Clicq-Base CF-NC 17°/ H 3 mm B Type
301	TBK-CF-NC1715R	Clicq-Base CF-NC 17°/ H 1.5 mm Rotational
302	TBK-CF-NC1730R	Clicq-Base CF-NC 17°/ H 3 mm Rotational
303	TBK-CF-NC3015A	Clicq-Base CF-NC 30°/ H 1.5 mm A Type
304	TBK-CF-NC3030A	Clicq-Base CF-NC 30°/ H 3 mm A Type
305	TBK-CF-NC3015B	Clicq-Base CF-NC 30°/ H 1.5 mm B Type
306	TBK-CF-NC3030B	Clicq-Base CF-NC 30°/ H 3 mm B Type
307	TBK-CF-NC3015R	Clicq-Base CF-NC 30°/ H 1.5 mm Rotational
308	TBK-CF-NC3030R	Clicq-Base CF-NC 30°/ H 3 mm Rotational
309	TBK-SLVTIAR	Clicq-Base Ti-Sleeve

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

 <p>Clicq-Base 2 Piece Ti-Base (EM)</p>		
No	REF/ Product name	Product Description
310	TBK-SLVTIUG	Clicq-Base Ti-Sleeve Ti-Sleeve (Dual Hex/Unigrip)
311	TBK-SLVTITX	Clicq-Base Ti-Sleeve Ti-Sleeve
312	TBK-TBANT	Titanium Ti Base Sleeve for Clicq-Base Anterior
313	TBK-TBAR	Titanium Ti Base Sleeve for Clicq-Base
314	TBK-TBRAR	Titanium Ti Base Sleeve for Clicq-Base Rotational
315	TBKU-TBAR	Titanium Ti Base Sleeve for Clicq-Base
316	TBK-TBTX	Titanium Ti Base Sleeve for Clicq-Base
317	TBK-TBRTX	Titanium Ti Base Sleeve for Clicq-Base Rotational
318	TBKU-TBTX	Titanium Ti Base Sleeve for Clicq-Base
319	TBK-TBUG	Titanium Ti Base Sleeve for Clicq-Base
320	TBK-TBRUG	Titanium Ti Base Sleeve for Clicq-Base Rotational
321	TBKU-TBUG	Titanium Ti Base Sleeve for Clicq-Base
322	TBK-KS-MULAR	Multi-Clicq Multi Unit Abutment for Clicq-Base
323	TBK-NB-MULAR	"Classic" Multi unit Abutment for Clicq-Base
324	TBK-LOCW-AR	Pop-Clicq Abutment Clicq-Base
325	TBK-KS-MULUG	Multi-Clicq Multi Unit Abutment for Clicq-Base
326	TBK-NB-MULUG	"Classic" Multi Unit Abutment for Clicq-Base

*Tabelle 37 Abutments zum Angießen*

 <p>Abutments for Casting</p>		
No	REF/ Productname	Product Description
1	EAPC-15-1	Burn-It Esthetic Angled Plastic Abutment for Casting 15 Degree 1 mm Shoulder


Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28




Abutments for Casting

No	REF/ Productname	Product Description
2	EAPC-15-2	Burn-It Esthetic Angled Plastic Abutment for Casting 15 Degree 2 mm Shoulder
3	EAPC-15-3	Burn-It Esthetic Angled Plastic Abutment for Casting 15 Degree 3 mm Shoulder
4	EAPC-25-1	Burn-It Esthetic Angled Plastic Abutment for Casting 25 Degree 1 mm Shoulder
5	EAPC-25-2	Burn-It Esthetic Angled Plastic Abutment for Casting 25 Degree 2 mm Shoulder
6	EAPC-25-3	Burn-It Esthetic Angled Plastic Abutment for Casting 25 Degree 3 mm Shoulder
7	PAC	Burn-It Plastic Sleeve Abutment for Laboratory Casting/ non Hex
8	PAC-H	Burn-It Plastic Sleeve Abutment for Laboratory Casting with Hex
9	PA-H	Burn-It Plastic Abutment Laboratory with Hex
10	PA-N	Burn-It Plastic Abutment Laboratory/ non Hex
11	AZA	Titanium Abutment for Casting with Plastic Sleeve
12	PACA	Plastic Connection Abutment/ non Hex
13	PACA-H	Plastic Connection Abutment with Hex
Abutments for Casting Straight		
14*	AZA-L	Titanium Abutment Long with Plastic Casting Sleeve
15*	NL-AZA-L	Titanium Abutment Long with Plastic Casting Sleeve, Narrow Line
16*	AZA-CC	Chrome Cobalt Abutment with Plastic Casting Sleeve
17*	NL-AZA-CC	Chrome Cobalt Abutment with Plastic Casting Sleeve, Narrow Line
18*	AZA-CC-L	Chrome Cobalt Abutment Long with Plastic Casting Sleeve
19*	NL-AZA-CC-L	Chrome Cobalt Abutment Long with Plastic Casting Sleeve, Narrow Line
20	SAPC-1	Burn-It Straight Anatomic Plastic Abutment for Casting 1 mm Shoulder
21	SAPC-2	Burn-It Straight Anatomic Plastic Abutment for Casting 2 mm Shoulder
22	SAPC-3	Burn-It Straight Anatomic Plastic Abutment for Casting 3 mm Shoulder


Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

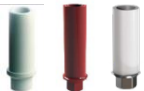 Abutments for Casting		
No	REF/ Productname	Product Description
23	COD-CPC	Plastic Abutment for Casting for Clicq Overdenture
24	COD-CPK	Clicq Overdenture Casting Kit (MCL, SCL-T, SCL-P, COD-CPC)


*Tabelle 38 Castable & UCLA Abutments (EM)*

 Castable & UCLA Abutments (EM)		
No	REF/ Productname	Product Description
1	ABZ-SV-RPCC	Chrome Cobalt Base SV-RP/ UCLA
2	ABZ-SV-RPCCR	Chrome Cobalt Base SV-RP/ UCLA Rotational
3	ABZ-SV-WPCC	Chrome Cobalt Base SV-WP/ UCLA
4	ABZ-SV-WPCCR	Chrome Cobalt Base SV-WP/ UCLA Rotational
5	ABZ-SV-NLCC	Chrome Cobalt Base SV-NL/ UCLA
6	ABZ-SV-NLCCR	Chrome Cobalt Base SV-NL/ UCLA Rotational
7	ABZ-SV-NPCC	Chrome Cobalt Base SV-NP/ UCLA
8	ABZ-SV-NPCCR	Chrome Cobalt Base SV-NP/ UCLA Rotational
9	ABZ-NA-NPCC	Chrome Cobalt Base NA-NP/ UCLA
10	ABZ-NA-NPCCR	Chrome Cobalt Base NA-NP/ UCLA Rotational
11	ABZ-NA-RPCC	Chrome Cobalt Base NA-RP/ UCLA
12	ABZ-NA-RPCCR	Chrome Cobalt Base NA-RP/ UCLA Rotational
13	ABZ-TS-NPCC	Chrome Cobalt Base TS-NP/ Mini/ UCLA
14	ABZ-TS-NPCCR	Chrome Cobalt Base TS-NP/ Mini/ UCLA Rotational
15	ABZ-TS-RPCC	Chrome Cobalt Base TS-RP/ UCLA
16	ABZ-TS-RPCCR	Chrome Cobalt Base TS-RP/ UCLA Rotational
17	ABZ-OS-SPCC	Chrome Cobalt Base OS-SP/ UCLA


Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

		
Castable & UCLA Abutments (EM)		
No	REF/ Productname	Product Description
18	ABZ-OS-SPCCR	Chrome Cobalt Base OS-SP/ UCLA Rotational
19	ABZ-OS-WPCC	Chrome Cobalt Base OS-WP/ UCLA
20	ABZ-OS-WPCCR	Chrome Cobalt Base OS-WP/ UCLA Rotational
21	ABZ-AR-RPCC	Chrome Cobalt Base AR-RP/ UCLA
22	ABZ-AR-RPCCR	Chrome Cobalt Base AR-RP/ UCLA Rotational
23	ABZ-GM-RPCC	Chrome Cobalt Base GM-RP/ UCLA
24	ABZ-GM-RPCCR	Chrome Cobalt Base GM-RP/ UCLA Rotational
25	ABZ-CF-RCCC	Chrome Cobalt Base CF-RC/ UCLA
26	ABZ-CF-RCCCR	Chrome Cobalt Base CF-RC/ UCLA Rotational
27	ABZ-CF-NCCC	Chrome Cobalt Base CF-NC/ UCLA
28	ABZ-CF-NCCCR	Chrome Cobalt Base CF-NC/ UCLA Rotational
29	MUL-KS- SLVCCUG	Multi-Clicq Cobalt Chrome UCLA
30	MUL-KS- SLVPSUG	Multi-Clicq Castable Plastic Sleeve
31	TAP-WS	Clicq Pression Waxing Sleeve
32	PAC-SV-RP	Plastic Castable Abutment


*Tabelle 39 Milling Blanks*

		
Milling Blanks		
No	REF/ Productname	Product Description
1*	ML-10-23	Milling Blank for Casting with HEX 10 mm D/ 23 mm L
2*	NL-ML-10-23	Milling Blank for Casting with HEX 10 mm D/ 23 mm L, Narrow Line
3*	ML-10-24	Milling Blank for Casting with HEX 12mm D/ 24mmL
4*	NL-ML-10-24	Milling Blank for Casting with HEX 12mm D/ 24mmL, Narrow Line

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

 Milling Blanks		
No	REF/ Productname	Product Description
5*	ML-10-24R	Milling Blank for Casting with HEX 12mm D/ 24mmL
6*	NL-ML-10-24R	Milling Blank for Casting with HEX 12mm D/ 24mmL, Narrow Line
7*	ML-12-24	Milling Blanks for Casting with HEX 12 mm D/ 24 mm L
8*	NL-ML-12-24	Milling Blanks for Casting with HEX 12 mm D/ 24 mm L, Narrow Line

*Tabelle 40 Pre-Milled Blank (EM)*

 Pre-Milled Blank (EM)		
No	REF/ Productname	Product Description
1	PMAM-SV-RP	Pre-Milled Ti-Abutment SV-RP
2	PMMX-SV-RP	Pre-Milled Ti-Abutment SV-RP
3	PMAG-SV-RP	Pre-Milled Ti-Abutment SV-RP
4	PMZZ-SV-RP	Pre-Milled Ti-Abutment SV-RP
5	PMMD-SV-RP	Pre-Milled Ti-Abutment SV-RP
6	PMNT-SV-RP	Pre-Milled Ti-Abutment SV-RP
7	PMAM-SV-RPR	Pre-Milled Ti-Abutment SV-RP Rotational
8	PMMX-SV-RPR	Pre-Milled Ti-Abutment SV-RP Rotational
9	PMAG-SV-RPR	Pre-Milled Ti-Abutment SV-RP Rotational
10	PMZZ-SV-RPR	Pre-Milled Ti-Abutment SV-RP Rotational
11	PMMD-SV-RPR	Pre-Milled Ti-Abutment SV-RP Rotational
12	PMNT-SV-RPR	Pre-Milled Ti-Abutment SV-RP Rotational
13	PMAM-SV-WP	Pre-Milled Ti-Abutment SV-WP
14	PMMX-SV-WP	Pre-Milled Ti-Abutment SV-WP
15	PMAG-SV-WP	Pre-Milled Ti-Abutment SV-WP

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

 Pre-Milled Blank (EM)		
No	REF/ Productname	Product Description
16	PMZZ-SV-WP	Pre-Milled Ti-Abutment SV-WP
17	PMMD-SV-WP	Pre-Milled Ti-Abutment SV-WP
18	PMNT-SV-WP	Pre-Milled Ti-Abutment SV-WP
19	PMAM-SV-WPR	Pre-Milled Ti-Abutment SV-WP Rotational
20	PMMX-SV-WPR	Pre-Milled Ti-Abutment SV-WP Rotational
21	PMAG-SV-WPR	Pre-Milled Ti-Abutment SV-WP Rotational
22	PMZZ-SV-WPR	Pre-Milled Ti-Abutment SV-WP Rotational
23	PMMD-SV-WPR	Pre-Milled Ti-Abutment SV-WP Rotational
24	PMNT-SV-WPR	Pre-Milled Ti-Abutment SV-WP Rotational
25	PMAM-SV-NL	Pre-Milled Ti-Abutment SV-NL
26	PMMX-SV-NL	Pre-Milled Ti-Abutment SV-NL
27	PMAG-SV-NL	Pre-Milled Ti-Abutment SV-NL
28	PMZZ-SV-NL	Pre-Milled Ti-Abutment SV-NL
29	PMMD-SV-NL	Pre-Milled Ti-Abutment SV-NL
30	PMNT-SV-NL	Pre-Milled Ti-Abutment SV-NL
31	PMAM-SV-NLR	Pre-Milled Ti-Abutment SV-NL Rotational
32	PMMX-SV-NLR	Pre-Milled Ti-Abutment SV-NL Rotational
33	PMAG-SV-NLR	Pre-Milled Ti-Abutment SV-NL Rotational
34	PMZZ-SV-NLR	Pre-Milled Ti-Abutment SV-NL Rotational
35	PMMD-SV-NLR	Pre-Milled Ti-Abutment SV-NL Rotational
36	PMNT-SV-NPR	Pre-Milled Ti-Abutment SV-NL Rotational
37	PMAM-SV-NP	Pre-Milled Ti-Abutment SV-NP
38	PMMX-SV-NP	Pre-Milled Ti-Abutment SV-NP
39	PMAG-SV-NP	Pre-Milled Ti-Abutment SV-NP

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

 Pre-Milled Blank (EM)		
No	REF/ Productname	Product Description
40	PMZZ-SV-NP	Pre-Milled Ti-Abutment SV-NP
41	PMMD-SV-NP	Pre-Milled Ti-Abutment SV-NP
42	PMNT-SV-NP	Pre-Milled Ti-Abutment SV-NP
43	PMAM-SV-NPR	Pre-Milled Ti-Abutment SV-NP Rotational
44	PMMX-SV-NPR	Pre-Milled Ti-Abutment SV-NP Rotational
45	PMAG-SV-NPR	Pre-Milled Ti-Abutment SV-NP Rotational
46	PMZZ-SV-NPR	Pre-Milled Ti-Abutment SV-NP Rotational
47	PMMD-SV-NPR	Pre-Milled Ti-Abutment SV-NP Rotational
48	PMNT-SV-NPR	Pre-Milled Ti-Abutment SV-NP Rotational
49	PMAM-NA-NP	Pre-Milled Ti-Abutment NA-NP
50	PMMX-NA-NP	Pre-Milled Ti-Abutment NA-NP
51	PMAG-NA-NP	Pre-Milled Ti-Abutment NA-NP
52	PMZZ-NA-NP	Pre-Milled Ti-Abutment NA-NP
53	PMMD-NA-NP	Pre-Milled Ti-Abutment NA-NP
54	PMNT-NA-NP	Pre-Milled Ti-Abutment NA-NP
55	PMAM-NA-NPR	Pre-Milled Ti-Abutment NA-NP Rotational
56	PMMX-NA-NPR	Pre-Milled Ti-Abutment NA-NP Rotational
57	PMAG-NA-NPR	Pre-Milled Ti-Abutment NA-NP Rotational
58	PMZZ-NA-NPR	Pre-Milled Ti-Abutment NA-NP Rotational
59	PMMD-NA-NPR	Pre-Milled Ti-Abutment NA-NP Rotational
60	PMNT-NA-NPR	Pre-Milled Ti-Abutment NA-NP Rotational
61	PMAM-NA-RP	Pre-Milled Ti-Abutment NA-RP
62	PMMX-NA-RP	Pre-Milled Ti-Abutment NA-RP
63	PMAG-NA-RP	Pre-Milled Ti-Abutment NA-RP

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

 Pre-Milled Blank (EM)		
No	REF/ Productname	Product Description
64	PMZZ-NA-RP	Pre-Milled Ti-Abutment NA-RP
65	PMMD-NA-RP	Pre-Milled Ti-Abutment NA-RP
66	PMNT-NA-RP	Pre-Milled Ti-Abutment NA-RP
67	PMAM-NA-RPR	Pre-Milled Ti-Abutment NA-RP Rotational
68	PMMX-NA-RPR	Pre-Milled Ti-Abutment NA-RP Rotational
69	PMAG-NA-RPR	Pre-Milled Ti-Abutment NA-RP Rotational
70	PMZZ-NA-RPR	Pre-Milled Ti-Abutment NA-RP Rotational
71	PMMD-NA-RPR	Pre-Milled Ti-Abutment NA-RP Rotational
72	PMNT-NA-RPR	Pre-Milled Ti-Abutment NA-RP Rotational
73	PMAM-TS-NP	Pre-Milled Ti-Abutment TS-NP/ Mini
74	PMMX-TS-NP	Pre-Milled Ti-Abutment TS-NP/ Mini
75	PMAG-TS-NP	Pre-Milled Ti-Abutment TS-NP/ Mini
76	PMZZ-TS-NP	Pre-Milled Ti-Abutment TS-NP/ Mini
77	PMMD-TS-NP	Pre-Milled Ti-Abutment TS-NP/ Mini
78	PMNT-TS-NP	Pre-Milled Ti-Abutment TS-NP/ Mini
79	PMAM-TS-NPR	Pre-Milled Ti-Abutment TS-NP/ Mini Rotational
80	PMMX-TS-NPR	Pre-Milled Ti-Abutment TS-NP/ Mini Rotational
81	PMAG-TS-NPR	Pre-Milled Ti-Abutment TS-NP/ Mini Rotational
82	PMZZ-TS-NPR	Pre-Milled Ti-Abutment TS-NP/ Mini Rotational
83	PMMD-TS-NPR	Pre-Milled Ti-Abutment TS-NP/ Mini Rotational
84	PMNT-TS-NPR	Pre-Milled Ti-Abutment TS-NP/ Mini Rotational
85	PMAM-TS-RP	Pre-Milled Ti-Abutment TS-RP
86	PMMX-TS-RP	Pre-Milled Ti-Abutment TS-RP
87	PMAG-TS-RP	Pre-Milled Ti-Abutment TS-RP

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

 Pre-Milled Blank (EM)		
No	REF/ Productname	Product Description
88	PMZZ-TS-RP	Pre-Milled Ti-Abutment TS-RP
89	PMMD-TS-Rp	Pre-Milled Ti-Abutment TS-RP
90	PMNT-TS-RP	Pre-Milled Ti-Abutment TS-RP
91	PMAM-TS-RPR	Pre-Milled Ti-Abutment TS-RP Rotational
92	PMMX-TS-RPR	Pre-Milled Ti-Abutment TS-RP Rotational
93	PMAG-TS-RPR	Pre-Milled Ti-Abutment TS-RP Rotational
94	PMZZ-TS-RPR	Pre-Milled Ti-Abutment TS-RP Rotational
95	PMMD-TS-RPR	Pre-Milled Ti-Abutment TS-RP Rotational
96	PMNT-TS-RPR	Pre-Milled Ti-Abutment TS-RP Rotational
97	PMAM-OS-SP	Pre-Milled Ti-Abutment OS-SP
98	PMMX-OS-SP	Pre-Milled Ti-Abutment OS-SP
99	PMAG-OS-SP	Pre-Milled Ti-Abutment OS-SP
100	PMZZ-OS-SP	Pre-Milled Ti-Abutment OS-SP
101	PMMD-OS-SP	Pre-Milled Ti-Abutment OS-SP
102	PMNT-OS-SP	Pre-Milled Ti-Abutment OS-SP
103	PMAM-OS-SPR	Pre-Milled Ti-Abutment OS-SP Rotational
104	PMMX-OS-SPR	Pre-Milled Ti-Abutment OS-SP Rotational
105	PMAG-OS-SPR	Pre-Milled Ti-Abutment OS-SP Rotational
106	PMZZ-OS-SPR	Pre-Milled Ti-Abutment OS-SP Rotational
107	PMMD-OS-SPR	Pre-Milled Ti-Abutment OS-SP Rotational
108	PMNT-OS-SPR	Pre-Milled Ti-Abutment OS-SP Rotational
109	PMAM-OS-WP	Pre-Milled Ti-Abutment OS-WP
110	PMMX-OS-WP	Pre-Milled Ti-Abutment OS-WP
111	PMAG-OS-WP	Pre-Milled Ti-Abutment OS-WP


Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

 Pre-Milled Blank (EM)		
No	REF/ Productname	Product Description
112	PMZZ-OS-WP	Pre-Milled Ti-Abutment OS-WP
113	PMMD-OS-WP	Pre-Milled Ti-Abutment OS-WP
114	PMNT-OS-WP	Pre-Milled Ti-Abutment OS-WP
115	PMAM-OS-WPR	Pre-Milled Ti-Abutment OS-WP Rotational
116	PMMX-OS-WPR	Pre-Milled Ti-Abutment OS-WP Rotational
117	PMAG-OS-WPR	Pre-Milled Ti-Abutment OS-WP Rotational
118	PMZZ-OS-WPR	Pre-Milled Ti-Abutment OS-WP Rotational
119	PMMD-OS-WPR	Pre-Milled Ti-Abutment OS-WP Rotational
120	PMNT-OS-WPR	Pre-Milled Ti-Abutment OS-WP Rotational
121	PMAM-AR-RP	Pre-Milled Ti-Abutment AR-RP
122	PMMX-AR-RP	Pre-Milled Ti-Abutment AR-RP
123	PMAG-AR-RP	Pre-Milled Ti-Abutment AR-RP
124	PMZZ-AR-RP	Pre-Milled Ti-Abutment AR-RP
125	PMMD-AR-RP	Pre-Milled Ti-Abutment AR-RP
126	PMNT-AR-RP	Pre-Milled Ti-Abutment AR-RP
127	PMAM-AR-RPR	Pre-Milled Ti-Abutment AR-RP Rotational
128	PMMX-AR-RPR	Pre-Milled Ti-Abutment AR-RP Rotational
129	PMAG-AR-RPR	Pre-Milled Ti-Abutment AR-RP Rotational
130	PMZZ-AR-RPR	Pre-Milled Ti-Abutment AR-RP Rotational
131	PMMD-AR-RPR	Pre-Milled Ti-Abutment AR-RP Rotational
132	PMNT-AR-RPR	Pre-Milled Ti-Abutment AR-RP Rotational
133	PMAM-GM-RP	Pre-Milled Ti-Abutment GM-RP
134	PMMX-GM-RP	Pre-Milled Ti-Abutment GM-RP
135	PMAG-GM-RP	Pre-Milled Ti-Abutment GM-RP


Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

 Pre-Milled Blank (EM)		
No	REF/ Productname	Product Description
136	PMZZ-GM-RP	Pre-Milled Ti-Abutment GM-RP
137	PMMD-GM-RP	Pre-Milled Ti-Abutment GM-RP
138	PMNT-GM-RP	Pre-Milled Ti-Abutment GM-RP
139	PMAM-GM-RPR	Pre-Milled Ti-Abutment GM-RP Rotational
140	PMMX-GM-RPR	Pre-Milled Ti-Abutment GM-RP Rotational
141	PMAG-GM-RPR	Pre-Milled Ti-Abutment GM-RP Rotational
142	PMZZ-GM-RPR	Pre-Milled Ti-Abutment GM-RP Rotational
143	PMMD-GM-RPR	Pre-Milled Ti-Abutment GM-RP Rotational
144	PMNT-GM-RPR	Pre-Milled Ti-Abutment GM-RP Rotational
145	PMAM-CF-RC	Pre-Milled Ti-Abutment CF-RC
146	PMMD-CF-RC	Pre-Milled Ti-Abutment CF-RC
147	PMNT-CF-RC	Pre-Milled Ti-Abutment CF-RC
148	PMMX-CF-RC	Pre-Milled Ti-Abutment CF-RC
149	PMAG-CF-RC	Pre-Milled Ti-Abutment CF-RC
150	PMZZ-CF-RC	Pre-Milled Ti-Abutment CF-RC
151	PMAM-CF-RCR	Pre-Milled Ti-Abutment CF-RC Rotational
152	PMMD-CF-RCR	Pre-Milled Ti-Abutment CF-RC Rotational
153	PMNT-CF-RCR	Pre-Milled Ti-Abutment CF-RC Rotational
154	PMMX-CF-RCR	Pre-Milled Ti-Abutment CF-RC Rotational
155	PMAG-CF-RCR	Pre-Milled Ti-Abutment CF-RC Rotational
156	PMZZ-CF-RCR	Pre-Milled Ti-Abutment CF-RC Rotational
157	PMAM-CF-NC	Pre-Milled Ti-Abutment CF-NC
158	PMMD-CF-NC	Pre-Milled Ti-Abutment CF-NC
159	PMNT-CF-NC	Pre-Milled Ti-Abutment CF-NC


Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

 Pre-Milled Blank (EM)		
No	REF/ Productname	Product Description
160	PMMX-CF-NC	Pre-Milled Ti-Abutment CF-NC
161	PMAG-CF-NC	Pre-Milled Ti-Abutment CF-NC
162	PMZZ-CF-NC	Pre-Milled Ti-Abutment CF-NC
163	PMAM-CF-NCR	Pre-Milled Ti-Abutment CF-NC Rotational
164	PMMD-CF-NCR	Pre-Milled Ti-Abutment CF-NC Rotational
165	PMNT-CF-NCR	Pre-Milled Ti-Abutment CF-NC Rotational
166	PMMX-CF-NCR	Pre-Milled Ti-Abutment CF-NC Rotational
167	PMAG-CF-NCR	Pre-Milled Ti-Abutment CF-NC Rotational
168	PMZZ-CF-NCR	Pre-Milled Ti-Abutment CF-NC Rotational


*Tabelle 41 Ti-Base CAD/CAM*

 Ti-Base CAD/ CAM		
No	REF/ Productname	Product Description
1*	TBC-0.5	Ti-Base 0.5 mm Shoulder
2*	TBC-1.5	Ti-Base 1.5 mm Shoulder
3*	TBC-3	Ti-Base 3 mm Shoulder
4*	TBC-4.5	Ti-Base 4.5 mm Shoulder
5*	TBC-0.5R	Ti-Base 0.5 mm Shoulder Rotational
6*	TBC-1.5R	Ti-Base 1.5 mm Shoulder Rotational
7*	TBC-3R	Ti-Base 3 mm Shoulder Rotational
8*	TBC-4.5R	Ti-Base 4.5 mm Shoulder Rotational
9*	NL-TBC-0.5	Ti-Base 0.5 mm Shoulder Narrow Line
10*	NL-TBC-1.5	Ti-Base 1.5 mm Shoulder Narrow Line


Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

		
Ti-Base CAD/ CAM		
No	REF/ Productname	Product Description
11*	NL-TBC-3	Ti-Base 3 mm Shoulder Narrow Line
12*	NL-TBC-4.5	Ti-Base 4.5 mm Shoulder Narrow Line
13*	NL-TBC-0.5R	Ti-Base 0.5 mm Shoulder Rotational Narrow Line
14*	NL-TBC-1.5R	Ti-Base 1.5 mm Shoulder Rotational Narrow Line
15*	NL-TBC-3R	Ti-Base 3 mm Shoulder Rotational Narrow Line
16*	NL-TBC-4.5R	Ti-Base 4.5 mm Shoulder Rotational Narrow Line
17*	TBC-10-1.5A	Ti-Base 10° 1.5 mm Shoulder A
18*	TBC-10-3A	Ti-Base 10° 3 mm Shoulder A
19*	TBC-10-1.5B	Ti-Base 10° 1.5 mm Shoulder B
20*	TBC-10-3B	Ti-Base 10° 3 mm Shoulder B
21*	TBC-10-1.5R	Ti-Base 10° 1.5 mm Shoulder R
22*	TBC-10-3R	Ti-Base 10° 3 mm Shoulder R
23*	TBC-20-1.5A	Ti-Base 20° 1.5 mm Shoulder A
24*	TBC-20-3A	Ti-Base 20° 3 mm Shoulder A
25*	TBC-20-1.5B	Ti-Base 20° 1.5 mm Shoulder B
26*	TBC-20-3B	Ti-Base 20° 3 mm Shoulder B
27*	TBC-20-1.5R	Ti-Base 20° 1.5 mm Shoulder R
28*	TBC-20-3R	Ti-Base 20° 3 mm Shoulder R
29*	NL-TBC-10-1.5A	Ti-Base 10° 1.5 mm Shoulder A Narrow Line
30*	NL-TBC-10-3A	Ti-Base 10° 3 mm Shoulder A Narrow Line
31*	NL-TBC-10-1.5B	Ti-Base 10° 1.5 mm Shoulder B Narrow Line
32*	NL-TBC-10-3B	Ti-Base 10° 3 mm Shoulder B Narrow Line
33*	NL-TBC-10-1.5R	Ti-Base 10° 1.5 mm Shoulder R Narrow Line
34*	NL-TBC-10-3R	Ti-Base 10° 3 mm Shoulder R Narrow Line
35*	NL-TBC-20-1.5A	Ti-Base 20° 1.5 mm Shoulder A Narrow Line


Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

 Ti-Base CAD/ CAM		
No	REF/ Productname	Product Description
36*	NL-TBC-20-3A	Ti-Base 20° 3 mm Shoulder A Narrow Line
37*	NL-TBC-20-1.5B	Ti-Base 20° 1.5 mm Shoulder B Narrow Line
38*	NL-TBC-20-3B	Ti-Base 20° 3 mm Shoulder B Narrow Line
39*	NL-TBC-20-1.5R	Ti-Base 20° 1.5 mm Shoulder R Narrow Line
40*	NL-TBC-20-3R	Ti-Base 20° 3 mm Shoulder R Narrow Line

*Tabelle 42 Uni-Base One Piece Ti-Base (EM)*

 Uni-Base One Piece Ti-Base (EM)		
No	REF/ Productname	Product Description
1	TBC-SV-RP	UniBase SV-RP
2	TBC-SV-RP15	UniBase SV-RP H 1.5 mm
3	TBC-SV-RP30	UniBase SV-RP H 3 mm
4	TBC-SV-RP45	UniBase SV-RP H 4.5 mm
5	TBC-SV-RPR	UniBase SV-RP Rotational
6	TBC-SV-RP15R	UniBase SV-RP H 1.5 mm Rotational
7	TBC-SV-RP30R	UniBase SV-RP H 3 mm Rotational
8	TBC-SV-RP45R	UniBase SV-RP H 4.5 mm Rotational
9	TBC-SV- RP1015A	UniBase SV-RP 10° H 1.5 mm A Type
10	TBC-SV- RP1030A	UniBase SV-RP 10° H 3 mm A Type
11	TBC-SV- RP1015B	UniBase SV-RP 10° H 1.5 mm B Type
12	TBC-SV- RP1030B	UniBase SV-RP 10° H 3 mm B Type


Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

		
Uni-Base One Piece Ti-Base (EM)		
No	REF/ Productname	Product Description
13	TBC-SV- RP1015R	UniBase SV-RP 10° H 1.5 mm Rotational
14	TBC-SV- RP1030R	UniBase SV-RP 10° H 3 mm Rotational
15	TBC-SV- RP2015A	UniBase SV-RP 20° H 1.5 mm A Type
16	TBC-SV- RP2030A	UniBase SV-RP 20° H 3 mm A Type
17	TBC-SV- RP2015UA	UniBase SV-RP 20° H 1.5 mm A Type Anterior
18	TBC-SV- RP2015B	UniBase SV-RP 20° H 1.5 mm B Type
19	TBC-SV- RP2030B	UniBase SV-RP 20° H 3 mm B Type
20	TBC-SV- RP2015UB	UniBase SV-RP 20° H 1.5 mm B Type Anterior
21	TBC-SV- RP2015R	UniBase SV-RP 20° H 1.5 mm Rotational
22	TBC-SV- RP2030R	UniBase SV-RP 20° H 3 mm Rotational
23	TBC-SV- RP2015UR	UniBase Shape SV-RP 20° H 1.5 mm Rotational Anterior
24	TBC-SV-WP	UniBase SV-WP
25	TBC-SV-WP15	UniBase SV-WP H 1.5 mm
26	TBC-SV-WP30	UniBase SV-WP H 3 mm
27	TBC-SV-WP45	UniBase SV-WP H 4.5 mm
28	TBC-SV-WPR	UniBase SV-WP Rotational
29	TBC-SV-WP15R	UniBase SV-WP H 1.5 mm Rotational
30	TBC-SV-WP30R	UniBase SV-WP H 3 mm Rotational
31	TBC-SV-WP45R	UniBase SV-WP H 4.5 mm Rotational
32	TBC-SV- WP1015A	UniBase SV-WP 10° H 1.5 mm A Type


Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

 Uni-Base One Piece Ti-Base (EM)		
No	REF/ Productname	Product Description
33	TBC-SV-WP1030A	UniBase SV-WP 10° H 3 mm A Type
34	TBC-SV-WP1015B	UniBase SV-WP 10° H 1.5 mm B Type
35	TBC-SV-WP1030B	UniBase SV-WP 10° H 3 mm B Type
36	TBC-SV-WP1015R	UniBase SV-WP 10° H 1.5 mm Rotational
37	TBC-SV-WP1030R	UniBase SV-WP 10° H 3 mm Rotational
38	TBC-SV-WP2015A	UniBase SV-WP 20° H 1.5 mm A Type
39	TBC-SV-WP2030A	UniBase SV-WP 20° H 3 mm A Type
40	TBC-SV-WP2045A	UniBase SV-WP 20° H 4.5 mm A Type
41	TBC-SV-WP2015UA	UniBase SV-WP 20° H 1.5 mm A Type Anterior
42	TBC-SV-WP2015B	UniBase SV-WP 20° H 1.5 mm B Type
43	TBC-SV-WP2030B	UniBase SV-WP 20° H 3 mm B Type
44	TBC-SV-WP2045B	UniBase SV-WP 20° H 4.5 mm B Type
45	TBC-SV-WP2015UB	UniBase SV-WP 20° H 1.5 mm B Type Anterior
46	TBC-SV-WP2015R	UniBase SV-WP 20° H 1.5 mm Rotational
47	TBC-SV-WP2030R	UniBase SV-WP 20° H 3 mm Rotational
48	TBC-SV-WP2045R	UniBase SV-WP 20° H 4.5 mm Rotational
49	TBC-SV-WP2015UR	UniBase SV-WP 20° H 1.5 mm Rotational Anterior


Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

		
Uni-Base One Piece Ti-Base (EM)		
No	REF/ Productname	Product Description
50	TBC-SV-NL	UniBase SV-NL
51	TBC-SV-NL15	UniBase SV-NL H 1.5 mm
52	TBC-SV-NL30	UniBase SV-NL H 3 mm
53	TBC-SV-NL45	UniBase SV-NL H 4.5 mm
54	TBC-SV-NLR	UniBase SV-NL Rotational
55	TBC-SV-NL15R	UniBase SV-NL H 1.5 mm Rotational
56	TBC-SV-NL30R	UniBase SV-NL H 3 mm Rotational
57	TBC-SV-NL45R	UniBase SV-NL H 4.5 mm Rotational
58	TBC-SV- NL1015A	UniBase SV-NL 10° H 1.5 mm A Type
59	TBC-SV- NL1030A	UniBase SV-NL 10° H 3 mm A Type
60	TBC-SV- NL1015B	UniBase SV-NL 10° H 1.5 mm B Type
61	TBC-SV- NL1030B	UniBase SV-NL 10° H 3 mm B Type
62	TBC-SV- NL1015R	UniBase SV-NL 10° H 1.5 mm Rotational
63	TBC-SV- NL1030R	UniBase SV-NL 10° H 3 mm Rotational
64	TBC-SV- NL2015A	UniBase SV-NL 20° H 1.5 mm A Type
65	TBC-SV- NL2030A	UniBase SV-NL 20° H 3 mm A Type
66	TBC-SV- NL2015UA	UniBase Shape SV-NL 20° H 1.5 mm A Type Anterior
67	TBC-SV- NL2015B	UniBase SV-NL 20° H 1.5 mm B Type
68	TBC-SV- NL2030B	UniBase SV-NL 20° H 3 mm B Type


Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

		
Uni-Base One Piece Ti-Base (EM)		
No	REF/ Productname	Product Description
69	TBC-SV-NL2015UB	UniBase Shape SV-NL 20° H 1.5 mm B Type Anterior
70	TBC-SV-NL2015R	UniBase SV-NL 20° H 1.5 mm Rotational
71	TBC-SV-NL2030R	UniBase SV-NL 20° H 3 mm Rotational
72	TBC-SV-NL2015R	UniBase Shape SV-NL 20° H 1.5 mm Rotational Anterior
73	TBC-SV-NP	UniBase SV-NP
74	TBC-SV-NP15	UniBase SV-NP H 1.5 mm
75	TBC-SV-NP30	UniBase SV-NP H 3 mm
76	TBC-SV-NP45	UniBase SV-NPH 4.5 mm
77	TBC-SV-NPR	UniBase SV-NP Rotational
78	TBC-SV-NP15R	UniBase SV-NP H 1.5 mm Rotational
79	TBC-SV-NP30R	UniBase SV-NP H 3 mm Rotational
80	TBC-SV-NP45R	UniBase SV-NP H 4.5 mm Rotational
81	TBC-SV-NP1015A	UniBase SV-NP 10° H 1.5 mm A Type
82	TBC-SV-NP1030A	UniBase SV-NP 10° H 3 mm A Type
83	TBC-SV-NP1015B	UniBase SV-NP 10° H 1.5 mm B Type
84	TBC-SV-NP1030B	UniBase SV-NP 10° H 3 mm B Type
85	TBC-SV-NP1015R	UniBase SV-NP 10° H 1.5 mm Rotational
86	TBC-SV-NP1030R	UniBase SV-NP 10° H 3 mm Rotational
87	TBC-SV-NP2015A	UniBase SV-NP 20° H 1.5 mm A Type
88	TBC-SV-NP2030A	UniBase SV-NP 20° H 3 mm A Type

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

		
Uni-Base One Piece Ti-Base (EM)		
No	REF/ Productname	Product Description
89	TBC-SV- NP2015UA	UniBase Shape SV-NP 20° H 1.5 mm A Type Anterior
90	TBC-SV- NP2015B	UniBase SV-NP 20° H 1.5 mm B Type
91	TBC-SV- NP2030B	UniBase SV-NP 20° H 3 mm B Type
92	TBC-SV- NP2015UB	UniBase Shape SV-NP 20° H 1.5 mm B Type Anterior
93	TBC-SV- NP2015R	UniBase SV-NP 20° H 1.5 mm Rotational
94	TBC-SV- NP2030R	UniBase SV-NP 20° H 1.5 mm Rotational
95	TBC-SV- NP2015UR	UniBase Shape SV-NP 20° H 1.5 mm Rotational Anterior
96	TBC-NA-NP	UniBase NA-NP
97	TBC-NA-NP15	UniBase NA-NPH 1.5 mm
98	TBC-NA-NP30	UniBase NA-NP H 3 mm
99	TBC-NA-NP45	UniBase NA-NP H 4.5 mm
100	TBC-NA-NPR	UniBase NA-NP Rotational
101	TBC-NA-NP15R	UniBase NA-NP H 1.5 mm Rotational
102	TBC-NA-NP30R	UniBase NA-NP H 3 mm Rotational
103	TBC-NA-NP45R	UniBase NA-NP H 4.5 mm Rotational
104	TBC-NA- NP1015A	UniBase NA-NP 10° H 1.5 mm A Type
105	TBC-NA- NP1030A	UniBase NA-NP 10° H 3 mm A Type
106	TBC-NA- NP1015B	UniBase NA-NP 10° H 1.5 mm B Type
107	TBC-NA- NP1030B	UniBase NA-NP 10° H 3 mm B Type

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

 Uni-Base One Piece Ti-Base (EM)		
No	REF/ Productname	Product Description
108	TBC-NA-NP1015R	UniBase NA-NP 10° H 1.5 mm Rotational
109	TBC-NA-NP1030R	UniBase NA-NP 10° H 3 mm Rotational
110	TBC-NA-NP2015A	UniBase NA-NP 20° H 1.5 mm A Type
111	TBC-NA-NP2030A	UniBase NA-NP 20° H 3 mm A Type
112	TBC-NA-NP2015UA	UniBase NA-NP 20° H 1.5 mm A Type Anterior
113	TBC-NA-NP2015B	UniBase NA-NP 20° H 1.5 mm B Type
114	TBC-NA-NP2030B	UniBase NA-NP 20° H 3 mm B Type
115	TBC-NA-NP2015UB	UniBase NA-NP 20° H 1.5 mm B Type Anterior
116	TBC-NA-NP2015R	UniBase NA-NP 20° H 1.5 mm Rotational
117	TBC-NA-NP2030R	UniBase NA-NP 20° H 3 mm Rotational
118	TBC-NA-NP2015UR	UniBase Shape NA-NP 20° H 1.5 mm Rotational Anterior
119	TBC-NA-RP	UniBase NA-RP
120	TBC-NA-RP15	UniBase NA-RP H 1.5 mm
121	TBC-NA-RP30	UniBase NA-RP H 3 mm
122	TBC-NA-RP45	UniBase NA-RP H 4.5 mm
123	TBC-NA-RPR	UniBase NA-RP Rotational
124	TBC-NA-RP15R	UniBase NA-RP H 1.5 mm Rotational
125	TBC-NA-RP30R	UniBase NA-RP H 3 mm Rotational
126	TBC-NA-RP45R	UniBase NA-RP H 4.5 mm Rotational


Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

 Uni-Base One Piece Ti-Base (EM)		
No	REF/ Productname	Product Description
127	TBC-NA-RP1015A	UniBase NA-RP 10° H 1.5 mm A Type
128	TBC-NA-RP1030A	UniBase NA-RP 10° H 3 mm A Type
129	TBC-NA-RP1015B	UniBase NA-RP 10° H 1.5 mm B Type
130	TBC-NA-RP1030B	UniBase NA-RP 10° H 3 mm B Type
131	TBC-NA-RP1015R	UniBase NA-RP 10° H 1.5 mm Rotational
132	TBC-NA-RP1030R	UniBase NA-RP 10° H 3mm Rotational
133	TBC-NA-RP2015A	UniBase NA-RP 20° H 1.5 mm A Type
134	TBC-NA-RP2013A	UniBase NA-RP 20° H 3 mm A Type
135	TBC-NA-RP2015UA	UniBase NA-RP 20° H 1.5 mm A Type Anterior
136	TBC-NA-RP2015B	UniBase NA-RP 20° H 1.5 mm B Type
137	TBC-NA-RP2030B	UniBase NA-RP 20° H 3 mm B Type
138	TBC-NA-RP2015UB	UniBase NA-RP 20° H 1.5 mm B Type Anterior
139	TBC-NA-RP	UniBase NA-RP 20° H 1.5 mm Rotational
140	TBC-NA-RP	UniBase NA-RP 20° H 3 mm Rotational
141	TBC-NA-RP2015UR	UniBase Shape NA-RP 20° H 1.5 mm Rotational Anterior
142	TBC-TS-NP	UniBase TS-NP/ Mini
143	TBC-TS-NP15	UniBase TS-NP/ Mini H 1.5 mm
144	TBC-TS-NP30	UniBase TS-NP/ Mini H 3 mm


Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

 Uni-Base One Piece Ti-Base (EM)		
No	REF/ Productname	Product Description
145	TBC-TS-NP45	UniBase TS-NP/ Mini H 4.5 mm
146	TBC-TS-NPR	UniBase TS-NP/ Mini Rotational
147	TBC-TS-NP15R	UniBase TS-NP/ Mini H 1.5 mm Rotational
148	TBC-TS-NP30R	UniBase TS-NP/ Mini H 3 mm Rotational
149	TBC-TS-NP45R	UniBase TS-NP/ Mini H 4.5 mm Rotational
150	TBC-TS- NP1015A	UniBase TS-NP/ Mini 10° H 1.5 mm A Type
151	TBC-TS- NP1030A	UniBase TS-NP/ Mini 10° H 3 mm A Type
152	TBC-TS- NP1015B	UniBase TS-NP/ Mini 10° H 1.5 mm B Type
153	TBC-TS- NP1030B	UniBase TS-NP/ Mini 10° H 3 mm B Type
154	TBC-TS- NP1015R	UniBase TS-NP/ Mini 10° H 1.5 mm Rotational
155	TBC-TS- NP1030R	UniBase TS-NP/ Mini 10° H 3 mm Rotational
156	TBC-TS- NP2015A	UniBase TS-NP/ Mini 20° H 1.5 mm A Type
157	TBC-TS- NP2030A	UniBase TS-NP/ Mini 20° H 3 mm A Type
158	TBC-TS- NP2045A	UniBase TS-NP/ Mini 20° H 4.5 mm A Type
159	TBC-TS-NP- 2015UA	UniBase TS-NP/ Mini 20° H 1.5 mm A Type Anterior
160	TBC-TS-NP- 2015B	UniBase TS-NP/ Mini 20° H 1.5 mm B Type
161	TBC-TS- NP2030B	UniBase TS-NP/ Mini 20° H 3 mm B Type
162	TBC-TS- NP2015UB	UniBase TS-NP/ Mini 20° H 1.5 mm B Type Anterior

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

 Uni-Base One Piece Ti-Base (EM)		
No	REF/ Productname	Product Description
163	TBC-TS-NP-2015R	UniBase TS-NP/ Mini 20° H 1.5 mm Rotational
164	TBC-TS-NP2030R	UniBase TS-NP/ Mini 20° H 3 mm Rotational
165	TBC-TS-NP2045R	UniBase TS-NP/ Mini 20° H 4.5 mm Rotational
166	TBC-TS-NP2015UR	UniBase Shape TS-NP/ Mini 20° H 1.5 mm Rotational Anterior
167	TBC-TS-RP	UniBase TS-RP
168	TBC-TS-RP15	UniBase TS-RP H 1.5 mm
169	TBC-TS-RP30	UniBase TS-RP H 3 mm
170	TBC-TS-RP45	UniBase TS-RP H 4.5 mm
171	TBC-TS-RPR	UniBase TS-RP Rotational
172	TBC-TS-RP15R	UniBase TS-RP H 1.5 mm Rotational
173	TBC-TS-RP30R	UniBase TS-RP H 3 mm Rotational
174	TBC-TS-RP45R	UniBase TS-RP H 4.5 mm Rotational
175	TBC-TS-RP1015A	UniBase TS-RP 10° H 1.5 mm A Type
176	TBC-TS-RP1030A	UniBase TS-RP 10° H 3 mm A Type
177	TBC-TS-RP1015B	UniBase TS-RP 10° H 1.5 mm B Type
178	TBC-TS-RP1030B	UniBase TS-RP 10° H 3 mm B Type
179	TBC-TS-RP1015R	UniBase TS-RP 10° H 1.5 mm Rotational
180	TBC-TS-RP1030R	UniBase TS-RP 10° H 3 mm Rotational
181	TBC-TS-RP2015A	UniBase TS-RP 20° H 1.5 mm A Type
182	TBC-TS-RP2030A	UniBase TS-RP 20° H 3 mm A Type

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

		
Uni-Base One Piece Ti-Base (EM)		
No	REF/ Productname	Product Description
183	TBC-TS-RP2015UA	UniBase TS-RP 20° H 1.5 mm A Type Anterior
184	TBC-TS-RP2015B	UniBase TS-RP 20° H 1.5 mm B Type
185	TBC-TS-RP2030B	UniBase TS-RP 20° H 3 mm B Type
186	TBC-TS-RP2015UB	UniBase TS-RP 20° H 1.5 mm B Type Anterior
187	TBC-TS-RP2015R	UniBase TS-RP 20° H 1.5 mm Rotational
188	TBC-TS-RP2030R	UniBase TS-RP 20° H 3 mm Rotational
189	TBC-TS-RP2015UR	UniBase Shape TS-RP 20° H 1.5 mm Rotational
190	TBC-OS-SP	UniBase OS-SP
191	TBC-OS-SP15	UniBase OS-SP H 1.5 mm
192	TBC-OS-SP30	UniBase OS-SP H 3 mm
193	TBC-OS-SP45	UniBase OS-SP H 4.5 mm
194	TBC-OS-SPR	UniBase OS-SP Rotational
195	TBC-OS-SP15R	UniBase OS-SP H 1.5 mm Rotational
196	TBC-OS-SP30R	UniBase OS-SP H 3 mm Rotational
197	TBC-OS-SP45R	UniBase OS-SP H 4.5 mm Rotational
198	TBC-OS-SP1015A	UniBase OS-SP 10° H 1.5 mm A Type
199	TBC-OS-SP1030A	UniBase OS-SP 10° H 3 mm A Type
200	TBC-OS-SP1015B	UniBase OS-SP 10° H 1.5 mm B Type
201	TBC-OS-SP1030B	UniBase OS-SP 10° H 3 mm B Type

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

 Uni-Base One Piece Ti-Base (EM)		
No	REF/ Productname	Product Description
202	TBC-OS-SP1015R	UniBase OS-SP 10° H 1.5 mm Rotational
203	TBC-OS-SP1030R	UniBase OS-SP 10° H 3 mm Rotational
204	TBC-OS-SP2015A	UniBase OS-SP 20° H 1.5 mm A Type
205	TBC-OS-SP2030A	UniBase OS-SP 20° H 3 mm A Type
206	TBC-OS-SP2015UA	UniBase OS-SP 20° H 1.5 mm A Type Anterior
207	TBC-OS-SP2015B	UniBase OS-SP 20° H 1.5 mm B Type
208	TBC-OS-SP2030B	UniBase OS-SP 20° H 3 mm B Type
209	TBC-OS-SP2015UB	UniBase OS-SP 20° H 1.5 mm B Type Anterior
210	TBC-OS-SP2015R	UniBase OS-SP 20° H 1.5 mm Rotational
211	TBC-OS-SP2030R	UniBase OS-SP 20° H 3 mm Rotational
212	TBC-OS-SP2015UR	UniBase OS-SP 20° H 1.5 mm Rotational Anterior
213	TBC-OS-WP	UniBase OS-WP
214	TBC-OS-WP15	UniBase OS-WP H 1.5 mm
215	TBC-OS-WP30	UniBase OS-WP H 3 mm
216	TBC-OS-WP45	UniBase OS-WP H 4.5 mm
217	TBC-OS-WPR	UniBase OS-WP Rotational
218	TBC-OS-WP15R	UniBase OS-WP H 1.5 mm Rotational
219	TBC-OS-WP30R	UniBase OS-WP H 3 mm Rotational
220	TBC-OS-WP45R	UniBase OS-WP H 4.5 mm Rotational

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

		
Uni-Base One Piece Ti-Base (EM)		
No	REF/ Productname	Product Description
221	TBC-OS-WP1015A	UniBase OS-WP 10° H 1.5 mm A Type
222	TBC-OS-WP1030A	UniBase OS-WP 10° H 3 mm A Type
223	TBC-OS-WP1015B	UniBase OS-WP 10° H 1.5 mm B Type
224	TBC-OS-WP1030B	UniBase OS-WP 10° H 3 mm B Type
225	TBC-OS-WP1015R	UniBase OS-WP 10° H 1.5 mm Rotational
226	TBC-OS-WP1030R	UniBase OS-WP 10° H 3 mm Rotational
227	TBC-OS-WP2015A	UniBase OS-WP 20° H 1.5 mm A Type
228	TBC-OS-WP2030A	UniBase OS-WP 20° H 3 mm A Type
229	TBC-OS-WP2015UA	UniBase OS-WP 20° H 1.5 mm A Type Anterior
230	TBC-OS-WP2015B	UniBase OS-WP 20° H 1.5 mm B Type
231	TBC-OS-WP2030B	UniBase OS-WP 20° H 3 mm B Type
232	TBC-OS-WP2015UB	UniBase OS-WP 20° H 1.5 mm B Type
233	TBC-OS-WP2015R	UniBase OS-WP 20° H 1.5 mm Rotational
234	TBC-OS-WP2030R	UniBase OS-WP 20° H 3 mm Rotational
235	TBC-OS-WP2015UR	UniBase OS-WP 20° H 1.5 mm Rotational Anterior
236	TBC-AR-RP	UniBase AR-RP
237	TBC-AR-RP15	UniBase AR-RP H 1.5 mm

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

 Uni-Base One Piece Ti-Base (EM)		
No	REF/ Productname	Product Description
238	TBC-AR-RP30	UniBase AR-RP H 3 mm
239	TBC-AR-RP45	UniBase AR-RP H 4.5 mm
240	TBC-AR-RPR	UniBase AR-RP Rotational
241	TBC-AR-RP15R	UniBase AR-RP H 1.5 mm Rotational
242	TBC-AR-RP30R	UniBase AR-RP H 3 mm Rotational
243	TBC-AR-RP45R	UniBase AR-RP H 4.5 mm Rotational
244	TBC-AR- RP1015A	UniBase AR-RP 10° H 1.5 mm A Type
245	TBC-AR- RP1030A	UniBase AR-RP 10° H 3 mm A Type
246	TBC-AR- RP1015B	UniBase AR-RP 10° H 1.5 mm B Type
247	TBC-AR- RP1030B	UniBase AR-RP 10° H 3 mm B Type
248	TBC-AR- RP1015R	UniBase AR-RP 10° H 1.5 mm Rotational
249	TBC-AR- RP1030R	UniBase AR-RP 10° H 3 mm Rotational
250	TBC-AR- RP2015A	UniBase AR-RP 20° H 1.5 mm A Type
251	TBC-AR- RP2030A	UniBase AR-RP 20° H 3 mm A Type
252	TBC-AR- RP2015UA	UniBase AR-RP 20° H 1.5 mm A Type Anterior
253	TBC-AR- RP2015B	UniBase AR-RP 20° H 1.5 mm B Type
254	TBC-AR- RP2030B	UniBase AR-RP 20° H 3 mm B Type
255	TBC-AR- RP2015UB	UniBase AR-RP 20° H 1.5 mm B Type Anterior
256	TBC-AR- RP2015R	UniBase AR-RP 20° H 1.5 mm Rotational

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

		
Uni-Base One Piece Ti-Base (EM)		
No	REF/ Productname	Product Description
257	TBC-AR- RP2030R	UniBase AR-RP 20° H 3 mm Rotational
258	TBC-AR- RP2015UR	UniBase AR-RP 20° H 1.5 mm Rotational Anterior
259	TBC-GM-RP	UniBase GM-RP
260	TBC-GM-RP15	UniBase GM-RP H 1.5 mm
261	TBC-GM-RP30	UniBase GM-RP H 3 mm
262	TBC-GM-RP45	UniBase GM-RP H 4.5 mm
263	TBC-GM-RPR	UniBase GM-RP Rotational
264	TBC-GM-RP15R	UniBase GM-RP H 1.5 mm Rotational
265	TBC-GM-RP30R	UniBase GM-RP H 3 mm Rotational
266	TBC-GM-RP45R	UniBase GM-RP H 4.5 mm Rotational
267	TBC-GM- RP1015A	UniBase GM-RP 10° H 1.5 mm A Type
268	TBC-GM- RP1030A 1	UniBase GM-RP 10° H 3 mm A Type
269	TBC-GM- RP1015B	UniBase GM-RP 10° H 1.5 mm B Type
269	TBC-GM- RP1030B	UniBase GM-RP 10° H 3 mm B Type
270	TBC-GM- RP1015R	UniBase GM-RP 10° H 1.5 mm Rotational
271	TBC-GM- RP1030R	UniBase GM-RP 10° H 3 mm Rotational
272	TBC-GM- RP2015A	UniBase GM-RP 20° H 1.5 mm A Type
273	TBC-GM- RP2030A	UniBase GM-RP 20° H 3 mm A Type
274	TBC-GM- RP2015UA	UniBase GM-RP 20° H 1.5 mm A Type Anterior

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

 Uni-Base One Piece Ti-Base (EM)		
No	REF/ Productname	Product Description
275	TBC-GM-RP2015B	UniBase GM-RP 20° H 1.5 mm B Type
276	TBC-GM-RP2030B	UniBase GM-RP 20° H 3 mm B Type
277	TBC-GM-RP2015UB	UniBase GM-RP 20° H 1.5 mm B Type Anterior
278	TBC-GM-RP2015R	UniBase GM-RP 20° H 1.5 mm Rotational
279	TBC-GM-RP2030R	UniBase GM-RP 20° H 3 mm Rotational
280	TBC-GM-RP2015UR	UniBase GM-RP 20° H 1.5 mm Rotational Anterior
281	TBC-CF-RC	UniBase CF-RC
282	TBC-CF-RC15	UniBase CF-RC H 1.5 mm
283	TBC-CF-RC30	UniBase CF-RC H 3 mm
284	TBC-CF-RC45	UniBase CF-RC H 4.5 mm
285	TBC-CF-RCR	UniBase CF-RC Rotational
286	TBC-CF-RC15R	UniBase CF-RC H 1.5 mm Rotational
287	TBC-CF-RC30R	UniBase CF-RC H 3 mm Rotational
288	TBC-CF-RC45R	UniBase CF-RC H 4.5 mm Rotational
289	TBC-CF-RC1015A	UniBase CF-RC 10° H 1.5 mm A Type
290	TBC-CF-RC1030A	UniBase CF-RC 10° H 3 mm A Type
291	TBC-CF-RC1015B	UniBase CF-RC 10° H 1.5 mm B Type
292	TBC-CF-RC1030B	UniBase CF-RC 10° H 3 mm B Type
293	TBC-CF-RC1015R	UniBase CF-RC 10° H 1.5 mm Rotational

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

		
Uni-Base One Piece Ti-Base (EM)		
No	REF/ Productname	Product Description
294	TBC-CF-RC1030R	UniBase CF-RC 10° H 3 mm Rotational
295	TBC-CF-RC2015A	UniBase CF-RC 20° H 1.5 mm A Type
296	TBC-CF-RC2030A	UniBase CF-RC 20° H 3 mm B Type
297	TBC-CF-RC2015UA	UniBase Shape CF-RC 20° H 1.5 mm A Type Anterior
298	TBC-CF-RC2015B	UniBase CF-RC 20° H 1.5 mm B Type
299	TBC-CF-RC2030B	UniBase CF-RC 20° H 3 mm B Type
300	TBC-CF-RC2015UB	UniBase Shape CF-RC 20° H 1.5 mm B Type Anterior
301	TBC-CF-RC2015R	UniBase Shape CF-RC 20° H 1.5 mm Rotational
302	TBC-CF-RC2030R	UniBase Shape CF-RC 20° H 3 mm Rotational
303	TBC-CF-RC2015UR	UniBase Shape CF-RC 20° H 1.5 mm Rotational Anterior
304	TBC-CF-NC	UniBase CF-NC
305	TBC-CF-NC15	UniBase CF-NC H 1.5 mm
306	TBC-CF-NC30	UniBase CF-NC H 3 mm
307	TBC-CF-NC45	UniBase CF-NC H 4.5 mm
308	TBC-CF-NCR	UniBase CF-NC Rotational
309	TBC-CF-NC15R	UniBase CF-NC H 1.5 mm Rotational
310	TBC-CF-NC30R	UniBase CF-NC H 3 mm Rotational
311	TBC-CF-NC45R	UniBase CF-NC H 4.5 mm Rotational
312	TBC-CF-NC1015A	UniBase CF-NC 10° H 1.5 mm A Type

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

 Uni-Base One Piece Ti-Base (EM)		
No	REF/ Productname	Product Description
313	TBC-CF-NC1030A	UniBase CF-NC 10° H 3 mm A Type
314	TBC-CF-NC1015B	UniBase CF-NC 10° H 1.5 mm B Type
315	TBC-CF-NC1030B	UniBase CF-NC 10° H 3 mm B Type
316	TBC-CF-NC1015R	UniBase CF-NC 10° H 1.5 mm Rotational
317	TBC-CF-NC1030R	UniBase CF-NC 10° H 3 mm Rotational
318	TBC-CF-NC2015A	UniBase CF-NC 20° H 1.5 mm A Type
319	TBC-CF-NC2030A	UniBase CF-NC 20° H 3 mm A Type
320	TBC-CF-NC2015UA	UniBase Shape CF-NC 20° H 1.5 mm A Type Anterior
321	TBC-CF-NC2015B	UniBase CF-NC 20° H 1.5 mm B Type
322	TBC-CF-NC2030B	UniBase CF-NC 20° H 3 mm B Type
323	TBC-CF-NC2015UB	UniBase Shape CF-NC 20° H 1.5 mm B Type Anterior
324	TBC-CF-NC2015R	UniBase CF-NC 20° H 1.5 mm Rotational
325	TBC-CF-NC2030R	UniBase CF-NC 20° H 3 mm Rotational
326	TBC-CF-NC2015UR	UniBase Shape CF-NC 20° H 1.5 mm Rotational Anterior
	TB-SV-RP	Ti-Base CAD/CAM Interface SV-RP
	TB-SV-RP15	Ti-Base CAD/CAM Interface SV-RP H 1.5 mm
	TB-SV-RP30	Ti-Base CAD/CAM Interface SV-RP H 3 mm
	TB-SV-RPR	Ti-Base CAD/CAM Interface SV-RP H 1.5 mm Rotational

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

 Uni-Base One Piece Ti-Base (EM)		
No	REF/ Productname	Product Description
	TB-SV-RP15R	Ti-Base CAD/CAM Interface SV-RP H 1.5 mm Rotational
	TB-SV-RP30R	Ti-Base CAD/CAM Interface SV-RP H 3 mm Rotational
	TB-SV-WP	Ti-Base CAD/CAM Interface SV-WP
	TB-SV-WP15	Ti-Base CAD/CAM Interface SV-WP H 1.5 mm
	TB-SV-WP30	Ti-Base CAD/CAM Interface SV-WP H 3 mm
	TB-SV-WPR	Ti-Base CAD/CAM Interface SV-WP Rotational
	TB-SV-WP15R	Ti-Base CAD/CAM Interface SV-WP H 1.5 mm Rotational
	TB-SV-WP30R	Ti-Base CAD/CAM Interface SV-WP H 3 mm Rotational
	TB-SV-NL	Ti-Base CAD/CAM Interface SV-NL
	TB-SV-NL15	Ti-Base CAD/CAM Interface SV-NL H 1.5 mm
	TB-SV-NL30	Ti-Base CAD/CAM Interface SV-NL H 3 mm
	TB-SV-NLR	Ti-Base CAD/CAM Interface SV-NL Rotational
	TB-SV-NL15R	Ti-Base CAD/CAM Interface SV-NL H 1.5 mm Rotational
	TB-SV-NL30R	Ti-Base CAD/CAM Interface SV-NL H 3 mm Rotational
	TB-SV-NP	Ti-Base CAD/CAM Interface SV-NP
	TB-SV-NP15	Ti-Base CAD/CAM Interface SV-NP H 1.5 mm
	TB-SV-NP30	Ti-Base CAD/CAM Interface SV-NP H 3 mm
	TB-SV-NPR	Ti-Base CAD/CAM Interface SV-NP Rotational
	TB-SV-NP15R	Ti-Base CAD/CAM Interface SV-NP H 1.5 mm Rotational
	TB-SV-NP30R	Ti-Base CAD/CAM Interface SV-NP H 3mm Rotational
	TB-NA-NP	Ti-Base CAD/CAM Interface NA-NP
	TB-NA-NP15	Ti-Base CAD/CAM Interface NA-NP H 1.5 mm
	TB-NA-NP30	Ti-Base CAD/CAM Interface NA-NP H 3 mm
	TB-NA-NPR	Ti-Base CAD/CAM Interface NA-NP Rotational
	TB-NA-NP15R	Ti-Base CAD/CAM Interface NA-NP H 1.5 mm Rotational


Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

		
Uni-Base One Piece Ti-Base (EM)		
No	REF/ Productname	Product Description
	TB-NA-NP30R	Ti-Base CAD/CAM Interface NA-NP H 3 mm Rotational
	TB-NA-RP	Ti-Base CAD/CAM Interface NA-RP
	TB-NA-RP15	Ti-Base CAD/CAM Interface NA-RP H 1.5 mm
	TB-NA-RP30	Ti-Base CAD/CAM Interface NA-RP H 3 mm
	TB-NA-RPR	Ti-Base CAD/CAM Interface NA-RP Rotational
	TB-NA-RP15R	Ti-Base CAD/CAM Interface NA-RP H 1.5 mm Rotational
	TB-NA-RP30R	Ti-Base CAD/CAM Interface NA-RP H 3 mm Rotational
	TB-TS-NP	Ti-Base CAD/CAM Interface TS-NP/ Mini
	TB-TS-NP15	Ti-Base CAD/CAM Interface TS-NP/ Mini H 1.5 mm
	TB-TS-NP30	Ti-Base CAD/CAM Interface TS-NP/ Mini H 3 mm
	TB-TS-NPR	Ti-Base CAD/CAM Interface TS-NP/ Mini Rotational
	TB-TS-NP15R	Ti-Base CAD/CAM Interface TS-NP/ Mini H 1.5 mm Rotational
	TB-TS-NP30R	Ti-Base CAD/CAM Interface TS-NP/ Mini H 3 mm Rotational
	TB-TS-RP	Ti-Base CAD/CAM Interface TS-RP
	TB-TS-RP15	Ti-Base CAD/CAM Interface TS-RP H 1.5 mm
	TB-TS-RP30	Ti-Base CAD/CAM Interface TS-RP H 3 mm
	TB-TS-RPR	Ti-Base CAD/CAM Interface TS-RP Rotational
	TB-TS-RP15R	Ti-Base CAD/CAM Interface TS-RP H 1.5 mm Rotational
	TB-TS-RP30R	Ti-Base CAD/CAM Interface TS-RP H 3 mm Rotational
	TB-OS-SP	Ti-Base CAD/CAM Interface OS-SP
	TB-OS-SP15	Ti-Base CAD/CAM Interface OS-SP H 1.5 mm
	TB-OS-SP30	Ti-Base CAD/CAM Interface OS-SP H 3 mm
	TB-OS-SPR	Ti-Base CAD/CAM Interface OS-SP Rotational
	TB-OS-SP15R	Ti-Base CAD/CAM Interface OS-SP H 1.5 mm Rotational
	TB-OS-SP30R	Ti-Base CAD/CAM Interface OS-SP H 3 mm Rotational


Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

 Uni-Base One Piece Ti-Base (EM)		
No	REF/ Productname	Product Description
	TB-OS-WP	Ti-Base CAD/CAM Interface OS-WP
	TB-OS-WP15	Ti-Base CAD/CAM Interface OS-WP H 1.5 mm
	TB-OS-WP30	Ti-Base CAD/CAM Interface OS-WP H 3 mm
	TB-OS-WPR	Ti-Base CAD/CAM Interface OS-WP Rotational
	TB-OS-WP15R	Ti-Base CAD/CAM Interface OS-WP H 1.5 mm Rotational
	TB-OS-WP30R	Ti-Base CAD/CAM Interface OS-WP H 3 mm Rotational
	TB-AR-RP4015L	Ti-Base CAD/CAM Interface AR-RP Ø 4.0 mm H Ø 0.6 mm
	TB-AR-RP4525L	Ti-Base CAD/CAM Interface AR-RP Ø 4.4 mm H Ø 1.5 mm
	TB-AR-RP4015LR	Ti-Base CAD/CAM Interface AR-RP Ø 4.0 mm H Ø 0.6 mm Rotational
	TB-AR-RP4525LR	Ti-Base CAD/CAM Interface AR-RP Ø 4.4 mm H Ø 1.5 mm Rotational
	TB-AR-RP	Ti-Base CAD/CAM Interface AR-RP
	TB-AR-RP15	Ti-Base CAD/CAM Interface AR-RP H 1.5 mm
	TB-AR-RP30	Ti-Base CAD/CAM Interface AR-RP H 3 mm
	TB-AR-RPR	Ti-Base CAD/CAM Interface AR-RP Rotational
	TB-AR-RP15R	Ti-Base CAD/CAM Interface AR-RP H 1.5 mm Rotational
	TB-AR-RP30R	Ti-Base CAD/CAM Interface AR-RP H 3 mm Rotational
	TB-GM-RP	Ti-Base CAD/CAM Interface GM-RP
	TB-GM-RP15	Ti-Base CAD/CAM Interface GM-RP H 1.5 mm
	TB-GM-RP30	Ti-Base CAD/CAM Interface GM-RP H 3 mm
	TB-GM-RPR	Ti-Base CAD/CAM Interface GM-RP Rotational
	TB-GM-RP15R	Ti-Base CAD/CAM Interface GM-RP H 1.5 mm Rotational
	TB-GM-RP30R	Ti-Base CAD/CAM Interface GM-RP H 3 mm Rotational
	TB-CF-RC	Ti-Base CAD/CAM Interface CF-RC
	TB-CF-RC15	Ti-Base CAD/CAM Interface CF-RC H 1.5 mm

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28


 Uni-Base One Piece Ti-Base (EM)		
No	REF/ Productname	Product Description
	TB-CF-RC30	Ti-Base CAD/CAM Interface CF-RC H 3 mm
	TB-CF-RCR	Ti-Base CAD/CAM Interface CF-RC Rotational
	TB-CF-RC15R	Ti-Base CAD/CAM Interface CF-RC H 1.5 mm Rotational
	TB-CF-RC30R	Ti-Base CAD/CAM Interface CF-RC H 3 mm Rotational
	TB-CF-NC	Ti-Base CAD/CAM Interface CF-NC
	TB-CF-NC15	Ti-Base CAD/CAM Interface CF-NC H 1.5 mm
	TB-CF-NC30	Ti-Base CAD/CAM Interface CF-NC H 3 mm
	TB-CF-NCR	Ti-Base CAD/CAM Interface CF-NC Rotational
	TB-CF-NC15R	Ti-Base CAD/CAM Interface CF-NC H 1.5 mm Rotational
	TB-CF-NC30R	Ti-Base CAD/CAM Interface CF-NC H 3 mm Rotational

*Tabelle 43 Scan Bodies*


 Scan Bodies		
No	REF/ Productname	Product Description
1	3DSPA-8	Peek Abutment for 3D Scanner
2	3SDPA-8C	Peek Abutment for 3D Scanner with 1.5 mm Collar
3	NL-3DSPA-8	3D Scan Straight Peek Abutment, Narrow Line
4	NL-3DSPA-8C	3D Scan Straight Peek Abutment with Collar, Narrow Line
5	MU-KSSB	Peek Scan Body for KS Multi Unit System

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28


*Tabelle 44 Scan Abutments (EM)*

 Scan Abutments (EM)		
No	REF/ Productname	Product Description
1	3DSPA-SV-RP8C	Ritter Classic CAD/CAM Scan Body SV-RP
2	3DSPA-SV-NL8C	Ritter Classic CAD/CAM Scan Body SV-NL


*Tabelle 45 Clicq-Scan Abutment (EM)*

 Clicq-Scan Abutment (EM)		
No	REF/ Productname	Product Description
1	SBC-U	Clicq-Scan Body for TBK/ TBC Ti-Base Universal
2	SBC-TBK-A	Clicq-Scan Body for TBK/ TBC Ti-Base Type A
3	SBC-TBK-B	Clicq-Scan Body for TBK/ TBC Ti-Base Type B
4	SBC-TBK-U	Clicq-Scan Body for TBK/ TBC Ti-Base Type Universal

*Tabelle 46 Multi-Clicq Scan Abutments (EM)*

 Multi-Clicq Scan Abutment (EM)		
No	REF/ Productname	Product Description
1	MUL-KS-SBUG	Multi-Clicq CAD/CAM Peek Scan Abtument (Dual Hex/ Unigrip)
2	MUL-KS-SBAR	Multi-Clicq CAD/CAM Peek Scan Abtument Hex 1.2 mm
3	MUL-KS-SBTX	Multi-Clicq CAD/CAM Peek Scan Abtument (Torx)
4	MUL-NB-SBUG	"Classic" Multi Unit CAD/CAM Peek Scan Abutment (Dual)
5	MUL-NB-SBTX	"Classic" Multi Unit CAD/CAM Peek Scan Abutment (Torx)


Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

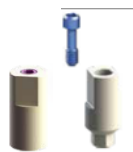

Multi-Clicq Scan Abutment (EM)

No	REF/ Productname	Product Description
6	MUL-NB-SBAR	"Classic" Multi Unit CAD/CAM Peek Scan Abutment HEX 1.2 mm
7	SB-SV-RPC	"Classic" CAD/CAM Scan Body SV-RP
8	SP-SV-RP	CAD/ CAM Scan Post SV-RP Compatible to Sirona-Cerec
9	SB-SV-RP	CAD/ CAM Scan Body SV-RP
10	SP-SV-WP	CAD/ CAM Scan Post SV-WP Compatible to Sirona-Cerec
11	SB-SV-WP	CAD/ CAM Scan Body SV-WP
12	SB-SV-NLC	"Classic" CAD/CAM Scan Body SV-NL
13	SP-SV-NL	CAD/ CAM Scan Post SV-NL Compatible to Sirona-Cerec
14	SB-SV-NL	CAD/ CAM Scan Body SV-NL
15	SP-SV-NP	CAD/ CAM Scan Post SV-NP Compatible to Sirona-Cerec
16	SB-SV-NP	CAD/ CAM Scan Body SV-NP
17	SP-NA-NP	CAD/ CAM Scan Post NA-NP Compatible to Sirona-Cerec
18	SB-NA-NP	CAD/ CAM Scan Body NA-NP
19	SP-NA-RP	CAD/ CAM Scan Post NA-RP Compatible to Sirona-Cerec
20	SB-NA-RP	CAD/ CAM Scan Body NA-RP
21	SP-TS-NP	CAD/ CAM Scan Post TS-NP/ Mini Compatible to Sirona-Cerec
22	SB-TS-NP	CAD/ CAM Scan Body TS-NP/ Mini
23	SB-ET-NP	CAD/ CAM Scan Body ET-NP/ Mini
24	SP-TS-RP	CAD/ CAM Scan Body TS-RP Compatible to Sirona-Cerec
25	SB-TS-RP	CAD/ CAM Scan Body TS-RP
26	SB-DE-RP	CAD/ CAM Scan Body DE-RP
27	SB-AO-RP	CAD/ CAM Scan Body AO-RP
28	SB-ET-RP	CAD/ CAM Scan Body ET-RP
29	SP-OS-SP	CAD/ CAM Scan Body TS-SP Compatible to Sirona-Cerec

Ritter Implants GmbH & Co. KG	<b>SSCP Part B</b>	<b>I55-0526-2301</b>
		Datum: 2023-02-28

 <p>Multi-Clicq Scan Abutment (EM)</p>		
No	REF/ Productname	Product Description
30	SB-OS-SP	CAD/ CAM Scan Body OS-SP
31	SP-OS-WP	CAD/CAM Scan Post OS-WP Compatible to Sirona-Cerec
32	SB-OS-WP	CAD/ CAM Scan Body OS-WP
33	SP-AR-RP	CAD/CAM Scan Post AR-RP Compatible to Sirona-Cerec
34	SB-AR-RP	CAD/ CAM Scan Body AR-RP
35	SP-GM-RP	CAD/CAM Scan Post GM-RP Compatible to Sirona-Cerec
36	SB-GM-RP	CAD/ CAM Scan Body GM-RP
37	SP-CF-RC	CAD/CAM Scan Post CF-RC Compatible to Sirona-Cerec
38	SB-CF-RC	CAD/ CAM Scan Body CF-RC
39	SP-CF-NC	CAD/CAM Scan Post CF-NC Compatible to Sirona-Cerec
40	SB-CF-NC	CAD/ CAM Scan Body CF-NC

## 6 Verweis

I55-0525-2301EN\_SSCP\_Part\_A (für Angehörige von Gesundheitsberufen)